展开尺寸 Expand Size	/
成型尺寸 Molding size	200mm*100mm
材质要求 MATERIAL	128g亮面铜版纸 128g glossy coated paper
印刷要求 PRINT	双面、四色印刷 Double-sided, four-color printing
工艺要求 Process requirements	胶装 Binding
公差要求 Process requirements	+/-1mm

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Writer

Name	Position	Date and Signature
Davy Lam	Product Manager	

Verification

Name	Position	Date and Signature
Andrey Mostovov	CTO	

Approval

Name	Position	Date and Signature
Stephane Mazet	CQO	

Revision history

Revision	Written by	Date	Modification description Impact
Rev00	Sabine Ferré		mise à jour adresse du siège FeetMe (last page)
Rev01	Sabine Ferré		Rajouts information charging case
Rev02	Sabine Ferré		ajout mention (section 3 User profile, page 5) "Referring health professional Prerequisite : health professional acknowledges having read and fully understood the information on uncertainty of measurement (section 15 page 13 Technical Specifications). For any further information concerning interpretations of uncertainty of measurement of the device, contact FeetMe at contact@feetme.fr "
Rev03	Sabine Ferré		multilangue
			ajouts mention Juridique, obligation contractuelle renforcée, section 10 page 10 d) Data backup "When the user is hosting the gait recordings, he is in charge of performing appropriate and regular data backups in order to prevent any destruction or loss of information. FeetMe cannot be held responsible for any accidental or unauthorised destruction or accidental loss of user data."
			Change IP57 ==> IP55 page 13
			Change the place of the reset button

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			Added training requirement for patients who use Rehabilitation
			+various changes (position of warning and caution messages, wording...)
			Added a spanish version
Rev06	Sabine Ferré	21/07/2020	Added warning about supply mains p.8 Modified case description (no more lid) p.8 Modified specifications: weight and thickness of the insoles Modified material of insole
Rev 07	Davy Lam Sabine Ferré Rebecca Davies	15/03/2021	Correction of Intended use and intended users Correction of wording that have been changed in the app buttons Modification of wordings FeetMe Rehabilitation app Add information about FCC Rules Added Rx only statement Added contraindications Addition of DC symbol to Section 1 (NCQMS-35) Addition of polyester to table of characteristics (NCQMS40)
Rev08	Davy Lam	23/03/2021	Update of Pressure measurement accuracy and minimum OS version required
Rev09	Davy Lam	06/04/2021	Removal of blank pages for printing, correction of insoles materials
Rev10	Davy Lam	22/04/2021	Fenda request : blank page at the end of Spanish version is moved as a separation page between French and Spanish version. Removal of the page number.
		23/04/2021	From FeetMe side : update of FCC information based on LCIE feedbacks. Removal of section 19 (specific to the US, from French and Spanish versions)

[Instruction for use](#)

FeetMe® Monitor

FeetMe® Evaluation & FeetMe® Rehabilitation

EN FR SP

Instruction for Use

English

Table of Contents

Revision history.....	2
1. Labels and symbols.....	3
2. About this manual	4
3. Product scope and intended use	4
4. User profile	5
5. Safety instructions.....	5
6. Product description	7
7. Operating principle.....	7
8. Product setup.....	7
9. Use of insoles with FeetMe® Evaluation	9

10. Use of insoles with FeetMe® Rehabilitation	11
11. Troubleshooting and shutdown procedure	13
12. Storage and transport.....	13
13. Cleaning and maintenance	14
14. Disposal	14
15. Technical characteristics	15
16. Recommended minimum phone features	16
17. Electromagnetic compatibility.....	16
18. Lifetime	21
19. Specific regulation	21

1. Labels and symbols

	Name and address of the manufacturer
	Date of manufacture
	Limiting temperatures for storage and transport
	Serial number
	Important safety instructions or operating instructions
	Double insulation
	Refer to the instruction manual / brochure
	Electronic equipment subject to selective collection
IP55	IP information

	Keep it dry
	Moisture limit for storage and transport
	Limiting pressure for storage and transport
	Type BF Applied Parts (this device is protected from leakage currents and electric shocks).
	Electromagnetic non-ionizing radiation. This equipment includes RF transmitters. Interference may occur around the equipment that has this symbol.
	These devices uses Bluetooth
	Medical devices in compliance with the directive 93/42 / EEC established by notified body No 0459
5V === 1A	Direct current

2. About this manual

We recommend that you read this entire manual carefully before starting to use the devices. This manual will provide you with the necessary information for a proper use and maintenance of the devices. For more information, please contact FeetMe by email at : contact@feetme.fr

- ! A **Warning** explains dangers that might result in personal injury or death.
- ! A **Caution** explains hazards that could damage a product, including data loss.

3. Product scope and intended use

FeetMe® Monitor connected insoles measure spatio-temporal gait parameters and plantar pressure to support the mobility assessment of individuals

FeetMe® Evaluation mobile application allows the real-time visualization and/or recording of spatio-temporal gait parameters

and plantar pressure to support the mobility assessment of individuals

FeetMe® Rehabilitation mobile application allows patients to perform rehabilitation exercises with real-time biofeedback

FeetMe® Monitor insoles can be used with its accessory: FeetMe®

Charging case (optional)

Contraindications :

These devices are contraindicated in:

- Patients not able to walk (e.g. wheelchair bound or bedridden),
- Patients with poor wound healing or broken skin on their feet.

Warning

! In case FeetMe® Monitor Insoles are used in patients using a walking frame, see section 15-Technical characteristics, to make sure that the pressure applied on the insoles answer the conditions of use of the insoles.

Special precautions regarding the environment of use:

- None for FeetMe® Monitor insoles
- See Section 18-Compliance for FeetMe® Charging case

4. User profile

There are two types of users:

- Health professionals with education on gait and/or posture (for example, but not limited to physicians, physiotherapists, ergotherapists, ...)
- Individuals, healthy or not, wearing the insoles and performing the tests and/or the exercises.

Individuals User Profile	
Weight	>40 kg and <120 kg
Sex	F/M
Health condition	All

Referring health professional prerequisite : health professional acknowledges having read and fully understood the information on uncertainty of measurement (Section 15 - Technical Characteristics).

For any further information concerning the interpretation of uncertainty of measurement for those devices, contact FeetMe at contact@feetme.fr.

The installation and use of FeetMe® Monitor insoles and FeetMe® Evaluation do not require a specific training. Patients using FeetMe® Rehabilitation should be trained by their health professional.

5. Safety instructions

Warning

- ! Do not use the device under any condition other than those specified in this manual.
- ! Do not put the power cable around your neck.
- ! The charger must be connected to a wall outlet.
- ! Charging the battery must be performed with the cable connected to the wall adapter specified in Section 15 - Technical characteristics. The use of a wall adapter other than the one specified creates the risk of an electric shock.

- ! Do not position the wall adapter or charging cable in a location where the unplugging is made difficult to the user.
- ! To avoid the risk of electric shock, this equipment must only be connected to a supply mains with protective earth.
- ! All FeetMe® products require special precautions with regards to electromagnetic compatibility. The devices must be installed and operated according to the electromagnetic compatibility information provided in Section 16 – Electromagnetic Compatibility of this manual.
- ! FeetMe® Monitor Insoles should not be used adjacent to or stacked with another unit. If adjacent or stacked use is necessary, the device should be observed to verify normal operation in the configuration in which it will be used.
- ! RF portable communications devices (including peripherals such as antenna cables and external antennas) should not be used closer than 30 cm (12 inches) to any parts of the FeetMe® Monitor insoles and FeetMe® Charging case, including the cables and accessories provided or specified by the

manufacturer. Otherwise, the performance of these devices may be impaired.

- ! The use of accessories and cables other than those specified or provided by the manufacturer may cause increased electromagnetic emissions or decreased immunity of those equipment and therefore cause improper operation.
- ! Do not puncture or cut the soles and their battery and do not access the electronic reading circuit directly.
- ! Do not touch the magnetic elements of the insole directly.
- ! If the device is defective, do not use it. Notify the clinician who prescribed its use, or the manufacturer or the distributor for a repair or replacement.
- ! Any modification made to FeetMe® Monitor insoles and FeetMe® applications is forbidden.
- ! Do not wear FeetMe® Monitor insoles in contact with bare skin nor wounds. Wear socks when using the device.
- ! Do not use FeetMe® Monitor insoles while they are charging.
- ! If FeetMe® Monitor insoles do not fit comfortably into your

shoes or if they are not comfortable when you wear them, change shoes before use.

- ! It is forbidden to dispose of FeetMe® Monitor insoles or FeetMe® Charging case along with household waste.

6. Product description

FeetMe® Monitor Insoles consist of a pair of connected insoles. FeetMe® Evaluation and FeetMe® Rehabilitation are mobile applications running on a smartphone.

The mobile applications are downloadable on a smartphone.

Detachable parts:
Double head charge cable (induction charger)
USB wall adapter (see Section 15)

7. Operating principle

FeetMe® Monitor insoles are powered by a battery and measure pressure on the plantar surface and spatiotemporal parameters. These data are calculated locally within the insole and transmitted by Bluetooth to the smartphone.

FeetMe® Evaluation and Rehabilitation applications can be used on a smartphone to receive, store process and display the pressure and spatiotemporal parameters measured by the insoles.

8. Product setup

Setup does not require any tool other than those provided with the device.

Caution

- ! Do not drop FeetMe® Monitor insoles and avoid shocks.

a. Charging the insoles with a unit charger

To charge the insoles, connect the charger to a power outlet then position the magnetic charger heads on the area identified by a round mark. Use the wall adapter specified in Section 15 - Technical Characteristics (order from FeetMe if needed).

Warning

- ! Do not use FeetMe® Monitor insoles while they are charging
- ! Do not put the power cable around your neck.
- ! The charger must be connected to a wall outlet.
- ! Charging the battery must be performed with the cable connected to the wall adapter specified in Section 15 - Technical characteristics. The use of a wall adapter other than the one specified creates the risk of an electric shock.
- ! Do not position the wall adapter or charging cable in a location where the unplugging is made difficult to the user.

b. Charging the insoles with the charging case

Open the case. The insoles won't charge if the lid is closed. Take out the power cable from the case and connect it to the case, close to the On/Off switch. Plug the other end to a power outlet. Switch it on (position I). If the charging slots are empty, the LED will flash white for a few seconds then go off.

The charging case should be at a temperature below 35° Celsius.

To position a pair of insoles in a charging slot, identify the insole size and locate the corresponding slot. Position each insole on the edge, top side facing outwards.

c. Charge light indicators

Continuous green : charged
Continuous white : charging
Discontinuous white : Insoles poorly positioned, reposition
Light off : power cable disconnected or improper positioning of the charger or the insole. Check position to make the light indicator steady white.

It takes approximately 3 hours for a complete charge of the insoles.

d. Bluetooth connection to the insoles

Once charged, the insoles are detectable via Bluetooth, using FeetMe® applications. The insoles may enter sleep mode automatically when not in use. In order to get them out of sleep mode, shake them for 3 seconds. No manipulation is therefore necessary from the user, to turn the insoles on and off.

Caution

- ! Be careful not to use the insoles alongside other Bluetooth devices.
- ! For optimal Bluetooth connection, the distance between phone and insoles should be less than 5m.

9. Use of insoles with FeetMe® Evaluation

Warning

- ! Do not wear FeetMe® Monitor insoles in contact with bare skin nor wounds. Wear socks when using the device.

- ! Do not use FeetMe® Monitor insoles while they are charging.
- ! If FeetMe® Monitor insoles do not fit comfortably into your shoes or if they are not comfortable when you wear them, change shoes before use.

Caution

- ! Make sure that FeetMe® Monitor insoles are charged before starting an examination.
- ! Do not drop FeetMe® Monitor insoles and avoid shocks.
- ! Always apply a hygienic protection to the insoles when using them.

a. First use or reinstallation

Create a Gmail account if you do not have one. Go to the « Play Store » and search for the application : « FeetMe Evaluation ».

Download the application and follow the installation instructions.

b. Perform a gait recording

1. Connect to the insoles : Click on the button « Connect to the

insoles », select the right and left soles of the corresponding size and then click on « I have selected my insoles ». If the insoles are not detected, see Section 11 - Troubleshooting.

2. Insoles calibration: For this step, the shoes must be laid flat and on a horizontal surface, the insoles must be placed inside the shoes. Do not put the shoes on. Click on the « Calibration of my insoles » button.

3. Select a test : Click on the chosen test. N.B. You can also acquire gait parameters without the phone and synchronize the data at the end of the recording by clicking on « Import a record ».

4. Perform a recording : Put on shoes and press « Continue ». if needed, follow the instructions displayed on the screen. Press « Start the test » to start recording. For « Free walking » tests, press « Pause » to stop the test and « Save record » to save the data. For other tests, stopping the recording and saving the data are automatic.

5. Make a new recording : You remain connected by default to the last insoles, so you can make other recordings without redoing the

steps of connection and calibration (repeat steps from step 3). To make a new recording with other insoles, click on « Connect to the insoles » and repeat the above steps from the beginning.

c. View the data acquired from the mobile application

The data collected for a given patient can be viewed from the application. To access it, go to the « Records » tab to access all the records. To access the data of a record, click on the associated record.

d. Data backup

In case the user hosts the recordings, he is in charge of performing appropriate and regular data backups in order to prevent any destruction or loss of information. FeetMe cannot be held responsible for any accidental or unauthorized destruction or accidental loss of user data.

e. View battery level

The battery level of the insoles is visible on the insoles selection page and also, once the insoles are connected, on the recording screen.

f. Update the application

Caution

- ! An internet connection is required to transfer data from the mobile application.
- ! All mobile application updates must be accepted.

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It is also possible to configure the automatic update installation.

10. Use of insoles with FeetMe® Rehabilitation

Warning

- ! Do not wear FeetMe® Monitor insoles in contact with bare skin nor wounds. Wear socks when using the device.
- ! Do not use FeetMe® Monitor insoles while they are charging.
- ! If FeetMe® Monitor insoles do not fit comfortably into your shoes or if they are not comfortable when you wear them, change shoes before use.

Caution

- ! Make sure that FeetMe® Monitor insoles are charged before starting an examination.
- ! Do not drop FeetMe® Monitor insoles and avoid shocks.
- ! Always apply a hygienic protection to the insoles when using them.

a. First use or reinstallation

1. Create a Gmail account if you do not have one.
2. Go to the « Play Store » and search for the application : « FeetMe Rehabilitation ».
3. Download the application and follow the installation instructions.

b. Perform a rehabilitation exercise

1. Select an exercise
2. Select the right and left insoles from the list then click “Connect insoles”. If the insoles are not detected, see Section 11 - Troubleshooting.

3. For this step, the shoes must be laid flat and on a horizontal surface, the insoles must be placed inside the shoes. Do not put the shoes on. Click on the « Calibrate my insoles » button.
4. Start exercising: Put on shoes and press « I put my insoles on » then follow the instructions.

c. View the data acquired from the mobile application

The data collected for a given patient can be viewed from the application. To access it, go to the « Results » tab to access all the exercises. To access the data of a specific exercise, click on the associated exercise.

e. View battery level

The battery level of the insoles is visible on the insoles selection page as well as in the profile page.

f. Update the application

Caution

! All mobile application updates must be accepted.

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It is also possible to configure the automatic update installation.

11. Troubleshooting and shutdown procedure

Defect	Potential cause	Apply the following and re-try
Insole is not detected by the application	Insole battery is too low	Fully charge insoles
	Insoles are in sleep mode	To take them out of sleep mode shake them during 3 seconds
	Bluetooth is Inactive	Activate the Bluetooth of your smartphone
	Electromagnetic noise	Do not wear the insoles in an electromagnetically noisy place
	Error on the insoles	 Reset the insole by pressing the « reset » button (with the tip of a pen for example) for 10 seconds. The reset button is located on the top of the insole, near the logo.

	Phone Bluetooth mode does not work	Disable the Bluetooth mode on your phone and then turn it back on
--	------------------------------------	---

To shut down the insoles, the user must disconnect the charging cable and the wall adapter, remove the insoles and put them in a place that is free of mechanical and electrical stress.

12. Storage and transport

See Section 15- Technical Characteristics for storage and transport conditions.

Caution

! When not in use, keep the device away from dust and in a clean and dry place. Do not store FeetMe® insoles near sources of heat or fire.

During transport, put the insoles in their packaging. Insoles may be carried in either checked or carry-on baggage. When carried in checked baggage, the device must be completely switched off.

Caution

- ! Do not drop FeetMe® Monitor insoles and avoid shocks.

13. Cleaning and maintenance

Clean the insoles regularly following the steps hereafter:

- Clean FeetMe® insoles with a wet cloth and apply the antibacterial, antifungal and antiviral product. Cloth must not be damp.
- Allow them to air dry.
- The insoles must be cleaned in-between patients.

Caution

- ! Do not use corrosive products, oil, acetone or petroleum products to clean FeetMe® insoles. Do not put FeetMe® insoles in a dishwasher, a tumble dryer or other water or steam appliance. Do not immerse FeetMe® insoles.

The maintenance of the insoles must be carried out by an authorized person. In case of a malfunction, please contact your distributor or the manufacturer.

14. Disposal

The device and its accessories contain electronic components.

Warning

- ! It is forbidden to dispose off FeetMe® insoles or FeetMe® Charging case along with household waste.

The device must be brought back to your healthcare provider or to its manufacturer to ensure its recycling according to WEEE Directive (Waste Electrical and Electronic Equipment).

15. Technical characteristics

Conditions of use of the insoles	Working temperature		+5° to +40°C
	Storage and transport temperature		-25° to +70°C
	Humidity for use, storage and transport		15 % to 93 %
	Altitude range		0 to 3000 m
	Pressure range		700 hPa to 1060 hPa
	Charge range		40 to 120 kg
	Measured pressure		0,55 to 7 kg/cm ²
	Power supply		3,7 V DC, 110mAh
Protection index of the insoles	IP55	IP5X: Protected against dust (limited ingress, no harmful deposits). IPX5: Protected against low pressure water jets from any direction. Limited ingress permitted.	
Measured parameters	Plantar pressure		In kg/cm ² ($\pm 0.55\text{kg/cm}^2$ below 2kg/cm^2 , $\pm 0.75\text{kg/cm}^2$ between 2kg/cm^2 and 5kg/cm^2 and $\pm 0.85\text{kg/cm}^2$ above 5kg/cm^2)
	Stride length		In meters ($\pm 0,1\text{m}$)

Insoles characteristics	Walking speed		In meter/second $\pm 0,1\text{m/s}$
	Available sizes		35 to 46
	Applied parts		Type BF, in contact with: foot
	Material		Polycarbonate or polyamide, Thermoplastic Elastomer (TPE), Polyester Fabric
	Dimensions		Thickness : 5mm
Lithium ion type battery (insoles)	Weight		[80 ; 120]g
	Life time		1 year
	Certification		IEC 62133-2: 2017
Wall adapter for charging cable		(Accessible part)- Standard IEC 60601-1 Manufacturer : Globtek, Inc – Ref : GTM41078-0605-USB 5 V DC	
Double head charging cable	(Accessible part) Induction charging	5V $\overline{=}$ 1A	
Charging case	Power supply		12V $\overline{=}$ 6.67A
	Operating temperature		+5° to +35°C
	Storage and transport temperature		-25° to +70°C
	Dimensions (cm)		52.5 x 37.5 x 15.2

	Weight	7kg
	Case Power cable	CEE 7/7 to IEC C13
	Phone charging cable	USB type A to USB type C
	Material	Polypropylene

16. Recommended minimum phone features

Designation	Value
OS	Android version 6
	iOS version 12
Bluetooth	V 4.0

17. Electromagnetic compatibility

Module	Characteristics	Value
Bluetooth v5.0 low energy	Emitted power	+4 dBm
	Frequency	2400-2483,5 Mhz

Portable and mobile RF communication devices may compromise the operation of other medical electrical devices.

Warning

- ! FeetMe® Monitor Insoles should not be used adjacent to or stacked with another unit. If adjacent or stacked use is necessary, the device should be observed to verify normal operation in the configuration in which it will be used.
- ! RF portable communications devices (including peripherals such as antenna cables and external antennas) should not be used closer than 30 cm (12 inches) to any parts of the FeetMe® Monitor insoles and FeetMe® Charging case, including the cables and accessories provided or specified by the manufacturer. Otherwise, the performance of these devices may be impaired.
- ! The use of accessories and cables other than those specified or provided by the manufacturer may cause increased electromagnetic emissions or decreased immunity of those equipment and therefore cause improper operation.

Electromagnetic Emissions

FeetMe® Monitor Insoles and their accessories are intended for use in the electromagnetic environment specified below. The customer or the user of FeetMe® Monitor Insoles should ensure that it is used in such an environment

Emissions Test	Compliance	Electromagnetic Environment (guidance)
RF Emission CISPR11	Group 1	FeetMe® Monitor Insoles and its accessories use RF energy only for its internal function. Therefore its radiated emissions are very low and not likely to cause interference in nearby electronic equipment.
RF Emission CISPR11 for FeetMe® Monitor Insoles	Class B	FeetMe® Monitor Insoles are suitable for use in all establishment, including domestic establishment and those directly connected to the public low voltage power supply network that supplies buildings.
RF Emission CISPR11 for FeetMe® Charging Case	Class A	FeetMe® Charging case is suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required) this equipment might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the equipment.
Harmonic Emissions – IEC 61000-3-2	Class A	
Voltage Fluctuations Flicker emissions – IEC 61000-3-3	Pass	

Electromagnetic Immunity

FeetMe® Monitor Insoles and their accessories are intended for use in the electromagnetic environment specified below. The customer or the user of FeetMe® Monitor Insoles should ensure that it is used in such an environment

Immunity Tests	IEC60601-1-2 Ed4 Test Level	Compliance Level	Electromagnetic environment/ Guidance
Electrostatic Discharges (ESD) (IEC61000-4-2)	± 8 kV in Contact $\pm 2, 4, 8, 15$ kV air	* ± 8 kV in contact * $\pm 2, 4, 8, 15$ kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical Fast Transient Burst (IEC61000-4-4)	± 2 kV for power supply lines	± 2 kV for power supply lines	FeetMe® Monitor Insoles are suitable for use in all establishment, including domestic establishment and those directly connected to the public low voltage power supply network that supplies buildings. FeetMe® Charging case is suitable for use in industrial areas and hospitals.
Surges (IEC61000-4-5)	± 0.5 et 1 kV differential mode $\pm 0.5, 1$ et 2 kV common mode	± 0.5 et 1 kV differential mode $\pm 0.5, 1$ et 2 kV common mode	

Power frequency (50 Hz/60Hz) magnetic field (IEC61000-4-8)	30 A/m	30 A/m	For FeetMe® Charging case, power frequency magnetic fields should be at levels characteristic of a typical hospital or industrial environment
Voltage dips, short interruptions and voltage variation on power supply input lines (IEC61000-4-11)	0% UT for 0.5 cycles at 0°, 45°, 90°, 135°, 180°, 225°, 270°, 315° 0% UT for 1 cycle 70% UT For 25 cycles at 50 Hz For 30 cycles at 60 Hz Monophased: at 0°	0% UT for 0.5 cycles at 0°, 45°, 90°, 135°, 180°, 225°, 270°, 315° 0% UT for 1 cycle 70% UT For 25 cycles at 50 Hz For 30 cycles at 60 Hz Monophased: at 0°	FeetMe® Monitor Insoles are suitable for use in all establishment, including domestic establishment and those directly connected to the public low voltage power supply network that supplies buildings. FeetMe® Charging case is suitable for use in industrial areas and hospitals
Voltage interruptions (IEC61000-4-11)	0 % UT; For 250 cycles à 50 Hz for 300 cycles à 60 Hz	0 % UT; For 250 cycles à 50 Hz for 300 cycles à 60 Hz	

Radiated RF (IEC61000-4-3) for FeetMe® Monitor Insoles	10 V/m 80 MHz at 2.7 GHz 80 % MA at 1 kHz	10 V/m 80 MHz at 2.7 GHz 80 % MA at 1 kHz	Recommended separation distance $d = 1.2\sqrt{P}$ $d = 1.2\sqrt{P}$ (80MHz to 800MHz) $d = 2.3\sqrt{P}$ (800MHz to 2.7GHz) where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF (IEC61000-4-3) for FeetMe® Charging Case	3 V/m 80 MHz at 2.7 GHz 80 % MA at 1 kHz	3 V/m 80 MHz at 2.7 GHz 80 % MA at 1 kHz	
Proximity fields From RF wireless communications equipment (IEC 61000-4- 3 provisional method)	9 V/m - 710 MHz, 745 MHz, 780 MHz, 5240 MHz, 5550 MHz, 5785 MHz 27 V/m - 385 MHz 28 V/m - 450 MHz, 810 MHz, 870 MHz, 930 MHz, 1720 MHz, 1845 MHz, 1970 MHz, 2450 MHz	9 V/m - 710 MHz, 745 MHz, 780 MHz, 5240 MHz, 5550 MHz, 5785 MHz 27 V/m - 385 MHz 28 V/m - 450 MHz, 810 MHz, 870 MHz, 930 MHz, 1720 MHz, 1845 MHz, 1970 MHz, 2450 MHz	
Conducted RF (IEC610004-6)	3 V - 150KHz à 80MHz 6 V in ISM bandwidth and bandwidth between 0.15 MHz and 80 MHz, (amateur radio bandwidth included for FeetMe® Monitor Insoles) 80% MA at 1 KHz	3 V - 150KHz à 80MHz 6 V in ISM bandwidth and bandwidth between 0.15 MHz and 80 MHz, amateur radio bandwidth included 80% MA at 1 KHz	

18. Lifetime

The lifetime of the device FeetMe® Monitor is the smallest duration between two years or one million steps made by the user with the product – i.e. 500 000 steps by insole -.

This duration does not apply to consumable parts, such as batteries or protective materials, which are expected to wear out over time, unless the damage is due to a defect in materials or manufacturing.

19. Specific regulation

This device FeetMe® Monitor Insoles complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the

FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

CAUTION: This equipment may not be modified, altered, or changed in any way without signed written permission from FEETME. Unauthorized modification may void the equipment authorization from the FCC and will void the FEETME warranty.

This device complies with FCC radiation exposure limits set forth for general population.

This device must not be co-located or operating in conjunction with any other antenna or transmitter.

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