

④ Alarm Sound Guide

Categories			Operating Criteria	Sound Type Frequency Pitch	Notes
App	GW	band battery	Battery remaining: less than 20%	Single sound (Ding-dong) Once Low pitch	Gateway alarm through the App
gateway		band battery	Battery remaining: less than 20%	Single sound (Ding-dong) Once Low pitch	Gateway alarm

⑤ Battery Status Displaying LED

LED Status	Band Battery	Gateway Power	Notes
Off	Power off	Power off	
On	80% or more	Power on	
slow flashes	20% or more and less than 80%		
frequent flashes	Less than 20% Or Charging		Less than 20% : short sound ringing once(Ding-dong)


⑥ Communication Status Displaying LED

LED status	Communication status between gateway and wifi AP	BLE communication status between gateway and App	BLE communication status between gateway and band	notes
slow flashes	Please put the two devices closer as the signal is weak during standby or operation.			
normal flashes	Normal state (Can be connected for communication or it is in normal state.)			

3.2 Information Required for Band Use

① Power on (charging required)

- Please fully charge the battery before first use. If the battery has remaining charge, the red LED is ON while charging it after coupling it with the charging cradle.
- (If the remaining charge of the battery is very low, in 10 to 20 minutes after coupling with the charging cradle, the red LED lights up and charges.)
- Disconnect it from the charger when charging is complete

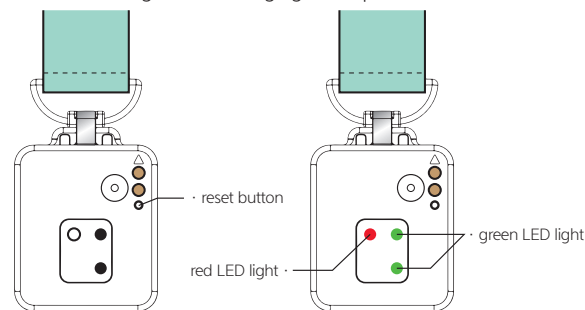

② How to solve when the Band is not working

- Press reset button for 8 seconds. After 1~2 seconds, press reset button once more for a second. Then green light will flash twice, and the band will restart with set values.


③ Band operation and charging indication

- How to turn on : It automatically turns on when combined with the charging cradle which is connected to the power.
- How to check Power-On: If you shake the band lightly , green LED will flash briefly.
- Reset : Use reset button only when band is not working.(Refer to ② How to solve when the Band is not working)
- !Do Not press the reset button too often with a pointed object. It may cause damage.
- LED displaying while charging : Red LED light on
- LED displaying when charging completed : Green LED light on

3.3 Acc band Components and How to Wear


① Step 1. Select acc band combinations for your baby

· Combination for 0 to 3 Months(acc band Small)

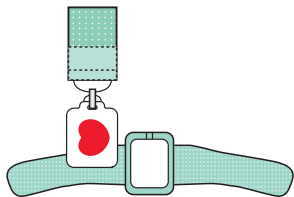

· Acc band S-T1 : Recommended for 0 to 3 months

+

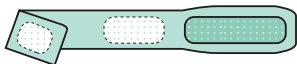


· Acc band S-T2 : Recommended for 0 to 3 months

+




· Connector S : Recommended for 0 to 3 months when wearing outside of the ankle

· Combination for 3 to 36 Months(acc band Large)

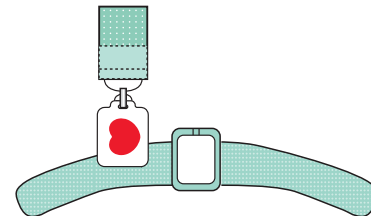


· Acc band L-T1 : Recommended for 3 to 36 months

+

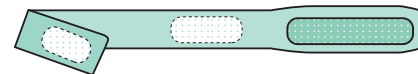

· Acc band L-T2 : Recommended for 3 to 36 months

+

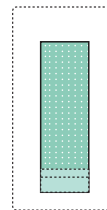

· Connector S
: Recommended for 3 to 5 months when wearing inside of the ankle




· Connector L
: Recommended for 3 to 36 months when wearing outside of the ankle
: Recommended for 5 to 36 months when wearing inside of the ankle

※ It varies according to the length of the baby's leg, thickness of the leg, and the position of wearing it (inside or outside of the ankle).


※ For more information on how to combine them, please refer to our homepage FAQ or Gallery's PIC.

② Step 2. Wear Acc Band

How to wear Acc band S(or L)-T1

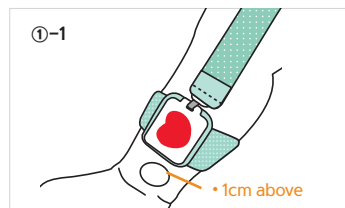


- Place acc band 1cm above the baby's outside ankle bone, in the center. (1-1 front side)

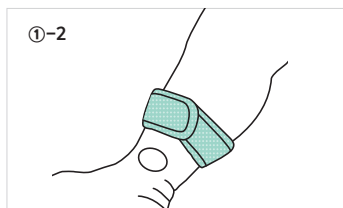


- Wrap the ankle with the wings of the acc band T1 to form X shape and secure it with velcro. (1-2 back side)

How to wear Acc Band S (or L) – T2

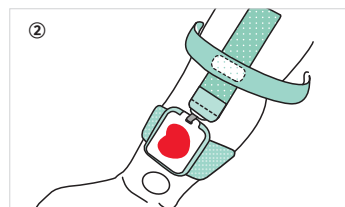

- Combine the two ends of acc band S (or L) – T2, wrapping the calf. (2)

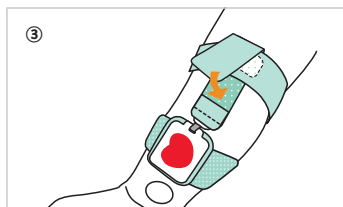

- Fold the connector onto the center velcro on T2. (3)

③ Step 3. Make sure it is attached properly

[wearing on the outside of the ankle]

	Proper Wearing (O)	Wrong wearing (X)	
Connector	The connector and the band body are on a straight line (O)	The connector and the band body are not on a straight line (X)	
Back side	Combining in X shape (O)	• Less than 1cm • wearing too low (X)	Combining in X shape (O)
Front side	Position 1~1.5cm (O)		The two ends of the acc band are aligned on a straight line (X)

[wearing on the inside of the ankle]

	Proper Wearing (O)	Wrong wearing (X)	
Connector	The connector and the band body are on a straight line (O)	The connector and the band body are not on a straight line (X)	
Back side	Combining in X shape (O)	• Less than 1cm • wearing too low (X)	Combining in X shape (O)
Front side	Position 1~1.5cm (O)		The two ends of the acc band are aligned on a straight line (X)

※Installation conditions

[product compatibility] AI,MON smart band product can be connected to a smartphone that meets the specifications below.

iOS 10 or later, Android 6.0 or later, Bluetooth 4.1 or Later

4.1 Introduction to setting up procedures for App operation

- Step 1: Install the app on your phone and connect it to the gateway.
 Step 2: Create an account following the instructions on the app.
 Step 3: Go to the baby registration and range settings.
 Step 4: Put the product on the baby's ankle and check the app screen.
 Step 5: Check the communication condition and ensure that the acc band are properly attached.

Step 1 : Band, gateway, and smart phone connection guide

- Power on the band and the gateway and follow the app's instructions to connect communications.
- If the connecting WiFi changes by moving to another place, you need to set it again.

Step 2 : Account creation

- Complete Step 1, and enter your desired ID, password, and email in <<Account>>/<Register a new account>
- Follow the instructions on the app, then a new account will be created.

Step 3 : Baby registration, the date of birth, and range settings

- Complete Step 2, and when <<baby info settings>>/<baby registration> done, basic operation will start according to the date of birth.
 For more specific registration, use the range settings using the references.

Step 4 : Wearing the band and measurement

- Select acc band that fits the baby's age-in-months and its body and combine the acc band with the band body. Put it around the baby's ankle.
 (See at the bottom of the <FAQ-How to wear properly> at the bottom of the <Menu>screen)
- You can check the real time information being measured on the main screen of the app.

You can check detailed usage in the FAQs.

- beginner → basic FAQ : 1~6 (app and home page)
- troubleshooting → additional FAQ : A1~A6 (home page)
- useful info → Tip FAQ : B1~B6 (home page)

4.2 Paired App (phone) and Shared App (phone)

- ① The phone that connects to the gateway is the paired App (phone).
 - Paired App (phone) is a phone that communicates directly with the gateway.
 - It initially connects to the gateway and becomes a paired App(phone).
 It registers the gateway and creates an account.
- ② You can change the paired App while the device is operating.
 - Paired App with the gateway can be changed in <<settings>>/<pairing and unpairing>.
 - Paired App (phones) performs the following roles:
 It sends your baby's information to you when WiFi is not available.
 You can check the wearing check and communication conditions.
- ③ Shared App (phone) is an app that accesses your account with your account ID and password.
 - Create an account with a paired phone first. Next, access the account and password with another phone. It becomes a shared app (phone), and all the information that the paired app receives will be shared.