

Model NO.: 131

操作指南 – 说明书 – 英文

封底

Bluetooth Connection

1. Please charge Go2Silence when you use it for the first time the device will be activated automati- cally

2. Turn on the Bluetooth in your phone Settings, open the SLEEPON APP, and follow the instructions to sign up and log in.

3. Connect your device according to the prompts, or you can click “my”-“Add Device” and select “Go2Silence”-“Add”

4. Confirm whether the device has vibration feedback when you see the notification. Click “Yes” to complete the device connection.

Unable to Search or Connect Your Device?

1. Make sure the device is powered and has been charged first.

2. Keep the device within 10 meters of your phone.

3. Make sure your phone’s Bluetooth is turned on, and the app permission is allowed.

4. You may need to restart the Bluetooth once more to try again if you use certain Android phones.

5. Please contact us if you still have problems. We’d be more than happy to help you out.

Application Renewal

SLEEPON APP will be updated at irregular intervals for the better user experience. Please make sure you have the latest version of the software available for your phone and enable automatic app updates. We will inform you when the new version is released!

We hope Go2Silence can give you a good user experience.

Most issues are easily resolved by phone or email. If you have any questions, please contact our support team. Our customer support representative is more than happy to help you out!

Specifications

Product Name: GO2SILENCE

Product Model: 131

Bluetooth Version: 5.0

Bluetooth Range: 10m

Battery: Rechargeable lithium-ion battery

Battery capacity: 130mAh

Input Charging Voltage: 5V

Operating Temperature: -10°C~50°C

Support Version: Android 5.0 / iOS 9.0 and above

Contact Us

Email: service@sleepon.us

Website: https://www.sleepon.us

SLEEPON

GO2SILENCE

Smart Positional Therapy Anti-Snoring Ball

User Guide | Please read it before using

封面

Important Statement

Go2Silence is based on scientific principles and the study of human physiology and cogni- tion. It is not a medical device. Please consult your doctor when you consider accepting the feedback from Go2Silence for your health activities.

What’s in the Box

GO2SILENCE	× 1
PU Ball	× 1
Type-C Charging Cable	× 1
Flannelette Sticker	× 2
User Manual & Instructions	× 1

What’s in the box

Go2Silence

PU Ball

Flannelette Sticker

User Manual

Charging Cable

Accessories

LED Indicator

Charging Port

PU Ball

Interaction Instruction

The Built-in LED and vibration motor will remind you according to your activities. The prompts are shown in the following table.

Function	Device Prompt
Charging	Flashes Red
Charge complete	Solid Green
Lower than 5% power	Constant Flashes Red
Bluetooth connection	Flashes Green and twice vibration
Wear detected	Flashes Green and threetimes vibration
Full storage	Solid Red
Firmware updating	Solid Green

* The functions such as sleeping positions are controlled by the APP

Download SLEEPON APP

Android device: search for “SLEEPON” in Google Play store and click “Install”

iOS device: search for “SLEEPON” in App Store and click “Get” to download.

Sleepon

SLEEPON

Better Sleep Better Life

★★★★★

GET

Download on the App Store

Get it on Google Play

SLEEPON

Better Sleep Better Life

www.sleepon.us

颜色

- Pantone 3262U
- 其他为四色印刷

