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### FCC Compliance Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

### FCC Interference Statement

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Change or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment under FCC rules.

This equipment complies with the FCC RF exposure limits set forth for an uncontrolled environment. this equipment must not be re-located or operating in conjunction with other antenna or transmitter.

THE YEOLRIM CO., LTD.  
D-311, H BusinessPark, 26, Beobwon-ro 9-gil  
Songpa-gu, Seoul 05836 Korea  
support@yeolrim.com (02-6281-8719)

www.yeolrim.com / www.orangeaid.co.kr

Please check product model name, S/N and trouble conditions before asking A/S.

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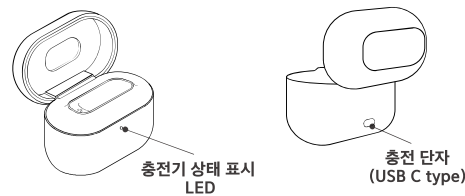
- If water or sweat enters product, parts inside the product may be damaged and may malfunction.
- Do not drop or give impact to the product. And keep the product away from the place of severe vibration or where magnet exists.
- Do not put the product into a microwave oven or an oven.
- Take care for the infants, children, demented elderly or pets not to swallow this product or related accessories. Swallowing the product may cause the danger of suffocation, respiratory difficulty or sticking in the gullet. So immediately consult with a doctor in that case.

**3. Possible Adverse Effects**

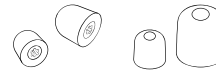
- Ear-tip of the product may accelerate accumulation of earwax.
  - Skin irritation or other adverse effects may be rarely caused depending on the skin sensitivity of the user.
- In case of any adverse effect, immediately stop using the product and consult with a medical staff.

**4. Requirements for Preventing Safety Accident**

- Please use original parts supplied from us to safely protect hearing of users. The guarantee will not cover any malfunction or safety accidents caused by using non-original parts.
- You may experience temporary dizziness when you put on the product. In that case, stop driving or operating precision machinery and take some rest before putting it on again. If the dizziness continues, consult with a doctor.

- In general, you need to keep the distance of minimum 15cm off the medical devices such as pacemakers or defibrillators, however, you first need to follow the instruction or recommendation of an expert.
- Diagnostic radiology such as X-ray, CT, MRI and PET Scanning, etc. and electrotherapy
  - As the product may be damaged if it is exposed to strong magnetic field, it should be removed before X-ray, CT, MRI, PET Scanning, electrotherapy and surgery.
- Chemical Product
  - As the chemical products, such as cosmetics, hair spray, perfume, lotion and insecticide, etc., may damage Orangeaid, remove the product before using the chemical product, and put on it under dry condition.
- Interference
  - This product may be interfered by other wireless products or devices supporting wireless functions (e.g. mobile phone). If there occurs any interference, keep longer distance off the device.
- Using in the Airplane
  - As Bluetooth is built in this product, do not use it in the airplane except when the use is allowed.
- Do not keep this product in the place of much moisture, dust or soot.
- Do not disassemble, assemble or remodel this product arbitrarily.
- Do not use the product in the hot and humid place such as bathroom and sauna.
- Do not use the product in the environment of high temperature such as inside of a car under sunlight or near to a heating device.

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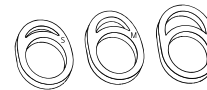
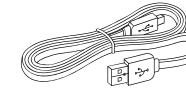
- The user need to be well-informed of the precautions to use the product safely and as intended.
- This product does not recover the hearing of the user to normal, nor prevent or improve hearing impairment caused due to damage to bodily tissue.
- It is recommended that the product should be regularly used as it takes time to adapt the user to the product.
- Please consult with an expert in case of unexpected operation or abnormal condition occurs with the product.

##### 1. Using

- Before using the product, the user need to put on it and make Fitting Setting on the dedicated Orangeaid App to fit hearing condition of the user.
- Abuse of product may cause temporary or permanent hearing loss.
- Do not have another person use the product, as incorrect use of the product may cause adverse effect to the health.
- If there is any signal of health problem during use of the product, immediately stop using the product and consult with an expert.

##### 2. Handling

- Active implants
  - Please always take care in handling active implants.

### Warranty and Product A/S

- ◆ Any trouble occurring during normal use of the product within warranty period will be repaired free of charge.
- ◆ Even if the trouble occurs within warranty period, repair costs could be charged in following cases.  
Any malfunction or damage due to careless handling or incorrect way of using of the user(Dropping, Impact, Pressurizing, Dust, Mud, Sand, Flooding, Place of high temperature and high moisture and Severe cold, etc.)  
Any malfunction or damage due to arbitrary repair, remodeling, disassembly and incorrect way of cleaning, etc.  
Any malfunction due to natural disaster such as fire, earthquake, flood and lightning, etc.  
Any malfunction or damage due to use of non-original consumables or products.

### Precautions for Safety

#### ◆ Important Safety Information

Please handle Orangeaid and cradle with care. This product contains sensitive electronic parts including battery. If you drop, burn, give impact to or disassemble it, or expose it to the environment of highly dense chemical components including excessive heat or liquid or evaporating liquefied gas, the product may be damaged or the function may be impaired or the user may get injured due to the damaged product.  
Never use the damaged Orangeaid or cradle.

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Model Name : YSH-C2000

corporate name : THE YEOLRIM CO., LTD.

Manufacturing Date : Separately Marked

Country of Origin / Manufacturer : Korea / THE YEOLRIM

Certification No. : R-R-YRO-YSH-C2000 / FCC ID : 2AYGZ-YSH-C2000

**Product Warranty Card**

The free warranty period of this product is 1 year from the purchasing date.

Purchase receipt is necessary to get free warranty service.

In case purchase receipt could not be checked, the free warranty period  
of the product is 1 year from the manufacturing date.

**Product Name Orangeaid**

Model Name YSH-C2000

Purchase Date

Name of User

Serial Number (S/N)

Place of Purchase

**Product Specification**

Specification	Battery	Li-ion 120mAh (Earset) / Li-Po 350mAh (Cradle)
	Input Voltage	DC 5V=90mAh (Earset) DC 5V=150mAh (Cradle)
	Size(mm)	17.00 x 39.15 x 27.82(Earset) 71.85 x 42.30 x 52.60(Cradle)
	Weight(g)	8.5g(Earset) / 73.5g(Cradle)
	(1)Continuous Service Hours / Charging Time(Earset)	12h / 1.5h
	(1) Continuous Service Hours / Charging Time (Cradle)	2.5times / 3.5h
Performance	Max. Output Sound Pressure Level(OSPL90)	124±5 dB SPL @ 1400 Hz
	High Frequency Average Max. Output Sound Pressure Level(HFA-OSPL90)	114±5 dB SPL
	High Frequency Average Max. Gain (HFA Full-on Gain)	42±6 dB
	Frequency Range	200 ~ 6500Hz
	Equivalent Input Noise	25(+5dB) dB SPL
	Total Harmonics Distortion (THD)	Under 2.0%
	(2)Current Consumption	9mA(under + 20%)

(1) Operation time may vary depending on the environment.

(2) Measuring value under amplification mode.

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**Maintaining Orangeaid**

1. Turn off Orangeaid while you are not using it to save battery.
2. It is important to clean and maintain the product periodically to extend product life and maximize the function.
3. Charge the product on the table or desk to protect the product against dropping and breakage due to inobservance during charging.
4. Wipe moisture or dust with soft dry cloth on the product surface.
5. Remove wax and powdery dust on the main parts of the product with the brush supplied together with the product.
6. Always keep the product in the dry place.
7. Avoid keeping the product in too hot or high place.  
(Recommended temperature 10~40℃)
8. In case Orangeaid is not used for a long time, battery may be naturally discharged. In that case, charge the battery again before using Orangeaid.

**⚠ Caution**

Regular replacement of ear-tip is recommended.  
Ear wax and foreign substances falling into the device can cause the failure or degraded performance of the device. Keep the device clean regularly with enclosed brush.

**Caution**

Telephone call and music listening settings for a smart device could be made on the smart device. The set up menu may vary depending on the smart device models and s/w versions.

**6. Changing Orangeaid Name(Only Android)**

- Step 1. Run Android Setting
- Step 2. Select Connect-Bluetooth
- Step 3. Select Setting Button on YSH-C2000
- Step 4. Change name to a desired one on Change Name menu.
- Step 5. Run the App again after changing the name to connect to the changed name.

**Caution**

When you use more than one product, you need to give different names to the products for product settings with no trouble.

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**! Caution**

To use 'Searching Orangeaid' function, keep GPS function and Orangeaid App in a smart device being enabled.

**! Warning**

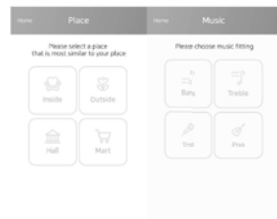
If you operate Alarming, with the product being put on, your ear may get damaged.

**5. Listening to Music**

When the product is wireless-connected with a smart device, you may listen to music from Orangeaid by playing music play software in the smart device.  
 While you are listening to music, voice amplification function is turned off.

### 3. Music Setting

This function may be used when the product and a smart device is connected through 'Media Audio' (Bluetooth). After music starts to play, you may change timbre on the lists by selecting Low Pitch Hardening, High Pitch Hardening, Trot and Song. Please refer to [4. Liking with Smart Device] of [Before using Orangeaid] for how to connect the product with a smart device for Music Setting.

### 4. Searching Orangeaid

Finally connected place is shown by periodically checking the connection status between a smart device and Orangeaid.

Move to the place where you lost the product referring to the map and press  icon to try connection.

When the product is connected, 'Alarm' button is activated. If you press the button, Alarm window is shown. Press 'Yes', and alarm sounds and LED blinks for 2~3 minutes on the product.

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**Tip.**

Selecting the expert mode allows a more detail fitting.  
Mode change can be selected by entering Setting > Fitting Mode.  
The number of channels applied according to the model is as follows.  
Orangeaid Lite/Pro/Premium : 8/16/32 ch

**! Warning**

The user must run automatic setting at the first time of using. Default value would be different according to the automatic setting value. Automatic setting is recommended to be done in the calm place with no noise.

**2. Fuction Setting**