- Tip 1.**
After the link is completed, Orangeaid and the smart device are automatically connected whenever the App is executed.
- Tip 2.**
In case YSH-C2000 does not exist on the linkable Orangeaid?
Check if Bluetooth function is turned on in the smart device.
Check if the power of Orangeaid is turned on.
Please try again, referring to 'No. 4 Linking to Smart Device'.

Caution

Link may be limited due to surrounding environment interrupting Bluetooth link, S/W version of smart device and type and conditions of smart device.

Using Orangeaid

- 1. Setting the Fitting**
Step 1. The user may run automatic setting at the first time of using.
Step 2. After completing automatic setting, the user may adjust setting values on manual setting.
Step 3. After completing the setting, enter save location, wearing location and place, and then, save the settings. (Max. 6 fields)
Step 4. The saved setting values may be deleted on 'Setting Information Edit'.

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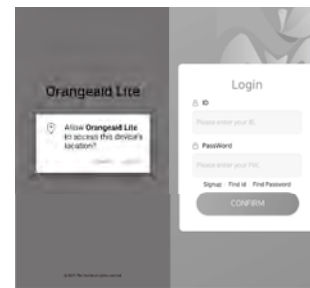
- First user needs to press Sign-in and proceed, referring to the sign-in procedure.
- Enter ID and Password to log in.

[Cautions at the time of signing-in]

ID : 4 letters or more, No same letter

Password : 4 letters of more

E-mail Address : In case the password is forgotten, temporary password will be sent to the e-mail address.

Step 3. Linking the Product

- Turn on Orangeaid and press 'Search Device' button to search the product.
- Press the searched device on the linkable device lists to make a link (YSH-C2000)

2. . Putting on the Product

Step 1. Press the ear-tip to make it small before putting on it.

Step 2. Put on it in the direction as shown in the image.

Step 3. Move the body to stabilize the ear-tip.

Step 4. The product is shipped, with S size ear-tip and wing-tip being connected. If they do not fit your ear, replace them with another size.

! Caution

Beep sound (feedback) may be heard during putting on the product.
In case the ear-tip is not fully swelled up or it does not fit your ear, beep sound (feedback) may be heard.

3. . Running the App

Step 1. Run the installed Orangeaid+ App.

Step 2. Signing up for a Membership and Logging-in

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⚠ Caution

When you first use Orangeaid after getting Orangeaid, make full charge of the battery.
 Make sure to connect Orangeaid with a smart device to use the functions of it.
 How to connect and screen may vary depending on the software version of a smart device. version.
 Supported functions and details may vary depending on the models, network operators or manufactures.

Starting Orangeaid

1. . Turning On/Off the Power/Pairing
 Turning on the Power (Green LED lights up) : Make a long pressing of the power switch (for About 2 seconds)
 Turning off the Power (Red LED lights up) : Make a long pressing of the power switch (for about 1 second)
 Pairing Mode (Green LED blinks) : Make a short pressing of the power switch twice

⚠ Caution

Turn off the power while you are not using Orangeaid.
 When Orangeaid is put in the cradle after use, the power is turned off automatically.
 But the battery power is insufficient, Orangeaid may not be turned off automatically.

3. Charging

< Before start >

Remove protection tape

Peel off the protection tape below the power button and charge the Orangeaid.

a. Charging the Cradle

Step 1. Connect USB C type Cable supplied together with the cradle.

Step 2. When charging starts, the red LED lights up.

Step 3. After full charging, the blue LED lights up.

b. Charging Orangeaid

Step 1. Put Orangeaid in the cradle and close the cover.

* Battery of Orangeaid may be checked on the app(App>Setting>info.Orangeaid).

(Cannot check while charging in cradle. Check after connecting with the App.)

! Caution

Peel off the protection tape below the power button of the Orangeaid before charging.
Check if the cover of the cradle is completely closed when charging Orangeaid.
In case the power of the cradle is insufficient, Orangeaid may not be charged.

4. Linking with Smart Devices

Step 1. Smart device Setting App > Select Connect (or BLuetooth) > Activate Bluetooth

Step 2. Press Orangeaid Power switch for About 2 seconds, and the LED lights up.

Step 3. Press Power switch twice, and the device enters Pairing Mode, with the green LED blinking

Step 4. When the device name is display on the screen, touch on it to make pairing.

(YSH-C2000) · Calling and music listening functions could be used when the product is linked with a smart device.

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Description of Markings

- ⚠ Warning : In case of non-compliance, direct impact may be caused.
- ⚠ Caution : In case of non-compliance, abnormal operation or malfunction may be caused.

Before Using Orangeaid

1. Becoming familiar with general descriptions before using

- Step 1. Please be familiar with how to use the product and the descriptions in the user manual.
- Step 2. Check if there is a wound or foreign substances in the ear.
- Step 3. If foreign material is on the product, brush off it with the brush supplied together with the product.

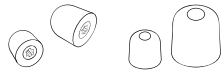
2. App Installation

Search "Orangeaid+" on the Google Play or App store and install the App in the mobile device.

• Orangeaid+ supports Android 4.4 or higher
iOS 10 or higher version.

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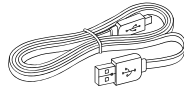
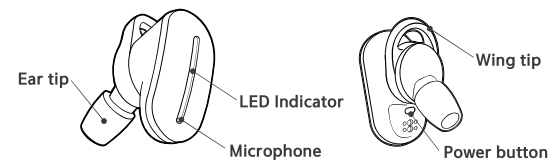
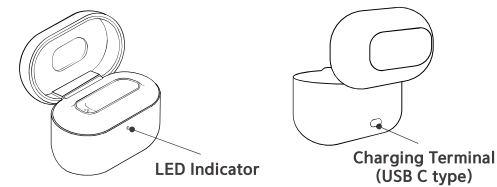
Main Functions of Orangeaid

◆ Performance

Sound processing technologies developed by The Yeolrim being applied, you may hear the ambient sounds louder and clearer.

◆ Personal Customization

Orangeaid provide a function to adjust the performance through the dedicated app.

◆ Telephone Calls

You may make telephone calls with Orangeaid, being linked with a smart device.

◆ Listening to Music

You may listen to music with Orangeaid, being linked with a smart device.

◆ Locating the Product

Orangeaid provide functions of product locating and notice searching, being linked with a smart device.

◆ Convenience

The product could be charged anytime and anywhere with the portable charging cradle.

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Orangeaid
Orangeaid

User Manual YSH-C2000

- This product could not be used to determine if you have a disease.
- This product is not a medical device.

Orangeaid