

 **amplicomms**

TV3500



SUMMARY

- Getting Start -----3
- Description of headset -----3
- Description of base -----4
- Description of remote control -----5
- How to install Amplicomms TV3500 -----5
- Connecting the base -----5
- Base set up -----6
- Changing the rubber ear buds -----7
- Installing batteries in the remote control -----7
- Registering the handset -----7
- Operation -----8
- Using the volume keys -----8
- Adjusting the sound level balance -----8
- Select audio profile (Equalizer) -----9
- Confirmation Tone -----9
- Using microphone button -----9
- Power save mode -----9
- Troubleshooting -----10
- Safety Information -----10
- Technical data -----12
- Warranty -----12
- Environment -----14
- Conformity -----14

Thank you for having purchased the Amplicomms TV3500!

Amplicomms TV3500 enables to listen to a TV or HiFi device with a headset, using cordless transmission.

If you use hearing aids, you need to remove them before using the headset.

IMPORTANT: PLEASE MAKE SURE YOU HAVE READ ALL SAFETY INFORMATION ON PAGE 10 BEFORE YOU SET UP YOUR DEVICE.


NEVER LET CHILDREN USE THIS DEVICE.

Getting Start

Box contents

- 1 headset with rubber ear buds size "S" (small)
- 1 additional pair of rubber ear buds size "L" (large)
- 1 base
- 1 infrared remote control
- 1 power adapter for the base
- 1 optical TOSLINK Digital Audio cable (with plastic protection)
- 1 analog audio cable (both ends with 3.5 mm jack stereo plug)
- Multi-language user guides
- 2 AAA size batteries for remote control


Description of headset

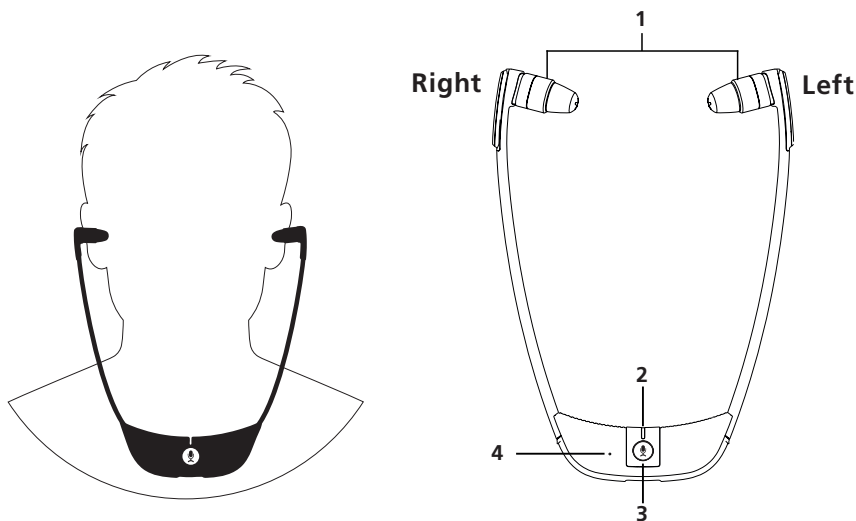

1. Left / Right earpiece
2. LED indicator
3. Microphone button for room monitoring / Power ON/OFF / TV listening mode
4. Microphone hole


Description of base

Base top view

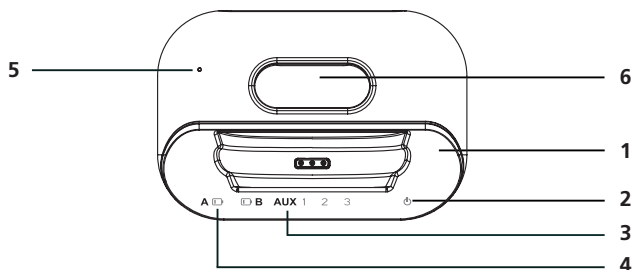



1. Charging cradle
2. Power LED indicator
3. Input LED indicator (3 = Microphone, 2 = Analog input, 1 = Optical input/ TOSLINK)
4. Battery low LED indicator
5. Microphone hole
6. Remote control support

Base rear view

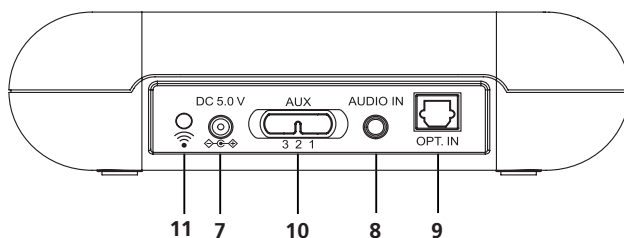

7. Power supply socket
8. Analog Audio In Socket (3.5 mm)
9. Optical Audio In Socket TOSLINK
10. 3 = Microphone, 2 = Analog input (3.5mm), 1 = Optical input/TOSLINK Aux Switch
11. Pairing button

Description of remote control

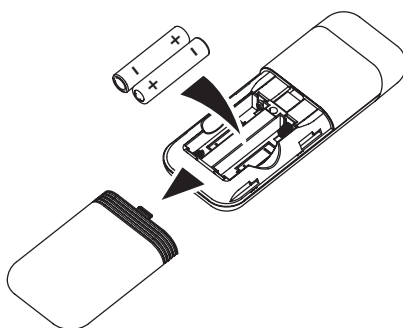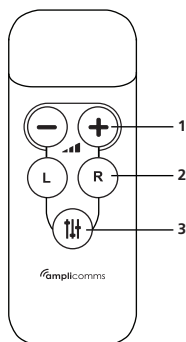


1. Volume control
2. Headset left/right balance
3. Equalizer (Sound Clarity mode)

How to install Amplicomms TV3500

Connecting the base

1. Connect the power adapter to the DC input socket located on the base rear panel.
2. Connect the power adapter provided to an electrical outlet.
3. Connect the base to your audio source.


There are 2 ways to connect the base to the audio source (TV/Hifi): either using one of the two supplied cables (analog or optical), or using TV/Hifi audio sound (if no audio output socket on your TV/Hifi device). Cable mode is preferred. Check on your TV / Hifi set which audio output solution is available (analog or optical).

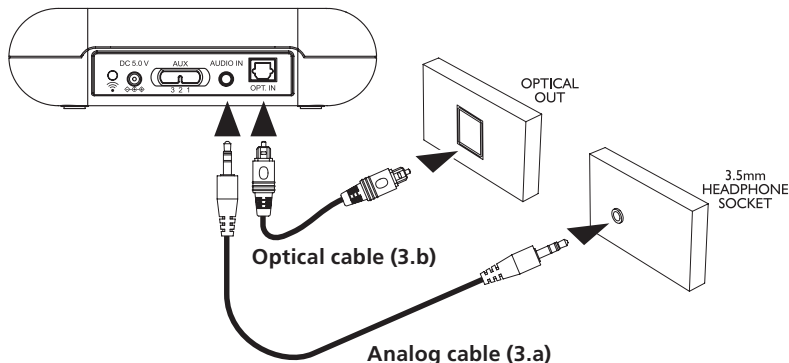

3.a. Using the analog audio cable.

Connect one side of the analog cable to the base and the other side to the analog audio output TV/Hifi socket. You may have to purchase adaptor plugs if your TV/Hifi output socket has no 3.5mm audio output (see drawings below).

Then on the base, move the switch (AUX) on position 2 = Analog input.

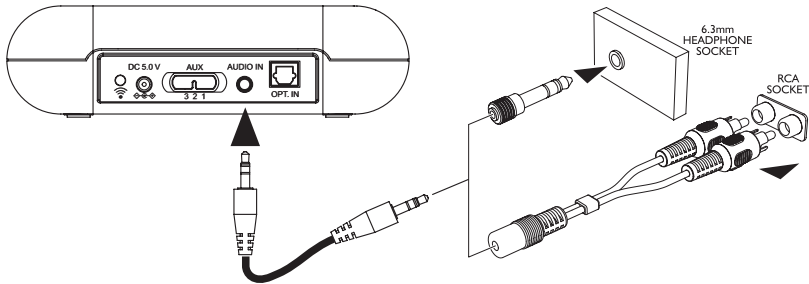

Note : Adaptor plugs are not provided

3.b Using the optical audio cable

Connect one side of the optical cable to the base and the other side to the TV audio optical output.

Then on the base, move the switch (AUX) on position 1 = Optical input.

This connection offers a higher sound quality and a lower susceptibility to electrical and magnetic interference.

Please make sure the optical TOSLINK digital audio cable does not get bent or folded! Avoid tight bending radii.

3.c Using base microphone (only if no audio output on TV).

1. If your TV set does not have any audio jack, you should use the microphone to pick up the signal. Put base close to your TV/Hifi loudspeaker for better signal pickup. Then on the base, move the switch (AUX) on position 3 =Microphone.
2. Please start with low volume on the TV. Adjust the TV volume level up slowly so as to determine the suitable volume level. The position of the base also has a decisive influence on the volume – please determine the best position by experimentation.

Base set up

Charge the headset for about 2 hours and 30 minutes before first use. Place the headset on the charging cradle of the base. The headset LED indicator will be red and will turn to purple when the battery is fully charged.

- A fully charged battery provides up to 6 hours of listening enjoyment.

Advice: Always have the headset fully charged and operational, put the headset back on its charging base after use.

Changing the rubber ear buds

1. For optimum user experience, we have provided 2 sizes of ear buds. Large and Small. If the small size fitted to the headset does not suit you, please try the alternative large size ear bud supplied in the box.
2. For changing the ear buds pick the mounted ear buds at the end and gently pull them out. Then position the alternative ear buds onto the "open ends" and let them snap into the place.

Installing batteries in the remote control

1. Insert AAA size batteries into the remote control with correct polarity.
2. To test the functions, put the headset on your ears and turn on the TV. Point the remote control to the base and press the volume $\oplus$ or $\ominus$ buttons, $\textcircled{L}$ or $\textcircled{R}$ balance buttons or $\textcircled{\text{III}}$ button to adjust the sound level and quality.

Registering the headset

The base and the headset are paired in advance. Normally, there is no need to pair them again.

1. Press and hold the Pair button at the back of the base for 10 seconds. The power LED will flash continuously.
2. Press and hold the $\textcircled{\text{M}}$ button on the headset for 10 seconds. The LED will flash continuously.
3. When registration is successful, the base and the headset LED will become steady on.

Operation

Important: Hearing damage due to high volumes!

This headset allows you to set higher volumes than conventional headset (up to 125 dB!). This can cause hearing damage for persons with healthy hearing and especially children.

- Do not continuously expose yourself to high volumes.
1. Before using the headset, set the volume on the remote control to a low value.
 2. Switch ON the audio source (TV/Hifi).
 3. If the headset is OFF, the LED will be OFF. Short press the  button on the headset to turn on it.
 4. Spread the headset arms until the ear buds can be inserted in the ears. Please note the ear buds can be angled through a movement up, down or sideways so as to enter the ear a little deeper. This will improve the sound quality and volume. Gently press the ear buds further into the ear to find a comfortable position for the ear buds and  or  buttons from the remote control.

Note: When the headset battery level is low you will hear 2 beeps every minute in the headset.

Using the volume keys

Point the remote control at the base, press the  or  buttons to achieve a comfortable loudness.

Adjusting the sound level balance

Point the remote control at the base, press the  or  buttons to adjust the headset left or right receivers sound level balance.

Select the audio profile (Equalizer)

Point the remote control at the base, press the  button to select one of the 4 available audio profiles.

Setting	Effect
1	Flat
2	Bass Boost
3	Treble Boost
4	Clarity

Note : To go back to factory default settings, short press the pair button at the back of the base.

Confirmation Tone

There is a “beep” tone as confirmation for every effective key press on TV3500 remote control. Some special tone will be used for below listed case:

1. 2 beeps will be heard when the volume level is reaching the maximum or minimum.
2. 2 beeps will be heard when the left and right sound level difference is reaching the maximum.
3. A long beep will be heard when the left and right sound level is equal balanced.
4. 2 beeps will be heard when EQ1 (Flat) is selected.

Using microphone button

In active mode, press the  button will switch from TV listening mode to transparency mode. The headset indicator will switch from blue to red. The microphone hole is at the front of the headset to pick up the sounds of your surroundings. Make sure that the microphone opening is not covered!.

When the microphone is switched ON, you will hear your surroundings louder. You can hear for example your door bell or your conversational partner even when you are wearing the headset.

To switch back to the TV listening mode, press the  button again. The headset indicator will turn to blue and you will hear the sound from the audio source again.

Power save mode:

To save power, the headset will enter sleep mode when there is no signal detected for approximately 5 minutes. To wake up the headset, press  button.

Note : To turn OFF the headset, press and hold the  button for 3 seconds. The headset LED will turn OFF.

Troubleshooting

No sound

- Ensure that the mains adapter is fully pushed into the power outlet and that its cable is properly connected to the DC IN connector on the base.
- Ensure that the headset ON. Either red or blue LED is steady on.
- The charge level of the headset battery may be too low. Put the headset on the charging cradle for 2 hours.
- Ensure that the TV set, Hi-Fi system or audio device is switched ON and the input source selection/connection is correct (AUX switch).
- The connected audio/video equipment may not be in playback mode. Start playback on the equipment.
- The headset output level is set too low. Adjust the volume to a suitable level with the remote control.

Distortion

- The headset may be too far away from the base. Move closer to it.
- The audio signal input level is too low/high. Increase/decrease the volume from the audio source TV/Hifi.

Safety Information

Please read this operating instruction manual thoroughly.

Intended use

This device was developed to transmit audio signals from an audio source like TV or HiFi device to a headset, using cordless transmission. Any other use is considered unintended use. Unauthorised modification or reconstruction is not permitted. Under no circumstances open the devices or complete any repair work yourself.

Installation location and safety information

- Prevent excessive exposure to smoke, dust, vibration, chemicals, moisture, heat and direct sunlight.

- Furniture polish, over time, may disintegrate the rubber feet of the base which may stain your furniture. To avoid a potential stain, you may want to place the base on a pad or mat.
- This listener allows you to amplify sounds to a loud volume, but listening to earphones at a high volume over an extended period of time may damage your hearing; especially children. Thus, we suggest that you set the volume to a level that is comfortable to your hearing.

For users wearing a pacemaker

It is recommended that, if you have a pacemaker fitted, you check with a medical expert or your health adviser before using this product. Switch the device OFF immediately if you notice or suspect any adverse effects.

Power adapter plug

Attention: Only use the power adapter plug supplied since other power adapter plugs could damage the base. Ensure access to the power adapter plug is not obstructed by furniture or such.

Pay attention that the plug and cable are in perfect condition. Kinked or worn cable represents the risk of a fatal accident!

Maintenance

- Clean the housing surfaces with a soft, fluff-free cloth.
- Never use cleaning agents or solvents.

Technical data

Carrier frequency: 1.9GHz Digital

Range: Up to 30 meters (open area)

Frequency response: 30 Hz – 20 KHz

Max volume: 125 dB

Distortion: < 1.5%

Signal-to-noise ratio: > 70 dB

Operating voltage (Base): 5VDC, 550mA

Operating voltage (Headset): 3.7 V, 450 mA Lithium Polymer Battery

Operation time: With fully charged battery up to 6 hours

Charging time: For empty battery approx. 2.5 hours

Weight Headset: 61 g

Weight Base: 150 g

Warranty

Please read carefully the user guide included in this box. TV3500 is an approved equipment in accordance with the European regulations, attested by the CE marking. The product you just bought is a technological product, it must be handled with care.

Note: You have a legal warranty on this product in accordance with the regulations applicable to the sale of consumer goods in the country in which you made this purchase.

For any information regarding this legal warranty, please contact your dealer.

Warranty period, counted from the date of purchase of the new product, the date on your invoice or ticket checkout where is indicated the name of your reseller, and equal to: Twenty four (24) months for the headphone, excluding consumables, accessories and batteries. For any claim under this warranty during the warranty period, you must return the entire Product to your dealer, together with the proof of purchase, the invoice or receipt issued by your dealer, indicating the place of purchase and the serial number of the Product, following the appearance or discovery of the defect. Return unit will be repair any defective item due to a defect in design, material or workmanship, at its expense, or to replace, at its expense, by an identical or at least equivalent element in terms of functionalities and performance. If the repair or replacement is impossible under normal commercial conditions, the Product will be refunded or replaced by a Product equivalent. Within the maximum limit allowed by the applicable law, the product or element replacement, which may be new or reconditioned, is warranted during a period of ninety (90) days from the date of repair or until the end date of the initial warranty period,

whichever is the earlier greater than ninety (90) days, it being specified that any period of immobilization of the Product of at least seven (7) days will be added to the warranty period remaining to run.

This warranty does not apply in the following cases:

- Installation or use not in accordance with the instructions in the guide of the user,
- Incorrect connection or abnormal use of the Product, especially with noncompatible accessories, as indicated in the user guide,
- Product opened, modified or replaced by means of non-approved parts, serial number deleted, illegible, damaged,
- Normal wear, including normal wear and tear on accessories, batteries and displays,
- Failure to comply with the technical and safety standards in force in the area geographical use,
- Product having suffered a shock or a fall,
- Product damaged by lightning, electrical surge, heat source or radiation, water damage, exposure to conditions of temperature, humidity or other excessive ambient conditions or any other cause external to the product,
- Negligence or faulty maintenance,
- An intervention, a modification or a repair carried out by a not authorized person.

If the returned Product is not covered by the warranty, you will receive a quote from repair that will mention the cost of analysis and transportation costs that you will be charged if you want the Product returned to you. Subject to the legal provisions in force, all warranties other than described herein are expressly excluded.

TO THE MAXIMUM EXTENT PERMITTED BY APPLICABLE LAW,

- A) THIS WARRANTY IS EXCLUSIVE TO ALL OTHER WARRANTIES EXPRESS OR IMPLIED, INCLUDING, BUT NOT LIMITED TO, WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE PARTICULAR;
- B) We DISCLAIMS LIABILITY FOR LOSS OR DETERIORATION OF DATA, LOSS OF USE LOST TO WIN, LOSS OF CHANCE, DEREVENUES OR INCOME, LOSS OF OPERATION, DAMAGES INDIRECT, CONSEQUENTIAL, CONSEQUENTIAL OR INCIDENTAL;
- C) LIABILITY IS LIMITED TO THE VALUE OF PURCHASE OF PRODUCT EXCEPT HEAVIL OR UNINTENTIONAL FAULT AND EXCEPT DAMAGE BODY

Environment



This symbol means that your inoperative appliance must be collected separately and not mixed with the household waste.

Help us preserve the environment in which we live!

Conformity

The logo printed on the products indicates the conformity with all essential requirements (FCC Part 15B and 15 D). Y

Maximum Output Power (NTP) : 24dBm

Frequency Range : 1920-1930MHz

FCC Statement

1. This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

(1) This device may not cause harmful interference.

(2) This device must accept any interference received, including interference that may cause undesired operation.

2. Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

Reorient or relocate the receiving antenna.

Increase the separation between the equipment and receiver.

Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.

Consult the dealer or an experienced radio/TV technician for help.

SAR Information Statement

Your wireless phone is a radio transmitter and receiver. It is designed and manufactured not to exceed the emission limits for exposure to radiofrequency (RF) energy set by the Federal Communications Commission of the U.S. Government. These limits are part of comprehensive guidelines and establish permitted levels of RF energy for the general population. The guidelines are based on standards that were developed by independent scientific organizations through periodic and thorough evaluation of scientific studies. The standards include a substantial safety margin designed to assure the safety of all persons, regardless of age and health. The exposure standard for wireless mobile phones employs a unit of measurement known as the Specific Absorption Rate, or SAR. The SAR limit set by the FCC is 1.6 W/kg. * Tests for SAR are conducted with the phone transmitting at its highest certified power level in all tested frequency bands. Although the SAR is determined at the highest certified power level, the actual SAR level of the phone while operating can be well below the maximum value. This is because the phone is designed to operate at multiple power levels so as to use only the power required to reach the network. In general, the closer you are to a wireless base station antenna, the lower the power output. Before a phone model is available for sale to the public, it must be tested and certified to the FCC that it does not exceed the limit established by the government adopted requirement for safe exposure. The tests are performed in positions and locations (e.g., at the ear and worn on the body) as required by the FCC for each model. The highest SAR value for this model phone when tested for use at the ear is **0.025W/Kg** and when worn on the body, While there may be differences between the SAR levels of various phones and at various positions, they all meet the government requirement for safe exposure. The FCC has granted an Equipment Authorization for this model phone with all reported SAR levels evaluated as in compliance with the FCC RFexposure guidelines. SAR information on this model phone is on file with the FCC and can be found under the Display Grant section of <http://www.fcc.gov/oet/fccid> after searching on

FCC ID: **2AYP5TV3500** Additional information on Specific Absorption Rates (SAR) can be found on the Cellular Telecommunications Industry Association (CTIA) web-site at <http://www.wow-com.com>. * In the United States and Canada, the SAR limit for mobile phones used by the public is 1.6 watts/kg (W/kg) averaged over one gram of tissue. The standard incorporates a substantial margin of safety to give additional protection for the public and to account for any variations in measurements.