User Manual

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Follow along and setup
with a 5-minute video:
setup.smartnora.com

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**Sleep better.
Live better.**

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Wake up tomorrow on the right side of the bed.

Smart Nora was invented by a family of professional snorers. Sick of waking up tired, or sleeping on the couch, we designed Smart Nora as a non-invasive solution to help us sleep better and wake up feeling refreshed.

During the night, the Pebble, a microphone equipped with detection algorithms, listens for the sounds of snoring. Upon detecting snores, it activates the silent pump in the Base to gently inflate and deflate the Pillow Insert. This slight movement interrupts your snoring, helping you sleep more deeply and wake up as your best self.

Over 100,000 customers are sleeping better because of Smart Nora. Our mission is to help the 3.3 billion adults around the world who snore, improve their sleep, their health, and their relationships.

PILLOW INSERT

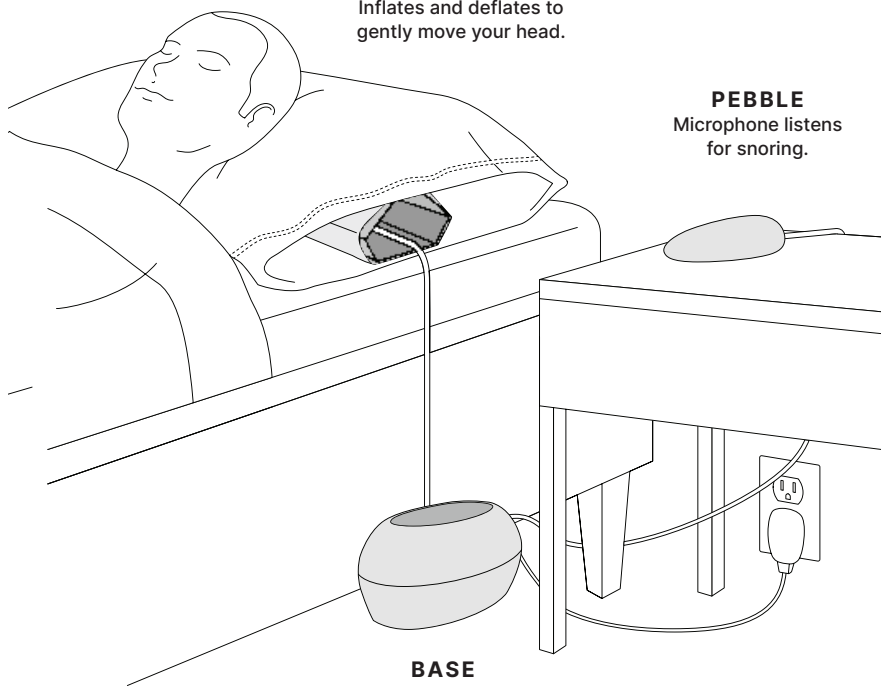
Inflates and deflates to
gently move your head.

PEBBLE


Microphone listens
for snoring.

BASE


Silent air pump sits
under your bed.


In the box

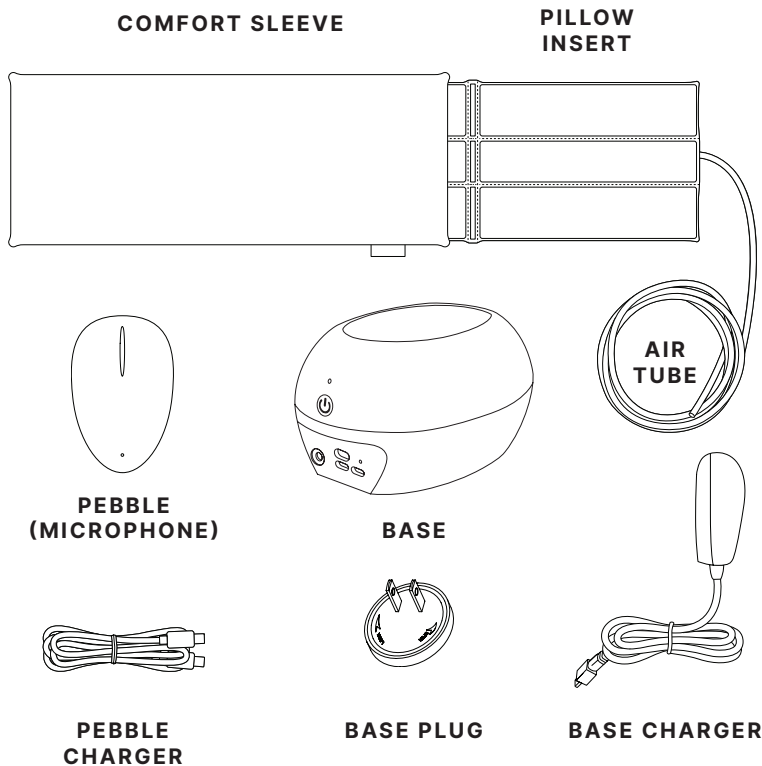Snoring Solution(PEBBLE)

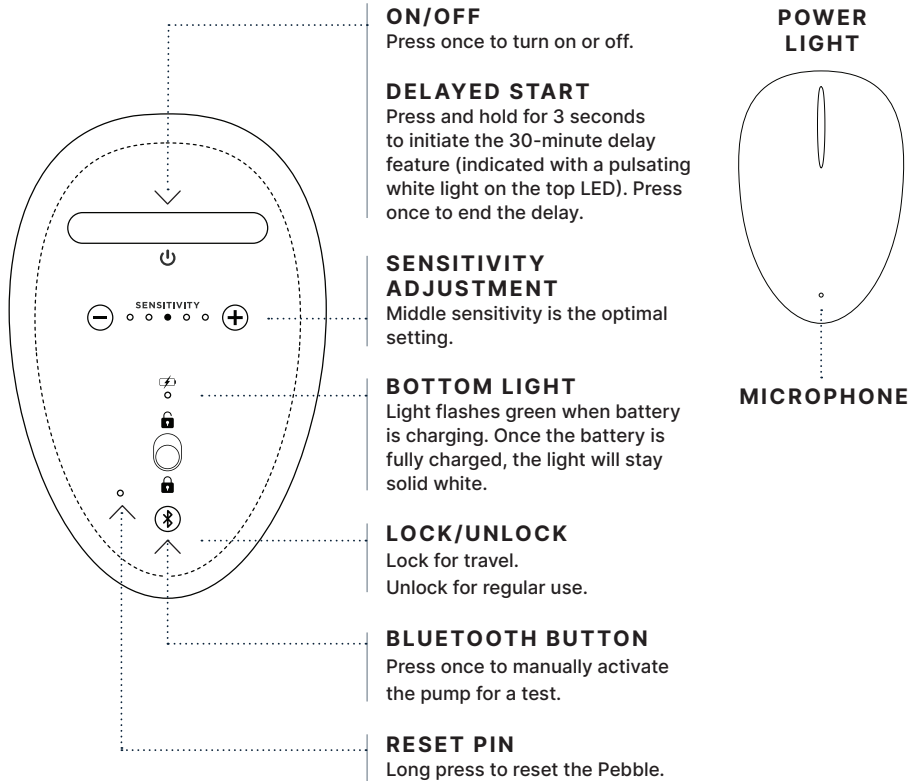Snoring Solution (BASE)

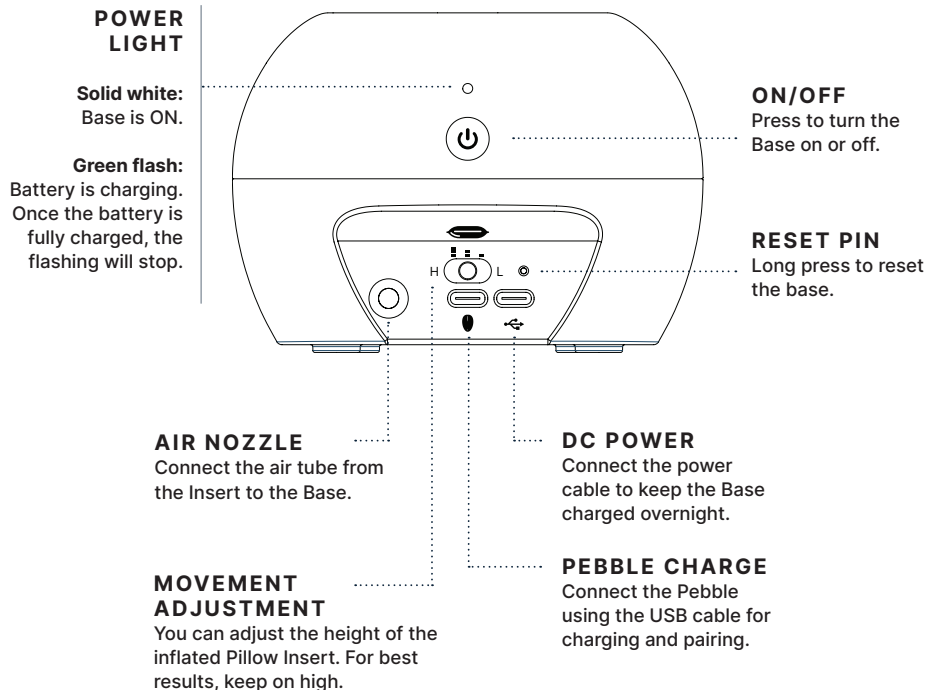Pebble & Base Lights

SOLID WHITE: ON/READY

Both Pebble and Base are on, paired, and fully charged.

PULSING WHITE

Pebble is in delayed start mode and will start listening for snoring in 30 minutes.

FLASHING RED: LOW BATTERY

Pebble and/or Base are low on battery. Plug in the Base to power. Connect the Pebble to the Base using the USB cable. Keep both plugged in for overnight use.

Pebble battery lasts for 1 night.
Base battery lasts for 1 night.

FLASHING GREEN: CHARGING

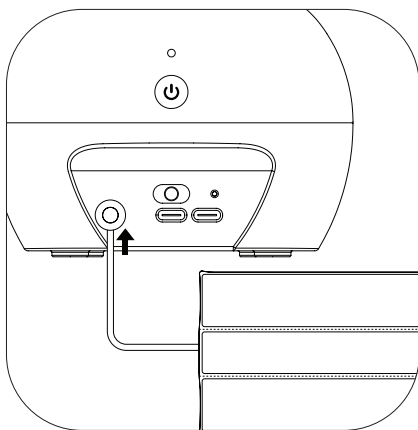
Pebble and Base are off and charging. The light will stop flashing once the battery is fully charged and ready to use.

FLASHING WHITE AND GREEN

On and Charging.


GETTING STARTED

Set up your Smart Nora.

1. Connect Air Tube.

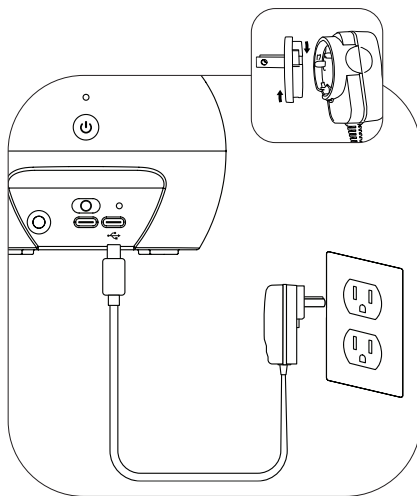
Connect the Air Tube from the Pillow Insert into the Base.

2. Connect Base Charger


Place Base Plug into Base Charger and twist to lock. Connect Base Charger into Base. Plug the Base Charger into a power source. Place Base on the floor under your bed.

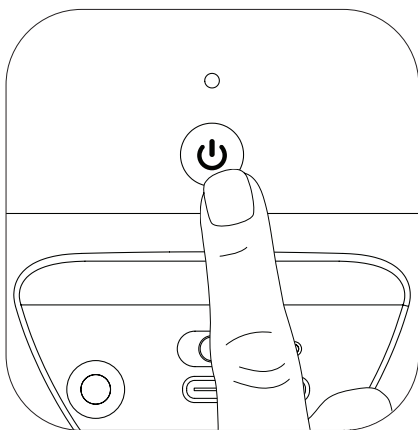- ✓ The LED on Base flashes green when charging.
- ✓ Once fully charged the battery lasts for 24 hours.

3. Turn the Base on.

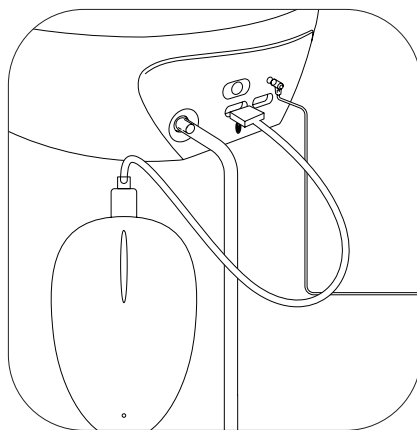
Press the power button to turn the Base On. White light will illuminate.

4. Connect the Pebble.


Connect the Pebble to the Base with the USB cable. Keep the Pebble plugged in for overnight use.

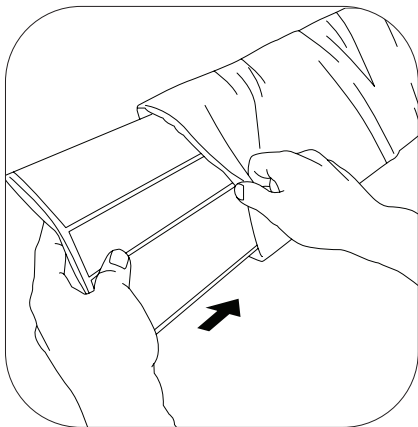- ✓ The LED on the bottom of the Pebble flashes green when charging.
- ✓ Once fully charged the battery lasts for 24 hours.

5. Slide Insert into Sleeve.

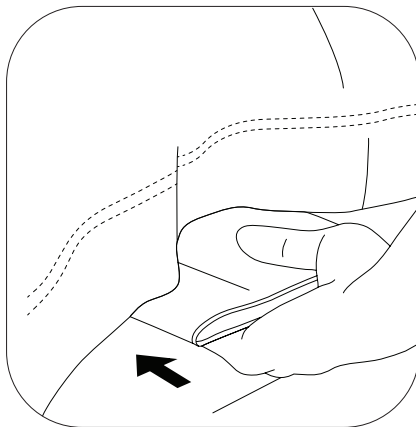Slide the Pillow Insert into the Comfort Sleeve.



6. Slide Insert under pillow.

Slide the Comfort Sleeve under the pillow inside the pillow case.

IMPORTANT:

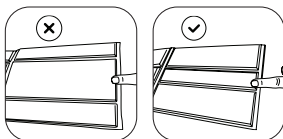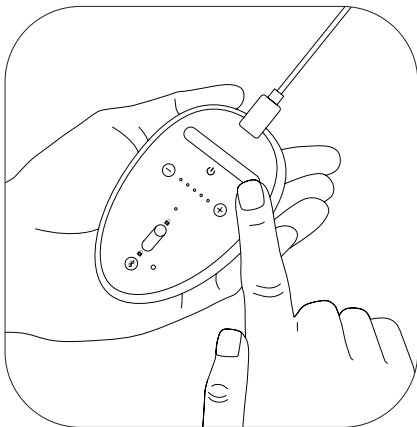Ensure the side with the thinner middle panel is facing up.


7. Turn the Pebble on.

Unlock the Pebble. Press ON/OFF button to turn the Pebble on.

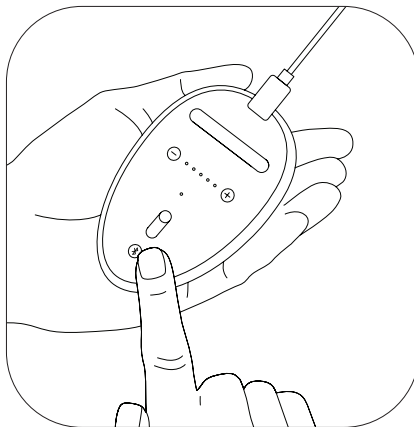
- ✓ Solid white light on top and bottom of the Pebble: On and ready.
- ✓ Blinking green and white light at bottom of the Pebble: On and charging.
- ✓ Blinking blue light: see support (page 26) for how to connect the Pebble and the Base.

8. Press Bluetooth.

Press the Bluetooth (ⓑ) button on the Pebble and watch for the white light to flicker. The pump is now activated.

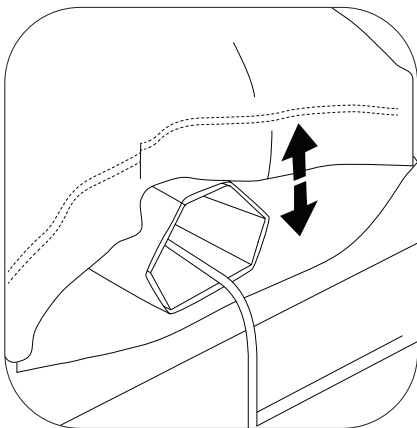- ✓ Note: You can not test Nora's snore detection by pretending to snore. Our algorithm is tuned to detect naturally occurring snores only.


9. Test before sleeping.

Put your head on your pillow. Press the Bluetooth  button on the Pebble. You should feel a gentle rise and fall (about 2 inches). Each rise and fall cycle takes 1 minute to complete. It will not deflate unless pressure is applied.

Setup is Complete!

Track Your Progress

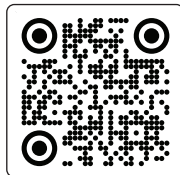Scan the QR code to download the Smart Nora App

Questions?

Contact us at:
support@smartenora.com
1-833-473-2550

Or chat with us live in
the Smart Nora app!

SLEEP WELL

Your New Nightly Routine

Keep the Base plugged in to the wall
and Leave it On.

Turn On the Pebble each night before
falling asleep.

Press and hold the large ON/OFF ⏻ button
on the Pebble for 3 seconds to initiate
30-minute delayed start (to allow yourself
to fall asleep before Nora listens for sounds
of snoring).

First 2 Weeks with Smart Nora:

What to expect

To experience best results, use Smart Nora for **at least two weeks**. The majority of users experience a reduction in their snoring after seven consecutive nights of using Smart Nora.

Download our app to track your progress and talk to the Smart Nora team directly any time.

Scan to start tracking your sleep journey today!

How do you know if Smart Nora is working for you?

1. Use the Smart Nora App

Whether you're the snorer or the partner, you can track the snoring patterns by downloading the Smart Nora app. We recommend using the app for 7 days without Nora and then with Nora to compare the difference. This will help you see improvements in snoring reduction and overall sleep quality.

2. Ask Your Partner

If you sleep with a partner, ask them about their quality of sleep. As snoring is reduced, they will experience fewer interruptions and wake up feeling more refreshed. Both the snorer and the partner benefit—while the snorer adjusts to Nora's gentle motion, they too will start sleeping better since they won't be nudged awake during the night to stop snoring.

For Snorer

NIGHTS 1-3

You may feel Smart Nora's slow movement a few times during the night.

To fall asleep easily, start the night with delayed start mode so you have 30 minutes to fall asleep before Smart Nora starts working.

NIGHTS 4+

Your body will adjust to the movements, and over time you will notice Smart Nora less and less.

NIGHTS 10+

Smart Nora will fade into the background, ensuring peaceful, uninterrupted sleep.

For Partner

NIGHTS 1-3

As your partner becomes accustomed to using Smart Nora, you will notice a reduction in intensity and frequency of their snoring.

If your bedroom is extremely quiet, you might notice the faint sound of the pump. But this will soon fade into the background.

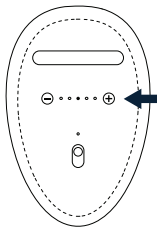
NIGHTS 4+

Your partner will be used to Smart Nora and you can both enjoy improved quality of sleep.

Customizing settings.

In the first few nights you can adjust the settings based on your comfort level and your bedroom environment.

Snore Detection

Start with the middle setting.

You can adjust the sensitivity setting to detect your snoring more or less often.

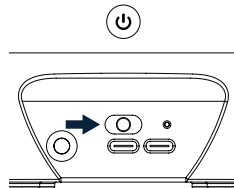
In a noisy bedroom you can decrease the sensitivity.

**DETECT
LESS
OFTEN**

**DETECT
MORE
OFTEN**


Pillow Height

Start with the middle setting.

According to your comfort, adjust the height setting to increase or decrease the movement of the pillow. It's best to get used to Medium or High setting.

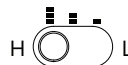**PILLOW
RISES
MORE**


**PILLOW
RISES
LESS**

EVERYDAY USE

**The optimal
experience.**

Pebble Placement

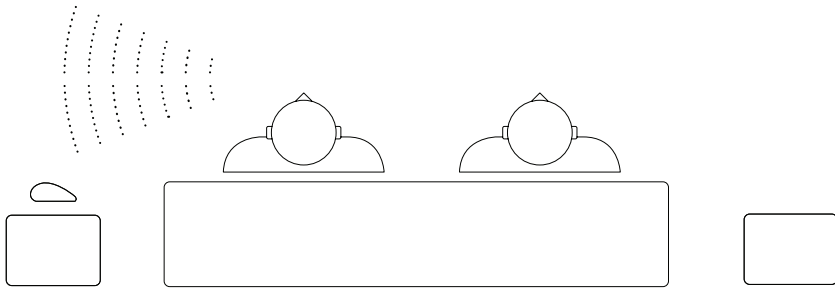✓ **Most Accurate**


For Accurate Detection

Place the Pebble close to the snorer, not the partner.


Place the Pebble within 3 feet (1 meter) or less of the snorer.

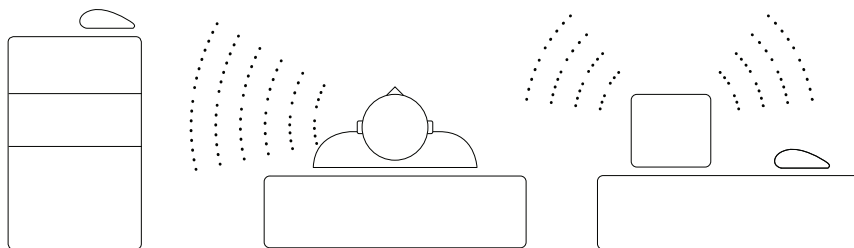⊗ Wrong Setup


- ⊗ Don't place the Pebble near the partner; place it close to the snorer.
- ⊗ Don't position the Pebble at a different level from your head—keep it at the same level.
- ⊗ Don't place the Pebble on an unstable or vibrating surface.
- ⊗ Don't position the Pebble close to noise sources like fans, AC vents, white noise machines, or radios. If you have such devices in the bedroom, place the Pebble on the snorer's side and keep the noisy device on the opposite side of the bed.

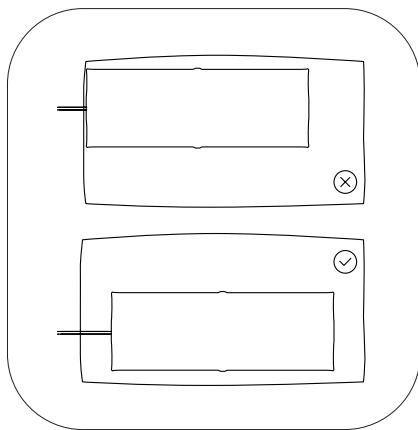Optimal Experience


Insert and Sleeve position

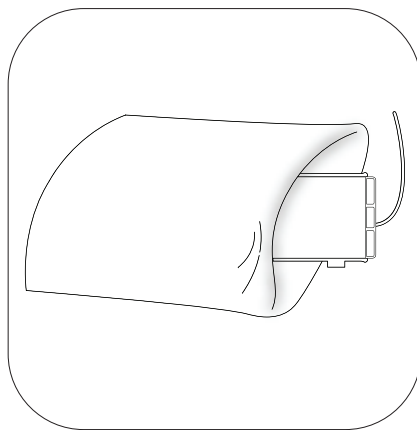Center the Pillow Insert and Comfort Sleeve inside your pillow. Position these towards the front edge of the pillow closer to your shoulder side.


Use the Comfort Sleeve

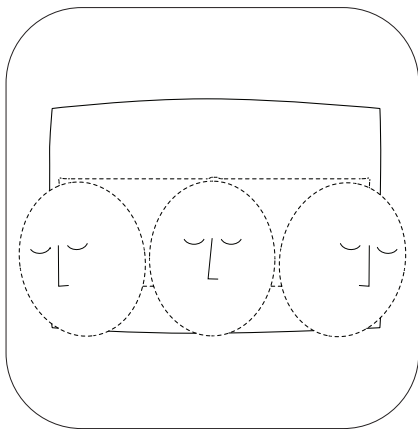This will make the Pillow Insert soft on all sides so you can comfortably put your hand under the pillow.

Make sure your head is on the pillow.

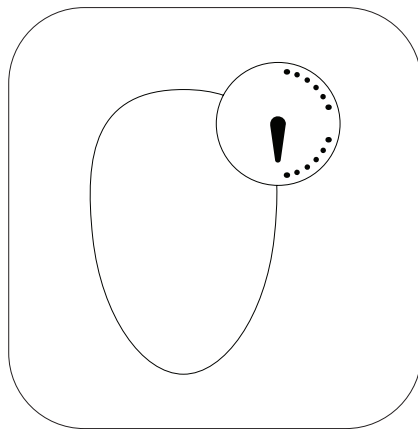You can place your head anywhere on the pillow as long as the Pillow Insert moves your head when it rises and falls.

Start the night with 30-minute delay

To fall asleep easily, start the night with the delayed start so you fall asleep before. Nora listens for snoring.

Support

Flashing Blue: Bluetooth issue

To pair the Pebble and Base

1. Connect the Pebble and Base with USB cable.
2. Lock the Pebble.
3. Turn the Base on.
4. Unlock the Pebble (Keep the Pebble off, all LED lights should be off)
5. **Press and hold the Bluetooth Button while the Pebble is off.** A purple light will flash.

When the purple light stops flashing, turn the Pebble ON, the LED should show a solid white light.

Nora is now ready.

If the purple light keeps flashing for more than 10 seconds, make sure there is a proper connection with the USB cable into the Pebble and Base. Make sure the Base is ON and you see the solid white light.

ADDITIONAL SUPPORT

Need help setting up your Smart Nora?
Have questions? Contact our support
and let's get you sleeping better as soon
as possible!

Call us: 1-833-473-2550

Web Support: support.smartnora.com

Email Support: support@smartnora.com

Or **chat with us live** in
the Smart Nora app!

Scan to
download the
app and get
help from our
support team.

Important Safety Information

READ CAREFULLY BEFORE USE.

Failure to follow these instructions could result in death or serious injury.

1. Smart Nora is intended to detect snoring and reduce snoring sounds by slowly moving the user's pillow by means of adjusting the volume of the air-inflated Pillow Insert. The device is indicated for those whose periodic snoring causes their partners to wake up. The device is not indicated for diagnosis or treatment of sleep apnea or any other medical condition.
2. **WARNING:** To avoid danger of suffocation, keep Smart Nora away from infants and children.
3. If your sleep habits and positions put you at risk of entangling a cord or tube around your neck while sleeping, do not use Smart Nora.
4. Smart Nora is not a monitoring device and will not warn you of any sleeping problems.
5. Avoid placing the power cable and air tube where it could present a tripping hazard.
6. Do not use Smart Nora around water or other liquids. Like any electronic device, there is an electrocution hazard if you get Smart Nora wet when it is plugged in.
7. Do not disassemble the Pebble, Base, or power adapter. There are no repairs that you can safely perform.

IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, basic safety precautions should always be followed, including the following:

READ ALL INSTRUCTIONS BEFORE USING

WARNING – To reduce the risk of burns, electrocution, fire, or injury to persons.

1. A product should never be left unattended when plugged in.
2. Close supervision is necessary when this product is used by, on, or near children or invalids.
3. Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
4. Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the product to a service center for examination and repair.
5. Keep the cord away from heated surfaces.
6. Never block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair, and the like.
7. Never drop or insert any object into any opening or hose.
8. Do not use outdoors or operate where aerosol (spray) products are being used or where oxygen is being administered.
9. Prevent unintentional starting. Ensure the switch is in the off-position before picking up or carrying the appliance. Carrying the appliance with your finger on the switch or energizing appliance that have the switch on invites accidents.
10. Recharge only with the charger specified by the manufacturer. A charger that is suitable for one type

of battery pack may create a risk of fire when used with another battery pack.

11. Under abusive conditions, liquid may be ejected from the battery; avoid contact. If contact accidentally occurs, flush with water. If liquid contacts eyes, additionally seek medical help. Liquid ejected from the battery may cause irritation or burns. (This advice is considered correct for conventional lithium-ion cell types. If this advice is incorrect for a cell design that differs from these, then the correct advice may be substituted.)
12. Do not use a battery pack or appliance that is damaged or modified. Damaged or modified batteries may exhibit unpredictable behavior resulting in fire, explosion or risk of injury.
13. Do not expose a battery pack or appliance to fire or excessive temperature. Exposure to fire or temperature above 130°C (266°F)

may cause explosion.

14. Follow all charging instructions and do not charge the battery pack or appliance outside of the temperature range specified in the instructions. Charging improperly or at temperatures outside of the specified range may damage the battery and increase the risk of fire.
15. Have servicing performed by a qualified repair person using only identical replacement parts. This will ensure that the safety of the product is maintained.
16. Do not modify or attempt to repair the appliance or the battery pack (as applicable) except as indicated in the instructions for use and care.
17. Use only with Gang Qi / GQ07-050100-DX charger or other chargers included with the product or provided by the manufacturer for use with this product.

This appliance is intended for household use only.

SAVE THESE INSTRUCTIONS

DISPOSAL INFORMATION FOR USA

- Do not dispose in household trash
- Recycle at local recycling centers or retailers
- Find recycling locations: www.call2recycle.org

DISPOSAL INFORMATION FOR OTHER COUNTRIES

Pursuant to and in accordance with Article 14 of the Directive 2012/19/ EU of the European Parliament on waste electrical and electronic equipment (WEEE), and pursuant to and in 2006/66/ EC of the European Parliament on batteries and accumulators and waste batteries.

The barred symbol of the rubbish bin shown on the equipment indicates that, at the end of its useful life, the product must be collected separately from other waste. Any products that have reached the end of their useful life must be given to waste discal and electronic equipment, or given back to the dealer when purchasing a new WEEE, pursuant to and in accordance with Article 14 as implemented in the country. The adequate separate collection for the sub-sequent start-up of the equipment sent to be recycled, treated and disposal of in an environmentally compatible way contributes to preventing possible negative effects on the environment and health and optimizes the recycling and reuse of components making up the apparatus. Abusive disposal of the product by the user involves application of the administrative sanctions according to the laws in force.

FCC STATEMENT

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

WARNING: Changes or modification to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

1. Reorient or relocate the receiving antenna.
2. Increase the separation between the equipment and receiver.
3. Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
4. Consult the dealer or an experienced radio/TV technician for help.

CAN ICES-003(B) / NMB-003(B) IC STATEMENT

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

- (1) This device may not cause interference.
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

DISCLAIMER

NON-MEDICAL DEVICE

Smart Nora is intended to rely on the detection of sounds consistent with snoring to slowly move the user's pillow as an aid in reducing snoring. The device is indicated for those whose periodic snoring causes their partners to wake up. The device is not indicated for diagnosis or treatment of sleep apnea or any other condition or disorder.

WARNING:

Potential indicators of sleep disorder/disease: Should any of the following symptoms or conditions co-exist with your snoring/sleeping difficulties, you should see your physician for an evaluation of obstructive sleep apnea.

1. Frequent excessive daytime sleepiness.
2. Periods of not breathing, as observed by bed-partners.
3. Awakening short of breath, choking or gagging.
4. History of heart disease.
5. Substantially overweight.

CONTRAINDICATIONS

- Smart Nora is not for use by infants or children.
- Smart Nora is not for continued use if ineffective.
- Smart Nora should not be used if it causes pain or other discomfort.

WARRANTY

Smart Nora Inc. offers a one-year manufacturer's warranty. Warranty does not apply to damage caused by accident, abuse, misuse, flood, fire, earthquake, other external causes or use with improper power supply.

MAINTENANCE

The Pillow Insert and the felt bag cannot be washed or dry-cleaned. The Base and the Pebble contain electronics; to avoid damage, do not expose any of the parts to liquids, including water and cleaning products. The Pillow Insert may be cleaned using a damp cloth.

CAUTION

Replacement of a battery with an incorrect type that can defeat a safeguard; disposal of a battery into fire or a hot oven, or mechanically crushing or cutting of a battery, that can result in an explosion; leaving a battery in an extremely high temperature surrounding environment that can result in an explosion or the leakage of flammable liquid or gas; a battery subjected to extremely low air pressure that may result in an explosion or the leakage of flammable liquid or gas.

ADDITIONAL BATTERY CHARGING & SAFETY INSTRUCTIONS

Your Smart Nora's Base Unit and Pebble contain rechargeable lithium-ion batteries.

OPERATION & CHARGING

- Use only provided charging cables and base charging adapter
- During charging green indicator light will blink
- When fully charged indicator light will be solid
- Keep at room temperature (50°F-95°F / 10°C-35°C)
- Normal discharge occurs during nightly use and typically lasts 1 night
- Recharge when indicator light blinks red

CAUTION: TO PREVENT FIRE/BURNS:

- Use only provided charging equipment
- Do not disassemble, crush, or expose to extreme heat
- Stop use if devices become unusually warm

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©2025