



Wireless Visual Ear Pen
Model: U0010

User Manual




App Install

1. Please scan QR code to download Android or IOS app. Or search "TKSEE" in App store or Google Play Store to download.



WIFI Connection

WIFI Name: HBY-*****
WIFI Password: 12345678
Please find this WIFI on device and connect.

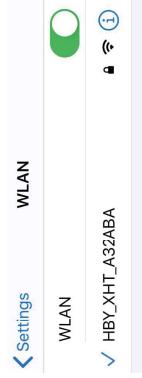

Product Features

- Wireless connection
- Built-in wifi-hotspot
- Real-time visualization
- Six-axis directional gyroscope, which can realize body rotation and keep the lens angle of view unchanged
- Lightweight aluminum body, can remain temperature of about 25 degrees celsius
- Otoscope 3.9mm ultra-thin design, easy to enter the ear
- Cold light led shadowless lamp
- Mouth, nose, ears, skin, multiple uses
- Suitable for smartphones and tablets
- HD 130W lens pixels




Product Details

Model: U0010
Lens diameter: D3.0mm
Lens focus: 1.4-1.8cm
Lens pixel: 130W
Gravity sensor: Six axis
Network standard: IEEE 802.11B/G/H
Wireless antenna: built-in 2.4GHz 3dbi copper tube antenna
Image transmission speed: 30fps
Battery Capacity: 200mAh
Battery working time: about 90 minutes
Charging input: DC5.0V/260mA
Charging port: Micro charging port
Charging time: about 1.5 hours
Product size: D13*120mm (20mm thin tube)
Product net weight: 0.032KG
Product material: aluminum + plastic
Color: black/red/green
Working temperature: -10-50 °C

Made In China




App Display

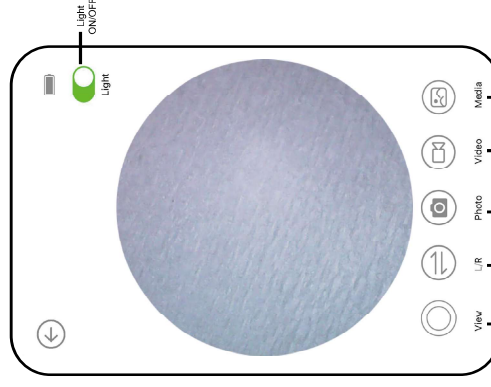

When WIFI connect successfully, we can use it normally via real-time view.


Product Structures

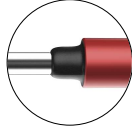

Otoscope Ultra-thin Design



Dedicated Ear Spoon
Two Longer & Two Shorter

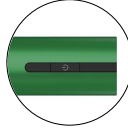

Aluminum Material
Lightweight & Fast Heat Dissipation

Warnings

- Please Keep out of reach of children
- Children are not allowed to use alone
- Lens can't be knocked with sharp objects
- If lens is dirty, please wipe with a small amount of alcohol
- Don't soak in water for a long time
- Please use and store this product at room temperature
- The first time you use it, you can gently deepen the ear spoon into your ear by feeling, and then look at the app, which can help eliminate the psychological fear and prevent misuse

Packing List

Ear Pen 1PCS
Ear Spoon 4PCS
USB-Micro Charging Cable 1PCS
User Manual 1PCS
Package Box 1PCS

FCC Caution:

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment.