# OSKAR SV-100

## PORTABLE TV VOICE ENHANCER

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Enjoy listening  
to the TV again  
with OSKAR.

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[faller-audio.com](http://faller-audio.com)

## THANK YOU FOR PURCHASING OSKAR

OSKAR is an innovative wireless TV speaker created for the hearing impaired that enhances the dialogue within TV programs and movies.

With the touch of a button, the superior audio technology refines voice and speech frequency bands while filtering out and minimizing distracting background noises.

This guide provides you with product details and user information. Please follow the instructions carefully to ensure OSKAR operates at its full capacity – providing value for many years to come. At faller audio, we pride ourselves in providing superior product design, engineering and service. If you have any questions about OSKAR, please reach out to faller audio customer support.

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Marcell Fallner  
Audio Expert and Founder of faller audio

#### Customizable speech filtering

Filtrage vocal personnalisable

#### Easy to operate

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#### Exceptional design, quality and durability

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#### Portable - bring to any room inside or outside

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## CONTENTS

OSKAR features .....	5
Product parts .....	6
Product overview .....	7
Setting up the product .....	8
Power supply and charging .....	9
Connecting with the optical audio cable .....	10
Connecting with 3.5mm audio cable .....	11
Using OSKAR .....	12
Speech filtering feature .....	13
Using headphones .....	14
Troubleshooting .....	15
Technical data .....	16
Important safety notes .....	16
General care .....	17
Electrical shock warning .....	17
Note on rechargeable lithium-ion batteries .....	18
Disposing of used electrical and electronic devices ...	18
Safe disposal of standard and rechargeable batteries	18
Disposing of used disposable and rechargeable	18
batteries .....	18
Warranty .....	18
Federal Communication Commission .....	18
Copyright and trademark .....	18
Contact and support .....	18

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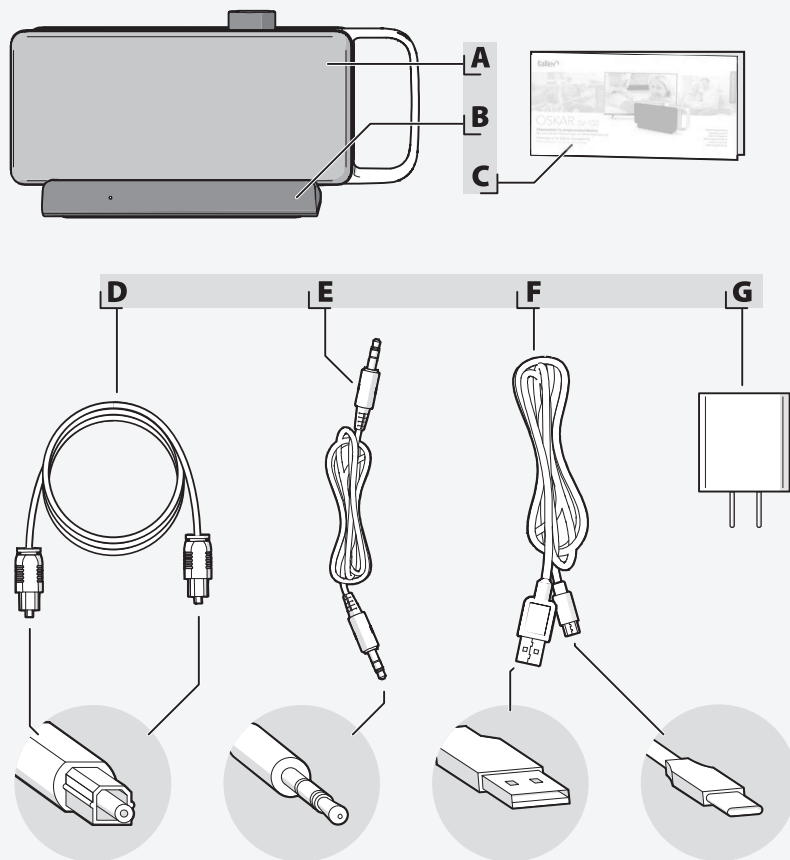
## OSKAR FEATURES

The OSKAR TV voice enhancer is a portable wireless TV speaker that brings clarity to TV programs and movie dialogue for the hearing impaired. Innovative audio technology unique to OSKAR heightens voices while filtering out and minimizing distracting background noise. This simple to operate device is easily adjustable to the user's personal preference for voice clarity and volume. Others listening to the TV through its standard speaker can also enjoy the same program at a lower sound level. When using the optical audio connection, OSKAR and the TV speaker is synchronized - without distortion or latency between them. The results: TV programming that is easy to understand at a reasonable volume, and harmony in the household.

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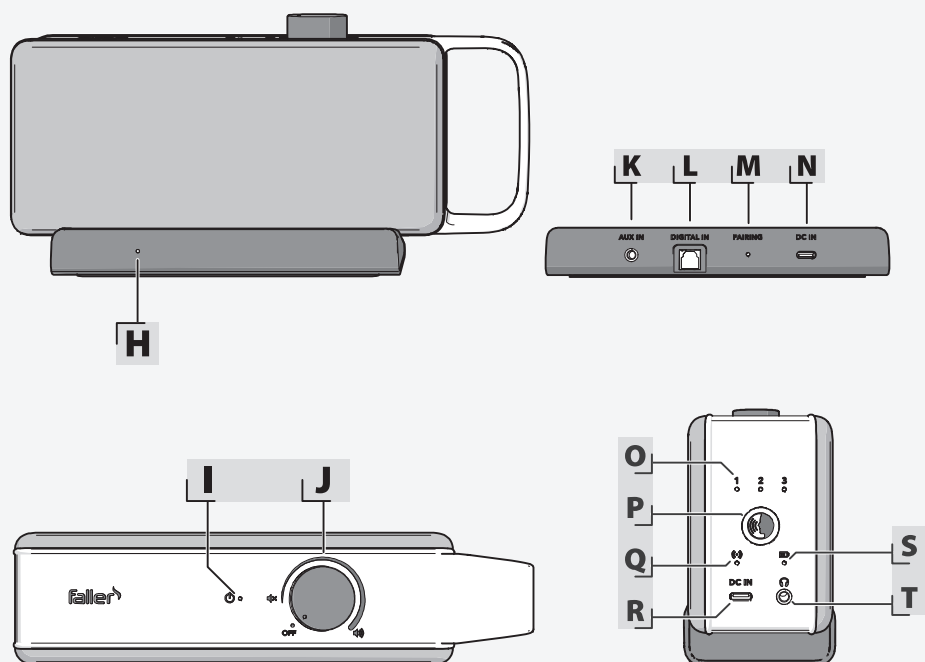
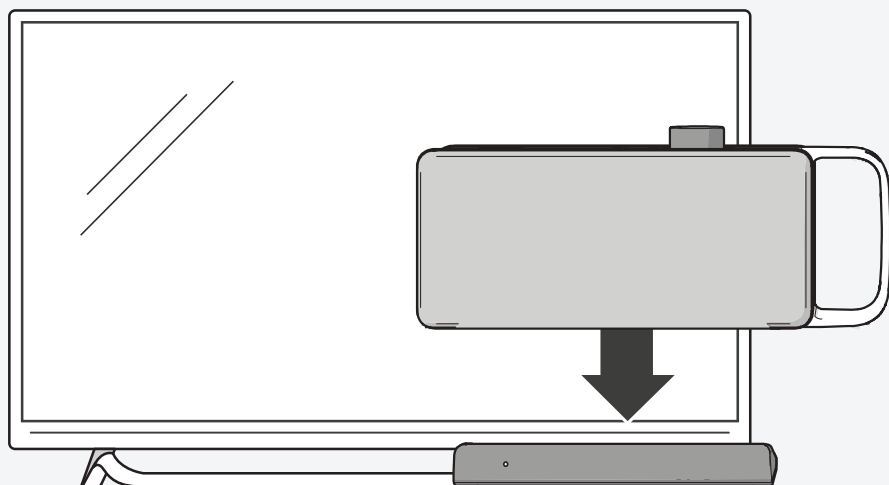
## SETTING UP THE PRODUCT

Place the docking station on a flat, stable surface close to the TV. Fully charge OSKAR before using it for the first time (**Fig. C**).

Use either the optical audio cable or the 3.5 mm audio cable to connect the device. For optimal sound quality use the optical audio cable. Use the 3.5 mm audio cable if the TV does not have an optical digital audio connection. Set OSKAR on the docking station.

If both cables are used, the docking station automatically selects the optical audio cable to transmit the TV audio.

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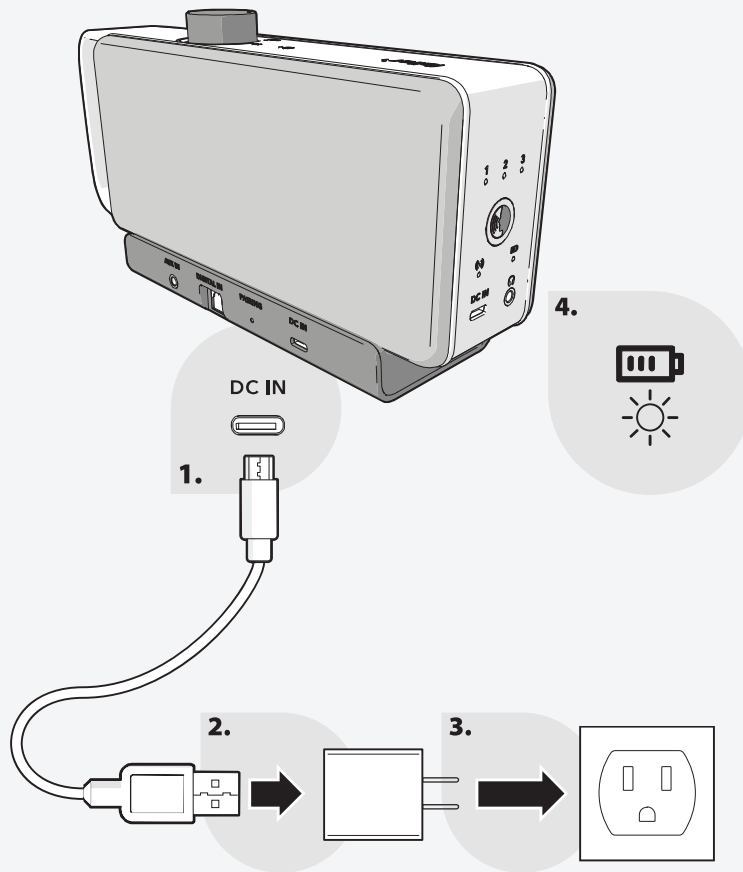
## POWER SUPPLY AND CHARGING

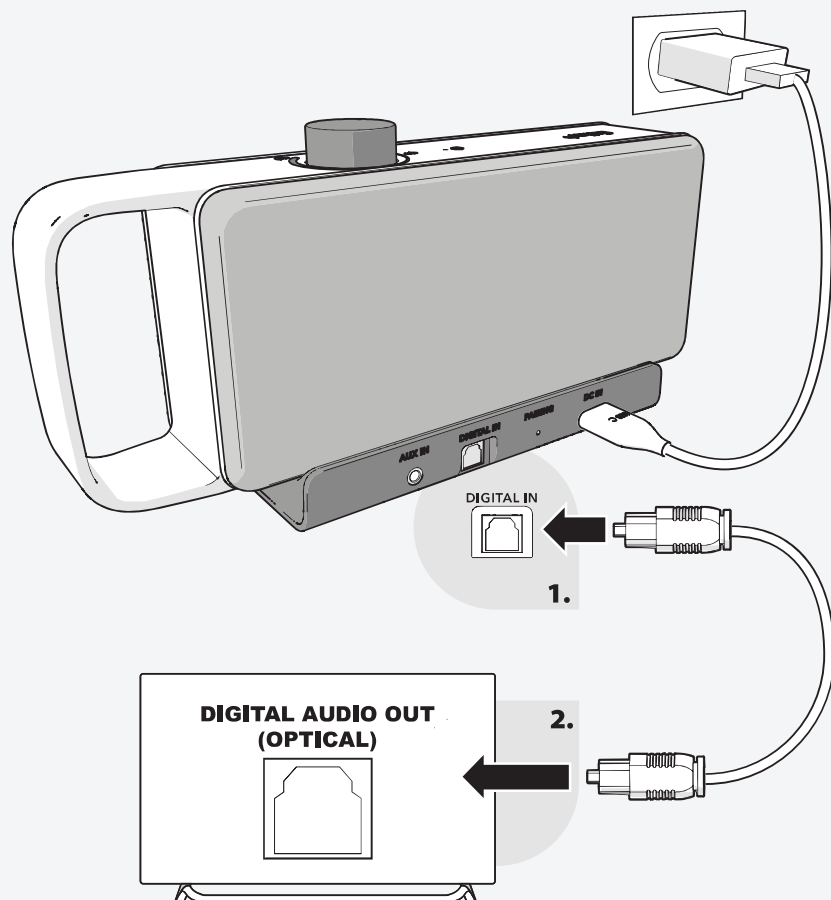
1. Connect the USB power cable to the DC IN 5 V connection on the docking station and to the USB power adapter. Then plug the power adapter into the wall socket (Fig. D).
2. Charge OSKAR before the first use by placing it on the docking station. **OSKAR does not have to be switched on to be charged.**

A red light on the docking station shows that the device is charging. A green light signifies that the device is fully charged. Charging takes about 4 hours. A single charge lasts for about 16 hours depending on use. If the red light flashes, the battery charge is less than 20%. Please place OSKAR back on the docking station to recharge.

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
## CONNECTING WITH THE OPTICAL AUDIO CABLE

1. Remove the protective covers from both ends of the optical cable.
2. Connect the audio cable to the docking station and the TV. Firmly plug in the connector until it locks in place (**Fig. E**).
3. Ensure that the digital audio setting in your TV is set to PCM (Pulse Code Modulation) output otherwise no sound may come from the device. For additional information on the PCM setting, please read the TV's operating manual.

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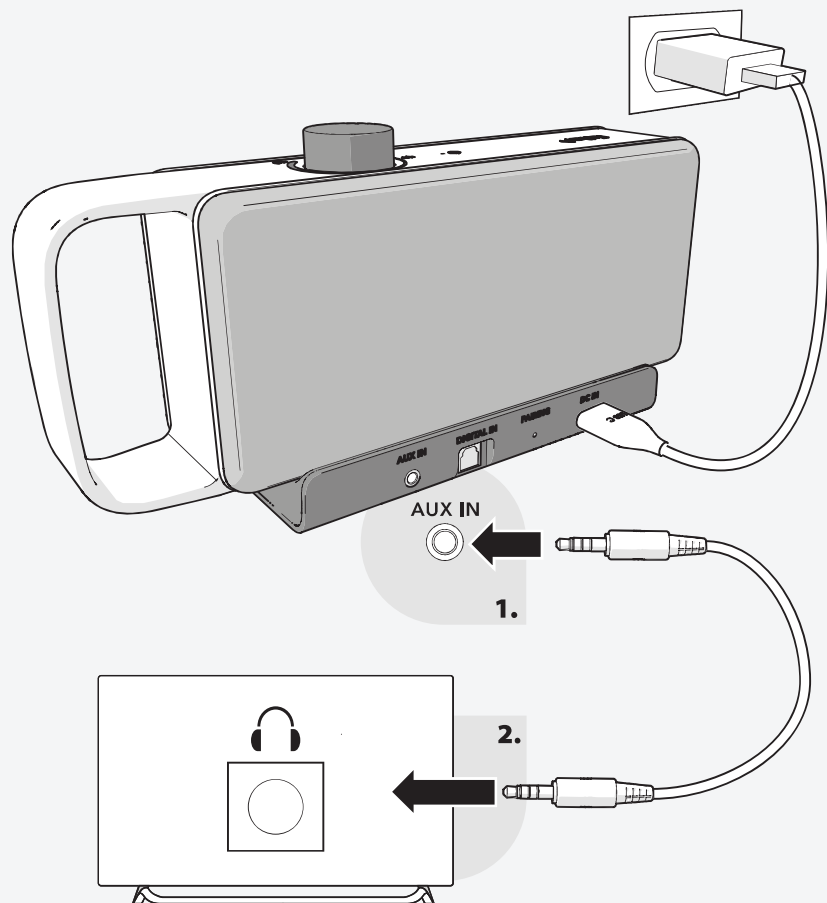### CONNECTING WITH THE 3.5 MM AUDIO CABLE

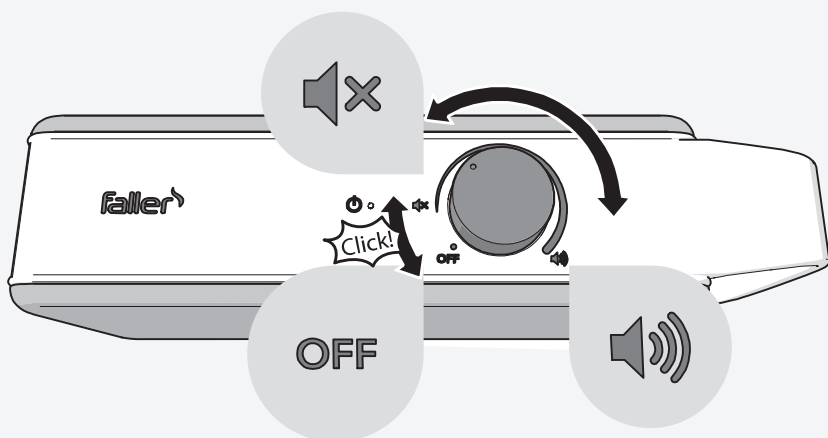
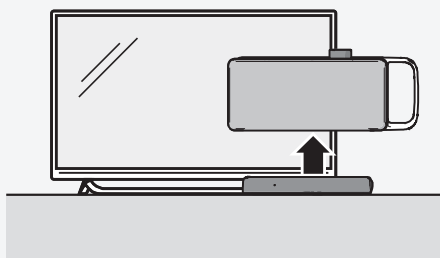
1. Connect the 3.5 mm audio cable to the AUX IN input on the docking station and to the audio output (headphone connection) on the TV (**Fig. F**). Firmly plug in the connector. For more information about this step please read the TV's operating manual.

**Note:** When using the TV headphone connection, the TV speaker may be disrupted/silenced. Please read your TV's operating manual to adjust the TV settings.

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## USING OSKAR

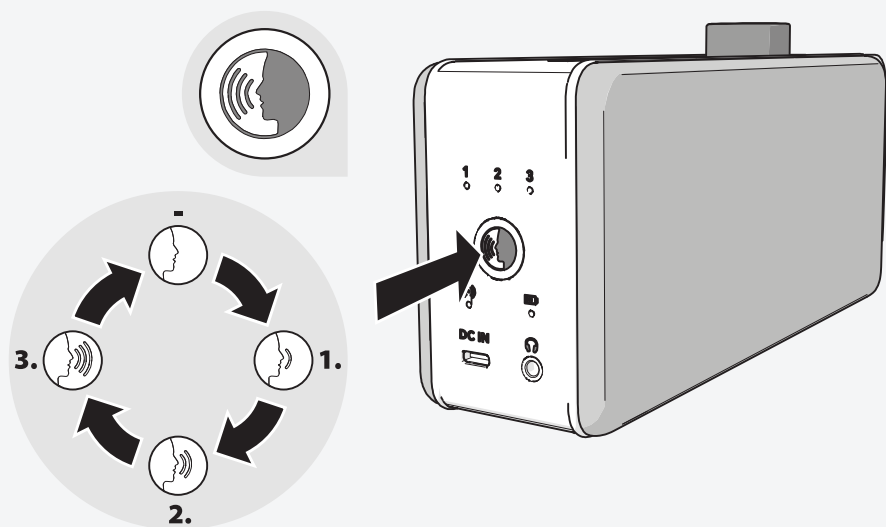
1. Remove the OSKAR speaker from the docking station.
2. Bring the OSKAR speaker to within an easy reach from the preferred TV viewing location.
3. Turn the OSKAR speaker on by rotating the top knob clockwise until a desired volume is reached (**Fig. G**).
4. To turn OSKAR off, rotate the top knob counterclockwise until it clicks. The light on top of the OSKAR speaker will go out. The light on the docking station will flash and then go out.

**Note:** OSKAR is an energy saving device. If the TV is off or if the OSKAR speaker and docking station do not receive an audio signal, OSKAR will automatically switch to standby mode after 5 minutes.

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## SPEECH FILTERING FEATURE

OSKAR uses unique technology to improve TV program dialogue. Voices are clearer, and disruptive ambient noises are filtered out and minimized. A simple to use button is located on the side of the device (**Fig. H**).

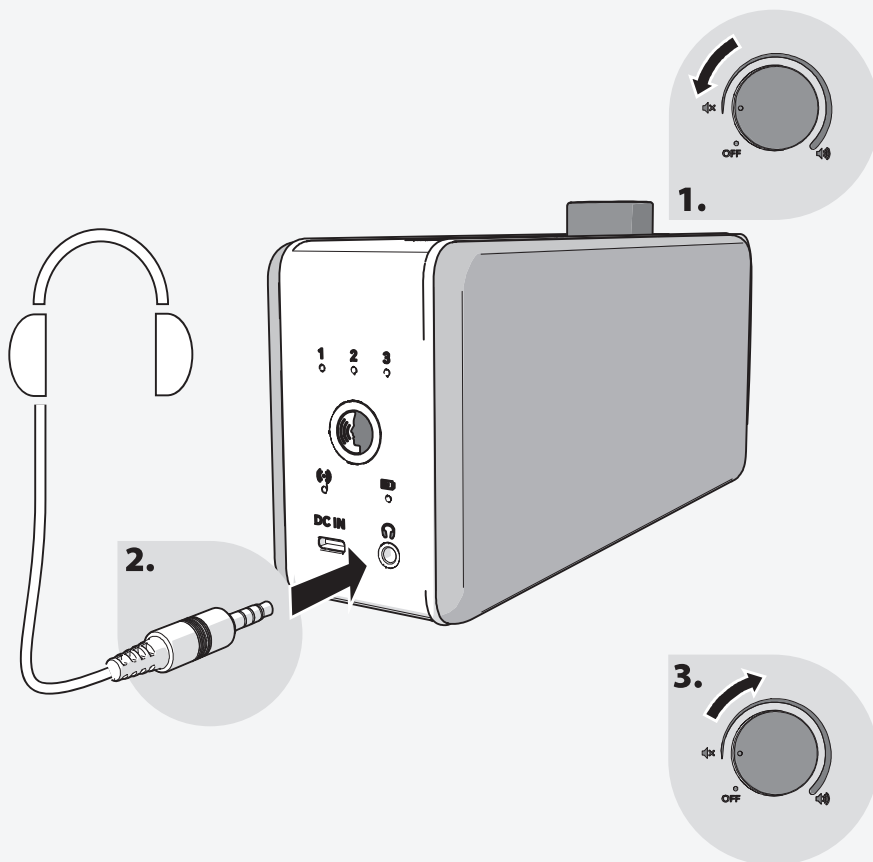
OSKAR provides three levels of varying degrees of speech filtering to suit a hearing-impaired person's individual preferences. A light indicates which level is operating. If no light appears, the dialogue clarifying function is turned off.

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## USING HEADPHONES

Headphones may also be used with OSKAR. Set the volume at the lowest level when first putting on the headphones then adjust the sound level (**Fig. I**).

The speech filtering feature will also work with headphones.

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## TROUBLESHOOTING

Symptom	Cause and remedy
OSKAR does not play the audio. (The light on the docking station and/or OSKAR is off or flashes)	<p>OSKAR is switched off.</p> <ul style="list-style-type: none"> <li>Switch on OSKAR by rotating the on/off volume knob clockwise.</li> </ul> <p>If the TV is switched off for more than 5 minutes or if it is muted, the docking station and the OSKAR speaker automatically switches to standby mode.</p> <ul style="list-style-type: none"> <li>Turn on or unmute your TV and then switch the OSKAR speaker off and on again.</li> </ul> <p>The cable connection is not correct.</p> <ul style="list-style-type: none"> <li>Check if the USB cable is firmly inserted into the docking station and the TV's USB power adapter.</li> <li>Make sure that the optical cable or the 3.5 mm audio cable is firmly inserted into the docking station and the TV.</li> </ul> <p>OSKAR is not paired to the docking station.</p> <ul style="list-style-type: none"> <li>Reset the docking station as follows: <ol style="list-style-type: none"> <li>Switch off OSKAR</li> <li>Disconnect the USB cable from the docking station and wait approximately 30 seconds before reconnecting the USB cable.</li> <li>The docking station resets automatically.</li> <li>Switch the OSKAR speaker back on again. When paired, the docking station light will come on.</li> </ol> </li> <li>Pair the OSKAR speaker to the docking station again: <ol style="list-style-type: none"> <li>Position the OSKAR speaker near the docking station.</li> <li>Use a pointed object (such as a straightened paperclip) to briefly press the PAIRING button. When connected, the light on the docking station will flash.</li> <li>Switch on the OSKAR speaker and then hold down the speech filter button for approximately 5 seconds until the light on the OSKAR speaker flashes. The lights on both the OSKAR speaker and the docking station will illuminate when paired.</li> </ol> </li> </ul>
OSKAR does not play the audio. (The light on the docking station and/or OSKAR illuminates or flashes)	<p>The TV audio has been muted.</p> <ul style="list-style-type: none"> <li>Increase the volume on the TV or switch off the mute button.</li> </ul> <p>The distance between OSKAR and the docking station is too large.</p> <ul style="list-style-type: none"> <li>Bring OSKAR closer to the docking station.</li> </ul> <p>The rechargeable battery is weak.</p> <ul style="list-style-type: none"> <li>Set OSKAR on the docking station to recharge.</li> </ul> <p>The digital audio is not set to PCM.</p> <ul style="list-style-type: none"> <li>Set the digital audio on the TV to PCM. Refer to the TV's operating manual for more information.</li> </ul> <p>The connection between the docking station and OSKAR is interrupted.</p> <ul style="list-style-type: none"> <li>Ensure that all cables are firmly inserted into the right outputs on the OSKAR docking station and into the correct inputs on the TV.</li> </ul>
TV audio is working through OSKAR, but the TV speaker is silent.	<p>When using the 3.5 MM cable/TV headphone connection, the TV speaker usually remains silent.</p> <ul style="list-style-type: none"> <li>Adjust the settings on the TV. Refer to the TV's operating manual.</li> </ul>
OSKAR's volume level is too low even when set to maximum.	<p>The headphone volume on the TV is set too low.</p> <ul style="list-style-type: none"> <li>Increase the headphone volume on the TV. Refer to the TV's operating manual.</li> </ul>
The sound fails frequently.	<p>Devices such as smart phones, Wi-Fi access points and microwave ovens generate radio waves and can cause radio interference.</p> <ul style="list-style-type: none"> <li>Position OSKAR and the docking station at a sufficient distance from such devices.</li> </ul>
There is interference when watching the TV.	<p>The 3.5 mm audio connection is not working correctly.</p> <ul style="list-style-type: none"> <li>Firmly connect the audio cable to the TV headphone connection and to the docking station. Occasionally unplugging and re-plugging the cables often improves the connection.</li> </ul>
The rechargeable battery does not charge.	<p>OSKAR has no charging contact to the docking station.</p> <ul style="list-style-type: none"> <li>Wipe the charging contacts on the docking station with a soft, clean, dry cloth. To prevent malfunctions, do not press hard on the charging points.</li> </ul>

TECHNICAL DATA

General	
Operating temperature	0 °C - 40 °C
Headphone output	3.5 mm stereo socket
Auxiliary input	3.5 mm socket
Auxiliary input sensitivity	300 mV
Optical input socket	Linear PCM
Optical sample rate	32 KHz; 44.1 KHz; 48 KHz; 96 KHz
Optical channel	Stereo 16 bit; stereo 24 bit
Frequency range	2.402 – 2.480 GHz
Maximum transmission power	< 10 dBm
Maximum operating distance	~ 30 m

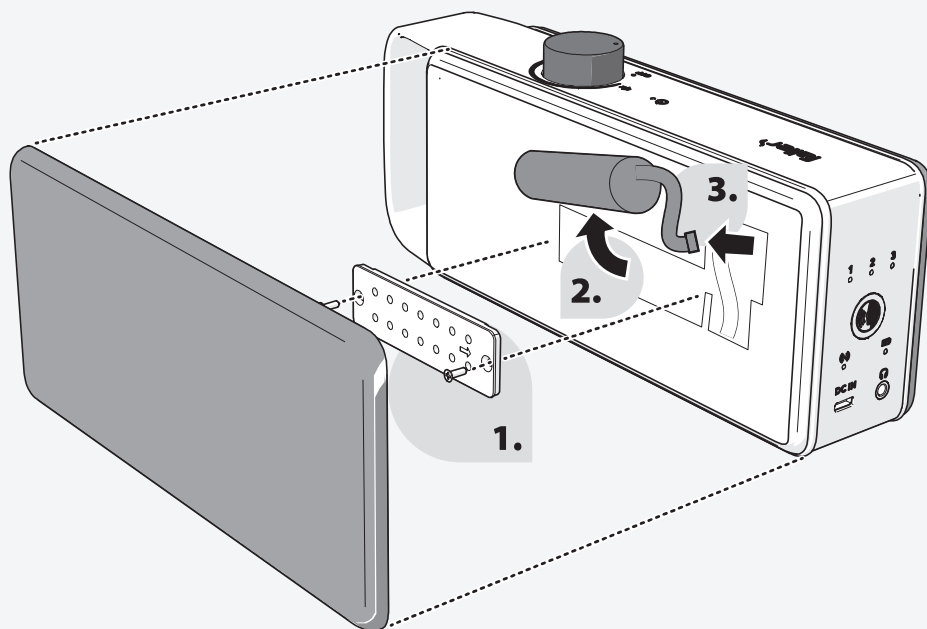
Voice enhancer Includes Alango VoiceFirst™ Technology	
Article number	SV-100
Dimensions (W × D × H)	240 x 102 x 60 mm
Weight	770 g
Battery type	Lithium ion rechargeable
Charging time	4 hours

Transmitter	
Article number	SV-101
Dimensions (W × D × H)	166 x 66 x 25 mm
Weight	107 g
Power consumption in standby	< 0.5 W

AC/DC Adapter	
Manufacturer	Shenzhen Keyu Power Supply Technology Co. 2~3/F, No. 13, Lane 3, Yuquan East Road, the 2nd Industrial part, Yulv, Guangming District, Shenzhen, China
Model number	KA12C-0502000US
Input voltage	100 – 240 VAC; 50 – 60 Hz
Output voltage	5.0 VDC
Output current	2.0 A
Output power	10.0 W
Average active efficiency	> 78.7 %
Efficiency at low load (10%)	75 %
No-load power consumption	< 0.10 W

IMPORTANT SAFETY NOTES

- Before using OSKAR for the first time, read through this manual carefully and follow all instructions. User accepts no liability for damage caused by failure to observe the safety instructions and information in this manual.
- Follow all safety instructions. Retain and share this operating manual with all users. Should this product be transferred to another user, be sure that the manual accompanies the device.
- Listening to loud music can cause long-term hearing damage. Never turn up the volume so high that listeners are no longer aware of the surroundings. When using headphones over prolonged periods, keep volume at a lower level. Always set the volume level low before turning on the device.
- The device may only be deployed for personal use and only for its intended purpose. It is not suitable for commercial use.
- Always keep the product, packaging and accessories (including the installed rechargeable battery) out of the reach of children and pets.
- People with impaired physical, sensory or mental abilities that cannot read, understand or set up OSKAR properly, should be supervised by a responsible person that can understand and operate this product safely. Keep the device out of their reach.
- The device and USB power adapter in particular are intended exclusively for use in dry interior spaces. Proper function is ensured at a room temperature from 5 °C to 35 °C. Do not expose the device to extreme heat or cold.
- Always handle the device and accessory parts with care and only set them up in clean, dust-free areas. To prevent the device from falling and causing injuries or property damage, place the device on a stable, level surface. Protect the device from intense mechanical shocks.
- The front side, edges and corners of the device are sensitive to pressure. Protect these areas from excessive loads. Do not place the device upside down. Do not place any heavy objects on top of the device and/or the USB power adapter.
- To prevent damage, injuries, fire and electrical shock, keep the device away from rain, humidity and excessively dusty areas. Protect the device from liquids, including dripping and splashing liquid, and do not place any objects filled with liquid, e.g. vases, on or near the device. As with all electronic products, make sure that liquids do not get into the parts of the system.
- Do not place any open flames such as lit candles on or near the device. Keep the device (including the USB

power adapter) away from flammable objects and from heat sources such as radiators, ovens and other devices that generate heat.

- Do not use the device outdoors (in direct sunlight), on a boat, inside a vehicle, in medical facilities, in unstable locations, or near water, rain, moisture or smoke. Keep the device away from any additional effects of weather.
- This device (including accessory parts) contains one or more magnets that could negatively affect pacemakers, programmable shunt valves for the treatment of hydrocephalus and other medical devices.
- Consult a physician before operating this product if using a medical device. Do not place this product near persons that use such medical devices.
- Do not open the device, remove the covers or expose the electronics. The device must not be maintained or repaired by the consumer.
- Repairs and maintenance of this device and any installed rechargeable batteries may only be performed by specialists authorized by faller audio. Repairs are necessary if the device is damaged in any way, for instance if the power cable or USB power adapter are damaged, liquid spills on the device, objects fall onto the device, the device is exposed to rain or moisture, the device does not work properly, or the device is dropped.
- Only use attachments and/or accessories that have been specifically designed for this device. Do not make any changes to the system or accessories. Unauthorized changes can negatively affect the safety, compliance with legal regulations, and the performance of the system.
- Only use the power supply provided with the device. An incorrect voltage can destroy the device. Do not make any modifications to the USB power adapter and do not use the supplied USB power adapter for other devices.
- Only operate the device if the electric voltage specified on the device and the power plug correspond to the voltage of your wall socket. An incorrect voltage can destroy the device.
- Make sure that the USB power adapter is easily accessible in case it needs to be disconnected from the power mains.
- Insert the USB power adapter all the way into the wall socket and only use power sockets in which the plug

fits snugly. Do not use extension cables or multi-socket adapters.

- The USB power adapter must be removed from the wall socket to completely disconnect the system from the power mains. Always disconnect the power adapter from the power mains when connecting the cable to the device.
- When not in use for multiple days or in the event of a storm, unplug the USB power adapter from the wall socket.
- Only use the device in environments where wireless 2.4 GHz radio technology is permitted.
- This device should be installed and operated at a minimum distance of 8 inches between the antenna of the device and a person's body.

#### GENERAL CARE

- Only clean the device on the outside with a soft, lint-free cloth or a slightly dampened rag. Do not use any wax sprays, harsh cleaning agents, cleaning agents that contain solvent or other substances as they could damage the surface.
- Lacquers and furniture polishes can chemically corrode the rubber feet of the docking station and the device, causing surface damage to furniture.
- Materials such as wooden surfaces, plastics, fabrics and metals can vary visually and darken or fade over a longer period of time, especially if exposed to sunlight.

#### ELECTRICAL SHOCK WARNING

- Do not use the device if there is obvious damage, as this could lead to electrical shock.
- Disconnect the device from the power supply before establishing or interrupting the connection to the device.
- Do not use the device in a wet or damp environment.
- Do not touch the USB power adapter with wet hands. Never plug in the connecting plug if the product or the charging cable is wet.
- Do not disconnect the device from the power mains by pulling on the cable. Always hold the USB power adapter.

#### NOTE ON RECHARGEABLE LITHIUM-ION BATTERIES

This device uses a rechargeable lithium-ion battery. Misuse or improper use can result in battery damage! In extreme cases, there is a risk of explosion, fire, or the development of heat, smoke or gases. In this case, faller

audio accepts no liability.

- Rechargeable batteries must not be exposed to excessive heat such as sunlight, fire or the like.
- Do not touch the battery with bare hands if it leaks or has been damaged. Leaking liquid can cause skin irritation and other injuries.
- If the device is not in consistent use, charge it at least every six months to ensure the rechargeable battery remains in good condition.
- Do not remove the rechargeable battery in the charged state from the device. Do not short-circuit it, dismantle it or damage it.
- If the device is faulty, do not recharge the battery. Terminate the charging process if the device becomes excessively hot.
- Observe the warranty and instructions for disposing of it properly.

#### DISPOSING OF USED ELECTRICAL AND ELECTRONIC DEVICES

(To be used in the countries of the European Union and other countries with a separate collection system for these products) This symbol indicates that this product must not be disposed of with other household wastes. Dispose of it in accordance with applicable legal regulations for electric waste. Protect the environment and the health of other people by disposing of this device properly. Please recycle when possible. To return the product, use the normal return and collection systems or contact the location where the product was purchased.

#### SAFE DISPOSAL OF STANDARD AND RECHARGEABLE BATTERIES

The lithium-ion battery in this device must be removed and recycled properly. See figure (J) on page 17.

**Warning:** Only remove the rechargeable battery before disposing the device. Make sure that the battery is completely discharged before removing it. Use a Philips and a flathead screwdriver to remove the rechargeable battery. Observe the basic safety instructions when performing the steps listed below. Protect eyes, hands and fingers and the working surface. Wear goggles to protect eyes from any breaking or flying parts.

1. To completely discharge the battery, take the device out of the docking station. Turn on the device and let it run until it enters standby mode. Repeat this step until the device can no longer be switched on.
2. Using a standard screwdriver remove the speaker cover on the back by pushing the screwdriver between the surrounding rubber cover and the fabric-wrapped speaker cover. Push the screwdriver away from the housing to detach the speaker cover leveraging it around the sides until it detaches.
3. Undo the two screws of the battery compartment with a Philips screwdriver.
4. Carefully pull the battery out of the housing by the plug connection cable.
5. Detach both plugs.
6. Cover the battery contacts with adhesive tape to prevent short circuits due to the residual battery charge. The rechargeable battery and the rest of the product can now be recycled and disposed of properly.

#### DISPOSING OF USED DISPOSABLE AND RECHARGEABLE BATTERIES

(To be used in the countries of the European Union and other countries with a separate collection system for these products). Discard disposable and rechargeable batteries at waste collection or recycling centers that properly dispose of electrical and electronic devices.

#### WARRANTY

faller audio guarantees that this product will function properly for one year from the date of purchase, providing that the operating instructions in this manual have been observed. This warranty becomes null and void in the event of malfunctions resulting from accidental and intentional damage of any kind, normal and excessive wear, negligence, or unauthorized modifications. In addition, this warranty does not apply to negligible defects. The warranty does not cover disposable or rechargeable batteries. For return information, please contact an authorized faller audio dealer or Customer Service. The warranty covers the cost of repairs of faults resulting from the material or manufacturing process, and includes return shipping. Repair or exchange does not

initiate a new warranty period. The warranty period of 1 year from the date of purchase applies.

Return the item in its original package with the sales receipt. Please send in the product in the original packaging along with the receipt of purchase. Please do not throw these items away. Claims do not affect and/or cancel the free of charge warranty rights of the customer defined by law.

#### FEDERAL COMMUNICATIONS COMMISSION (FCC)

This device complies with part 15 of the FCC Rules.

Operation is subject to the following two conditions:

1. This device may not cause harmful interference, and
2. this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

1. Reorient or relocate the receiving antenna.
2. Increase the separation between the equipment and receiver.
3. Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
4. Consult the dealer or an experienced radio/TV technician for help.

This equipment should be installed and operated with minimum distance 20cm between the radiator & your body.

FCC ID: 2BCJW-SV100  
FCC ID: 2BCJW-SV101  
IC: 31221-SV100  
IC: 31221-SV101

Complies with Canadian ICES ICES-3 (B)

This device contains license-exempt transmitter(s)/ receiver(s) that comply with Innovation, Science and Economic Development Canada's license-exempt RSS(s). Operation is subject to the following two conditions:

1. This device may not cause interference.
2. This device must accept any interference, including interference that may cause undesired operation of the device.

CAN ICES-003 (B) / NMB-003 (B)

This Class B digital apparatus complies with Canadian ICES-003. The following importer is the responsible party

Certifi Global Approvals, Ltd.  
380 Wellington Street Tower B, 6th floor  
London, Ontario, N6A 5B5, Canada  
Phone: 1 866 885 4575

#### COPYRIGHT AND TRADEMARK

©2023 sonoro audio GmbH. All rights reserved. "faller", the "faller" logo and other "faller audio" marks and symbols are owned by sonoro audio GmbH and are potentially registered. All other trademarks are the property of their respective owner. sonoro audio GmbH accepts no liability for errors in this product information guide.

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iCertifi Global Approvals, Ltd.