

# 尺寸720\*110mm 材质：80g哑光铜版纸

### Product Description

Phanpy E-joy S2 Wearable Intelligent Electric Breast Pump is an electric breast pump for breastfeeding women to pump and gather milk.





### Applicable Population





Adult lactating women.

### Product Features

- Newly upgraded wearable electric breast pump, freeing up your hands, instantly wearable, truly enabling discreet breastfeeding anytime, anywhere.
- Featuring a sleek and compact design, this portable and lightweight breast pump allows mothers to easily carry it with them.
- With the addition of a massage mode, this breast pump provides a SPA-level breastfeeding experience, offering intimate care for a mother's breasts.
- Equipped with a built-in 1500mAh large-capacity battery, this breast pump offers an extended battery life.
- Boasting a clear and user-friendly digital display, this breast pump features a fully touchscreen interface for easy and comfortable operation.
- The product comes with a 20mm breast pump flange insert, catering to the different needs of mothers.
- The innovative and extendable breast shield design features wide-angle openings and an elliptical curve for full coverage of the breast, providing increased suction comfort.
- With the one-touch pause button, you can quickly stop and resume the pumping process in case of an emergency, providing a more user-friendly experience.
- The memory program design automatically recalls the previous mode and suction level used, offering convenience and ease of use.
- The power-saving mode is designed to automatically shut down the device after a 5-minute pause in operation.
- The innovative smart APP design allows mothers to track and record their breast milk volume in real-time.







### Components


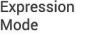
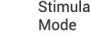






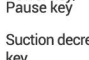

Name	Material	Heat-resistance temperature
Newcup/Tee joint connector	PP	120°C
Silicone airbag/24mm Breast shield/ Duckbill valve/20mm Breast pump flange insert	Silicone	220°C
Shell of pumping unit	ABS	90°C (prohibit washing)

### Display Instructions





(1)The display of battery:  
If the battery power is less than 20%, the interface shows "Lo"






Under such condition, Lo is displayed for 3 seconds, and mode indicator is not on.

Lo will return to normal display after it's displayed for 3 seconds.

Lo is displayed for 3 seconds after an interval of three minutes, and mode indicator is not on.

### Button Descriptions






Press short to decrease/increase levels

Long press for 1s to power on/off, short press to pause/exit to pause

Short Press To Switch Modes

### Installation Instructions






① Install the silicone airbag and press it in place. (The positioning groove of the silicone airbag is aligned with the positioning rib of the tee joint connector)

② Install the duckbill valve and press it in place.

③ Put the installed tee joint connector into the Newcup. (The positioning groove of the silicone airbag faces outward)




### Instructions For Use



① Determine your own nipple diameter. (Default breast shield size is 24mm, mothers with smaller nipples can stacked use a 20mm breast pump flange insert to ensure the breast shield fits the breast better)

② Slightly bend down to ensure that the nipple is in the center of the breast shield duct, so that the shield fits the breast tightly




③ After fixing the breast shield in place, wear a bra and keep your body upright with the main pump unit facing up.



④ Press the power key to start the electric breast pump and enter the stimulation mode. (the stimulation mode will last up to two minutes, and it will automatically switch to the expression mode after two minutes)

⑤ If breast milk comes out early, please press "+" key to switch to expression mode.



⑥ Via "+" "-" and "+" "-" keys to decrease/increase the suction level and adjust to the most comfortable state.



⑦ Under the expression mode, you can check the amount of breast milk by observing the scale line on the right side of the Newcup.

⑧ Hold the Newcup horizontally and remove the main pump unit of the breast pump from the side.




⑨ Keep the breast shield side of the Newcup up and lay it flat on the table top (do not put it upside down to prevent milk from flowing out)



⑩ Pour milk from the mouth of the Newcup into the breast milk storage bottle/bag. (You can view the scale to determine the amount of milk poured)

⑪ After use, remove and clean by pressing the bottom of the cup and the rim of the breast shield.

### How to choose the right breast shield



The nipple enters the duct, there is space on the left and right, no discomfort.

The nipple and areola all enter the breast shield duct, and there will be strong pain when pumping.

The nipple is full of breast shield duct, and there will be strong pain when pumping.

### Precautions for use

Do not put it upside down to avoid milk flowing out of the Newcup and causing damage to the main pump unit.




When the milk needs to be poured out after pumping, please make sure that the main pump unit is separated from the Newcup.

Stop pumping, after removing it, make sure that the breast shield is facing up.

Determine your own nipple diameter before use.

Correct posture, do not lie down.

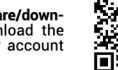

### App Instruction



① Enter the URL(<https://xcx.phanpy-inc.com/share-downloadApp.html>) and follow the steps to download the APP. Open the APP software and enter your account number and password to log in.

② Open the main console and access the app interface. You will be able to search for available devices. Choose Device 1 to enter the app control interface. **Note: The device name can be customized.**

③ On the "Home" screen of the app, the Device Name-1 area is shown as connected. Adjust the settings by long-pressing the vertical bar and switch between modes by tapping the icon button.




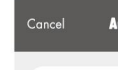
④ If simultaneous breast pumping is required, connect the second device. Tap the "Reconnect" option in the top right corner to return to Step 2 and select the second device. Once the second device is connected, the "Home" screen will display both Device Name 1 and Device Name 2.

⑤ During breast pumping, if a mother encounters an urgent situation, she can tap "Pause" to bring up the milk recording box. After recording the milk quantity, she can save and exit. Please note that both devices must have the milk quantity filled in before exiting. When the device's operation time is complete, a milk recording box will also appear to remind the mother to fill in the quantity.




### Phanpy

Wuxi Phanpy baby products Co., Ltd.  
Add: No. 2 Fengwei Road, Xishan District, Wuxi City, Jiangsu Province, China 214000  
Version: Version A  
Before use, please read this manual carefully and store the manual where it can be taken at any time

[share the love](#) [FROM PHANPY TO HAPPY](#)



⑥ To set an alarm, go to the Alarm interface. Tap the "+" icon in the top right corner to add a new time. This will remind the mother of the breastfeeding schedule. Tap "Save" to confirm. Swipe right to delete any added times.



⑦ Access the "Log" interface to view the breastfeeding sessions' time, milk quantity, and total milk production for the day. If the main console is closed during breast pumping without timely recording, the mother can make up for it by tapping "+" to access the milk recording dialog. (Need to fill in the pumping time first, and then fill in the breastmilk volume) If there are any errors in the entered milk quantity, the mother can also make corrections and save them by tapping "Confirm."

⑧ The "More Pages" has the functions of switching pumping units, software language option, software edition number, video for the installation and usage of E-joy S2, user agreement, privacy policy, log out and account log out.

### Operation Mode

#### Stimulation mode

Using high-frequency to stimulate lactation, 8 levels can be chosen, the higher the level is, the stronger the stimulation of lactation is.

#### Expression mode

Suction is gentle and powerful, breast milk is pumped out efficiently, 8 levels can be chosen, the higher the level is, the stronger the breast pumping is.

#### Massage Mode

Dredge the breasts while pumping and feel painless like doing spa. Intelligent massage, with 8 levels for suction, the higher the level is, the stronger the massage level is.

### Trouble Shooting

1. If the pumping unit is unable to work, please kindly check if it's due to the battery power is running low. If the interface shows Lo when turning on the pumping unit, then means low battery, pls kindly charge the breast pump in time.

2. The host cannot be charged: check whether the power adapter is damaged, you can use a power adapter that matches the voltage and interface type to try to charge.

3. The suction power is too small or no suction power:  
a. Check if the silicone airbag is installed in place.  
b. Check if the breast shield is installed in place.  
c. Check whether the duckbill valve is installed in place.

### Cleaning & Sanitizing

#### Sterilizable parts

- Silicone airbag/duckbill valve/24mm Breast shield/Tee joint connector/Newcup/20mm Breast pump flange insert

#### Cleaning

- Wash all parts with plenty of warm water (about 30°C). Use neutral detergents available on the market. It is recommended to use those that do not contain artificial fragrance and pigment.
- Rinse all parts with clean drinking-quality cold water (about 20°C). Dry after washing.

#### Sterilization

- Before the first use and after each use, need to be sterilized.

⑩ Put the sterilizable parts into hot water for 3-5 minutes or use a neutral detergent to wash in warm water, and use it after sterilization and drying (Do not directly put the parts in boiling water to avoid damage to the parts due to excessive temperature).

#### Dry

Wipe dry with a clean cloth, or place on a clean cloth to dry. Put the cleaned parts in a clean storage bag or in a clean environment.

Do not store wet parts in sealed containers or storage bags.

Be sure to dry all moisture and store after drying.

#### Note

Do not spray or pour the liquid directly on the main pump unit of the breast pump.

Only use drinking-quality water for cleaning.

After cleaning, be sure to dry all residual moisture.

### Attentions

- Please read this user manual carefully before using the product and keep it in a safe place.
- Please fully charge the device before the first use.
- Do not submerge the main unit of the breast pump in water or any other liquid for cleaning.
- Do not use a steam sterilizer to disinfect the main unit of the breast pump.
- When using a power adapter, please select the adapter whose input voltage matches the unit.
- If the main unit is damaged due to improper operation, our company will charge the corresponding repair fee.
- Only allow Phanpy or authorized service providers to replace the lithium battery. Replacing the battery with the wrong model may cause explosion hazards.
- It takes approximately 2.5 hours to fully charge the device; under normal use, a full charge can last for 6 cycles, with each cycle lasting 30 minutes.
- This product is not a toy and should be kept out of the reach of children.
- Please dispose of waste batteries or battery packs in accordance with national environmental regulations. (Note: Lithium batteries will discharge deeply if left unused for a long time, causing the device to shut down. To activate it, plug in the power and wait for the screen to light up after about ten minutes. You can then continue charging. If it remains inactive for more than half an hour, please contact customer service.)
- Do not use this device as a milk opener. The premise of using a breast pump is to have sufficient milk supply and unobstructed mammary glands. It is recommended to undergo professional breast examination before use to ensure normal lactation and unobstructed mammary glands.
- Do not use the breast pump during pregnancy, as pumping may cause miscarriage.
- If abnormal noise or excessive heat occurs during product use, please stop using it immediately and contact customer service personnel.

### FCC regulatory conformance:

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:  
(1) This device may not cause harmful interference.  
(2) This device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:  
• Reorient or relocate the receiving antenna.  
• Increase the separation between the equipment and receiver.  
• Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.  
• Consult the dealer or an experienced radio/TV technician for help.

NOTE: The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user's authority to operate the equipment.

#### RF Exposure

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment.

### Product Specification

Product name	Phanpy E-joy S2 Wearable Intelligent Electric Breast Pump
Package	2set
Rated Voltage	3.7V===
Input Voltage	5V ===1A
Warranty Period	12 Months
Executive Standard	GB 4806.11-2016 GB 4806.7-2016 GB 4706.1-2005

### AFTER SALES SERVICE GUARANTEE CARD

Return

Exchange

ID(required)

Shop name

Order number

Shipping address

Reason for return or exchange (required)

Customer's name

Telephone number


Product name

1. If users operate it in accordance with this instruction manual, the product shall be maintained free of charge within one year from the date of purchase. Please consult the customer service.

2. We will not provide free warranty service for the faults caused by the following users' reasons. Such as:  
A. Fault caused by unauthorized disassembly, modification of the product.  
B. Fault caused by accidentally hit, drop in the process of use and handling.  
C. Failure due to lack of proper maintenance.  
D. Fault caused by failure to operate properly according to instruction manual.  
E. Failure caused by improper repair of repair shop not authorized by our company.

3. When you need free warranty service, you must provide purchase invoice and warranty card.

4. Repair services outside the scope of the warranty will be charged according to regulations.

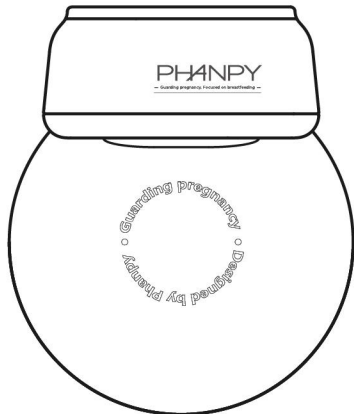

### INSTRUCTION MANUAL

1

2

3

4

5

6

7

8

9

10

11

12

13

14