# PEACHBITE WATCH G2 User Manual

## **Caution for Use**

• Please read all of the following to use this product safely.

We and the manufacturer are not liable for damage, failure, or accidents caused by failure to follow the prohibitions and recommendations while using this device.

#### **Prohibitions**

- Do not submerge the device in water or pour water over it.
- Do not disassemble internal circuit components when replacing the band.
- Do not use this device for purposes other than monitoring the health of your pet.
- Do not throw or subject the device to strong shocks.
- . Do not spray any substances onto the device.

#### Recommendations

- Avoid using outdoors on rainy days.
- Please use the included charging cable for charging.
- It is possible to charge the watch or power the relay using an external battery for charging.
   (USB 5V output).


#### **Customer Service**

#### MaxK IIc

9945 WARWANA RD, HOUSTON TEXAS 77080


**EMAIL** contact@peachbite.com **WEB** www.peachbite.com **TEL** 713-497-5012

#### **COMPOSITION AND PART NANES**

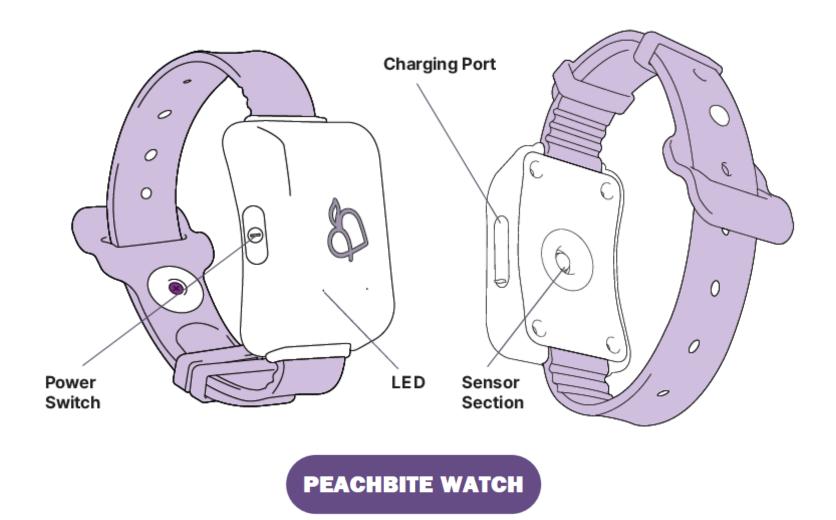

The Peachbite Watch is a pet wearable device that collects your pet's biological data through sensors.

#### **COMPOSITION AND PART NANES**

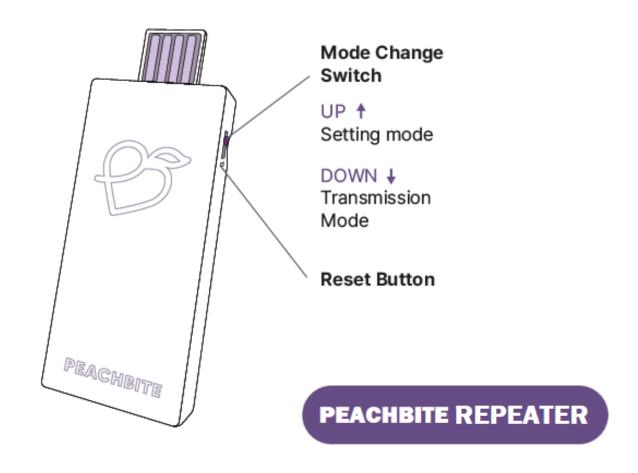

#### **DEVICE COMPOSITION**


- PEACHBITE WATCH
- PEACHBITE REPEATER
- 5pin USB Charging Cable
- Device QR Code
- User Manual

Receive your pet's data from the Peachbite Watch and transmit to the server.

The Peachbite Repeater helps you to check the pet's health status from anywhere any time.


Purple

# **LED LIGHTS** – Meaning of product LED lights

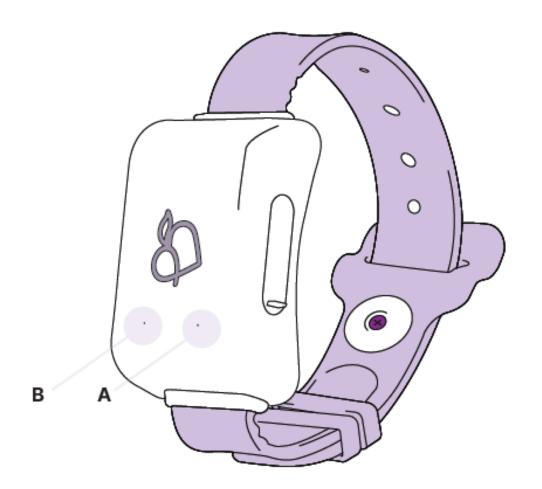

- When Charing: The red or orange light on A and the device does not operate.
- When charging is complete: A yellow-green light turn on.
- Device On-Off: Press and hold the power button, B will flash a blue light quickly.
- When the device is on: The light of B flashes weakly once every 0.5 seconds.

# **LED LIGHTS - Meaning of product LED lights**

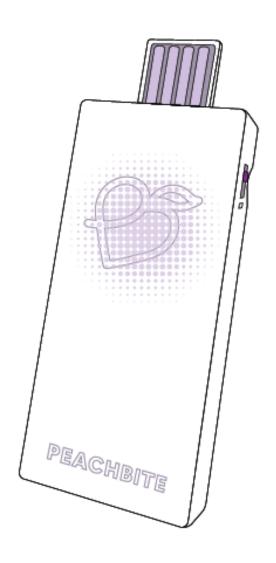

- Purple Light : The Repeater is in the setting mode.
- Blue Light: The Repeater has been set up correctly and is ready to receive data.
- When the blue light is blinking, data is being transmitted and received from the Watch.
- When the setting of the Repeater is incorrect, the Red light is turned on.

# **Prepare for using PEACHBITE WATCH**



Mobile phone, Peachbite watch, Peachbite repeater, power supply adapter or portable charger, home Wi-Fi ID and password (general Wi-Fi, not 5G)

# **Prepare for using PEACHBITE WATCH**


# Supply power to the repeater power adapter portable charger 4011 **Download PEACHBITE APP** Download on the Google Play App Store

- When using the repeater indoors or at home, connect it to a power source.
  - When walking with pet's, you can connect the repeater to a power bank and take it with you.
- The Repeater uses both Bluetooth and Wi-Fi.
   The Watch and Repeater are connected via Bluetooth.
   The Repeater transmits the data received from the
   Watch to the server via Wi-Fi.

You could access by https://app.peachbite.com, but some functions are not supported.

Please download the PEACHBITE application to use it comfortably.

#### **REGISTERING YOUR PET**

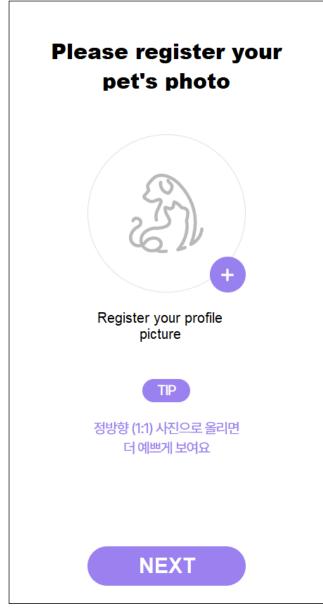



#### STEP 01

After completing the mobile registration (https://app.peachbite.co.kr/), start registering your pet! Register your profile picture and press the Next button.

TIP. It looks prettier if you upload it as a square (1:1) ratio photo.

#### **STEP 02**

Enter your pet information and click the Next button to complete your pet registration!

STEP 01 STEP 02

#### **REGISTERING A REPEATER**





STEP 01

After completing pet registration, press the start button for repeater registration


STEP 02

Enter the 12-digit ID attached to the repeater

Or, press the QR code button to recognize the provided repeater QR code.

STEP 01 STEP 02

#### **REGISTER YOUR WATCH**

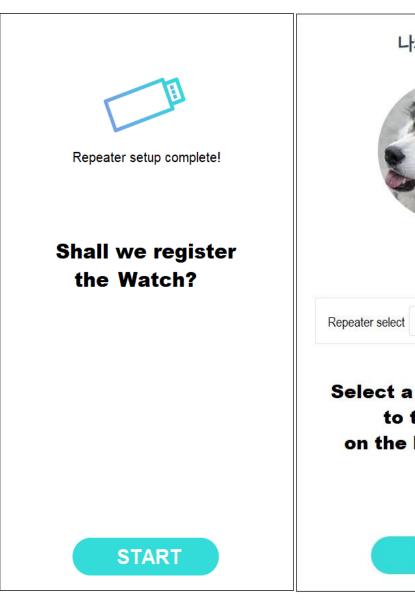





#### STEP 01

After completing the repeater registration, press the watch registration start button

#### **STEP 02**

Select t a pet to register to the watch on the home screen.

#### STEP 03

Enter the 12-digit ID attached to the Watch
Or, press the QR code button to recognize the provided
Watch QR code.

Watch ID is in the original box

STEP 01 STEP 02 STEP 03

#### **REGISTERING A WATCH & SETTING UP A REPEATER**




Basic setup is co plete!

Turn on the watch and follow the repeater and indoor Wi-Fi settings.

워치의 전원을 켜고 아이의 목에 채워주세요. 중계기 설정을 따라하시면서 우리 아이의 생체 정보를 가져오겠습니다.

**START** 

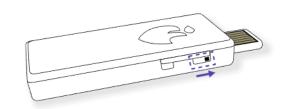

Move the switch
on the side of the provided
repeater to the terminal
to change to the setting
mode and connect it to the
adapter (or power bank)
to supply power.

When power is supplied, the center of the repeater emits a purple light.

**NEXT** 

STEP 04 STEP 05

#### **STEP 04**


Basic setup is complete.


Turn on the watch and follow the repeater and indoor Wi-Fi settings.


#### STEP 05

Move the switch on the side of the repeater up to change to [Setting Mode] and connect it to an adapter or power bank.

#### **REGISTERING A WATCH & SETTING UP A REPEATER**

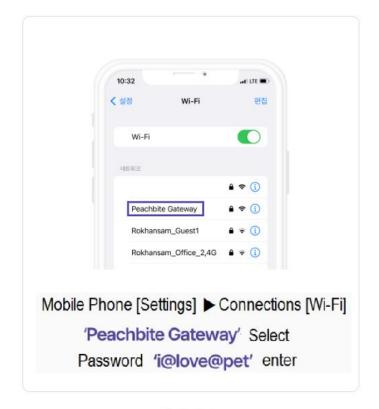

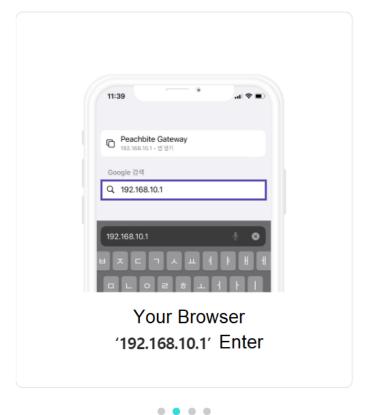

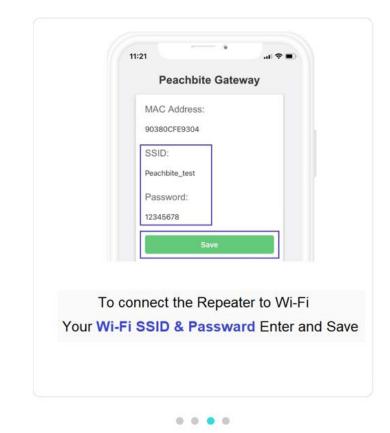

STEP 06

. . . .

Select 'Peachbite Gateway' in your phone's Wi-Fi settings and enter 'i@ love@pet' in the password field.


**STEP 07** 

Connect by entering '192.168.10.1' in the browser


**STEP 08** 

Enter the Wi-Fi name you are using in SSID and the Wi-Fi password in Password and save it. 5G Wi-Fi is not supported. Wi-Fi input requires case sensitivity.

#### **REGISTERING A WATCH & SETTING UP A REPEATER**

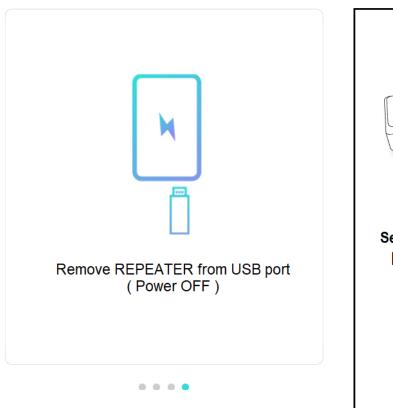

**STEP 09** 

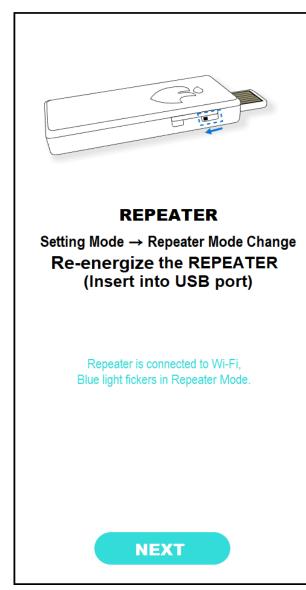


#### **STEP 09**


After cutting off the power supply to the repeater (removing it from the USB port), switch the repeater from setting mode to repeater mode, and supply power to the repeater again.


#### (Insert into USB port)


If the repeater is connected to indoor Wi-Fi, the **blue light** blinks in repeater mode.

# **SETTING UP A HOTSPOT FOR A WALK [ANDROID]**

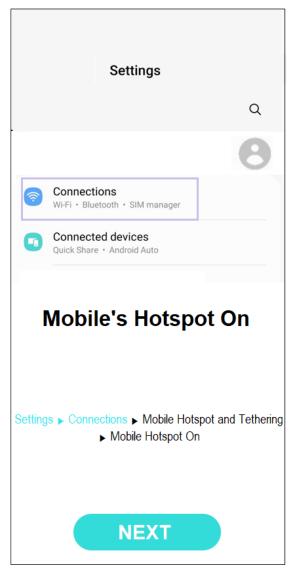

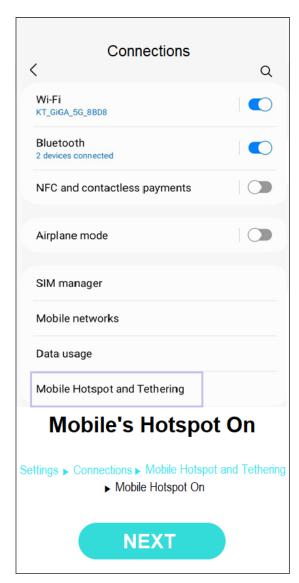


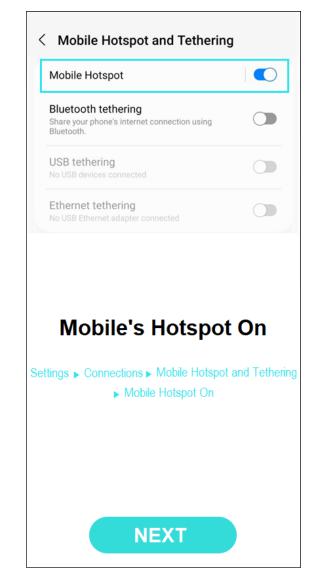


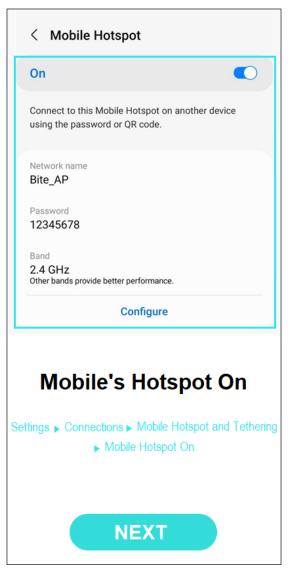


STEP 01 STEP 02 STEP 03 STEP 04

# **SETTING UP A HOTSPOT FOR A WALK [IOS]**

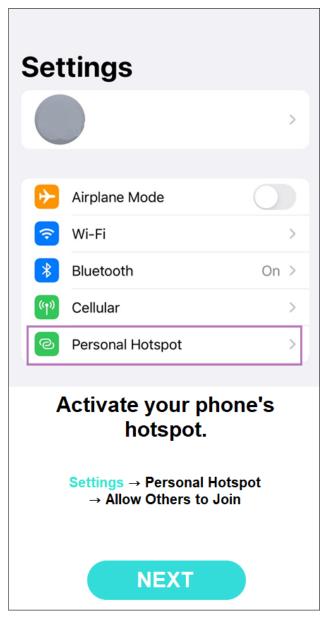


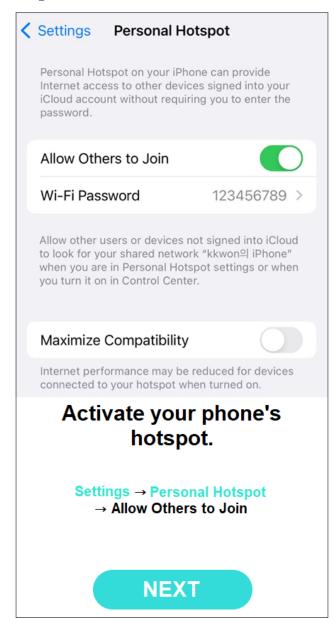

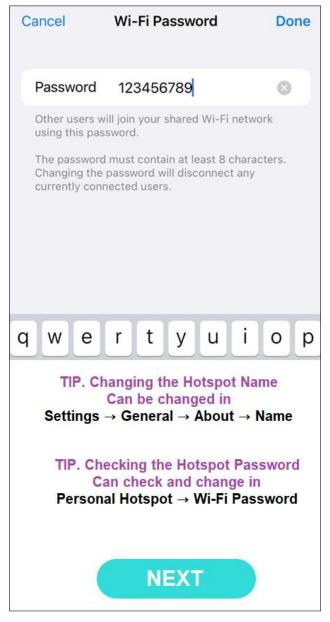

STEP 01 STEP 03

# **SETTING UP A HOTSPOT FOR A WALK[Android+IOS]**

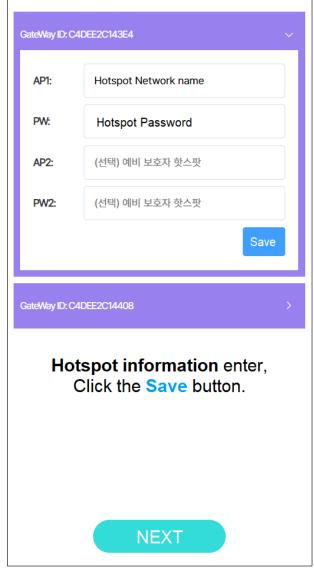



**STEP 04** 

You can designate a hotspot for walking with your dog when selecting a registered repeater

Please enter the hotspot information you checked earlier.

AP1: Enter parental hotspot name
PW: Enter parental hotspot password
Please save after entering


**STEP 05** 

After the initial setting, you can just connect the repeater to the power bank and leave immediately when walking.

Enjoy an exciting walk now!


STEP 04 STEP 05

# **SPECIFICATION** Product Specification

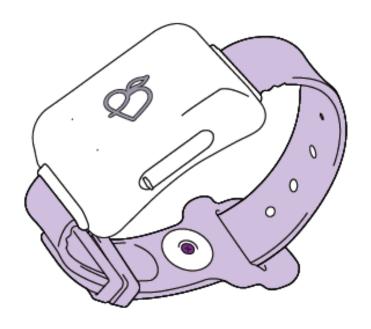

Model	PEACHBITE WATCH PbW-S1_KR		
Battery	Lithium Polymer Battery	Usage Time 10 Days	
Charging Time	4 Hour	X The usage time may vary depending on the number of times the battery is used.	
Charging Type	Micro USB 5pin	Body Material PC	
Communication	Bluetooth	Strip Material Silicon	
Weight	Large: 67g / Small 50g	KC Certification R-R-mkK-PbW	
FCC Certification	(추가 예정)		

# **SPECIFICATION** Product Specification

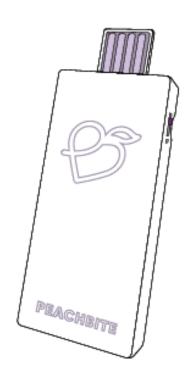

Model	PEACHBITE REPEATER PbG-S2
Battery	Constant power supply
Communication	Wi-Fi / Bluetooth
Weight	11g
<b>Body Material</b>	ABS
KC Certification	R-R-mkK-PbG-S2
FCC Certification	(추가예정)

# **Product A/S**

- For questions about device repair, handling, management, and apps, contact the customer service center or the place where you purchased the device.
- Please receive the warranty after confirming that the 'purchase date' and 'dealer information' are written at the place of purchase.
- In the case of a paid A/S request, if there is no purchase warranty, you must present the purchase receipt specifying the product name and place of purchase.
- The warranty period is one year from the date of purchase. If you request repair within the warranty period, please contact the customer service center or the place of purchase.
- The warranty period can be applied based on the contents of the warranty or purchase details.
- Repair fees will be charged for breakdowns caused by customer negligence, such as damage due to impact or submersion.
- After the warranty period has elapsed, paid repairs are provided through the customer service center.
- However, if 3 years have passed since the release date of the product, repair may be difficult depending on availability of parts.

# Warranty

Date of purchase / Place of purchase :

Model & Size:

**WATCH ID:** 

**REPEATER ID:** 

**※** Supplied with QR code.



#### **Customer Service**

MAX K Inc. (USA)

9945 Warwana Rd, Houston Texas 77080

**EMAIL** contact@peachbite.com **WEB** www.peachbite.pet **TEL** 713-497-5012

#### **NOTE**

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

### **FCC Radiation Exposure Statement:**

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with minimum distance 15 mm between the radiator & your body.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference.
- (2) This device must accept any interference received, including interference that may cause undesired operation.

CAUTION: Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.