

202306(Rev.0)

# ClipOn Hearing Aid


Model: PSA5035

## User's Guide


- Please read this instruction manual carefully and use it correctly.
- Please read before use.
- Please keep the instruction manual in a safe place and use it when necessary

### Package contents and names of each part


#### Main unit (front)

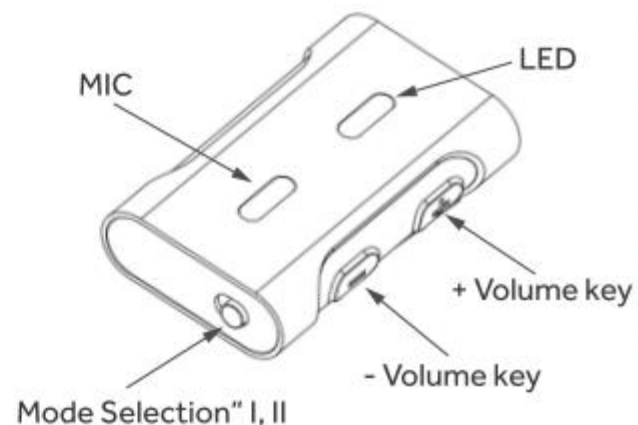

#### Main unit (back)

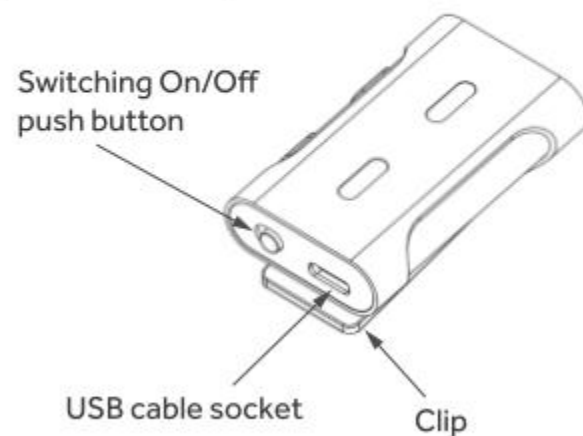

#### Earphone

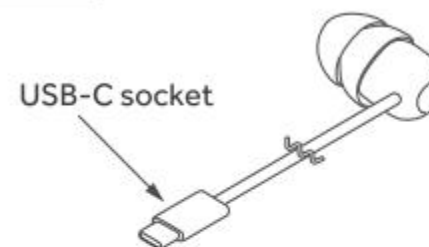

#### Charging cable




## earplugs



## Pouch




## Charging method

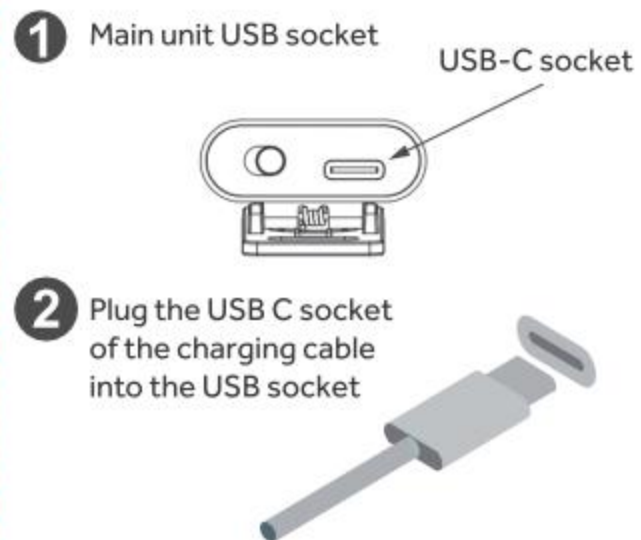

Charge once a day

## 4 How to replace hearing plugs

The earpiece can be removed by pinching the rubber part with your fingers and pulling

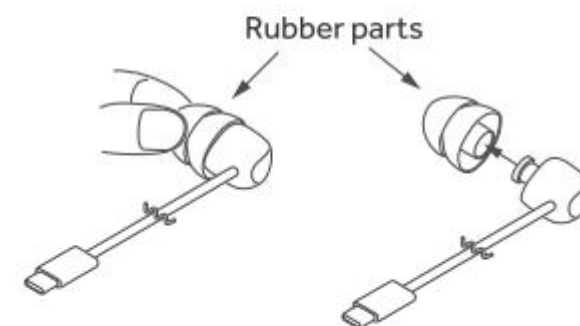

Do not hold the cable and pull it


## How to use:

1



Headphones connect USB socket

2

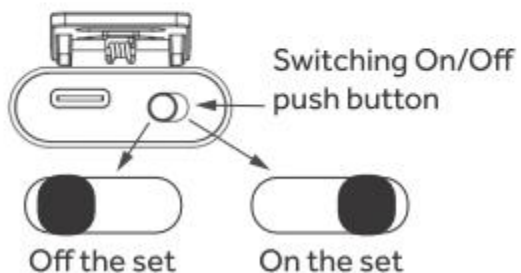

Switching On/Off  
push button

Off the set

On the set


3

Put the headphones into your ears and press the "+" and "-" buttons to adjust the appropriate volume

If the earpiece is small or large, replace it with another size

4

Hold the clip of the main unit between your fingers, then secure it to your breast pocket or anywhere on your clothing

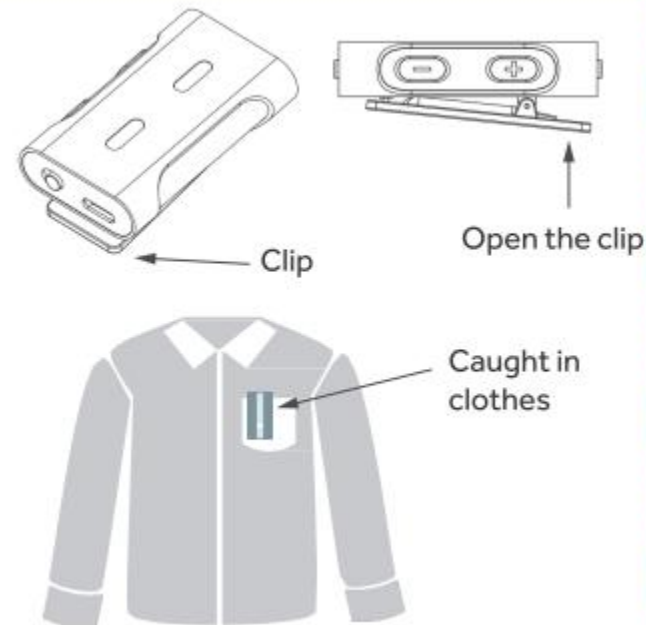


Clip

Open the clip

Caught in  
clothes

5

Select the switching mode according to the environment

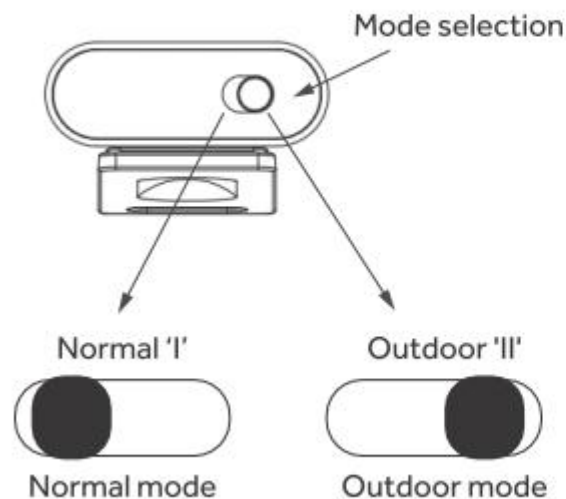

Mode selection


Normal 'I'

Outdoor 'II'

Normal mode

Outdoor mode

6

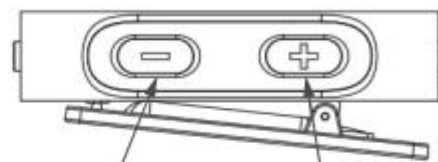

Press the '-' sign to  
make the sound smaller

Press the '+' sign to increase  
the volume

⑦. Power on the TV dongle and plug in the USB power input

Plug in the 3.5mm jack for TV output to dongle input. Turn on the Main unit power and turn to dongle mode. Check the main unit to hear the dongle connect voice prompt

List to the main unit and get the sound transfer form TV to main unit

## Remark

- Nightly charging guarantee extends battery life
- Free replacement for non-human damage within one year



## FCC Statement

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Caution: Any changes or modifications to this device not explicitly approved by manufacturer could void your authority to operate this equipment.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

### RF Exposure Information

The device has been evaluated to meet general RF exposure requirement.

The device can be used in 0mm exposure condition, compliance with exposure requirements.