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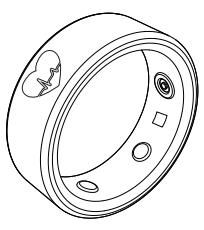
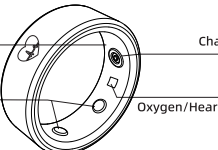
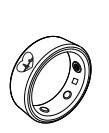
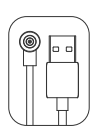
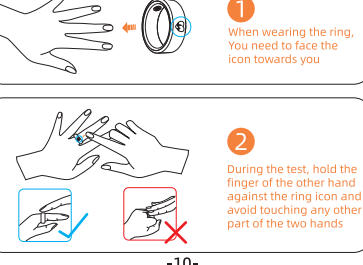
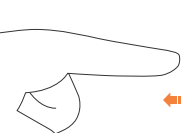
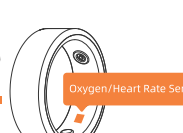
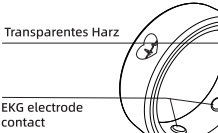
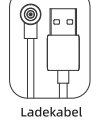
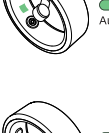
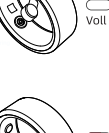
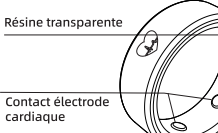
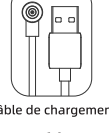
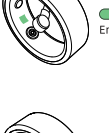
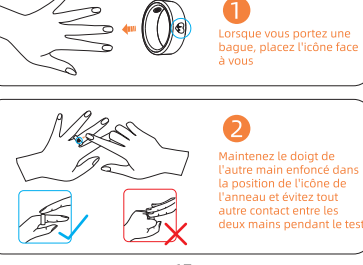
Component Description  Transparent Resin Charging port ECG electrode contact Oxygen/Heart Rate Sensor Packing List  Smart Ring  Charging cable  User manual -7-	Power On Connect the charging cable to power on the smart ring  Connection APP 1.Search and download Smarthealth APP, or scan the QR code on the right to select the appropriate APP version to download. 2.Open the app and follow the instructions to match the ring.  -8-	Lighting status The green light flashes when charging and turns off when fully charged The green light flashes during the heart rate test and the red light flashes during the blood oxygen test  Charging  Charged Over  Heart rate test  Blood oxygen Test -9-	ECG test ECG test can be carried out according to the operation mode below. It is recommended to wear the comfortable size on the index finger, middle finger and ring finger to better monitor the data  -10-	Wearing style Wear the ring on your own suitable finger, choose a comfortable size and wear it on the index finger, middle finger, and ring finger to better monitor the data  Wear the sensor on the inside of your finger Heart rate and blood oxygen measurements are more accurate  Oxygen/Heart Rate Sensor *Note: The measurement results of this product are for reference only and are not used for any medical purposes. Please follow the guidance of your doctor and do not use the measurement results for self-diagnosis and treatment. -11-	Note In daily use, please choose a size that fits your finger and avoid wearing too tight, keep the contact area of the finger ring dry, and wipe it in time when water enters. Wipe it off when water enters the ring. If symptoms such as redness and swelling occur at the contact area, stop using it immediately and consult a doctor. Secure Please keep the charging cable and the device in a dry environment when charging, do not use wet hands to touch the charging cable, please avoid the charging cable being immersed in liquids such as water. Do not touch the charging cable with wet hands, and avoid immersing the charging cable in water or other liquids. Do not disassemble or modify the battery, insert foreign objects, or immerse it in water or other liquids, as this may cause battery leakage, overheating, fire or explosion. Leakage, overheating, fire or explosion. Do not drop, crush or puncture the battery. Avoid subjecting the battery to large external pressures that may cause internal short-circuiting and overheating of the battery.  -12-
---	--	--	---	--	--

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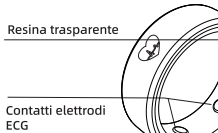
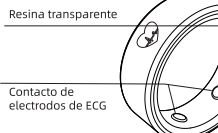
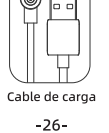
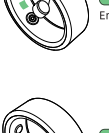
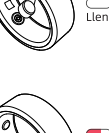
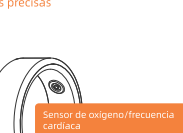
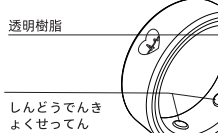
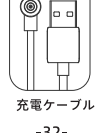
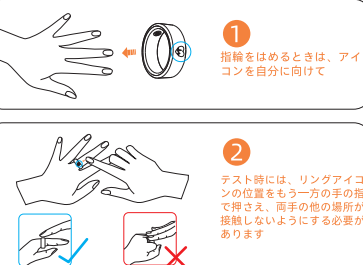
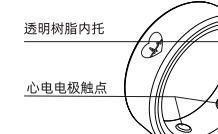
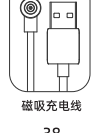
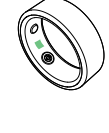
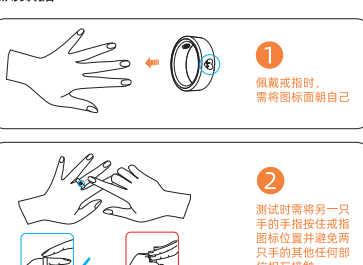
This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

FCC Radiation Exposure statement

The device has been evaluatec to meel general RF exposure requirement. The device can be used in portable exposure condition without restriction.