


EYE MASSAGER

Operating instruction

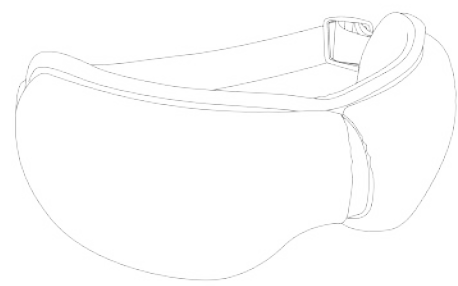

Please read the instruction carefully before use

PLEASE READ THIS INSTRUCTION BEFORE USE

Thank you for buying this eye massager. Please read the operation manual carefully before use, and pay special attention to safety matters, so as to operate and use this product correctly

This manual provides users with a safe and effective way of operation. Please keep it properly after reading it for future reference

ORDERS TO RECORD

- | | |
|---|-----|
| 1. Safety precautions | 2 |
| 2. Product profile | 2-3 |
| 3. Please confirm the commodity content | 3 |
| 4. Name and function of each part | 4 |
| 5. product usage | 5 |
| 6. Cleaning, maintenance and storage | 6 |
| 7. Faults and solutions | 7 |
| 8. Product specifications | 7 |

1

SAFETY PRECAUTIONS

If you wear contact lenses, please remove contact lenses before using this product. If there is any dirt or foreign matter in the mask, please wipe it with a wet cloth before using it to avoid accidental eye.

During the charging process, the product is not supported to start up (charging is conducted under the working state, the product starts the self-protection program and stops working automatically).

Do not use the eye massager during outdoor walking or driving.

Do not use eye massager in hot and humid places.

PRODUCT INTRODUCTION

Eye massager, USES the microcomputer chip control technology for the entire human nature simulation control, the application of air pressure that a whole new concept to the eye care, under the control of air pressure in the computer to a full range of muscles around the eye pressure, vibration and coupled with the internal function and the function of the far infrared heating, the eye acid bilges, the eye pain, relieve eye fatigue.

1. FEATURES:

If you buy a Bluetooth model Eye massage device can be connected to mobile phone bluetooth play music;

Portable wireless usage, USB charging mode;

Remote infrared heating microcomputer chip control technology;

5 massage techniques;

15 minutes automatic timing;

Air bag massage (massage the muscles around the eye area by proper matching between air filling and air releasing);

5V voltage output is more secure, one key operation is simple and convenient;

New vibration mode;

2

2. SCOPE OF APPLICATION:

For the eyes

3. CONTRAINDICATIONS:


Disables eye injuries or inflammation.

If there are symptoms of eye diseases can not be used, general implementation of eye surgery, glaucoma, cataract, retinal detachment and other major eye diseases patients should not be used.

For patients with deep stroke, hypertension and hypotension, please consult a professional physician first. If you feel any discomfort during use, please stop using immediately. Please avoid using the eye massager during drinking or physical discomfort


PLEASE CONFIRM THE COMMODITY CONTENT

Eye massage device, data cable, power plug, instruction manual ;

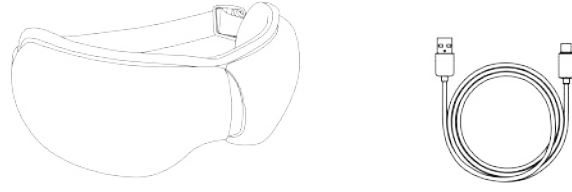

3

ONE KEY CONTROL LOOP

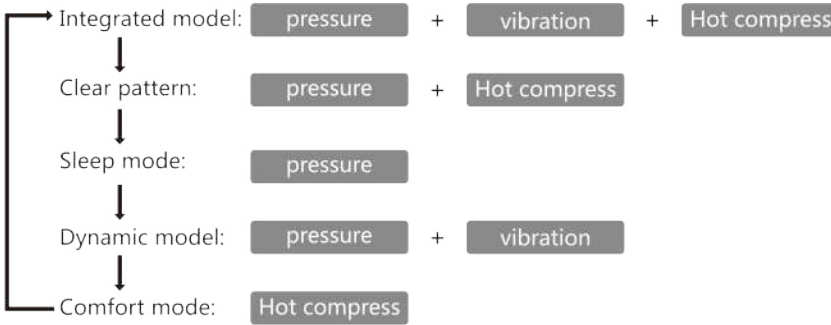

NAME AND FUNCTION OF EACH PART

- (1) long press the key for 3 seconds to start the comprehensive mode (pressure + vibration + hot compress)
- (2) press the key again to start clear mode (air pressure + heat compress)
- (3) continue to press the key to start sleep mode (air pressure)
- (4) continue to press the key to start vitality mode (pressure + vibration)
- (5) continue to open the key and start to stretch (hot compress)
- (6) press the button for 3 seconds to turn off the machine function
- (7) music pause and volume adjustment can be controlled at the playback end
- (8) If you buy a Bluetooth model the host starts bluetooth automatic open mobile search corresponding type connection

4

Warranty card

Purchase date: ____ Year ____ Month ____ Day	Good order: _____	
Product name/model	Cause of failure	Date of repair
Place of purchase: _____		
Contact address: _____		
Telephone: _____ Postal Code: _____		
Company Chop	Name and number of inspector	Customer signature

Certificate

Product name: Eye massager Quantity: 1 set

Date of production: Examination clerk:

9


PRODUCT USAGE

Method of use

1. Preparation before use

- (1) take out the eye massager gently
- (2) put the eye massage device on the eye position

2. Usage (as shown)

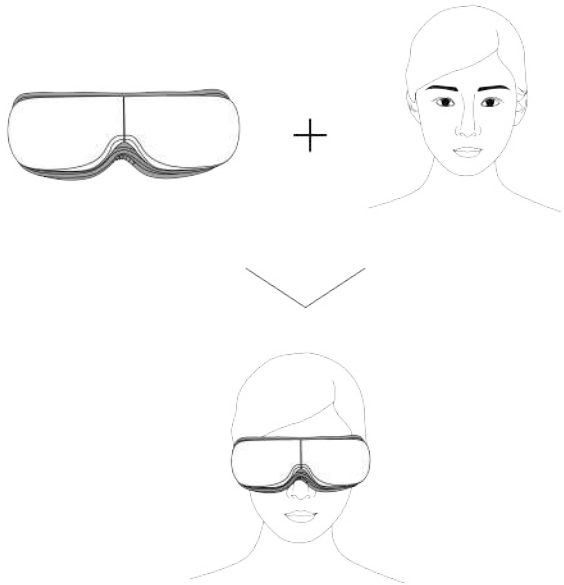


5

PRODUCT CLEANING, MAINTENANCE AND STORAGE

Product cleaning, storage and maintenance

1. Product cleaning and maintenance

If the product is stolen, please wipe it with a dry soft cloth. If it is particularly dirty, you can wipe it with medical alcohol.

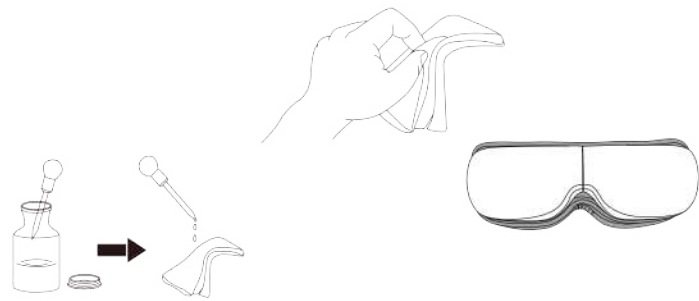

2. Keep

Keep it dry, ventilated and clean

Please keep out of reach of children

Do not disassemble the main engine without permission

6

COMMON FAILURES AND SOLUTIONS

Unable to boot:

1. If the battery power is too low, please charge immediately
2. Host protection (with electricity) connect the charger, charge, and allow the host to automatically remove the protection (charge for 5 seconds)
3. The battery is not charged and the charger is damaged
4. The pressure intensity is too weak

Note: all maintenance work must be done by our authorized maintenance personnel

PRODUCT SPECIFICATION

The product name	Eye massager
Automatic timing	15 minutes
The rated voltage	DC 5V
Rated power	3.4W
Power adapter	AC 100-240V DCSV

7

COMMON FAILURES AND SOLUTIONS

The fault	Possible cause of failure	The solution
Unable to boot	Low battery	Please charge immediately
	Motherboard protection (with electricity)	Connect the charger to charge, and allow the host to automatically remove protection (5 seconds after charging)
	The battery is not charged	Replace charger
	In charge	Unplug charger
The pressure is too weak	Low power	charging
Eye patch not hot	Low power	charging
	Model is wrong	Tune to the corresponding pattern

8

MESSAGE BRINGS WISDOM TO LIFE

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

FCC Radiation Exposure statement

The device has been evaluatec to meel general RF exposure requirement. The device can be used in portable exposure condition without restriction.