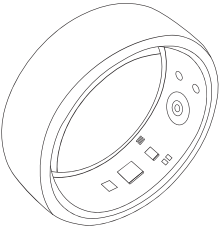
70 mm

70 mm

Smart Ring User manual

explain

Please note that the ring service is not intended to diagnose, treat, or prevent any diseases or medical conditions. The information and data provided in the service are for reference only and cannot replace the services of health professionals or physicians. If you have any questions about medical conditions, it is recommended to consult a professional physician.

Product Specification

- Product Name: Smart Ring
- Product weight: approximately 5.0 grams
- Outer surface material: stainless steel
- Main material: epoxy resin
- Memory: Built in 512KB
- CPU: PHY6222
- Heart rate sensor: HX3602
- Battery capacity: 15mAh per ring
- Standby time: 10-15 days
- Charging method: Magnetic charging (USB rated 5V)
- Connection method: Low power Bluetooth 5.1
- Operating System: Compatible with Android 6.0 and above, iOS 10.0 and above
- Waterproof level: IP68

Use requirement

This device can only be used on Android and iOS smartphones; System requirements: Android 6.0 and above, iOS 10.0 and above.

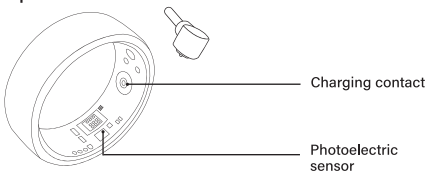
APP Download

Scan the QR code or enter the APP Store or App Store to download and install the APP. Please accept for all related permission requests. After installation, open the APP without login, and bind the device to use it normally.

Fitness Max APP

Component Introduction

Charging contact

Photoelectric sensor

Charge the ring

1. Magnetic linear charging

Use the equipped magnetic charging cable to attach to the metal point on the inner ring of the ring, and connect the other end of the charging cable to a USB charging head or computer USB interface. Please charge the device for the first time when using this product, and it will automatically turn on during charging.

2. Storage type charging compartment for charging

Remove the charging compartment and insert the provided USB charging cable into the charging port to fully charge it. The red light will remain on during charging, and the green light will remain on after charging is complete.

APP Function Introduction

Healthy Three Rings

- Exercise (upper half of the outer ring): The data comes from exercise, and the data for the day is cleared in the early morning (continuous walking, running, or exercising for more than 10 minutes is considered one exercise). Different exercise goals can be set in personal information.
- Calories (inner loop): The data comes from exercise and step count. Clear the data for the day in the early morning and set different calorie goals in personal information.
- Standing (lower half of the outer ring): Data source (if the number of steps exceeds 20 continuously within an hour, one standing will be displayed in hours). Clear the data for the day in the early morning and set different standing goals in personal information.

Step counting

Record daily real-time steps, reset to zero in the early morning, set target steps, and check completion rate.

Figure 1

Figure 1


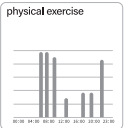
Figure 1

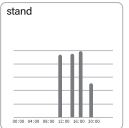sleep

Wear a ring to fall asleep, and when connecting to the app, you can check the sleep quality, sleep depth and wakefulness time, and view historical data such as sleep.

Heart rate/blood pressure/blood oxygen

When the ring is connected, measurements can be initiated by the APP, and historical data and curve graphs can be viewed. The ring is automatically measured every 2 hours by default. (Charging and sleep status stop monitoring)

Searching for devices

You need to connect to the app. When connected, your phone can search for the ring, and the ring LED light will flash to remind you.

Remote photography

When connected to the app, the app enters the remote control camera interface. Shake the ring and it will automatically take a photo after a countdown of 3 seconds, allowing the app to access the saved selfies in the album.

Other settings

Restore factory settings: Clear the current steps, calories, and some related settings of the app for the ring, and restore default settings.

Maintenance

- Keep the ring clean and hygienic;
- Do not use in high temperature and high humidity environment for a long time;
- Pay attention to the wiping and maintenance of the charging pilot;
- Long-term non-use should be charged every 20 days or so, and full power off storage.

Precautions for use

1. Bluetooth is disconnected beyond the distance during use and cannot initiate the ring search function until reconnected.

2. During the use of Bluetooth, there may be occasional disconnection. Please reconnect (if not connected for more than five minutes, manual connection is required).

Common troubleshooting

If there are any problems during the use of the ring, please refer to

the following terms to solve them. If the problem still exists, please contact the dealer or designated repair personnel.

1. Unable to search for ring Bluetooth.

① The battery level is too low, please connect a charger to charge.

② The APP automatically disconnects from the ring

③ The battery level is too low, please connect a charger to charge.

④ The distance between the phone and the ring is too far, disconnect, reduce the distance and automatically reconnect

⑤ The ring has a short lifespan.

⑥ The battery is not fully charged, please ensure sufficient charging time.

⑦ The ring cannot be charged properly

⑧ Please check if the battery is usable, as its performance may decrease after a few years of use.

⑨ Is the charger charging normal? You can try updating the charger.

Safety Warning

- Never allow children and pets to play with this product, which contains components that can cause choking.
- Do not replace the battery, open the shell and disassemble the product without authorization, otherwise it may bring safety risks;
- Do not place the product at very high or very low temperatures; Do not use this product in high temperature and humidity environment; Do not use chemical reagents to clean the product;
- Do not place the product in a high temperature and high pressure environment, the battery may explode; Do not wear the product while charging; Do not charge the product when it is wet.

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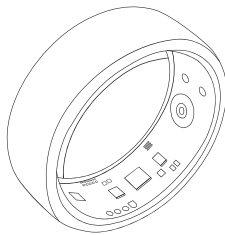
FCC Warning:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Caution: Any changes or modifications to this device not explicitly approved by manufacturer could void your authority to operate this equipment.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

(1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with minimum distance 0cm between the radiator and your body.