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
Wireless Bone Conduction Headphone

LUC-801 User Manual

1. Components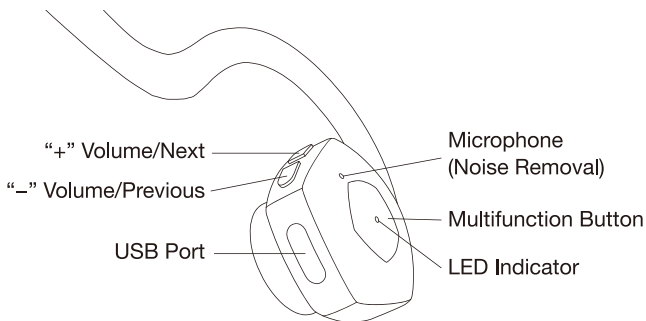

2. Accessories

Item	Quantity
Headphones	1 Set
Micro-USB Cable	1 Piece
Earplugs For Silence	1 Pair
Tightening Adjustment Band	1 Piece
User Manual	1 Piece

3. Charging Instruction

Connect your headphones to a standard USB port or wall charger via the USB cable we provided.

The standard charging time is 3 hours.

The LED is red while charging.

The LED turns blue when fully charged.

Reference:

It is highly recommended to fully charge the headphones before first use.

Do not use the headphones while charging.

To maintain long battery life, we recommend charging at least once a month.

4. Bluetooth Pairing

If you accidentally pair with an unintended device, press and hold the multi-function button for 3 seconds until the LED flashes blue and red. The headphones will switch to manual pairing mode.

Enable the Bluetooth function on your smartphone or other Bluetooth device and search for the device from the settings. After searching, select "LUC-801" in the device list.

The headphones will be automatically connected to the last connected device in about 10 seconds.

When successfully paired, the LED will flash blue.

Notice:

If the connected device exceeds the range (33 ft/10 meters), the connection will be disconnected. Within 10 minutes, the headphones will automatically reconnect if the previously connected device is within range. If the connection is lost for more than 10 minutes, please press the multi-function button to reconnect.

If the power of the connected device is turned off, it disconnects with the headphones. To reconnect, turn on your device and press any button on your headphones.

5. Status LED

Color	Situation
Red (flash)	Low power
Red (solid)	Charging
Blue (solid)	Charging completed
Blue and red (flash)	Pairing
Blue (Fast flash once every 2 seconds)	The connection between the headphones and the device has been lost. Device is powered off or out of range (33 feet/10 meters).
Blue (Fast flash once every 5 seconds)	Connected. Make a call or play audio.

6. How to wear


Place the neckband on the back of your neck, place the transducer in front of your ear, and place it over your cheekbone.

7. How to use the tightening adjustment band


This band is used to adjust the tightness of the headphones when they are not tight enough.


- ① Wrap it around the metal part of the headphone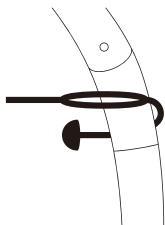

- ② Pass the hemispherical part through the ring part and secure it.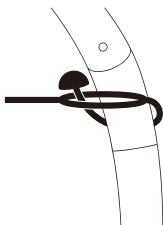

- ③ Do the same on the other side

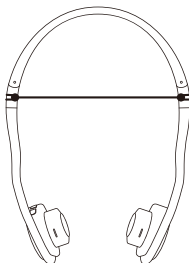8. Specification

Play Time	About 12 hours
Charging Time	About 3 hours
Standby Time	Over 1000 hours
Bluetooth Version	V4.0
Frequency Range	100Hz-18KHz
Radio Range	10 meters (or 33 feet)
Bluetooth Connection	A2DP, AVRCP, HSP, HFP
Type Of Speaker	Bone conduction Bluetooth headphones
Waterproof	IP55
Size	13*6cm (folded)
Weight	45g

9. Basic Operation

Operation	Headphone control	Sound
Answering A Call	Click the multi-function button.	Buzz
End A Call	Click the multi-function button.	Buzz
Reject A Call	Press the multi-function button for 1-2 seconds.	Buzz
Redial	Double-click the multifunction button.	Buzz
Volume Control	Click the volume button “+”/“-”.	
Play/Pause Music on Smartphone	Press the multi-function button once.	
Previous/Next Song	Press the volume button “+”/“-” for 2 seconds.	Buzz
Power On	Press the multi-function button for 3 seconds.	Buzz
Power Off	Press the multi-function button for 5 seconds.	Buzz
Bluetooth Disconnected		Buzz
Bluetooth Connected		Buzz
Low Power		Buzz

10. Storage and Maintenance

Store your headphones in a cool, dry place away from direct sunlight. Storage temperatures are 0-45°C (32-113°F). Battery life may be shortened if used in cold/hot/humid environments.

When unused for a long period of time, make sure to charge before use.

Do not leave the headphones in extremely hot or cold places.

Do not charge the headphones in humid environments.

Clean the headphones with a soft, dry cloth.

11. Troubleshooting

Problem	Possible problems	Solution
Can not turn on	Battery is low	Please charge the battery
Can't connect automatically	Is in standby mode	Double-click the multifunction button
Can't pair	Not in pairing mode	Press the multi-function button for 3 seconds
The person on the other end of the phone can't hear you	Microphone not enabled	Press the multifunction button to call

12. Limited Warranty

All products are covered by a 1 year limited warranty. Products with manufacturing defects may be replaced free of charge within one year from the date of purchase. If one year has passed since the date of purchase, there will be a charge for repair or replacement of parts.

13. Important Matters

Damage or failure caused by any of the following is not covered by the limited warranty:

- Misuse or negligent use;
- Use contrary to instructions
- Deviations from storage and transportation guidelines
- Repair or modification outside of an authorized service center
- Disassembly of the device

14. Warning

- Please refrain from disassembling, hitting, pressurizing, throwing into fire, etc.
- Do not use if severe bulges appear.
- Do not place in high temperature.


Dedicated Sound Collector

LUC-08 User Manual

1. Product Description

- This product is a set of wireless bone conduction headphones and sound collector LUC-08.
- This product assists you in hearing through Bluetooth pairing.
- The LUC-08 can be connected using an AV Cable to devices that do not support Bluetooth connectivity, such as TVs, computers, and DVD players.
- By transmitting sound from LUC-08 to bone conduction headphones and collecting the sound only to the user, you can enjoy the sound without disturbing others.
- The headphones can be used as standalone wireless headphones and can be connected to a smartphone or other device via Bluetooth (please refer to the headphone instruction manual for instructions on how to use them).

2. Parts Description

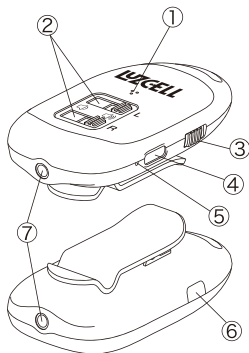- ① Sound Collection Microphone
- ② Volume slider ③ Power Switch
- ④ Charging Port ⑤ LED Indicator
- ⑥ Pairing Button ⑦ Audio Jack

- ④ Insert the USB Type-C terminal of the charging cable here to charge.
- ⑦ Insert the included AV Cable into this socket and plug the other end into your TV, radio, mobile phone, etc. to enjoy the sound.

3. Accessories

List	Quantity
Sound Collection/transmitter	1
Charging Cable	1
3.5mm AV Cable	1
User Manual	1

4. Battery Charging

- We recommend charging the headphones and bone conduction sound collector LUC-08 before using them for the first time.
- It takes About 3 hours to fully charge the battery.
- We recommend charging it even when not in use. Even if you do not use it for a long time, we recommend charging it once a month.
- To avoid the product from consuming power for a long time, please turn it off immediately after use.

• 5. Bluetooth Pairing

- (Two bone conduction headphones can be connected to LUC-08 at the same time.)
- First turn on the LUC-08.
- Turn on the power switch and press the pairing button on LUC-08 once to enter the pairing state.
- Turn on the bone conduction headphones you want to pair.
- With the power off, press and hold the multi-function button for 3 seconds to power on the bone conduction headphones.
- When you turn on the bone conduction headphones, the LED indicator on the headphones will quickly flash red and blue and automatically search for LUC-08.
- LUC-08 and bone conduction headphones will automatically connect in about 10 seconds.

- If the connection is successful, the LED on the LUC-08 will remain solid blue, the red light will flash slowly and the blue light on the bone conduction headphones will flash slowly.
- If you want to connect a second pair of bone conduction headphones, turn on the headphone and press the LUC-08 pairing button at the same time, the LEDs on the headphone and LUC-08 will flash to enter the pairing state.
- A second pair of bone conduction headphones connect to the LUC-08 device in about 10 seconds.
- Once the LUC-08 and two pairs of bone conduction headphones are connected, simply turn on each device and the LUC-08 and bone conduction headphones will be connected automatically.

Notice: Pairing of the LUC-08 and bone conduction headphones included in the set is completed before the product is shipped. Pairing is enabled automatically when the customer simply turns on the product. If the connection fails during use, please follow the steps above to configure again.

6. How to wear

7. Connecting LUC-08 with other Devices

- You can connect it to your cell phone, tablet, radio, etc. using the included 3.5mm AV Cable. You can listen to the audio and watch the program directly.
- By connecting to a TV or other device using an AV Cable, you can watch without disturbing the people around you.

Notice: Please note that if there is a wall between the LUC-08 and bone conduction headphones, the sound may be interrupted.

8. Description of LED Indicator

Headphone

LED Color	Situation	Explanation
Red	Flash Fast	Battery Level Low
Red	Always On	Charging
Blue	Always On	Charging Completed
Blue	Flash Fast	Pairing In Progress
Blue	Flash Once Every 5 Seconds	Pairing Connection Successful; Not Connected, Waiting
Blue-red	Shine For 1 Second	Off

Sound Collector

LED Color	Situation	Explanation
Red	Always on	Charging
Blue	Flash slowly	Waiting for automatic reconnection
Blue	Always on	Connection status
Blue	Flash fast	Pairing in progress

9. Operation Instructions

Headphone

Function	Operation
On (manual)	Press and hold the multi-function button for 3 seconds
Off (manual)	Press and hold the multi-function button for 5 seconds

Off (auto)	When waiting for more than 10 minutes without connection or when the battery level is low
Automatic reconnection	Automatically connects to paired equipment within 10 seconds after turning on the power
Pairing	Automatically enters pairing state 10 seconds after turning on the power. Automatically reconnect within 10 minutes after disconnecting.
Reconnect (auto)	Automatically reconnect within 10 minutes after disconnecting
Reconnect (manual)	If the connection is lost for more than 10 minutes, the headphones will automatically turn off and you will need to turn them on again.
Volume adjustment	Adjust the volume using the “-” and “+” buttons

Sound Collector

Function	Operation
On	Power on or enter auto-reconnect state
Off	Power off
Pairing button	Press once to enter pairing
Volume slider	Adjust the volume using the volume slider. Volume can be adjusted separately for left and right.

10. Standard Specs

Item	Headphone	Sound Collector
Usage Time	About 12 Hours	About 12 Hours
Charging Time	About 3 Hours	About 3 Hours
Standby Time	About 1000 Hours	About 1000 Hours
Bluetooth Version	4	4
Frequency Range	100hz-18khz	100hz-18khz
Effective Range	10 Meters	10 Meters
Microphone	-58db	-54db

11. FAQ

Problem	Possible cause	Solution
Can't turn on	Insufficient battery power	Please charge
Can't pair	Your headphones or LUC-08 may be connected to other Bluetooth devices.	Disconnect communication with the connected device and pair again.
Connection is unstable	The distance is too far or there is an obstacle between them.	Move obstacles and try short distances

12. Storage and Maintenance

- Please store in a well-ventilated place.
- Do not use during severe thunderstorms.
- When unused for a long period of time, please charge it before use.
- Please clean with a clean cloth and do not wash with water.

13. Quality Assurance

- The warranty period is one year from the date of purchase.
- Damages caused by human factors or force majeure are not covered by the warranty.
- Do not disassemble or modify the product without permission. The warranty will be invalidated if the product is disassembled or modified.
- If you experience any problems with your product, please contact your local retailer immediately.

14. Warning

- Please refrain from disassembling, hitting, pressurizing, throwing into fire, etc.
- Do not use if severe bulges appear.
- Please refrain from using it in high temperature environments.

Warranty Conditions

The product comes with a one year warranty from the date of purchase.

Warranty coverage includes manufacturing defects, but excludes misuse, loss, or theft.

During the warranty period, we provide repair or replacement services, but shipping and on-site service fees are not included.

If more than one year has passed since the date of purchase, products with manufacturing defects will be repaired and charged for parts cost or replaced if necessary.

This warranty does not include the following:

Use contrary to instructions

Misuse or careless operation

Violation of storage and transport instructions

If you are unable to provide a valid proof of purchase or valid warranty.

Repairs, dismantling, or alterations made by anyone other than an authorized service center

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This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Warning: changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment

The devices has been evaluated to meet general RF exposure requirement, the device can be used in portable exposure condition without restriction

LUZCELL®