## Lefy: Link for Excitement, Empowering Vitality, Shaping Health

We firmly believe that health is humanity's most valuable asset, and exercise is the best way to achieve it. Therefore, we are dedicated to leading the public towards a healthier and happier life through innovative products and services.

Our mission is to provide comprehensive and professional health and fitness solutions for fitness enthusiasts and health-conscious individuals. From high-quality wearable health and fitness products, personalized workout plans, and scientific nutrition guidance to AI-powered services, we strive to meet the diverse needs of different groups in the area of health and fitness.

In the future, Link for Excitement will continue to innovate, expand its business fields, and work with outstanding global partners to promote the development of the health industry. We are convinced that through our efforts, more people will experience the joy and vitality of life, as we together create a better tomorrow.

---

### Thank You for Choosing Ring X

Before using the ring, please carefully read this pamphlet and the accompanying documents. Use the ring, the app, mini-program, public account services, Lefy services, and any other related services with caution, as failure to follow the instructions may result in product damage, malfunction, or even injury.

### Support and Inquiries

If you have any questions regarding starting or using the ring, app, mini-program, public account services, or other Lefy services, or if you have any product-related concerns, please refer to the FAQ at [www.lefytech.com](http://www.lefytech.com).

### Safety Information

Please note that Lefy services are not intended for diagnosing, treating, curing, or preventing any disease or medical condition. Information and guidance provided by Lefy services are for reference only and should not replace the services of health professionals or physicians. If you have any medical concerns or plan to make any changes to your health, sleep, or activity based on Lefy services, always consult your doctor. Never ignore or delay seeking professional medical advice due to information from Lefy services.

We are not liable for any health issues resulting from the information or guidance provided by Lefy services. If you choose to make changes to your health, sleep, or activity based on Lefy services, you agree to assume all risks. It is important to be sensitive to your body's responses. For example, if you experience unexpected, recurrent, or prolonged pain, fatigue, or discomfort due to changes in health, sleep, or activity, consult a physician before continuing with these changes.

Be cautious when moving or handling heavy items, ensuring that the ring or any other Lefy products worn do not get caught in fixed structures or heavy objects. If your finger swells or becomes irritated due to the ring or any other Lefy product, remove it immediately. If symptoms persist after 2-3 days without using Lefy products, consult a dermatologist.

#### Usage, Care, and Maintenance

Finger size may vary throughout the day, making it difficult to remove the ring at times. If the ring gets stuck:

- Wet your finger with cold water and mild soap, then slowly twist the ring to remove it.
- Raise your hand above your heart to lower blood pressure, then attempt to remove the ring.
- In an emergency or if discomfort occurs and you're unable to remove the ring, seek medical attention immediately.

Clean the ring using a soft cloth or by washing your hands with mild soap and water.

The Ring X may wear out with showering, bathing, swimming, and snorkeling.

Avoid wearing the ring during heavy-duty activities such as strength training, shoveling, or handling heavy tools, as well as when carrying objects made of metal, ceramic, or stone. Additionally, avoid friction between the ring and other rings or objects made of metal, ceramic, stone, or diamonds. The Ring X can get scratched and may also scratch softer metal jewelry or items it comes into close contact with, such as gold, silver, or aluminum phone cases. Some ceramic phone cases with soft coatings may also get scratched.

Keep the ring away from children. This product is not intended for individuals under the age of 18. If you suspect a child has swallowed the ring, seek medical attention immediately.

Do not expose the ring to excessive heat, such as inside vehicles or under the sun.

Do not pierce the ring or its battery.

Avoid handling batteries or using devices and machinery containing batteries while wearing the Ring X. In some cases, if both the cathode and anode of another battery touch the ring, a short circuit may occur, similar to a standard metal ring. This could present a potential electric shock hazard. Take appropriate precautions to avoid such situations.

---

#### Safety and Product Information

##### Specifications:

- Outer Surface Material: Titanium, 18K Gold, or Ceramic, depending on the specific model.
- Inner Surface Material: High-density Ceramic.
- Memory Capacity: It is recommended to sync the ring's data daily to the Lefy app. The ring can store data for up to one week, depending on the type and frequency of usage.
- Battery: Rechargeable lithium-ion polymer battery, ranging from 15mAh-27mAh (depending on ring size), non-replaceable.
- Battery Life: Typically between 3-7 days, depending on the enabled functions, type, and frequency of use.

- Charging: Wireless charging using the Lefy charger.
- Charging Time: Typically between 30-120 minutes, depending on the initial charge level.
- LEDs: The red and green light-emitting diodes (LEDs) used in the product operate within the visible color spectrum, making them safe for both users and those nearby.
- Connectivity: Bluetooth Low Energy (Bluetooth Smart).
- Software Updates: Automatic software updates via the app.
- Lefy App Availability: Available on App Store, Google Play, app markets, or via the official website link.
- Security: Encrypted Bluetooth communication between the ring and the app.
- Water Resistance: Up to 100 meters/50 meters.
- Charger is not waterproof and is intended for indoor use only. Do not use or store the charger in humid environments such as bathrooms.
- Operating Temperature Range: -10-52°C/14-125°F .
- Charge at room temperature.
- Charger Input Connector and Voltage: USB Type-C connector. USB rated at 5V°.
- The device should be powered by an external 5Vdc power supply compliant with IEC62368-1 standards. The power supply for the device should not exceed 15W after 3 seconds.
- The charger complies with international safety standards for ICT and AV equipment (EN/IEC62368-1).
- Dispose of the ring (and its internal battery) according to applicable local laws and regulations. Special handling may be required. The charger is an electronic device and should follow local electronic product recycling regulations.
- Refer to [www.lefytech.com](http://www.lefytech.com) for instructions on performing a factory reset before disposing of the ring.

---

#### Disclaimer

The services are provided "as-is" and "as available" without any warranties of any kind. We and our affiliates, officers, directors, employees, representatives, agents, partners, and licensors disclaim all express or implied warranties in connection with the service and the use thereof, including warranties of merchantability, fitness for a particular purpose, and non-infringement.

Lefy does not guarantee that your use of the service will be uninterrupted, virus-free, or error-free. Recommendations made by Lefy or its representatives, either orally or in writing, do not constitute any warranty. The limited warranty provided by Lefy applies only to the original purchaser, ensuring that the product is free from defects in materials and workmanship for one (1) year from the date of purchase. If local laws provide for a longer warranty period, the warranty will be extended accordingly. Lefy will repair or replace any component that does not meet this limited warranty at no cost to you, except for shipping charges. Refurbished or new products may be used for replacement at Lefy's discretion. This warranty does not cover normal wear and tear, damage caused by misuse, accident, or unauthorized modifications.

---

#### Limitation of Liability

Lefy and its affiliates are not liable for any damages, including direct, indirect, incidental, or consequential damages resulting from your use of or inability to use the services. In no event will Lefy's liability exceed 100 RMB.

---

#### Intellectual Property

Lefy and Ring X are our trademarks. Lefy content, products, features, and services are protected by U.S., China, Hong Kong, and international copyright, trademark, and patent laws. All rights reserved.

---

#### Safety Information

This service is not a medical device. It is not intended to provide medical advice or diagnose, treat, cure, or prevent any disease or medical condition. Consult your doctor before making any health, sleep, or activity adjustments based on the information provided by this service. Lefy is not liable for any health issues that may arise from the use of this information.

---

#### Terms of Use

Your use of Lefy products and services is governed by our Terms of Use and Privacy Policy, available at [www.lefytech.com](http://www.lefytech.com). The disclaimers and limitations of liability stem from our Terms of Use, which may change if our terms are revised.

#### FCC Warning Statement

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- - Reorient or relocate the receiving antenna.
- - Increase the separation between the equipment and receiver.
- - Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- - Consult the dealer or an experienced radio/TV technician for help.

Unauthorized changes or modifications to this device could void your authority to operate the

equipment. This device complies with part 15 of the FCC Rules. Operation is subject to two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

This device has been evaluated to meet general RF exposure requirements and can be used in portable exposure conditions without restriction.

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