

# VIV Ring User Manual

## 1. Introduction

Thank you for choosing the VIV Ring. VIV is a state-of-the-art wearable device designed to monitor your biometric signals and provide valuable health insights. The VIV Ring are not medical devices and are intended solely for wellness purposes. Please read this manual carefully before using the device to ensure proper operation and safety.

## 2. Product Overview

Product Name: VIV Ring

Model Number: VIV-Ring

Manufacturer: VIV HEALTH INC.

Country of Manufacture: China

Design and Development: Seoul, South Korea

## 3. Safety and Compliance Information

-FCC Compliance: This device complies with Part 15 of the FCC Rules. Operation is subject to the following conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

-RF Exposure: This device has been evaluated to meet FCC radiation exposure limits.

-Battery Safety: Use only the supplied charger. Do not disassemble or expose the device to extreme temperatures.

## FCC Warning

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help. The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction

#### 4. Package Contents

VIV Ring

Charging Dock

USB-C Charging Cable

User Manual

#### 5. Device Specifications

Wireless Connectivity: Bluetooth Low Energy (BLE)

Battery Life: Up to 5~6 days

Charging Time: Approximately 90 minutes

Water Resistance: IPX8

Sensor Type: Optical Heart Rate Sensor, SpO2 Sensor

#### 6. Setup Instructions

-Download the VIV Ring App from the Apple App Store or Google Play Store.

-Create an account and follow the on-screen instructions.

-Charge the VIV Smart Ring fully before first use.

-Pair the ring with your smartphone via Bluetooth.

-Adjust device settings and measurement intervals through the app.

#### 7. Usage

Manual Measurement: Initiate real-time heart rate and SpO2 measurement and more via the app.

**Automatic Measurement:** The ring supports 24-hour continuous monitoring with adjustable measurement intervals.

**Data Access:** View your health data and insights in the VIV Ring App.

## 8. Troubleshooting

-Device Not Pairing: Ensure Bluetooth is enabled and the ring is charged.

-No Automatic Measurements: Confirm that measurement intervals are correctly set in the app.

-Low Battery: Recharge the ring using the supplied charging dock.


## 9. Maintenance and Care

-Clean the ring with a soft, dry cloth.

-Avoid exposure to chemicals or abrasive materials.

-Store in a cool, dry place when not in use.

## 10. Regulatory Information

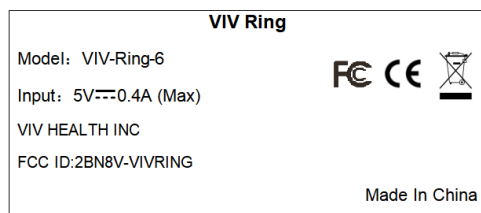

**CE Certification:** [CE ID]

## 11. Warranty and Support

**Warranty Period:** 1 year from the date of purchase

**Customer Support:** [viv@vivring.ai](mailto:viv@vivring.ai)

**Support Phone Number:** +82 2 6487 6878

Thank you for choosing VIV Ring. Stay connected to your