



AAVAA Software User Guide

Instructions for setting
up your AAVAA Devices.

Index

Device Set-Up	1. Turn on your AAVAA Device	3
	2. Pair your device	4
	3. Disconnect your device.	7
Functions & Settings	4. Settings Overview	8
	5. Keyboard & Demos	9
	6. Log-out & Report a problem	10

Warning:

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

NOTE: This device and its antenna(s) must not be co-located or operation in conjunction with any other antenna or transmitter. The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

1. Device Set-Up

- A** To turn on your **AAVAA Glasses**, switch the button at the bottom of the device. When your device enters **pairing mode** it will light a **purple light**.

○ – on

✕ – off

● Pairing

● Connected

● Low Battery




- B** To turn on your **AAVAA Headband**, press the button at the top of the device. When your device enters **pairing mode** it will light a **purple light**.

● Pairing


● Connected

● Low Battery

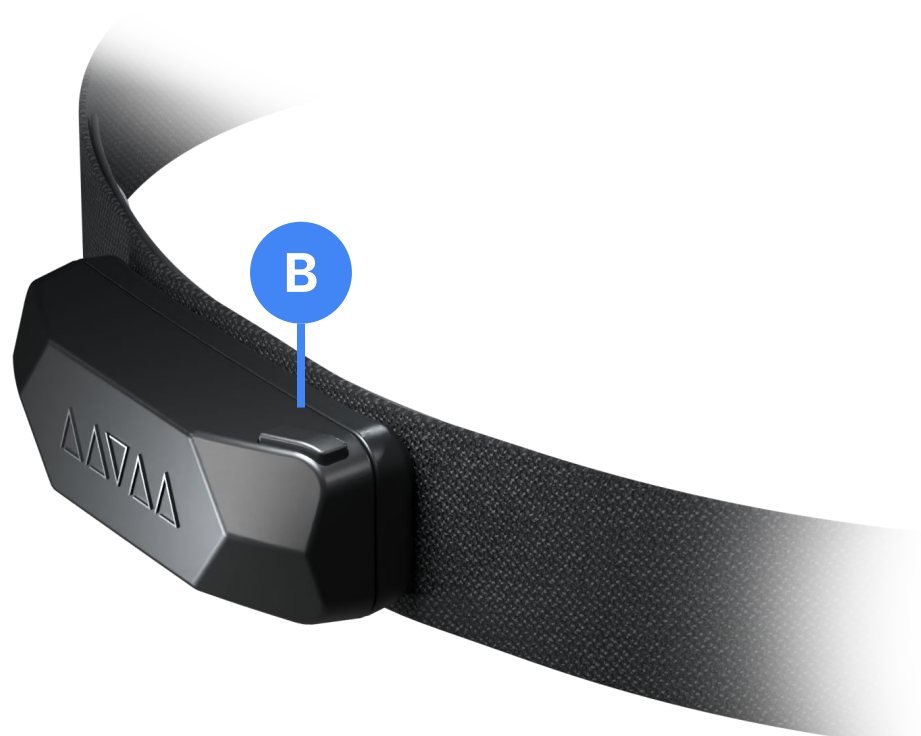


2. Pair Your Device

- C** Head to your computer's **Bluetooth Settings** and add your **AAVAA device**.

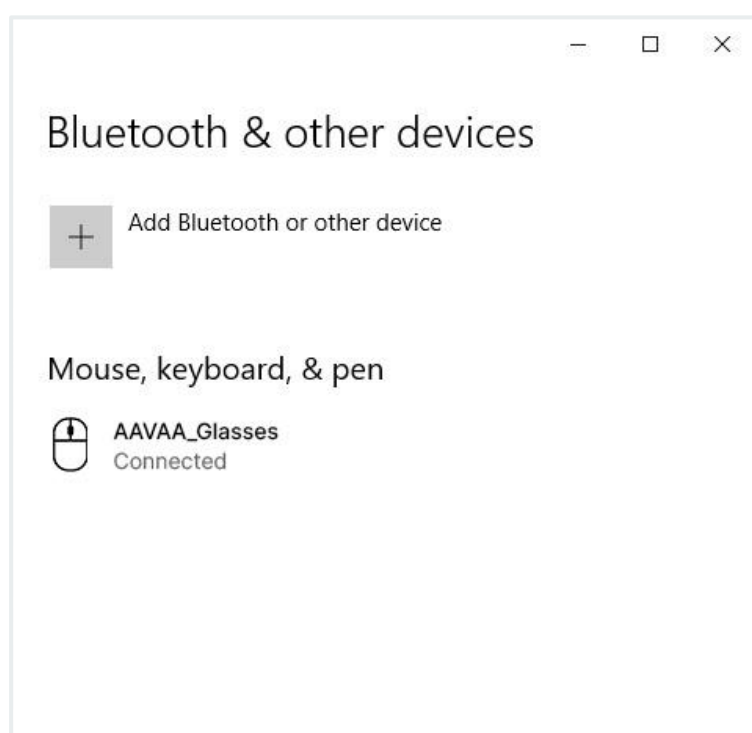


- D** Connect the **AAVAA USB key** to your computer and install "**AAVAA.exe**" from the files.

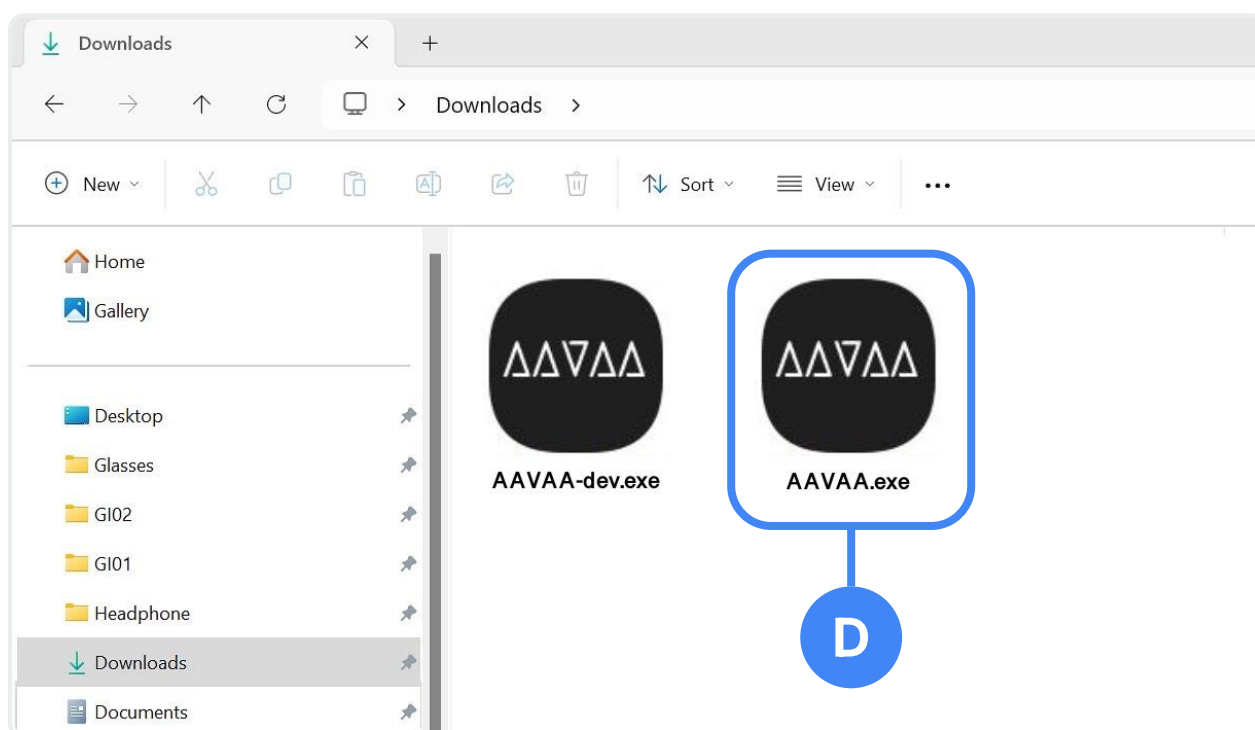


2. Pair Your Device

- E** Enter the Username and Password provided to you with your purchase.

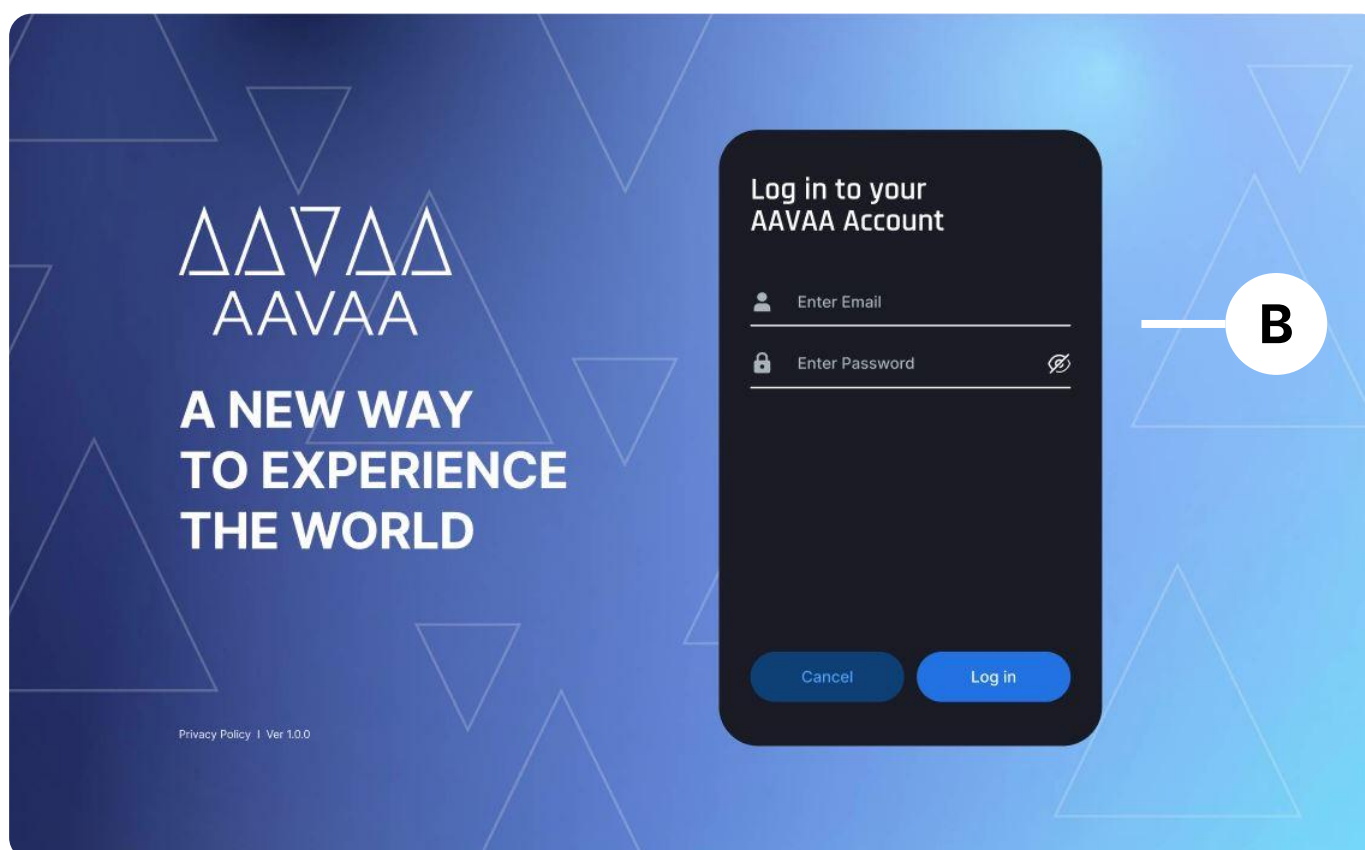


- F** To turn on your **AAVAA Glasses**, switch the button at the bottom of the device. For the **AAVAA Headband**, click the button at the top.

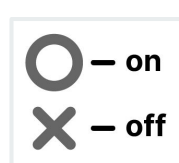

2. Pair Your Device



Select your **AAVAA device** from the list of **Available AAVAA Devices**. Hit "**Refresh**" to update the list.

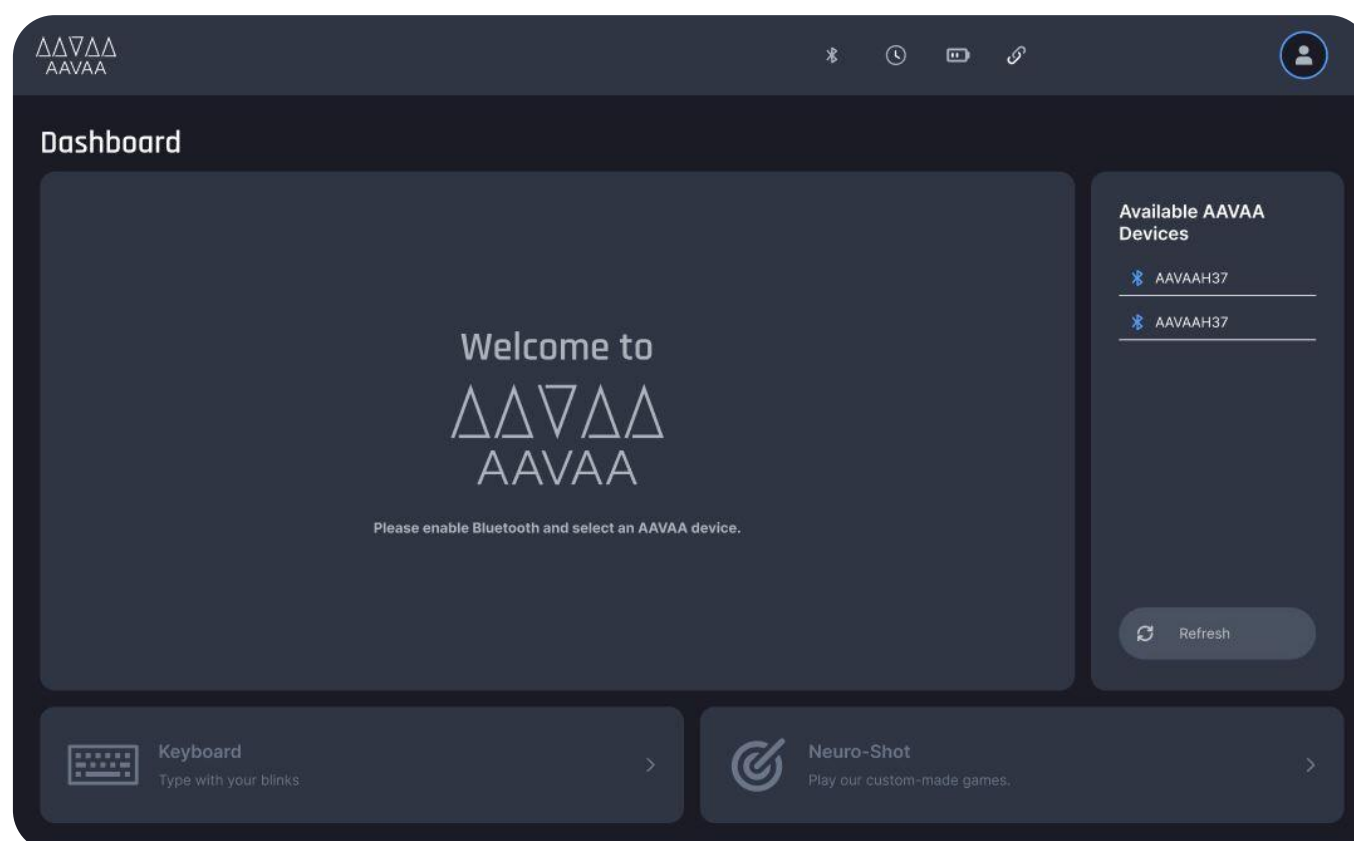


AAVAA Devices Names/Codes

Glasses: **AAVAAGLS**


Headband: **AAVAABND**

2. Disconnect Your Device


- H** To disconnect your AAVAA device or connect a different device, click on the “**Disconnect**” button.

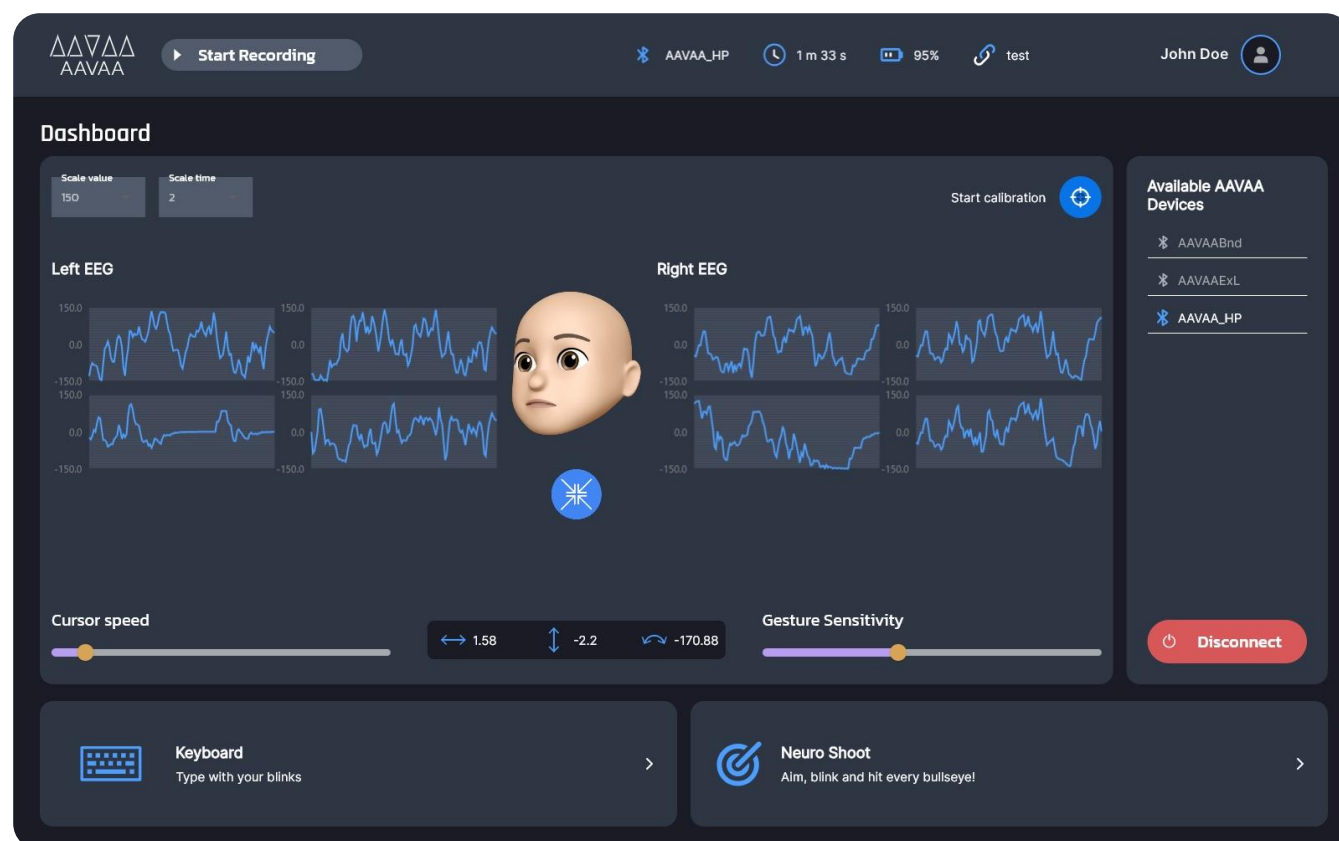

- I** The list of **Available AAVAA Devices** will appear again and you select a new device.

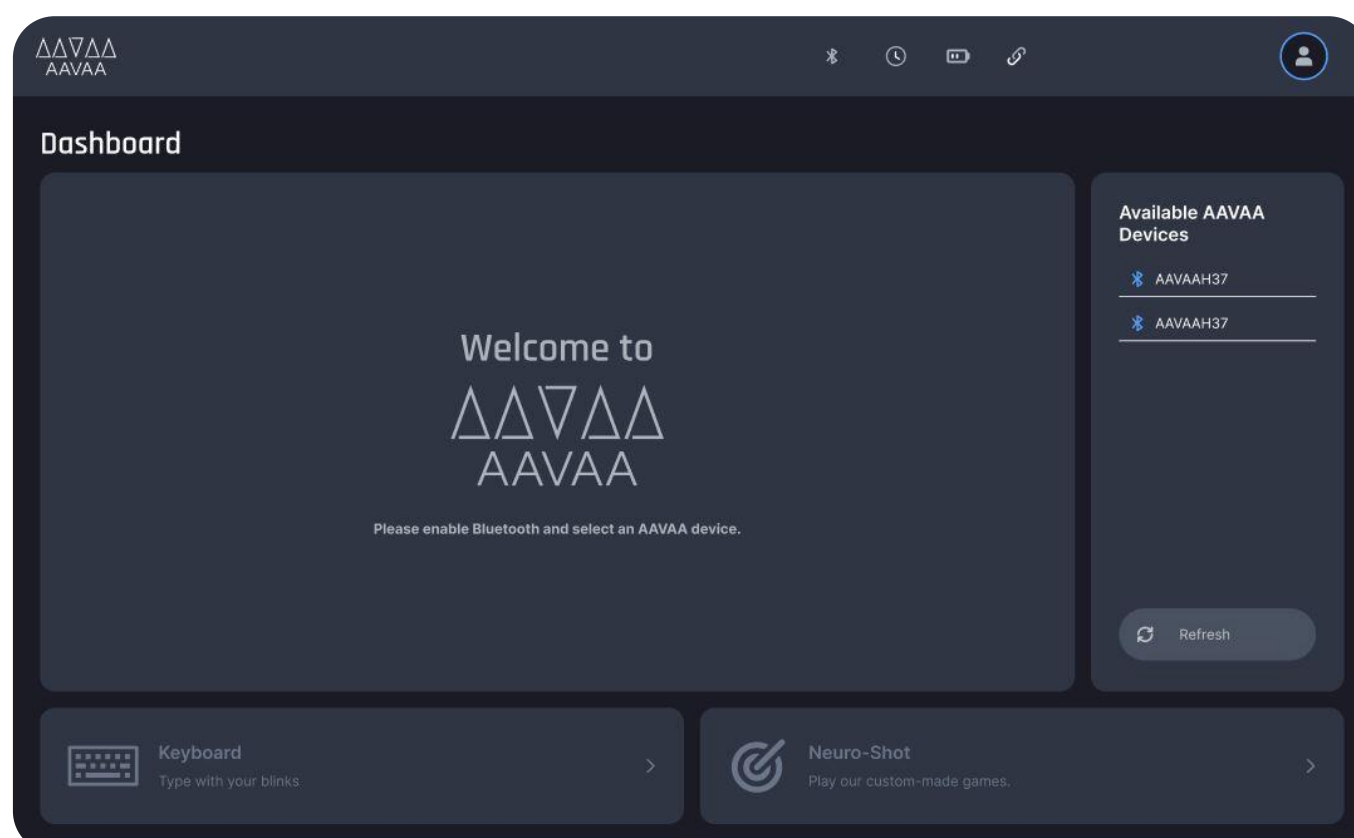

2. Functions & Settings

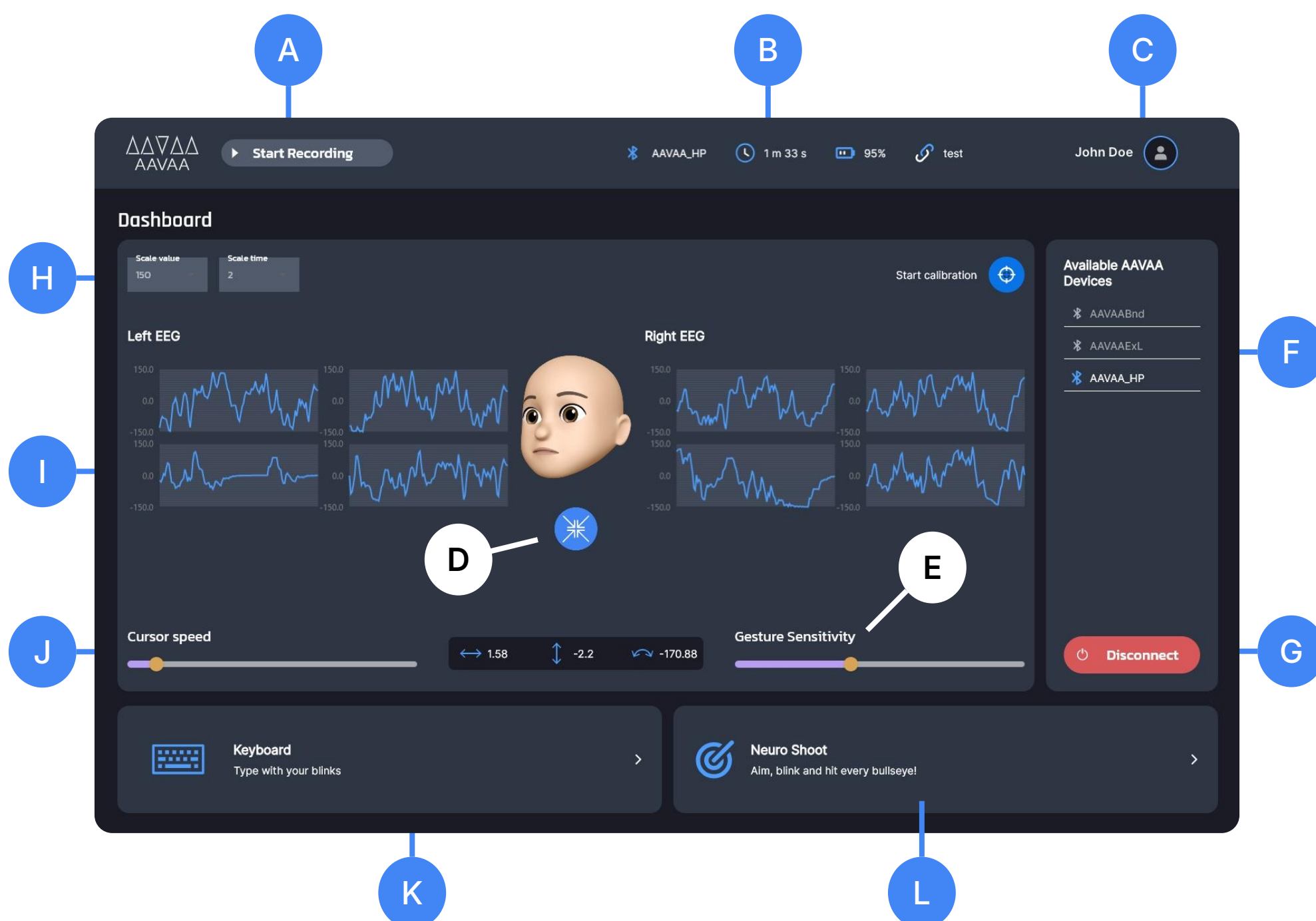

A - Start or stop sensor data recording.

B - Basic status on the connected device.

C - Log-out and report a bug settings.

D - Click the Re-Center button to center cursor.

E - Control sensitivity of facial gestures.

F - List of available bluetooth AAVAA devices.

G - Disconnect your device from the application.

H - Edit the scale and time value of the graphs.

I - Sensor Channel Graph

J - Control the speed of the head-cursor.


K - Click to access the AAC Keyboard

L - Click to access our Neuro-Shot game demo.

Warning: Data Recordings will be saved by default at this location on your PC:
C://Users/<your-user-name>/AppData/Local/AAVAA/data

2. Functions & Settings



To access the Keyboard and Game Demo simply click on their icons on the bottom section.

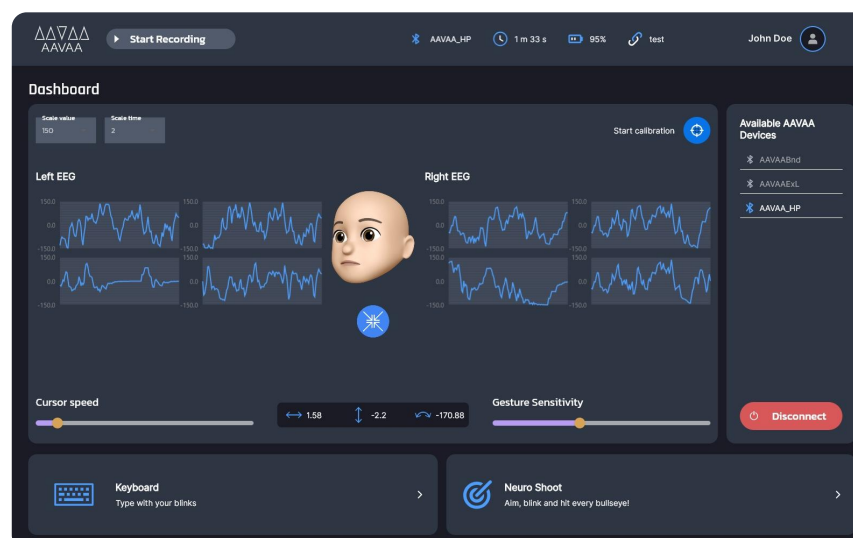

With the **AAC Keyboard** you can type and spell words out loud.




The **Neuro-Shoot** game is a demo custom made for our Neuro devices.

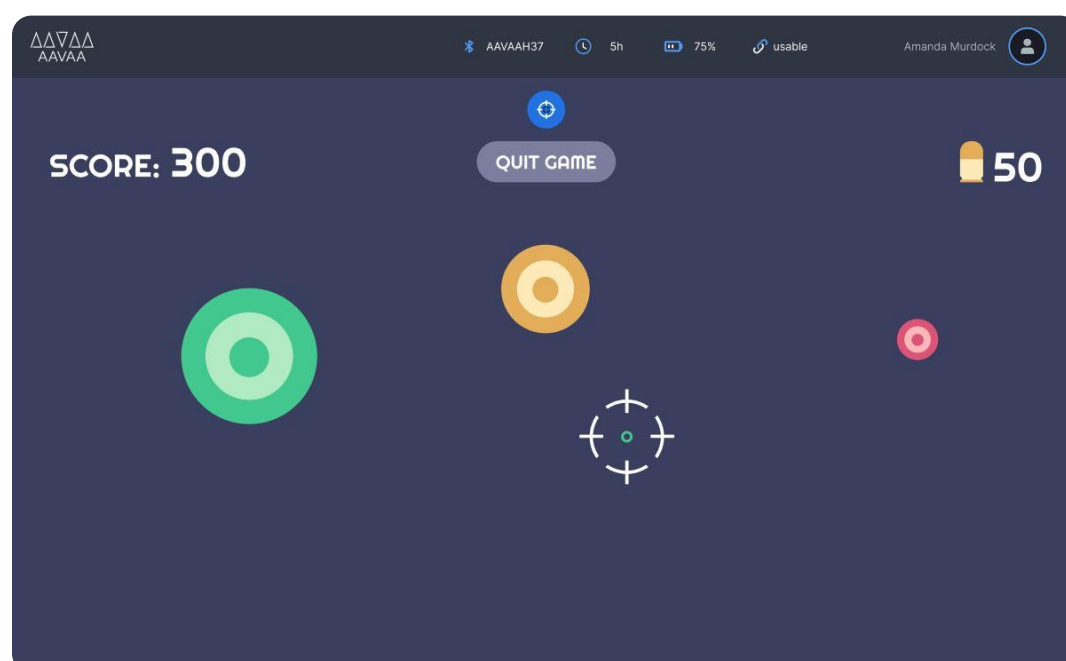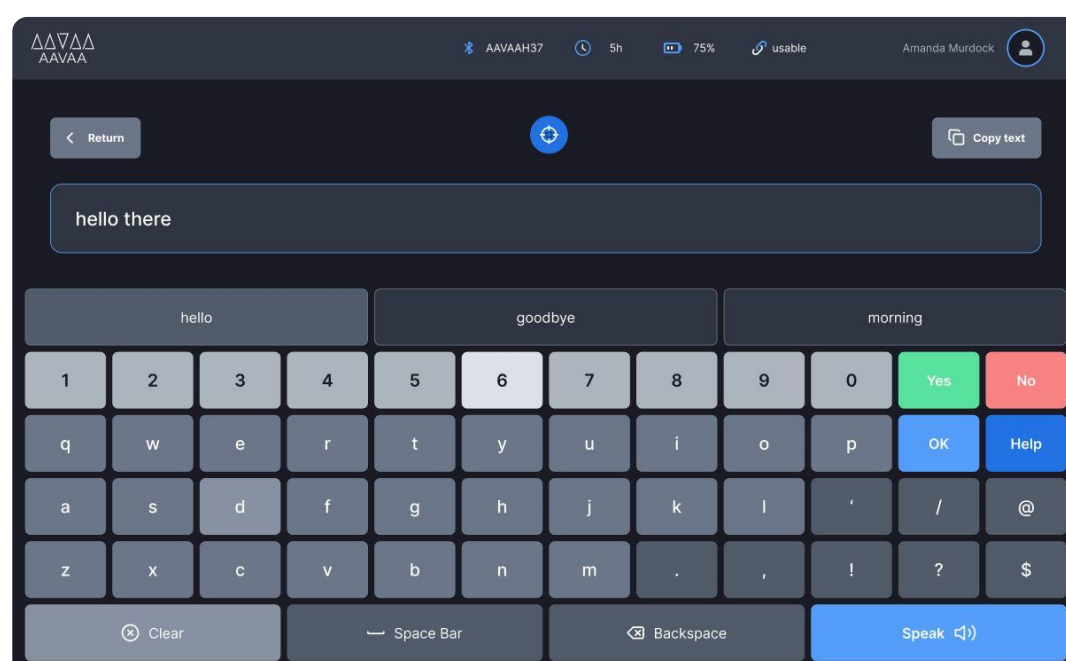

2. Functions & Settings

P To log-out or report a bug, click on the user icon and select between the two options.

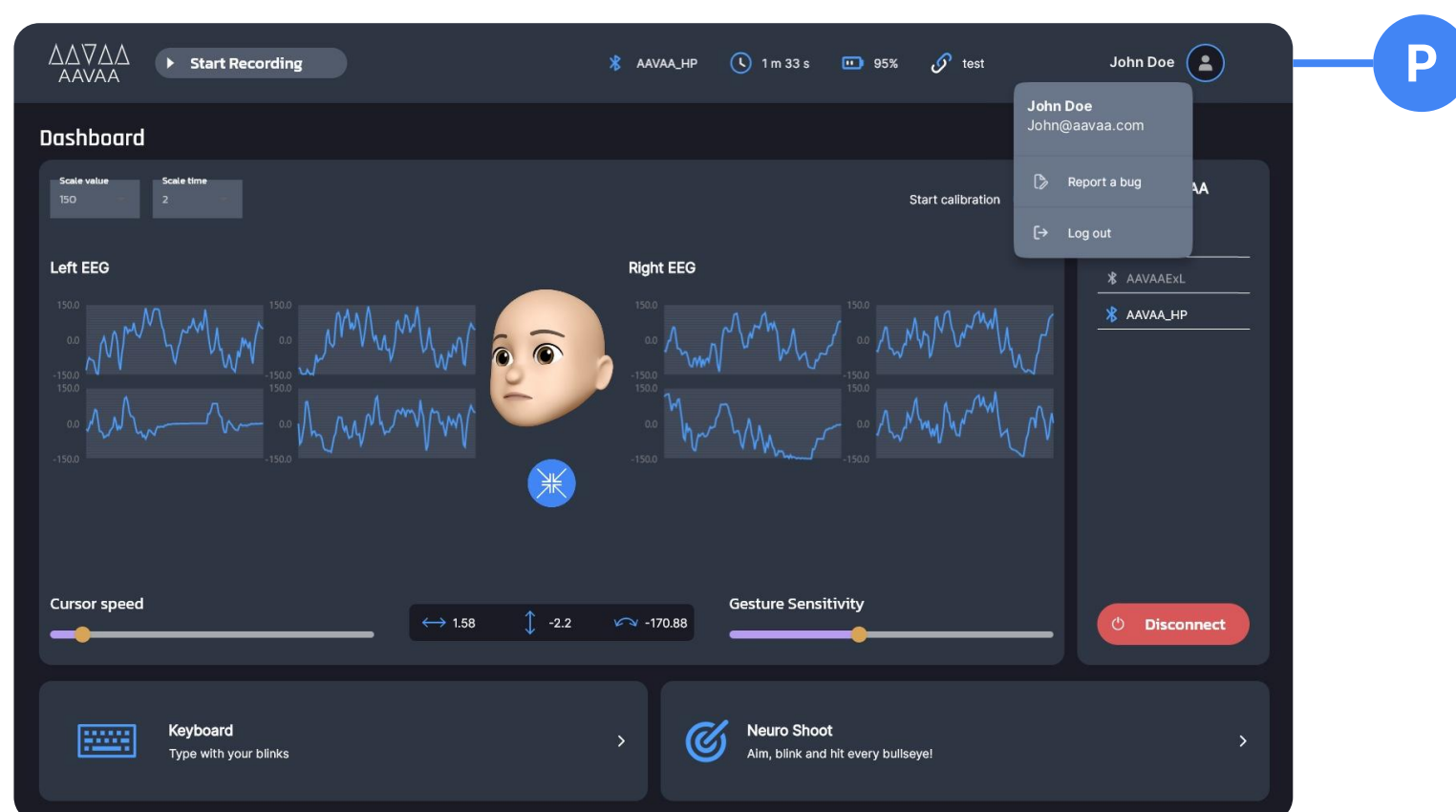



Visit us at AAVAA.COM
Or contact us at info@aavaa.com



info@aavaa.com
AAVAA.COM

(514)-524-2099