


720眼部护眼仪英文说明书

139MM

105MM

Instruction Manual

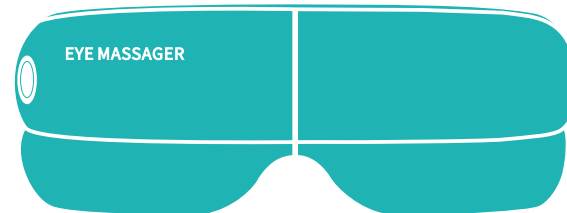

Shine Your Eyes

Stay true to your roots with light in your eyes


Safety Precautions

Please read carefully to understand all the instructions before use, and follow the instructions in use. Do not use the product when you have eye symptoms. Do not use the product if you had eye surgeries or have serious eye conditions such as glaucoma, cataract and retinal detachment.

- Clean the product with alcohol cotton. Do not use water.
- Do not disassemble the product by yourself.

- Pay attention to the surface of the product, which becomes hot in use, if you are insensitive to heat.
- The product is not suitable for those who are physically weak, slow in response or with mental disorders, unless with guidance or assistance from their guardians. The product is not designed for kids and the disabled to use alone. Keep the product out of the reach of the children.
- This is a massage product for personal health care. It can't replace the effects of medicines and medical appliances.
- Do not disassemble, crush and squeeze the lithium battery in the product, or put it into the fire. Stop using the product when the battery swells seriously.
- Do not use the battery after it is soaked in water.
- Do not place the product in an environment with high temperatures as it contains a lithium battery.
- Use the product under the doctor's guidance if you have sensitive skin, facial allergic dermatitis or plastic materials on your face.
- Remove contact lenses, hairpins and hair accessories, etc. before use.
- Do not use the product in hot and humid places.
- Avoid strong sunlight and gas corrosion.
- Do not use the product while walking or driving, as the product blocks vision.
- Charge the product with a mobile phone charger (5V/1A).
- Close your eyes in use.
- Do not use sharp objects to scratch and pierce the product.

Product Structure

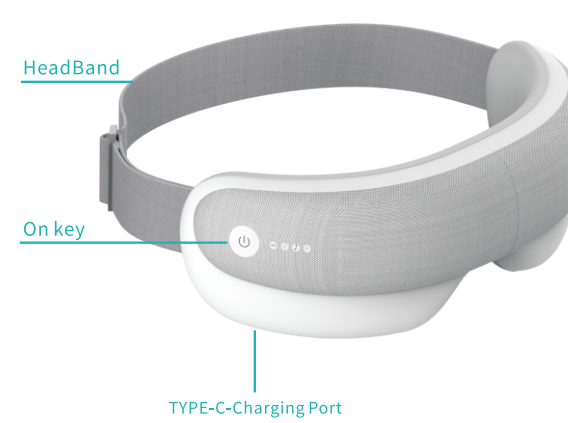

Product Characteristics

Voice Broadcast

180°Fold

Smart Air Pressure

Skin Friendly Leathe

Hot Compress

Music

Instructions

- Wearing the product**
Aim the product at your eyes, put it on, and adjust the elastic to the most comfortable position.
Adjust the main device to the most comfortable position by moving it up/down and left/right.
⚠ Attention
Remove glasses, contact lenses, hairpins, hair accessories, etc. before use to avoid discomfort. Do not use the product with wet hands.
Avoid dropping and treading on the product.
- Button Control**
Long press the power button for 2 seconds to power on and off.
Click the power button to switch to the next mode.
Double click the power on button to turn on the mute mode.
- Bluetooth music**
The Bluetooth is on by default. The product plays built-in music when turned on and will switch to music in the Bluetooth devices after connection. The product will switch back to the built-in music automatically when the Bluetooth disconnects.
⚠ Listening to music at a large volume for a long time may lead to hearing loss.

Before Use

- Fully charge the battery before use with a 5V mobile phone charger.
- Connect the mobile phone charger with a USB plug of the USB cable.
- Connect the main device with a micro USB plug, and plug the mobile phone charger into the socket.
- The battery indicator light flashes during charging and is on when the charging is finished.
- The product doesn't work during charging. It stops working automatically if charging starts in use.

Modes of Use

Default:Eye Protection Mode: Soothing hot compress mode.
One-touch Loop Mode:
There is a voice broadcast for the current mode after you press the mode button every time.
Each mode lasts for ten minutes by default. The product turns off automatically after ten minutes.
Soothing hot compress mode : Low Atmospheric Pressure+ Hot Compress+ Music
Strong hot compress mode : Barometric Pressure+ Hot Compress+ Music
Hot compress mode : Hot Pressure + Music
Soothing mode : Low Atmospheric Pressure+ Music
Power mode: Barometric Pressure + Music

Note: In any mode, double-click the button to enter silent mode.

Hazardous Substances in Products

Part Name	Hazardous substances						
	Lead Pb	Heavy Hg	Cadmium Cd	Hexavalent Chromium Cr(VI)	Polybrominated Biphenyl PBB	Polybrominated Diphenyl Ether PBDE	
Plastic Parts	○	○	○	○	○	○	
Circuit Board	×	○	○	○	○	○	
Pressure Control Component	○	○	○	○	○	○	
Battery Pack	×	○	○	○	○	○	
Straps	○	○	○	○	○	○	
Attachments	○	○	○	○	○	○	

○: the content of the hazardous substance in all homogeneous materials of the component is below the required limit according to GB/T 26572.
×: the content of the hazardous substance in at least one homogeneous material of the component exceeds the required limit according to GB/T 26572.

Cleaning and Storage

- Turn off the power before cleaning. Clean the surface of the product with a dry soft cloth.
- Cover the product with a dust shield (provided by yourself) when not in use for a long time.

⚠ Attention

Cut the power supply before cleaning. Do not use corrosive cleaners like benzene or diluent to clean the product, which may cause the product to fade and the structure to be corroded and cracked.
Do not expose the product to sunlight or high temperatures, which may cause the product to fade.

Common Problems and Solutions

Serial Number	Problems	Possible Causes	Suggested Solutions
1	Noises like "whew" and "da".	Normal working noises from the pump inflating/deflating, the motor spinning, the clothing cover rubbing, etc.	Unnecessary.
2	Automatic shutdown after startup.	The lithium battery is low.	Charge the battery.
3	The power button doesn't work.	The battery is not charged.	The charger may be broken.
4	Air pressure is too low.	The lithium battery is low.	Charge the battery.

- Turn off the product and contact the dealer or the company if problems still exist after conducting the suggested solutions.
- The company is not responsible for any loss or damage to products and people due to dismantling and repairing the product without authorization.


After-sales Service and Warranty

The product enjoys free maintenance for one year from the day it was purchased when the user follows this instruction manual.
Labor and material fees are covered by the free maintenance, but the delivery and door-to-door service fees are excluded.
Improper disposal of the product may lead to environmental pollution.

Charging Services

- Damage caused by improper use and storage from the users.
- Damage caused by non-appointed service outlets of the company.
- Product of which the warranty expires.

Battery Disassembly Diagram

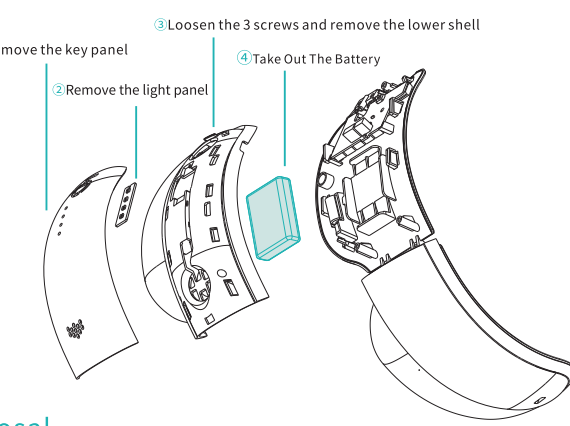

Disposal

- Remove the battery before discarding the product.
- Cut off the power supply before removing the battery.
- Do not change the battery without authorization.
- Use appropriate tools to remove the upper cover of the product, remove the battery and discard it in corresponding recycling facilities.

Transportation and Storage

Ambient temperature: -20℃ - 40℃
Relative humidity: less than 80% with good ventilation. Avoid corrosive gas, and is suitable for transportation by air, rail, road and ship. The product should avoid showers by rain and snow, and shouldn't be put upside down and crashed. Open the package and check the product when it has been stored for over three months and power on for half an hour.

Warranty Card (Warranty Record)

Full Name	Telephone	Address	Product Name	Date of Purchase

Date	Problems and Solutions	Remarks

Certificate

Product Model: _____

Product Specification: _____

Date of Manufacture: _____

Inspection Date: _____

Shine Your Eyes

Stay true to your roots with light in your eyes

1

2

3

4

5

6

7

8

FCC Warning Statement:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Caution: Any changes or modifications to this device not explicitly approved by manufacturer could void your authority to operate this equipment.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

The device has been evaluated to meet general RF exposure requirement.