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Photonic Eye Massager **ETA-100**

Manual

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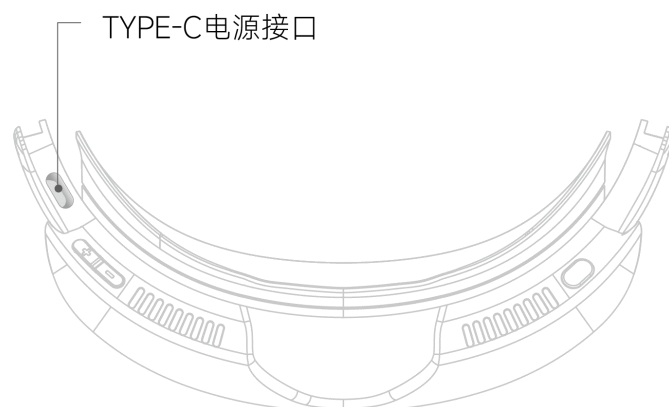
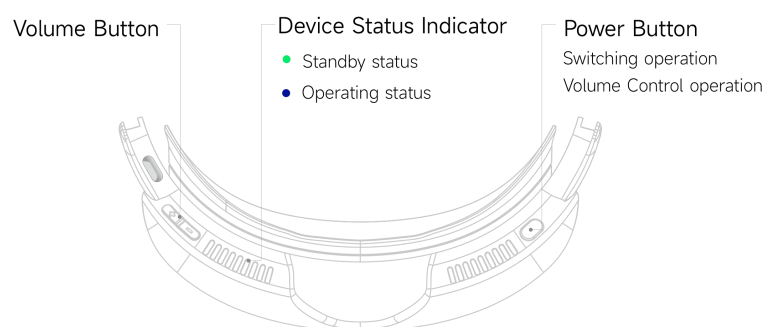
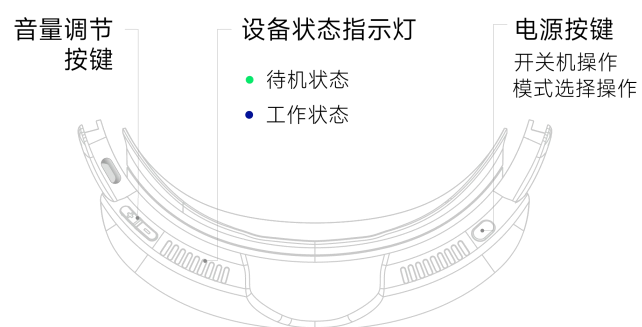
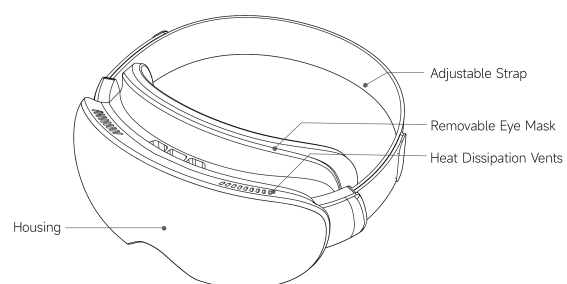
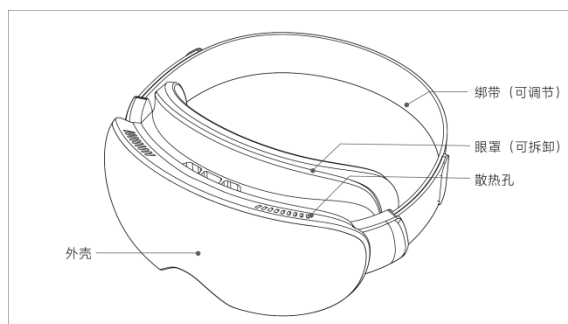
Thank you for choosing our Eye Light Therapy Device. To ensure optimal results, please carefully
read the instruction manual prior to use and retain it for future reference.

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Power On: Press and hold the power button for 3 seconds until the red indicator lights up.

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Auto Shutdown: The device powers off automatically after a 15-minute session.

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Manual Shutdown: Press and hold the power button for 3 seconds during operation/standby.

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Mode Selection

After Power On: Double-click the power button to cycle through 5 intensity levels (default: Level 3).

Switch Modes During Operation:

Press once to pause → Double-click to switch modes → Press once to resume.

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Pause/Interruption

During operation, press the power button once to pause or resume the session.

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Volume Control

Press “+” or “-” to adjust voice prompt volume.

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The device status indicator light signals the working status of the device.

- 绿灯常亮：设备处于待机状态 Constant green light: Standby state
- 蓝灯闪烁：设备正在运行 Flashing blue light: Working state.

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Precautions

- Before use, ensure the input power matches specifications (DC 9V/3A).
- Clean the eye area and remove makeup prior to use. Do not wear contact lenses or colored contacts during operation.
- Keep eyes closed during treatment. Avoid direct exposure to the light source.
- Discontinue use immediately and remove the device if discomfort occurs during operation.
- Do not exceed 2 sessions per day, with each session limited to 15 minutes.
- Operating temperature range: -5°C to 40°C; humidity must remain below 85%. Avoid prolonged use in high-temperature or humid environments.
- This product is not suitable for children under 12 years of age. Keep out of reach of children.
- If the power cord or device shows signs of damage or malfunction, cease use immediately and contact the authorized service center for inspection/repair. Do not attempt self-repair or modifications.
- Never disassemble the product. Unauthorized tampering may cause device damage or electric shock hazards.

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Maintenance and Care

- To ensure optimal performance, regularly wipe the main unit and remove the eye pads for periodic cleaning.
- Before cleaning or maintenance, confirm the power plug is unplugged and the device is powered off.
- Never immerse the power cord or main unit in water or rinse them under a tap.
- Clean the surface with a neutral detergent. The detachable eye pads may be washed separately. Avoid using acidic/alkaline or corrosive cleaners to prevent product damage.
- After cleaning, ensure the product is completely dry before reconnecting to the power supply.

- If the product will not be used for an extended period, store it in a cool, dry place and avoid dropping the device to maintain functionality.

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Warranty Card

For warranty service, fill in the following information and return with the product:

User Name: _____

Address: _____

Contact Person: _____

Phone: _____

Purchase Date: _____

Model: _____

Serial Number: _____

(Serial number located on the bottom of the packaging box, SN)

Warranty Coverage

1. Covers malfunctions under normal use during the warranty period.
2. Warranty void if:
 - 1) Warranty card and serial number mismatch.
 - 2) Damage from improper storage, maintenance, or misuse.
 - 3) Malfunctions caused by force majeure.
 - 4) User disassembly or unauthorized repairs.
 - 5) Expired warranty period.
3. Other terms comply with national regulations.

Mailing Address

After-sales Department, 3rd Floor, GAOXINQIKEJILUO Building, Block 67, Xingdong
community, Xin'an street, Bao'an District, Shenzhen, P.R. China
Contact: 4000078965

Federal Communications Commission (FCC) Statement.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

This device may not cause harmful interference, and
this device must accept any interference received,
including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide Reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Warning: Changes or modifications made to this device not expressly approved by Shenzhen Pinchuan Vision Health Technology Co., Ltd. may void the FCC authorization to operate this device. Note: The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user's authority to operate the equipment.

RF exposure statement:

This device complies with FCC RF radiation exposure limits set forth for an uncontrolled environment. The device is installed and operated without restriction.