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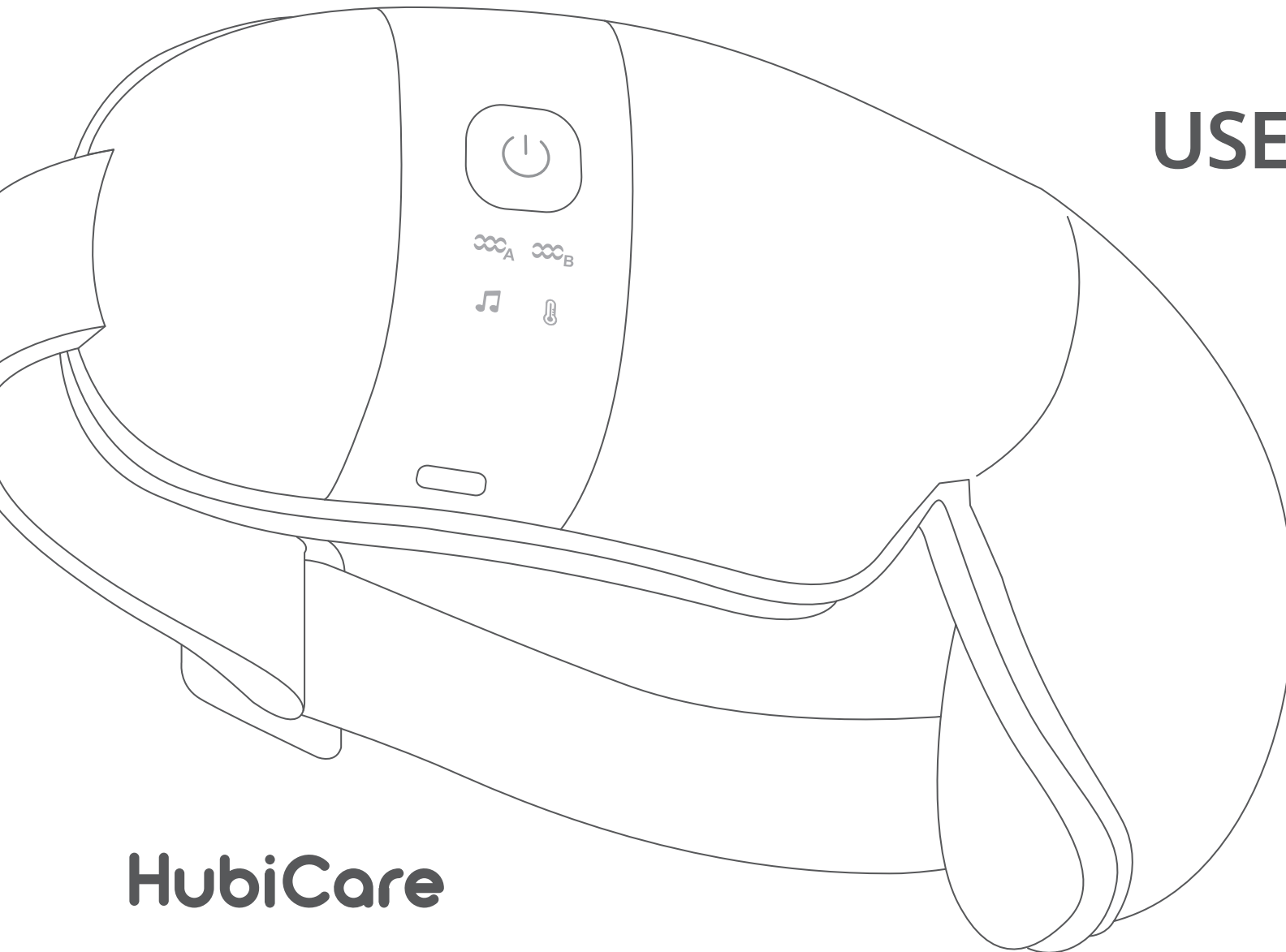
Please read before first use

This manual provides detailed instructions on the proper usage and care for **HubiCare** Premium Double Cushion Eye Massager (HEM-7)

Contents

- 1. Product introduction

- 2. Package Contents and Parts Description

- 3. How to use

- 4. Maintenance and Storage

- 5. Safety Precautions

- 6. Troubleshooting

- 7. Technical Specifications

1. Product Introduction

Thank you for purchasing the HubiCare Premium Double Cushion Eye Massager. This device is designed to provide 15 minutes of soothing eye relaxation and fatigue relief through five core functions : Gentle vibration, Air pressure massage, Heat compression, Soothing music, A removable COOL GEL PAD for refreshing cooling therapy. The Double Layer Eye Cushion ensures a comfortable and secure fit, while the ultra-low noise design allows for a quiet and peaceful experience.

Recommended for

- People who use computers for extended periods.
- Frequent smartphone users.
- Night-time drivers.
- Avid readers.
- Children attending remote classes via computers or laptops.
- Anyone seeking relaxation after a long day.
- Those who want to fall asleep comfortably.


Product Features

- Double Layer Eye Cushion
- Low noise (average $\leq 45\text{dB}$)
- 5 functional modes
- Optimized heat temperature
- High-capacity rechargeable battery
- Air pressure massage
- Cooling massage with removable gel pad

2. Package Contents and Parts Description

- **Package Contents** | Eye Massager, Storage Pouch, USB Type-C Charging Cable, User Manual, Removable COOL GEL PAD

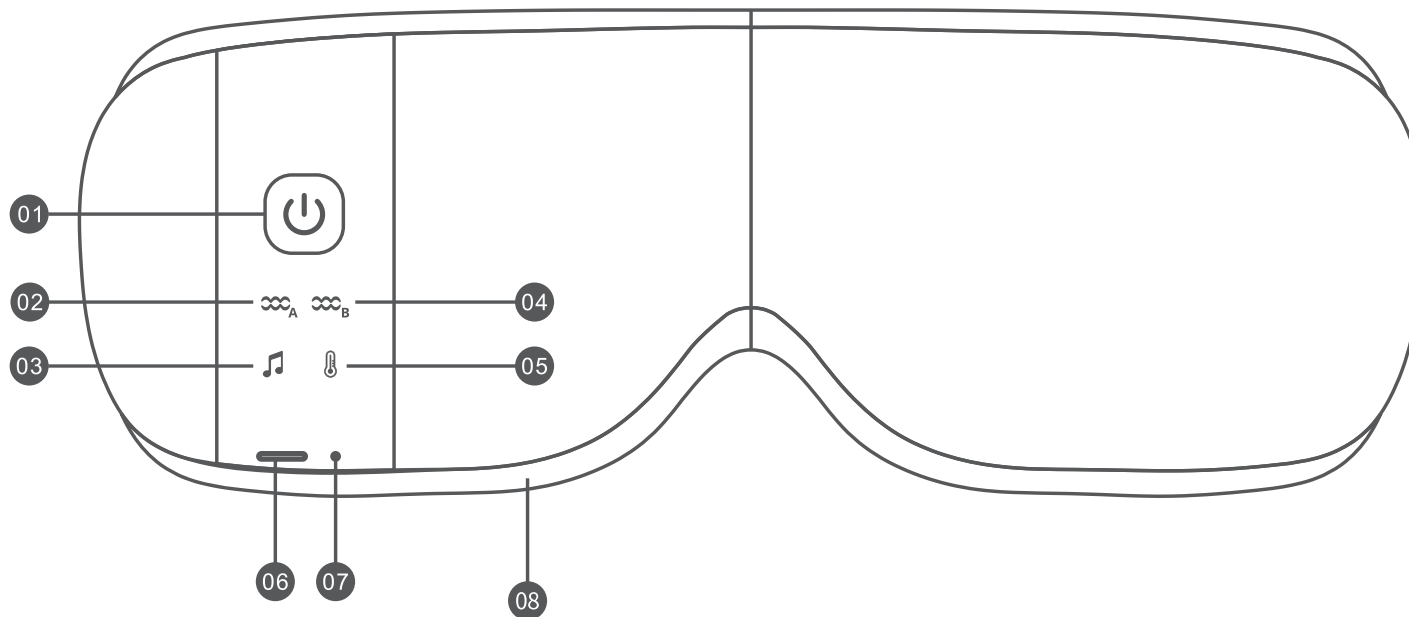- **Parts Description**

- 01 Power / Mode Switch Button
- 02 Air Pressure A Indicator
(Smart Mode)
- 03 Music Indicator
- 04 Air Pressure B Indicator
(Dynamic Mode)
- 05 Heat Indicator
- 06 USB Type-C Charging Port
- 07 Charging Indicator
- 08 Double Layer Eye Cushion

3. How to Use

■ Charging the Device

- 01 Connect the USB Type-C charging cable to the charging port located at the bottom of the eye massager.
- 02 The charging indicator will turn red during charging.
- 03 When fully charged, the indicator will turn blue.
- 04 Disconnect the charging cable after charging is complete. (A full charge takes approximately 2–3 hours, depending on the voltage.)
- 05 The device cannot be used while charging.

- Charging Indicator LED Colors: Charging – Red, Fully Charged – Blue
- Note: Fully charge the eye massager before first use.

■ Using the Eye Massager

- 01 Wear the eye massager properly over your eyes.
- 02 Turn on the device by pressing and holding the power button for more than 2 seconds.
A voice prompt will play, and the device will power on in the default Air Pressure + Heat mode.
- 03 To switch modes, press the power button briefly once.
- 04 The device will automatically power off after 15 minutes.
- 05 To turn off the device manually or restart it, press and hold the power button for 2 seconds.

3. How to Use

■ Massage Modes

Soothing mode : Air Pressure A + Heat compress + Music 1 ∞_A + 🌡 + 🎵

Cold compress mode: Air Pressure B + Music 2 (COOL GEL PAD usable) ∞_B + 🎵

Hot compress mode : Heat compress + Music 3 🌡 + 🎵

Sleep mode: Heat compress + Music 4 🌡 + 🎵

Vitality mode: Air Pressure B + Heat compress + Music 5 ∞_B + 🌡 + 🎵

■ Music Control

- Toggle music on/off by pressing the power button twice quickly during use.

■ Session Duration

- Each session lasts 15 minutes.
- To continue, press and hold the power button for 2 seconds to restart the device.
- When the battery is low, a voice prompt will notify you. Please recharge the device for continued use.

■ Bluetooth Function

- The Bluetooth name of this product is "HEM-7"
- Pair it with your device during the first use.
- Once paired, the eye massager will automatically reconnect when Bluetooth is enabled on your device.

Voice Prompts Include:

- "Bluetooth is connected"
- "Bluetooth is disconnected"

3. How to Use

■ Voice Prompts

-Welcome	-Bluetooth connected	-Soothing mode	-Sleep mode
-Music on	-Bluetooth disconnected	-Cold compress mode	-Vitality mode
-Music off	-Massage finished	-Hot compress mode	

■ How to Use the COOL GEL PAD

Refrigerate the COOL GEL PAD for 30 minutes.

Then, hook the straps onto both sides of the eye massager and wear it over your eyes.

After use, remove the gel pad and store it at room temperature.

Cautions for COOL GEL PAD

- Do not puncture or ingest the COOL GEL PAD
- Keep out of reach of children
- Store in a cool, dry place
- Do not microwave or wash with water
- Do not freeze
- Clean with a damp cloth

4. Maintenance and Storage

■ Storage

- 01 Store the eye massager in a well-ventilated area
- 02 Avoid direct sunlight, high humidity, or extreme temperatures.

■ Maintenance

- 01 Turn off the massager and disconnect the charging cable before cleaning.
- 02 Use a dry, soft cloth to gently wipe the surface.
- 03 Do not use any cleaning agents or harsh chemicals.
- 04 For stains, use a slightly damp tissue to gently clean the area.
- 05 After cleaning, make sure the device is completely dry before storing or using it again.
- 06 For the COOL GEL PAD, clean it gently with a damp cloth.

5. Safety Precautions

- 01 This product is intended for personal use only and is not designed to replace or serve as a medical device for treatment purposes.
- 02 Remove all contact lenses and accessories before use.
- 03 Close your eyes during usage. A slight blur may occur shortly after use. Open your eyes slowly after usage.
- 04 Do not use the device while walking or driving.
- 05 This product includes a heat function. The temperature is close to body temperature (99°F) and is not intended for therapeutic heat purposes.
- 06 Do not use it on infants or children.
- 07 Do not use it after consuming alcohol or while sleeping.
- 08 Do not use if you have a scar near the eyes, have had eye surgery, or suffer from any eye conditions such as glaucoma, cataracts, or retinal detachment. Consult with your doctor before use.
- 09 This product is equipped with a timer and will automatically turn off after each session. Continuous usage of the product may strain your eyes.
- 10 Stop immediately if you experience any pain or discomfort during use.
- 11 Do not use this product in a high-temperature or high-humidity environment.
- 12 This is not a medical device but a personal health care product and should not be used for treatment purposes.

5. Safety Precautions

■ COOL GEL PAD Precautions

- 01 Do not puncture or ingest the COOL GEL PAD.
- 02 Keep out of reach of children.
- 03 Store in a cool, dry place.
- 04 Do not microwave the COOL GEL PAD, nor wash it with water.
- 05 Do not freeze the COOL GEL PAD.
- 06 Clean the COOL GEL PAD with a damp cloth.

6. Troubleshooting

Issue	Possible cause	Solution
Power does not turn on	No battery	Charge the massager
	Power button was not pressed long enough	Press the power button for at least 3 seconds
Air pressure is weak	Low battery	Charge the massager
Loud noise during usage	Issue with the motor	If the noise increases, please contact the seller
Will not charge	Charging cable is not properly connected	Properly connect the cable
	Charging cable may be damaged	Replace with a new charging cable

7. Technical Specifications

Product	Premium Double Cushion Eye Massager	Size	L230xW105xH92 (mm)
Model	IFAN-E20H /HEM-7	Weight	278g
USB	Micro USB C-type	Battery	3.7V, 1500mAH
Voltage	DC5V/1A	Operating time	15 minutes
Charging time	about 2~3 hours	Power Consumption	5W
Product	COOL GEL PAD HEM-GEL	Size	L300×W100×8 (mm)
Material	TPU laminate Ice Silk Lycra	Weight	100g

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Manufacturer : SHEN ZHEN FANR TECHNOLOGY CO.,LIMITED

US Importer : HUBDIC USA INC. Carson, CA 90746, United States of America

Customer Service Center : support@hubdic.com

WEEE-Reg-Nr. DE 13009330

EU	REP
----	-----

eucomply OÜ

Pärnu mnt. 139b - 14

11317 Tallinn Estonia

hello@eucompliancepartner.com

+33757690241

FCC Caution:

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

The device has been evaluated to meet general RF exposure requirement. The device can be used exposure condition without restriction.

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Seller : HuBDIC Co.,Ltd.