

图纸编号：IFAN-R30说明书-01

尺寸：120*70g


材质：128g双铜



IFAN-R30

Noise-Reducing Eye sleep Aid Massager

User's Manual

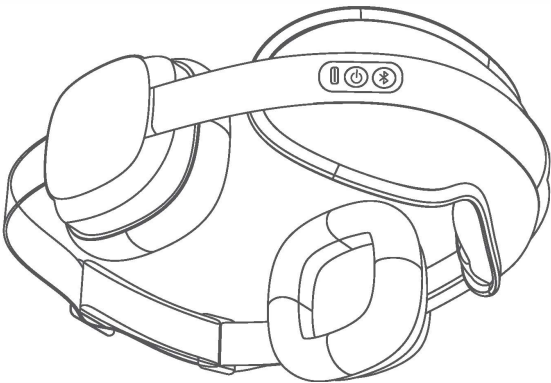

Introduction

Dear user:
Hello!
Welcome to use our products, in order for you to more fully experience our products, please be sure to carefully read and save this user guide.

- 01 -

Matters Needing Attention

 Danger Warning

Do not use this product under any of the following circumstances:
a. For contact lens wearers, remove contact lenses before use.
b. Patients in the rehabilitation stage after eye surgery.
c. Patients who are undergoing treatment and feel abnormal.
d. Patients with hypertension, ocular hypertension, arteriosclerosis or cerebral thrombosis.
e. Patients with high fever, malignant tumors and osteoporosis.
f. Patients with infectious diseases.

- 02 -

Matters Needing Attention

 Attention

01. Be sure to remove the plug from the socket before using or cleaning the product.
02. Please keep this product away from water, fire and corrosive environment.
03. If the product accidentally falls into the water while charging, please pull out the plug immediately and do not directly touch the product that falls into the water.
04. Do not use this product in humid, high temperature, high electromagnetic environment.
05. This product contains lithium batteries. Please do not disassemble, modify or throw them into the fire to avoid danger.
06. The product has a hot compress function and the appliance has a hot surface, which must be paid attention to when used by people who are not sensitive to heat.
07. Do not use this product continuously for a long time, otherwise it may cause fatigue and discomfort on the massage site.
08. This product is not intended for use by any sick or incapacitated person.
09. Do not use this product with children or persons with limited ability without adult supervision.
10. This product is a household electronic product, not as a medical device, not suitable for diagnosis, treatment and other commercial purposes.

- 03 -


Packing Instructions

Before using the product, please check the integrity of the internal accessories. In case of missing or damaged items, please contact the seller and replace the new items.

The list of contents is as follows:
01. IFAN-R30 host *1
02. Type-C charging cable *1
03. Instructions for use *1

- 04 -

Function Description

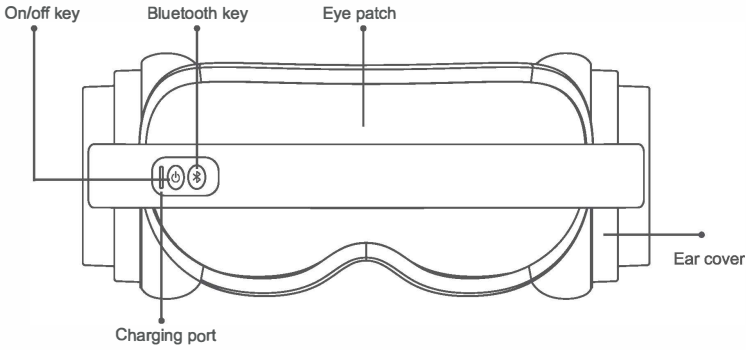

- 05 -

Functional Operation

 On/off key:
Turn on/Off: Long press the power button to turn on the device for 1.5s. After turning on the device, it will enter [Low Temperature] by default. The indicator will light up green and a voice announcement will be made saying "Turn on heating". Press and hold the power button again to shut down.
Switch temperature gear: Press the power button to switch the temperature gear, and the mode switching sequence is as follows: [low temperature] - [high temperature] - [Turn off heating], corresponding to the voice broadcast.
Double-click the key: enter [music mode], corresponding to the voice broadcast. (Enter this mode Bluetooth is on, no working time limit, free listening to songs, the default is no heating, you can short press the power button to turn on the heating, respectively: low - high - off, do not display the corresponding light, only broadcast the corresponding voice. Double-click the key again to exit the music mode and there is a corresponding voice broadcast).
● Startup status: Temperature Low: Steady orange ● Charging status: Charging: Red breathing light
High temperature: Steady red Full: Steady green
Turn off heat: Steady green
Music mode: Blue ice steady on

- 06 -

Functional Operation

 Bluetooth key:
When Bluetooth is not connected, the built-in music mode (blue light is on);
Long press -- Search for a new Bluetooth device (stop playing music, Bluetooth light flashes blue, voice announcement "Search for a new Bluetooth device"); Bluetooth music mode (Bluetooth light is blue on);
Bluetooth name: IFAN-R30;
Short press -- play the next song;
Double click -- Turn music off/on;
Long press -- Disconnect the currently connected Bluetooth device and search for a new Bluetooth device (stop music playing, Bluetooth light blinks blue, and voice broadcast "Disconnect the current Bluetooth device and search for a new device");
Connect mobile Bluetooth:
Android operation procedure:
1. Turn on the Bluetooth port on your phone.
2. In the Other Devices list, select IFAN-R30.
3. The Bluetooth Pairing Request page is displayed.
4. Click PAIR, The connection is successful.
Apple's steps
1. Turn on Bluetooth.
2. Select IFAN-R30 from the OTHER DEVICES list, the connection is successful.

- 07 -

Preparation Before Use

Please confirm whether the product has power, if the power is too low, the machine function is turned off or can not be turned on, please charge before use. Be sure to unplug the power adapter when the battery is fully charged.
Charging
1. Connect the USB charging cable to the charging port.
2. When the machine is in the charging state, the red breath light of the indicator light is lit, and the green light is lit long.
3. During the charging process, the product does not support startup work, and the product starts the self-protection program and stops working.
4. The host adopts built-in rechargeable battery, charging time ≤3.5 hours.
5. In the process of use, the voice prompts a low battery alarm, please suspend the use, and use it after full charge.
Multiple Charging Methods
Computer USB port/Mobile power supply /5V==1A power adapter
Note: The default working time is 15 minutes. If you want to extend the working time, you need to start the host again.

- 08 -

Preparation Before Use

Atmospheric pressure: 86kpa~106kpa
Ambient temperature: -10℃-65℃
Relative humidity: <80%RH, no corrosive gases and well-ventilated room.
Transportation: This product is equipped with simple shock-proof facilities, suitable for air, rail, road and ship transportation, should avoid rain, snow splash, inversion and collision. When the storage period exceeds 6 months, the box should be opened for inspection, and the power should be energized for half an hour, and it should not be close to the ground, walls and roof.
The room should be well ventilated to avoid strong sunlight and corrosive gas.

- 09 -

Product Information

Product name: Noise-Reducing Eye sleep Aid Massager
Product model: IFAN-R30
Rated input: 5V == 1A
Rated power: 5W
Battery capacity: 1200mAh
Charging interface: Type-C
Rated time: 15 minutes
Product size: 195*125*93mm

- 10 -

Troubleshooting and Maintenance

1. If a fault occurs during operation, perform the following check.
2. The basic fault of the product can not be solved after trouble shooting, please call after-sales service or pay attention to the public number to consult customer service staff. Do not disassemble this product yourself.

Basic Failure	Possible causes	Troubleshooting methods
Unable to boot	The power key is not turned on	Long press the power key for 1.5 seconds
	The product battery is low	Charge the product
	At the set time	Press the power key again
No charging	The USB cable is not plugged in properly	Re-plug the USB cable
	Don't use the original charge	Replace with the charging cable of the product
No music	Music Paused status	Press the music play button in the electronic device; Or double-click the Bluetooth key to open the music machine
Bluetooth connection failed	Matching of electronic device and massager distance is beyond the limit	Shorten the distance of electronic devices and machines
	The Bluetooth key is mistakenly pressed and the Bluetooth function is turned off	Press the power button to shut down, and then restart the Bluetooth function of the massager
Bluetooth has successfully connected, still can not play	Not connected to the corresponding Bluetooth	Turn on the Bluetooth device of the electronic device again and perform Bluetooth pairing again
	Bluetooth playback music sound is low	Adjust the volume of your electronic device
The use time is short	Non normal temperature	Please use in the ambient temperature of 10℃-35℃

- 11 -

FCC Caution:

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:
-Reorient or relocate the receiving antenna.
-Increase the separation between the equipment and receiver.
-Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
-Consult the dealer or an experienced radio/TV technician for help.

The device has been evaluated to meet general RF exposure requirement. The device can be used exposure condition without restriction.