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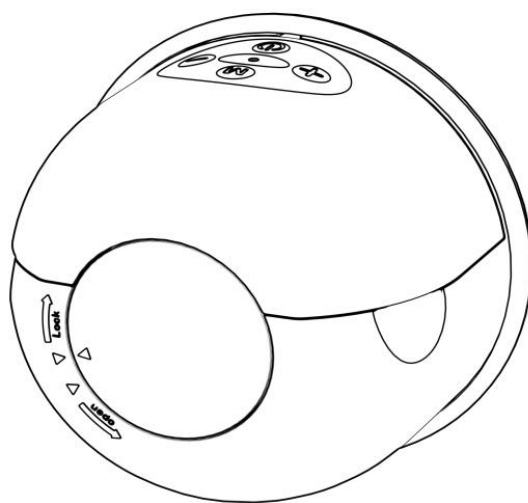
The best gift to a new mom. A perfect appliance of breast feeding

Mini Wearable Breast Pump (Model No. YY-A46)

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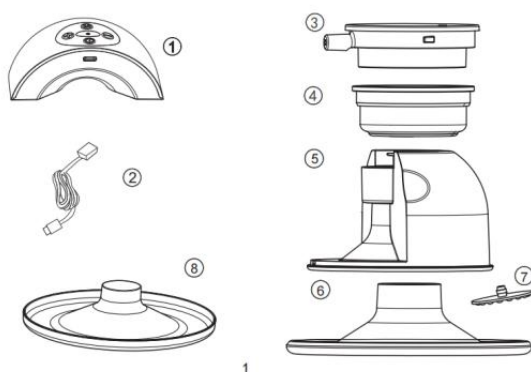
Congratulations on your new baby! It's our pleasure to offer you products and services. Please read this manual before use and keep it carefully for your future maintenance and care.

Introduction

Breast milk contains all the nutrients and antibodies the baby needs to grow up, and every mother wants her baby to enjoy the most natural, safest, and most nutritious natural food. Starting from the heart, Kidro strives to create the most suitable breastfeeding products for Chinese mothers, bring them the best breastfeeding experience, and help mothers adhere to breastfeeding.

Kidro electric breast pump simulates the baby's sucking rhythm, with six gears to adjust, allowing mothers to enjoy every breast-sucking time. The product is small, easy to carry and operate, built-in rechargeable battery, USB-Type C charging interface, and with intelligent APP to facilitate product operation.

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No.	Name	Material	Can be sterilized at high temperature
1	Pump Unit	ABS	×
2	USB Cable		×
3	Diaphragm Cap	PP	√
4	Diaphragm	Liquid silicone	√
5	Milk Bottle	PP	√

6	Breast Shield	Liquid silicone	√
7	Valve plate	Silicone	√
8	Dust Cover	PP	√ (Can without sterilization)

Cleaning and Sterilizing

① Cleaning

Separate all parts that can be sterilized at high temperature and clean them with a safety cleaner.

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3

② Sterilizing

- Soak all parts that come in contact with your breast and breast milk, and sterilize for at least 5 minutes.
- Can sterilize 1-2 times a day. For relatively frequent breast pumping (Performed once every 2-3 hours), not recommended to sterilize after each use. (Excessive sterilization may cause premature aging of the parts and reduce the service life of the breast pump) Recommend thoroughly cleaning and sterilizing the milk stains once a day.
- Before sucking breast milk, all breast suction accessories must be sterilized to prevent breast milk from deteriorating.

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Install the valve plate on the milk bottle and make sure they are fixed securely.

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Install diaphragm cap with the diaphragm into the milk bottle, and screw the diaphragm cap clockwise.

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Insert the breast shield into the milk bottle and snap it into the rim of the bottle to prevent spillage

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Put the pump unit into the milk bottle and confirm whether the product is flat

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Sterilize breast pump components according to the "Installation Guide".

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Make sure the product is fully charged to avoid lacking power during the use of the product.

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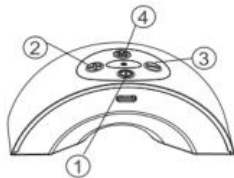
Choose a comfortable seat, relax and lean forward slightly.

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② Pump Controls and Display (操作面板及显示说明)

App 初始界面 App initial interface

查找设备 Find a Device

App 工作界面 App working interface

查找设备 Find a Device

Power Button: Press the POWER BUTTON for about 2 seconds to turn the unit on. (After the power is on, the product is in a suspended state.)

Need to touch the button again or touch the start button on the APP to turn on the working state of the product. (APP's shutdown button, can not completely turn off the host)

Suction Increase: Increase suction force

Suction Decrease: Reduce suction force

Mode Switch: Switch between massage mode and expression mode

Kidro (APP): Click "Find Device" on the initial interface of the APP (Kidro), automatically connect the first device (Kidro-L), and click "Find Device" on the work interface, automatically connect the second device (Kidro-R)

Pause/work indicator: Press this button on APP to pause or start the product

Massage mode indicator: Purple light indicates the mode (Suction level: 1-6)

Expression mode indicator: Blue light indicates the mode (Suction level: 1-12)

State indicator: Green light indicates the normal power-on status

Power indicator: Yellow light indicates the pump is being charged. (Green light when full of power)

Battery level indicator: Flashing red light means battery is low

Pumping with Kidro

- ① Place the breast pumps onto your breasts. Make sure that you center your nipples.
- ② Turn on the Kidro by holding the power button. When turning on, massage mode will begin. Use the increase and decrease buttons on pump controls or APP to find a comfortable suction level.
- ③ After 2 minutes, expression mode will automatically begin. (Press mode switch button will end massage mode early and enter expression mode) Again, use the

increase and decrease buttons on pump controls or APP to find a comfortable suction level.

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5

④ APP can not directly turn off the breast pump. After breast pumping, touch the power button ⏻ for about 2 seconds to turn off the unit.

(Doctor's advice: to protect your breasts, the best time for a single breast pumping time is no more than 20 minutes, the product will automatically shut down after a single working time of more than 30 minutes.)

Note:

Kidro has a memory function, after turning off the product and then turning it on again to use will remember the previous use of the gear.

Advice:

- If you can't suck breast milk, please don't keep forcing the breast pump for more than 5 minutes, try again at another time.
- Generally, it takes about 10 minutes to pump out 60-125ml of breast milk, but it will vary from person to person. Please use a larger milk bottle if more than 150ml of breast milk is pumped out at one time.
- Do not fill the milk bottle too full.

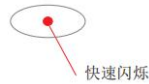
Power supply

- ① Use the charging adapter (5V ,1A)

- ②Build-in lithium battery(can be used for about 80 minutes at full power)
- ③Can be charged by power bank
- ④Use a USB cable to connect a computer for charging

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Battery Power Exhausts

Fast Flashing

When battery power is on and the pump is on any level of massage or expression mode, the red light flashes quickly and indicates that battery power is exhausted and needs to be charged.

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Being Charged

Slow Flashing

When power adapter is connected with pump, LED will light up in yellow until battery is fully charged.

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Fully Charged

Not Flash

When fully charged, the battery indicator will stop flashing and LED turns to green.

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*The above only applies for infants one month of older. Please consult a doctor for more details.

Warning:

- Fresh breast milk should be stored in refrigerator or freezer.
- Be sure to date the milk for storage. Milk should be stored in a refrigerator for no longer than 48 hours and the freezer for no longer than 3 months.
- When freezing, maximum milk volume should be 3/4 of storage bottle. Milk can expand during freezing.
- Do not store breast milk on the storage compartment on the inside of the cold room door of the refrigerator.
- Do not refreeze thawed milk.

Feeding Breast Milk

① Using warm water:

If the milk is refrigerated, put the milk container in warm water to heat the milk. Shake the container to warm the milk through.

If the milk is frozen, defrost in the refrigerator 24 hours prior to using. Once defrosted, warm it as refrigerated milk.

② Using milk heater:

Turn on milk heater and set it to 40°C. Put milk container into heater.

*Layers of liquid may be for when using frozen milk. This is normal. Please shake the bottle well before feeding.

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7

Warning:

- Do not heat thawed milk again.
- Do not use microwave oven to defrost breast milk.
- Do not heat milk with boiling water.
- If hot water is used to heat milk, please try milk temperature before feeding.

Troubleshooting

Problem	Possible causes	Solution
Pump does not work	Empty battery	Recharge battery
Low suction force	Air leakage at connection positions (Valve plate, diaphragm, breast shield)	Check all connecting parts to make sure they are installed properly
	The output air path of breast pump is blocked.	Straighten or clean air tubing
	No cause found	Please contact National Customer Service Tel: 400-666-8841
No suction	Valve plate is not installed or damaged due to deformation	Reinstall or replace valve plate
	No cause found	Please contact National Customer Service Tel: 400-666-8841

Warning:

- After touching the power button for about 2 minutes, this product automatically changes from massage mode to expression mode.
- The product will automatically shut down after a single working time of more than 30 minutes.

(Doctor's advice: to protect your breasts, the best time for a single breast pumping time is no more than 20 minutes)

- When sucking milk, the body leans slightly down to ensure that the milk flows smoothly into the bottle.

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The stage of using the breast pump

① If breastfeeding is going very well, it is recommended to wait until a regular pattern of breast milk secretion and breast-feed time has developed (unless specifically requested, usually started at least 2 to 4 weeks after the baby's birth).

② In the following cases, you can use it at any time:

① Unable to breastfeed immediately, regular breast pumping will help breast milk

secretion.

- ② To breastfeed baby in the hospital.
- ③ Distension and pain of the breasts. (You can suck out part of the breast milk between feeding to relieve the pain, which can also make the baby suck easier).
- ④ If the nipples are sore or cracked and unable to feed the baby directly, you can suck out the breast milk until the nipples return to normal.
- ⑤ Temporarily separated from the baby and want to continue breastfeeding in the future. Can suck out the breast milk regularly to keep the breast milk secretion.

Tips

- Choose a time of day when you are less likely to be disturbed and relaxed.
 - Keep the baby by your side or putting a picture of the baby while using the breast pump.
 - Put a hot towel on the breast for a while before starting to pump.
 - Suckle after a bath when your body is warm and relaxed.
 - Use a breast pump on the other breast while the baby is breastfeeding, or follow up with a breast pump to suck out the remaining milk after the baby has finished.
 - Use a breast pump on the other breast while your baby is breastfeeding or follow up with a breast pump to suck out the remaining milk after your baby has finished.
 - Change the position of the breast pump on the breast from time to time to stimulate milk secretion.
- * Practice using the breast pump until you find what works best for you, but if using the breast pump causes you pain, stop immediately and consult the doctor.

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Warnings

1. This product is a precision electronic appliance. Please do not use it during bathing.

Please keep it in a cool, dry and ventilated place, avoid direct sunlight, and keep it out of the reach of children.

2. Unplug the power adapter after use.
3. This product is for personal use only. Do not use it between more than one person to avoid cross infection.
4. Do not use a breast pump during pregnancy, because early breast milk expression will stimulate uterine contraction which may lead to abortion.
5. Do not forcibly remove breast shield from breast. Please power off and gently release the seal between the breast and the breast shield, and then remove breast shield from the breast.
6. When disassembling or assembling the pump, make sure to assemble each part according to the installation Guide. When disassembling and cleaning, please be careful not to misplace/lose the small parts.
7. Set the suction level according to individual physique to ensure comfortable milk expression.
8. Please clean with water and disinfect before use.
9. Doctor's advice: to protect your breasts, the best time for a single breast pumping time is no more than 20 minutes.
10. Please disassemble the product parts and disinfect them, and let them cool for about 20 minutes before assembling them for use.

Technical Information

Product Name	Mini Wearable Breast Pump
Model Number	YY-A46
Power Input	AC100-240V, 50-60HZ, 0.2A
Power Output	5V-1A
Rated Power	3.5W
Battery Capacity	3.7V/1000mAh
Executive Standard	GB 4806.7-2016,GB 4806.2-2015,GB 4806.11-2016

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Warranty

1. Warranty scope:

If the following problems occur in the use of this product during the warranty period, the company will provide a free warranty:

1. Due to the quality problems of the product itself, such as appearance damage found out of the box, etc.
2. Due to the quality problems of the product itself in the course of normal use.

2. Warranty period:

The product warranty period is one year from the date of purchase (If the purchase invoice is lost, the warranty period time from the date of departure.)

3. Expired warranty scope service:

If the product breaks down during the warranty period due to the following reasons, our company will repair it, but we need to charge a material fee:

1. All man-made damage that occurs not in accordance with the instructions.
2. The user repairs or modifies this product without authorization.

4. Expired warranty period service:

If the product breaks down after the warranty period, we will repair it and charge the maintenance and material fee.

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GUANGZHOU YONGYI INDUSTRIAL CO., LTD

Add: No.4, Majun Rd., Dongchong Town, Nansha District, Guangzhou, Guangdong, China.

National Service Hotline: 400-666-8841 020-39953345

In order to continuously improve user satisfaction, the product will continue to upgrade, the information may be different from the actual product, please take the physical object as standard.

FCC Requirement

changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) this device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

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GuanZhou Yongyi industrial Co.,Ltd