

Wireless Visual Ear
Pick manual



Please read this manual
carefully before use
and keep it properly for
future reference.

| APP download



Please use your mobile browser to
scan the QR code for downloading
and installation.

PicidaePro

► iOS app store

Android Google

Android China

Apple users
Select the los appstore option
after scanning the code.Open
the App Store to download and
install.
You can also directly search for
PicidaePro downloads in theApp
Store.

PicidaePro

IOS app store

► Android Google

Android China

Android users
A fter scanning the code, select
the Android Google option to enter
the next page,Click: Download
and install through third-party
browser options.

| APP Connection:

- [Power on]**
Press and hold the oval power button 2
at the bottom of the product body
seconds to boot;
- [Open APP]**
Click the Start button on the left and
wait for it to connect automatically
Connected products;
- [Get Started]**
Once the connection is successful, click
on the left side again to start
button to visualize the ear canal.

| Troubleshooting:

Screen freezing:
* Make sure the product is used in
close proximity to the phone within the
Wi-Fi signal transmis-sion range.
* Low battery can weaken the Wi-Fi
signal, so please charge it in a timely
manner.
*If there are many Wi-Fi signalsin the
environment, they may interfere with
the signal of the product. Please use it
in an area with fewer Wi-Fi signals.
No image display:
* Confirm that the app has been granted
permission to use network data.
* Check if the phone is connected to
the product's Wi-Fi network. If
connected but still no display, try
closing and reopening the app.

Unable to store images
or videos:
* Confirm that the app has been
granted permission to take photos and
record videos.

Unable to charge:
* Please use the provided charging
cable to charge.
* Try using a different charger if the
problem persists.

| Charging Instructions:

- Before initial use, please ensure that
the product is fully charged.
- It is not recommended to completely
drain the battery.
- When the indicator light flashesred,
please charge it promptly
- When charging, connect the Type-C
interface plug to this product.

| Indicator Light
Instructions:

- After powering on, the blue light will
flash.
- When connected to Wi-Fi, theblue
light will stay on.
- When the battery is low, the
redlight will flash.
- During charging, the red light
willstay on.
- When the battery is fully
charged,the red light will turn off.

| Precautions

- Please turn off the product before
charging. Do not use a fast charger
for charging.
- If the image is unclear, please use
a cotton swab to wipe the lens.
- Before use, observe the
surrounding environment to prevent
accidents.
- It is normal for the lens and body to
generate heat.
- This product is for personal care
use only and should not be used as a
medical device.
- Please keep the product in a cool,
dry place, away from fire and
corrosive substances.
- This product is not suitable for
children.
- Do not immerse the product in
water or rinse it under a faucet.
- Do not allow children to play with
this product.
- Do not take the product to an
unauthorized service center for
repairs.
- Do not use any accessories or
partsproduced by other
manufacturers, using such
accessories or parts will void your
warranty.

FCC Compliance Statement: This
device complies with Part 15 of the
FCC Rules. Operation is subject to
the following twoconditions:(1)this
device may not cause harmful
interference, and (2)this device must
accept any interference
received,including interference that
may cause undesired
operation.Changes or modifications
not expressly approved by the party
responsible for compliance couldvoid
the user's authority to operate the
equipment.

NOTE: This equipment has been tested and
found to comply with the limits for a Class B
digital device, pursuant to Part 15 of the FCC
Rules.These limits are designed to provide
reasonable protection against harmful
interference in a residential instak-ation. This
equipment generates,uses instructions, may
cause harmful interfe-rence to radio
communications.However there is no
guarantee that interference will not occur in a
particular installation. If this equipment does
cause harmful interference to radio or
television reception, which can be determined
by turning the equip-ment off and on,the user
is encouraged to try to correct the interference
by one or more of the following measures:
--Reorient or relocate the receiving antenna.
--Increase the separation between the
equipment and receiver.
--Connect the equipment into an outlet on a
circuit different from that to which the receiver
is connected.
--Consult the dealer or an experienced
radio/TVtechnician for help.

| Card After-sales Warranty

Product Warranty Policy:
1.Under normal use, the warranty period for
this product is one year.
2. The following situations are not covered by
the warranty (including but not limited to): a. No
warranty card or any proof of purchase. b.
Exceeding the scope of precautions.abnormal
use. c.Self-repair or damage caused by human
factors(including improper operation), aswell as
damage caused by natura disasters. d.
Exceeding the "Three Guarantees" period, or
using the product for purposes beyond its
intended use.
3. When returning the product for repair, please
include the warranty card.
4. The company reserves the right to modify the
warranty terms and the final interpretation of
the terms.

After-sales Warranty
Card

Customer Name _____

Customer Phone _____

Customer Address _____

Product Name _____

Product Specifications/Model _____

Remarks _____

Thank you for choosing our product. All our products
undergo rigorous quality testing before leaving the
factory. If you have any issues with the product,please
contact our after-sales service. We are committed to
providing you with assistance.

无线可视挖耳勺
说明书



请在使用前仔细阅读
本说明书, 请妥善保管
以备日后查询。

| APP下载方式



请使用手机浏览器扫二维码下载安装。

PicidaePro

► iOS app store (苹果商店)

Android Google (安卓国外)

Android China (安卓国内)

苹果用户
扫码后选择 iOS App Store 选
项打开 App Store 下载安装。
也可以直接在 App Store 搜索
PicidaePro 下载。

PicidaePro

IOS app store (苹果商店)

Android Google (安卓国外)

► Android China (安卓国内)

安卓用户
扫码后选择Android China (安
卓国内) 选项进入下一页面后点
击 :通过第三方浏览器下载选
项, 下载并安装。

| APP连接：

- 【开机】**
长按产品机身底部椭圆形电源键2
秒开机;
- 【打开APP】**
点击左侧开始按钮，等待自动连
接产品;
- 【开始使用】**
连接成功后，再次点击左侧开始
按钮，可视耳道画面即刻呈现。

| 故障排除

画面卡顿：

- 超过Wi-Fi信号传输距离, 产
品需与手机近距离使用;
- 电量不足导致Wi-Fi信号变
弱, 请及时充电;
- 环境存在较多Wi-Fi信号时,
会干扰本产品的信号, 请在
Wi-Fi信号偏少的区域使用。

无图像显示：

- 确认给予APP网络数据使用权限;
- 确认手机是否连接上产品
Wi-Fi名称;
- 确认连接后, 如还无显示: 关闭
APP重新打开APP。
若系统提示: 当前WLAN不可上
网, 是否继续使用此WLAN。请
不要切换Wi-Fi, 点击 “使用”
即可。

拍摄无法存储图像或视频：

- 确认给予APP拍照录像权限。

无法充电

- 请使用随产品附赠的充电线充电;
- 更换充电器后再试。

| 充电说明:

- 初次使用之前, 请将产品充满电;
- 不建议将电池电量完全耗尽;
- 当指示灯红灯闪烁时, 请及时充电;
- 充电时将Type-C接口端插头线
连接到本产品。

| 指示灯说明:

- 开机后蓝灯闪烁;
- Wi-Fi连接后蓝灯常亮;
- 电量低红灯闪烁;
- 充电时红灯常亮;
- 充满电后红灯熄灭。

| 注意事项

- 请将产品关机后, 再进行充电;
请勿使用快充充电器充电。
- 图像不清晰时, 请使用棉签擦拭
镜头。
- 使用前请观察周围环境, 防止
意外发生。
- 镜头、机身发热属于正常现象。
- 本产品仅作为个人护理用品, 不
能当作医疗设备使用。
- 请将产品放置在阴凉干燥处, 远
离火源及腐蚀性物品。
- 本产品不适合儿童使用。
- 请勿将本产品浸入水中; 请勿在
自来水龙头下冲洗。
- 不得让儿童玩耍本产品。
- 请勿将本产品送到非厂家授权的
服务中心检修。
- 切勿使用由其他制造商生产的
任何附件或部件; 如果使用此类附
件或部件, 您的保修将失效。

| 售后保障卡

产品保修政策：

- 本产品在正常使用情况下, 保修
期限为一年。
- 出现以下情况不属于保修范围
(包括但不限于):
a. 无保修卡或任何凭证;
b. 超出注意事项范围、非正常使用;
c. 自行修理或人为损坏 (含操作不
当); 自然灾害对产品造成的影响;
d. 超过“三包”期限; 产品用于其他
用途等。
- 产品返修时, 应连同保修卡一同
寄回。
- 本公司保留对保修条款的修改
权及最终解释权。

保修卡

客户姓名 _____

客户电话 _____

客户地址 _____

产品名称 _____

产品规格/型号 _____

备注 _____

感谢您选择本产品, 本公司所有产品
出厂前均经过严格的质量检测,
如有任何产品问题, 请联系售后。

正面

反面

FCC Requirement

changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) this device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.