H2AD

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Twitoo

Telemedicine Gateway

FCC regulation :

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

FCC Radiation Exposure Statement:

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment .

This equipment should be installed and operated with minimum distance 20cm between the radiator& your body.

This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

H2AD

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Twitoo

Telemedicine Gateway

Manual - English

I.	Introduction.....	2
1.	Precautions.....	2
II.	Installation.....	2
1.	Package contents	2
2.	Start-up.....	2
III.	Utilization.....	3
IV.	Maintenance	3
1.	Battery	3
2.	Positioning in the home.....	3
3.	Indicators:.....	3
4.	Flashes.....	3
5.	Multifunctional button	3
V.	Technical specifications and standards	4

I. Introduction

You just bought a Twitoo and we thank you. Twitoo is a radio gateway that allows you to automatically transfer your data to a secured server. The measured data will only be accessible to you and the persons that you choose (family, doctor, entourage...).

1. Precautions

- Read this manual entirely before using Twitoo
- Twitoo is made out of high technology. Please avoid extreme temperatures and humidity, direct solar light, shocks, mold and dust.

II. Installation

1. Package contents

- Twitoo

- Power adapter

2. Start-up

- Plug the Twitoo to the power adapter
- Plug the power adapter to the mains
- After the initialization sequence (about 1 minute), the 4 LEDs will be turned on continuously.
- In case of problem, please see the **Maintenance** section.

III. Utilization

In normal case, Twitoo has to be plugged in continuously.

If needed, to link a new medical Bluetooth peripheral, please see the peripheral specific manual. In some cases, the multifunctional button could be used.

To clean the Twitoo, you can use damp cloth with special attention on the multifunctional button.

IV. Maintenance

1. Battery

Twitoo contains a backup battery that allows it to continue operating in case of mains failure for about 1 hour. The battery automatically recharges when Twitoo is plugged to the mains again. It is possible that the battery is not fully charged when you received your Twitoo. You have to let it plugged at least 12 hours so that it is completely operational.

2. Positioning in the home

Twitoo use wireless communication system, so that you should avoid proximity with TV, microwave-oven, cellular phones, or other devices that emit strong electromagnetic fields. The nature and thickness of the walls can also disrupt the transmission. We advise you to place your Twitoo in a living room on a visible furniture.

3. Indicators:

Twitoo has 4 LEDs of different colors:

- Green: power status
- Blue: Bluetooth status
- Orange: GPRS status
- White: Button status

Intensity of these indicators can be set remotely by a technician.

In case of mains power failure, the green light will blink.

4. Flashes

There are 4 types of flashes on the indicators of Twitoo: On, Off, Slow flash, Quick flash. These flashes could help the diagnostic of remote assistance.

5. Multifunctional button

When the multifunctional button indicator is On, that shows the good status of the Twitoo.

If this indicator is Off, that indicates a problem. To solve this, try the following actions:

- Check that the power adaptor is correctly plugged in and that there is no mains failure.
- Unplug, then plug in again the Twitoo and wait the end of the initialization sequence.
- If you do not see amelioration, please contact technical service.

V. Technical specifications and standards

Product dimensions: 240 x 95 x 63 mm

Package dimensions: 315 x 235 x 88 mm

Power sources: mains + battery

Life time on battery: 1h

Battery maximum charge time: 12h

Wireless communication: Bluetooth class 2 (5 to 10m)
GSM-GPRS quad band (800-900-1800-1900Mhz)

Operating temperature range: 0°C à 45°C

CE certified

Classification: Medical device of class 1

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