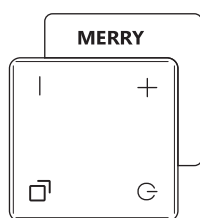
Sonata
Sounds & Clear
Wireless My Personal Sound Amplifier

Model No.: ME-700



CONTENT	PAGE
QuickGuide - English	2
QuickGuide - Japan	9
QuickGuide - 中文	17
QuickGuide - 简体中文	24
FCC Statement	31

Inside Package

Personal Sound Amplifier
(Receiver:ME-700R)

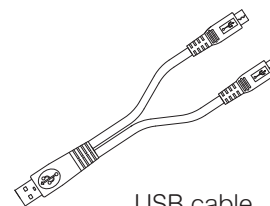
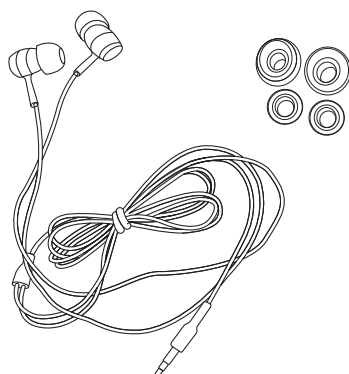
Binaural Earphone
(Including: a pair of Large,
Medium, Small ear buds)


USB cable

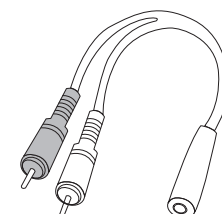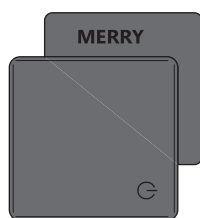
Wireless Microphone
(Transmitter: ME-700T)


RCA stereo audio adapter

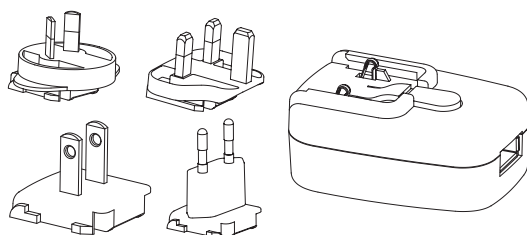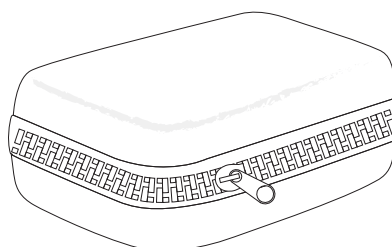Universal Adapter
(Including US, EU, UK, and AU adapter)

Carry Case

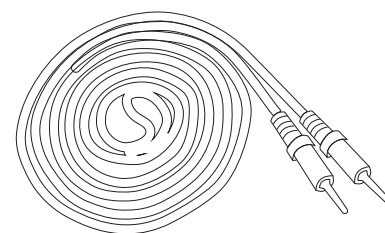3.5mm stereo audio cable

Specification

Personal Sound Amplifier (Receiver)

Color

White

Max Sound Pressure Level

118±4 dBSPL@2,900Hz

Max Full-on Gain

60±5dB

Size

32.5 mm (L) * 36 mm * (W) *14.9 mm (H)

Weight

12.2g

Operation time

Fully recharge 1 hour; Operating 3 hours.

Binaural Earphone

Wireless Microphone (Transmitter)

Color

Black

Microphone

Omnidirectional

Operating Voltage

3.1~4.2V

Max Line-In Voltage

1.2 V

Size

32.5 mm (L) * 36 mm * (W) *14.9 mm (H)

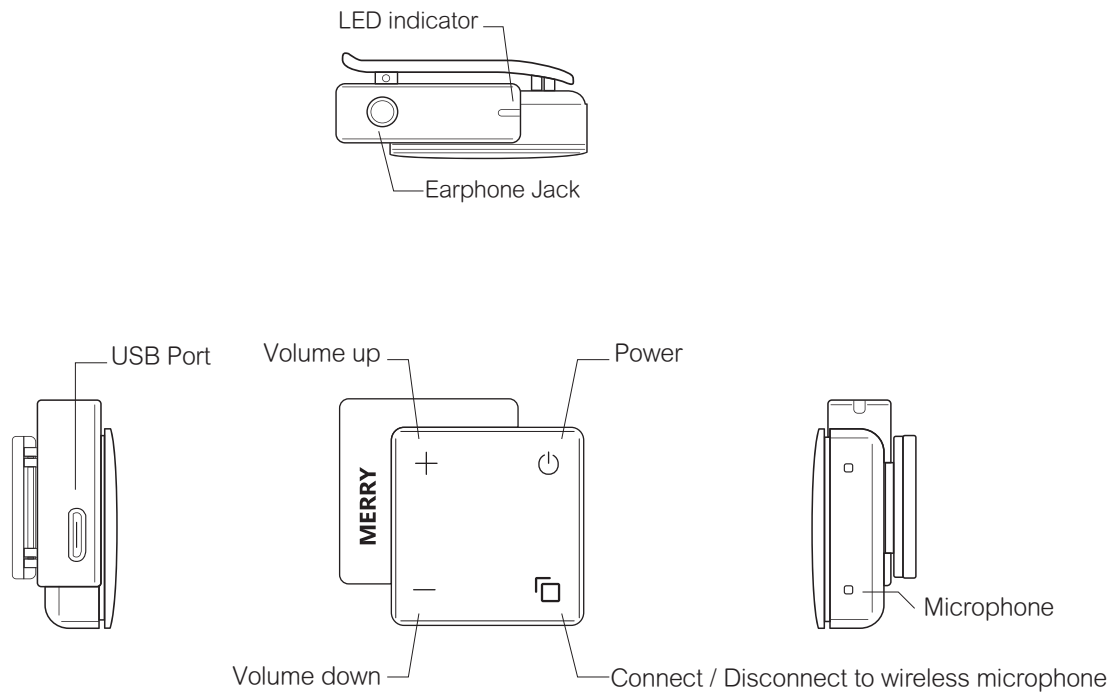
Weight

12.2g


Operating Time

Fully recharge1 hour ; Operating 3 hours.

Functions

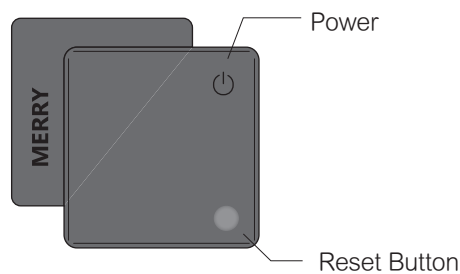Personal Sound Amplifier (Receiver : ME-700R) - White

Wireless Microphone(Transmitter : ME-700T) - Black

ME-700

Thank you for choosing ME-700 !

Please read this manual guide before using ME-700 to avoid the risk of improper using.

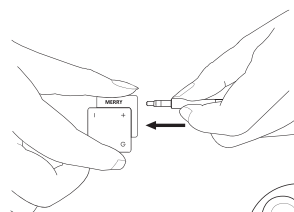
ME-700 is built in Li-ion rechargeable battery.

The battery must be full charged for the first time. In order to extend the battery lifetime effectively, we suggest the product ME-700 should be charged during 5~40 °C.

How To Use

A.Plugging the Earphone into Personal Sound Amplifier (ME-700R).

Please insert earphone into control box.


B.Wearing Earphone

Put the ear buds on your ear.

Mark “L” represents left side.

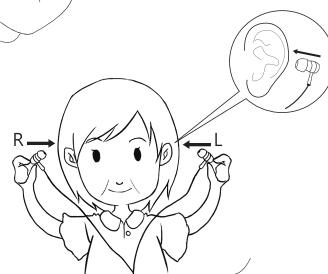
Mark “R” represents right side.

C.Powering On

Press the “On/Off switch” button for 3 seconds to turn on the Personal Sound Amplifier (ME-700R) and Wireless microphone (ME-700T). When the battery is fully charged, the LED indicator is Green.

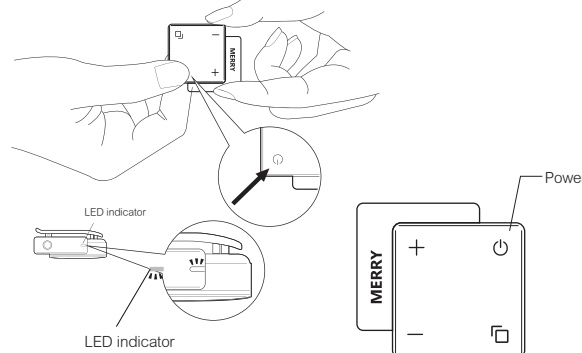
※ME-700 could not be turned on if the battery is too low.

D.Mode Switching(Connect/ Disconnect)


Press connect/disconnect button on Personal Sound Amplifier (ME-700R), it will connect automatically. It will be disconnect automatically when you press the same button again. When disconnect, the personal sound amplifier can work individually as sound amplifier devise.

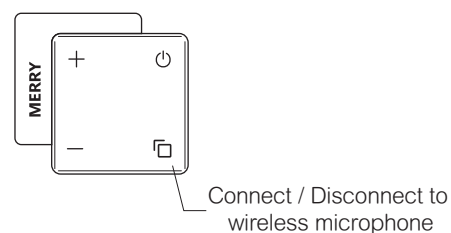※Notice : When connect, the LED indicator is Blue.


E.Volume Control

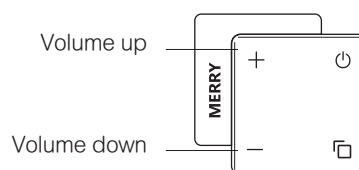Press “+” or “-” to adjust the level of volume.

F.Turning the Power Off

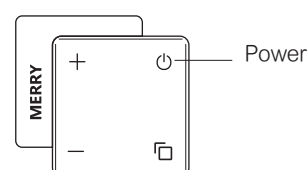
Press the “On/Off switch” button for 3 seconds to power off the ME-700. When power is off, the LED indicator light is off.

How to connect to TV

STEP 1:Checking Accessories

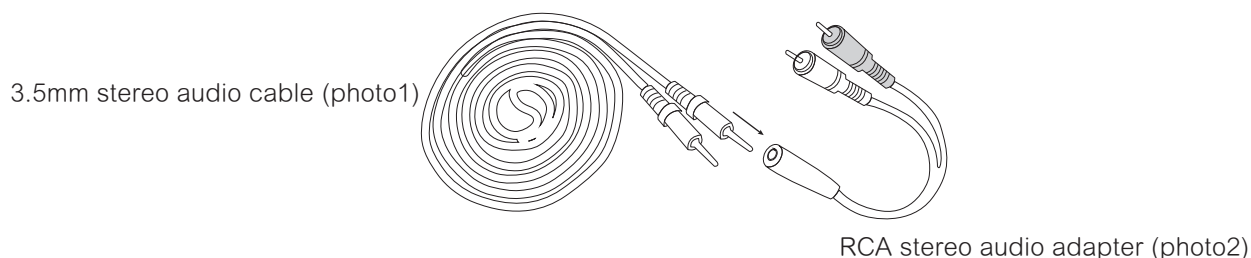
Please take out 3.5mm stereo audio cable (photo1) and RCA stereo audio adapter(photo2) from box, and connect RCA stereo audio adapter with stereo audio cable from 3.5mm port.

STEP 2:RCA stereo audio adapter connects to TV

Connect RCA plugs to Audio out (Right->Red, Left->White) on the television. (photo 3)

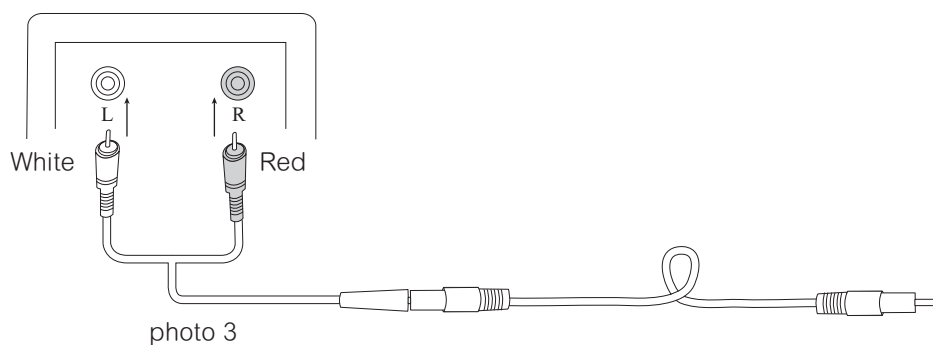※TV volume will be delivered to wireless microphone of ME-700T, and the TV audio still works.

STEP 3:3.5mm stereo audio cable connects to Wireless Microphone

Connect audio cable to the wireless microphone and power on the wireless microphone. (photo 4)

※Check if the LED indicator of wireless microphone is green.

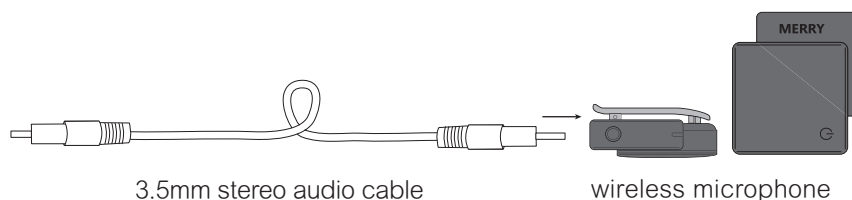
photo 4

STEP 4:Power on the Personal Sound Amplifier.

Turn on the personal sound amplifier (ME-700R) and click the “connect/disconnect” to connect with the wireless microphone (ME-700T). When the LED indicator turns blue, it connects successfully and can start to work.

LED Indicator

The color of LED indicator illustrate the power status of ME-700 :

A. Personal Sound Amplifier (ME-700R)(White)

	Status	Color of LED Indicator
Power On	Disconnection with full charged	Green
	Disconnection with low battery / charging	Green with Blinking Orange
	Connection with full charged	Blue
	Connection with low battery / charging	Blue with Blinking Orange
	Reset	LED Off
Power Off	Not charged	LED Off
	Charging	Orange
	Full charged	Led Off
	Reset/ charging	Orange Flash

B. Wireless Microphone (ME-700T)(Black)

	Status	Color of LED Indicator
Power On	Disconnection with full charged	Green
	Disconnection with low battery / charging	Green with Blinking Orange
	Connection with full charged	Blue
	Connection with low battery / charging	Blue with Blinking Orange
	Reset	LED Off
Power Off	Not charged	LED Off
	Charging	Orange
	Full charged	Led Off
	Reset/ charging	Orange Flash

Trouble Shooting

Please check below items see if any possibilities of malfunction. If you can not find a solution from the following Q&A, please contact the local distributor for customer service

Questions	Answers
1.Can not turn on?	▲ Please check the power status ? ▲ Please make sure to press 3 seconds for turing on?
2.No Voice?	▲ Please make sure the earphone is inserted in the personal sound amplifier (white). ▲ Try to adjust the volume °

Trouble Shooting

Question	Answer
3.Can not connect?	▲ Please check both personal sound amplifier and wireless microphone are powered on. ▲ Please press the “connect/disconnect” button again, and wait for 5 secs. In the connection mode, the LED indicator is blinking blue. The LED indicator is blue when connects successfully. ▲ During the regular connection process, the blue indicator will be blinking. After the connection confirmed, the indicator is blue.
4.High-pitched noises in earphone	▲ Insert ear buds tightly into ear canal or change the different size of ear buds. ▲ Reduce volume or increase the distance between the earphone and the microphone of control box.
5.How to use personal sound amplifier?	▲ Please check the power status of personal sound amplifier (white)? ▲ Please check if it disconnected with wireless microphone. ▲ Please check if the LED indicator is Green.
6.System Reset	▲ Personal Sound Amplifier (White), please press connect button for 10 seconds till it to turn off. Turn on the device, and it will start to run normally. ▲ Wireless Microphone (Black), please press “ reset button” for 10 seconds till it to turn off. Turn on the device, and it will start to run normally. ▲ If still cannot turn on or turn off the product after undergoing the above actions. Please kindly repeat again, or call Merry service phone number for further assistance.

Notice

- ※ This product is not a hearing aid. If you have any hearing problems, please see doctor for advising.
- ※ This symbol indicates that integral electrical equipment must not be treated as household waste. It should be taken to the relevant collection depot for the recycling of electrical and electronics equipment.

Eco-friendly

Warning

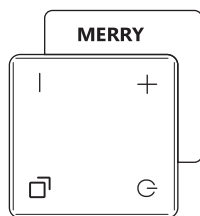
- ▲ If you feel uncomfortable or tinnitus, please stopping using this product and having hearing check.
- ▲ Please keep ME-700 away from children and pets. In case children or pets swallow it. If swallows it by accident, please consult a doctor immediately.
- ▲ Do not use ME-700 where it is forbidden to use electronic device.
- ▲ Please keep ME-700 away from fire (high temperature), water(humid environment), or any device generating high voltage of electronics.
- ▲ Please use the AC adapter provided by Merry Electronics to recharge your ME-700.

MERRY ELECTRONICS CO., LTD
 No.22, 23rd Road, Taichung Industrial Park,
 Taichung, 40850 Taiwan, R.O.C.
 Hear@ merry.com.tw
<http://www.merryhealthcare.com/>
 Merry Service Phone Number:+886289132381

Manufacturer
 MERRY ELECTRONICS (SHENZHEN) CO., LTD.
 Merry Ind. Park Hua Rong Rd., DaLang, BaoAn,
 ShenZhen, 518109 China

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FCC Statement

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

CAUTION:

To assure continued FCC compliance:

Any changes or modifications not expressly approved by the grantee device could void the user's authority to operate the equipment.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

FCC RF Radiation Exposure Statement:

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment.

End users must follow the specific operating instructions for satisfying RF exposure compliance.

This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.