

**SMARTSTOCK, LLC**



**TECHNOLOGY LEADER IN LIVESTOCK  
HEALTH MANAGEMENT**

**User's Manual. Bolus Installation instructions.**

**FCC ID: LPH-SSL001**

The bolus is designed to be ingested into ruminant animals. The bolus is approximately 1 1/8" in diameter and approximately 3" in length. The bolus is installed into the bolus gun or balling gun (figure 2). The balling gun is inserted into the esophagus of the ruminant animal. When the balling gun is inserted up to the handle, squeeze the handle to release the bolus. Assure that the flat end of the bolus is inserted into the balling gun first. The rounded end will thus go into the animals mouth first.

Warning: if you are not familiar with installing boluses into ruminant animals, seek help from a professional, like a herd health management specialist or a Doctor of Veterinary Medicine (DVM). For further information view the installation video at [www.smartstock-usa.com](http://www.smartstock-usa.com).



Figure 1



Van Arman Photography

Figure 2

**P.O. Box 337  
1310 N. Sewell  
Pawnee, OK 74058  
918.762-1065  
[www.smartstock-usa.com](http://www.smartstock-usa.com)**