

TO STOP THE ALARM

On the Parent Unit, press any key to stop the alarm.


OR

Press the **Pager/Hold** button on the Nursery Unit.



EN

TO ADJUST THE SENSOR PAD SENSITIVITY LEVEL:

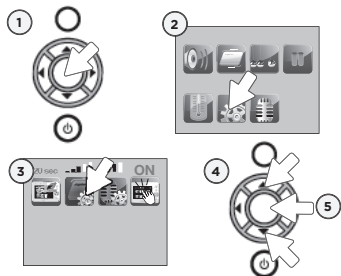

- 1) Press **Menu/Select** button to access main menu.
- 2) Select the **Settings** icon to access sub-menu.
- 3) Select the **Movement Sensitivity** icon.
- 4) Use the up and down arrows to choose the desired sensitivity level (1 to 5).




- 5) Press **Menu/Select** button to confirm.

NOTE: If you are receiving false alarms, adjust the sensitivity level.

TO ACTIVATE THE TIC FUNCTION:

Allows you to hear a **TIC** sound when Sensor Pad function is activated.

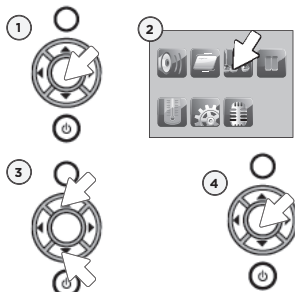

- 1) Press **Menu/Select** button to access the main menu.
- 2) Select the **TIC** icon.
- 3) Use the up and down arrows to **activate** or **deactivate** the **TIC** function.
- 4) Press **Menu/Select** button to confirm.


Regular **TIC** sound will emit from Parent Unit and a pendulum will appear on the Parent Unit indicator bar.

IMPORTANT

- The Sensor Pad is not designed for use with adults, children or infants older than 24 months.
- Once you are finished using the Sensor Pad, deactivate the movement function and please remove it from the baby's crib and store it away.

5.2 - SOUND

TO ACTIVATE SOUND:

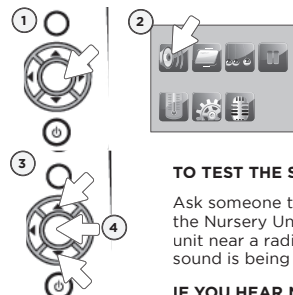

TO TEST THE SOUND:

Ask someone to speak into the Nursery Unit or place the unit near a radio to ensure that sound is being picked up.

IF YOU HEAR NO SOUND:

- 1) Press **Menu/Select** button to access main menu.
- 2) Select the **Sound** icon.
- 3) Use the up and down arrows to **activate** or **deactivate** the sound.
- 4) Press **Menu/Select** button to confirm.

- Check that both units are powered **ON**.
- Check that both units are plugged into functioning electrical outlets.
- Check the batteries in both units (replace or recharge them if necessary).
- Bring the Parent Unit closer to the Nursery Unit.

IMPORTANT

To avoid feedback (whistling), keep both units at least 10 feet apart (3 meter).

ADJUST SOUND LEVEL



Increase volume
(Up arrow)

Lower volume
(Down arrow)

PUT IN SILENT/MUTE MODE



Press the down arrow for 2 seconds to mute sound. **Mute** icon will flash.

5.3 - BATTERIES


HOW TO CHECK THE BATTERIES:

Unplug the adapters from both units.

- If the green light on the Nursery Unit does not show, replace the batteries.
- If the Parent Unit and its screen do not light up, recharge the batteries.

HOW TO CHARGE THE BATTERIES IN THE PARENT UNIT:

Leave the Parent Unit plugged to fully charge the unit. Although the batteries at 1/3 capacity works normally, we recommend that you fully recharge them.

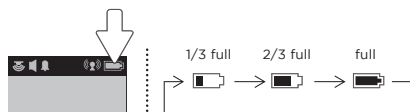

PARENT UNIT BATTERY ALARM

- Alarm activates when the batteries need to be recharged.
- Parent Unit will emit a beep sound every 15 seconds.
- **Battery Alarm** icon will show on LCD screen.

YOU ARE NOW READY TO USE YOUR MONITOR

The Angelcare® monitor can be used with the pre-program factory settings or you can also customise your settings to your preference.

DEFAULT SETTINGS



Sound Mode: ON
Speaker Volume: Level 3 (of 6)



Movement Mode: ON
Sensor Pad Sensitivity: Level 3 (of 5)
TIC: ON



Temperature Unit: Fahrenheit USA/Celsius Canada
Temperature Alarm: OFF



VOX: OFF
VOX Level: Level 2 (of 4)




Screen Saver Timer: ON
(after 3 minutes)



LCD Brightness: Level 3 (of 5)



Key Tone: ON

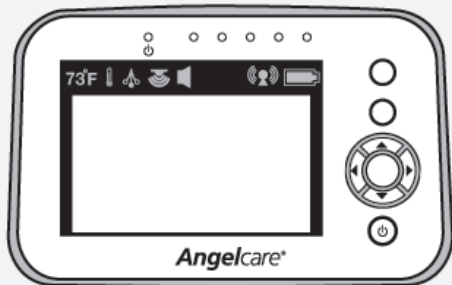

IMPORTANT

- Test your monitor before first use.
- Test your monitor regularly to ensure it is functioning correctly.
- Test your monitor every time you move the Sensor Pad to a different place.

STEP 6 - ACTIVATING AND DEACTIVATING ADDITIONAL FUNCTIONS

IMPORTANT

When in the menu or sub-menu modes, sound transmission between Parent and Nursery Units will be temporarily deactivated. ONLY the movement detection alarm will remain active (if function is activated).

6.1 - VOICE ACTIVATION (VOX) OR CONTINUOUS SOUND MODE

Allows you to select your option on how you monitor sound: **VOX** (voice activation picks up sound based on the setting of the sensitivity level) or **Continuous** (sound monitoring is continuously on, picking up all sounds).


TO ACTIVATE VOX MODE:

NOTE:

By deactivating **VOX** the Continuous setting is **ON**.

TO ADJUST THE SOUND SENSITIVITY (VOX):

Allows you to determine the sound level needed to trigger sound transmission to the Parent Unit. This means that sound will only transmit based on the level that you set.

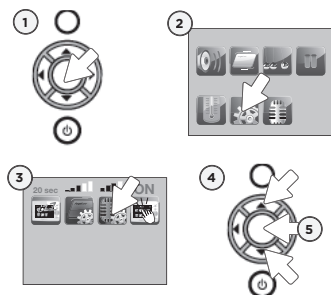

- 1) Press **Menu** icon to access the main menu.
- 2) Select the **Settings** icon to access sub-menu.
- 3) Select the **VOX** icon to **activate** or **deactivate** the VOX function.
- 4) Press **Menu/Select** button to confirm.


- 1) Press **Menu/Select** button to access the main menu.
- 2) Select the **Settings** icon to access sub-menu.
- 3) Select the **VOX Sensitivity** icon.
- 4) Use the up and down arrows to choose the desired sensitivity level (1 to 5)



- 5) Press **Menu/Select** button to confirm.

6.2 - HOLD

Allows you to put your monitor on hold.

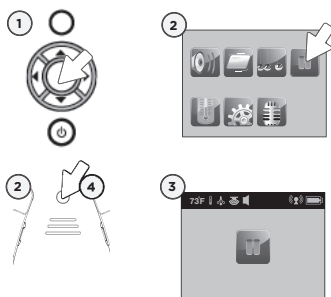

- 1) Press **Menu/Select** button to access main menu.
- 2) Select the **Hold** icon.
OR
Press and hold the Nursery Unit **Pager/Hold** button for at least 2 seconds.
- 3) **Hold** icon will appear on the screen.
- 4) Press and hold the Nursery Unit **Pager/Hold** again to return to normal operation.




Nursery Unit **Night Light** will blink. Parent Unit will emit 3 consecutive beeps every 30 seconds to remind you that monitor is on **Hold**.

IMPORTANT

When your monitor is on **Hold**, the movement-detection feature and movement alarm are disabled.

6.3 - THERMOMETER


TO CHANGE THE MEASUREMENT UNIT:

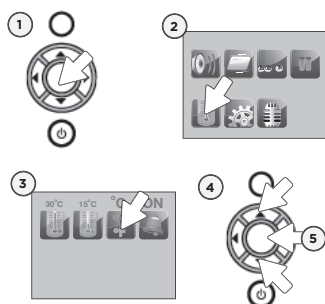

- 1) Press **Menu/Select** button to access main menu.
 - 2) Select the **Temperature** icon to access sub-menu.
 - 3) Select the **°C °F** icon.
 - 4) Use the up and down arrows to change from **Celsius** to **Fahrenheit**.
- Celsius


Fahrenheit
- 5) Press **Menu/Select** button to confirm.

TO ACTIVATE THE TEMPERATURE ALARM:

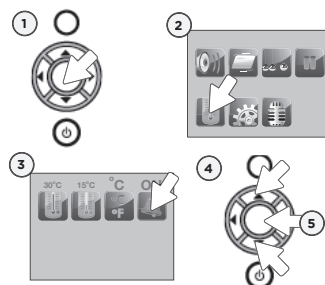

- 1) Press **Menu/Select** button to access main menu.
 - 2) Select the **Temperature** icon to access sub-menu.
 - 3) Select the **Temperature Alarm** icon.
 - 4) Use the up and down arrows to **activate** or **deactivate** the **Temperature Alarm**.
- Temperature Alarm icon:** Indicates temperature alarm is **ON**.
- 5) Press **Menu/Select** button to confirm.

TO SET THE TEMPERATURE RANGE:

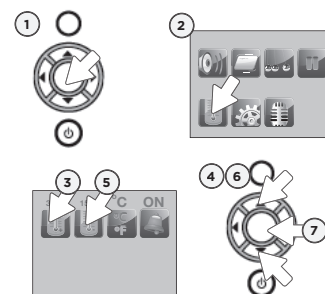

- 1) Press **Menu/Select** button to access main menu.
- 2) Select the **Temperature** icon to access sub-menu.
- 3) Select the **High Temperature** icon. to set **maximum** temperature range (59 to 90 °Fahrenheit/15 to 32 °Celsius).
- 4) Use the up and down arrows to set the highest temperature preferred.
- 5) Select the **Low Temperature** icon. to set **minimum** temperature range (41 to 68 °Fahrenheit/5 to 20 °Celsius).
- 6) Use the up and down arrows to set the lowest temperature preferred.
- 7) Press **Menu/Select** button to confirm.

The range between the minimum and maximum temperature must be at least 7 °F/4 °C. An error tone will be heard if you try to set it up below this minimum level.




TEMPERATURE ALARM

- Alarm activates when temperature reaches maximum or minimum set temperature.
- Parent Unit will emit 4 consecutive beeps, every 5 seconds.
- **Temperature Alarm** icon will show on LCD screen.

6.4 - SCREEN SAVER

Allows you to set the screen saver starting point.

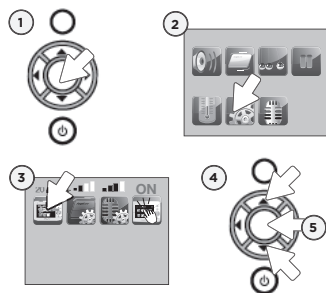

MANUAL SETTING:

To trigger manually the **Screen Saver** mode press and hold the Parent Unit **VIDEO ON/OFF** button.

- 1) Press **Menu/Select** button to access main menu.
- 2) Select the **Settings** icon to access sub-menu.
- 3) Select the **Screen Saver** icon.
- 4) Use the up and down arrows to select the delay before **Video** mode turns into **Screen Saver** mode.
- 5) Press **Menu/Select** button to confirm.

NOTE:

The LCD screen will turn black when video is in **Screen Saver** mode. Press any button to return to **Video** mode.

6.5 - INFRARED NIGHT VISION

Allows you to capture image when lighting is insufficient.



A **Night View** icon will appear at the top of the Parent Unit screen.



The **Night View** icon will disappear at the top of the Parent Unit screen.

- 1) The infrared LEDs will turn **ON** automatically when the lighting is not sufficient, to give a night view on LCD display.

NOTE:

When the **Infrared Night Vision** is **ON**, only black and white image will be displayed on the LCD screen.

- 2) If the light is bright enough in the room, the infrared LED on the Nursery Unit will turn **OFF** automatically.

6.6 - NIGHT LIGHT

Allows you to use your monitor as a night light.

TO ACTIVATE THE LIGHT:

Push the button on the back of the right "wing" to turn the **Night Light ON** or **OFF**.



6.7 - PAGER

Allows you to locate the Parent Unit.

TO LOCATE PARENT UNIT:


Press the **Pager/Hold** button for less than 2 seconds. The Parent Unit will emit a paging sound for one minute.



IMPORTANT

If you press the **Pager/Hold** button on the back of the Nursery Unit for more than 2 seconds, you will put the movement monitoring on **Hold**. To unhold, push the button again.

TO STOP THE PAGING SOUND:

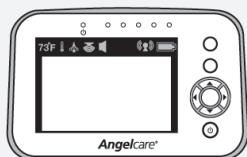

Press any button on the Parent Unit.


OR

Press the Nursery Unit's **Pager/Hold** button



6.8 - ZOOM MODE


Allows you to zoom in to twice the normal view.

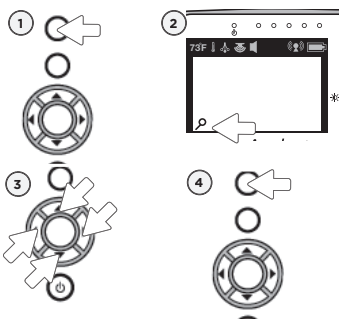

- 1) Press and release the **Zoom** button on the Parent Unit.
- 2) The **Zoom** icon will appear at the bottom left corner of the screen.
- 3) Use the **arrow keys** to zoom in on what you want to see more closely.
- 4) Press and release the **Zoom** button on the Parent Unit again to return to normal view.

6.9 - KEY TONE


Allows you to activate or deactivate key tone sound.

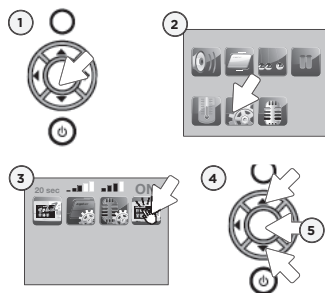

- 1) Press **Menu/Select** button to access main menu.
- 2) Select the **Settings** icon to access sub-menu.
- 3) Select the **Key Tone** icon.
- 4) Use the up and down arrows to **activate** or **deactivate Key Tone** sound.
- 5) Press **Menu/Select** button to confirm.

6.10 - LCD SCREEN BRIGHTNESS

Allows you to adjust the LCD screen brightness on the Parent Unit.

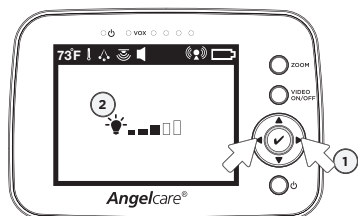

- 1) Use the left and right arrows to adjust the screen brightness level (**1** to **5**).
- 2) A light bulb will be shown on the LCD screen.

TAKING CARE OF YOUR RECHARGEABLE BATTERIES

- Ensure that you fully charge your batteries for 9 hours before their first use.
- To maintain maximum battery life it is recommended that you fully discharge your batteries periodically. If your Parent Unit is not in use, please turn it **OFF**.

FALSE ALARMS

- Turn **OFF** the Nursery Unit.
- Verify the connection between the Sensor Pad and the Nursery Unit. Unplug and reconnect the cord.
- Make sure there is no bedding between the mattress and the Sensor Pad.
- If the crib has a spring or slat base, place a hardboard between the crib base and the Sensor Pad measuring at least 13" x 13" (33 x 33 cm) and 1/4 inch (6 mm) thick.
- Make sure the Sensor Pad is positioned as described under the setup requirements. Sensitivity may need to be adjusted, refer to **Step 5.1**.

ALARM WILL NOT SOUND

- Avoid contact with the crib when the monitor is **ON**.
- Place crib near a solid supporting wall to avoid Sensor Pad from detecting movement from outside the crib.
- You may need to decrease the sensitivity, refer to **Step 5.1**.

NO SOUND IS BEING TRANSMITTED

- Bring the Parent Unit closer to the Nursery Unit.
- Recharge the Parent Unit.
- Check connections or change to another electrical outlet.
- Ensure both units are **ON**.
- Ensure mains supply units are activated.
- Check on the top bar of the Parent Unit to see if the monitor is on **MUTE** instead of **ON**. Refer to **Step 5.2**.

NURSERY UNIT POWER ON INDICATOR (GREEN LIGHT) DOES NOT LIGHT UP

- Check connections or change to another electrical outlet.
- Push and hold the button on the back of the **left 'wing'** for 2 seconds to turn the Nursery Unit **ON**.

WEAK RECEPTION

- Bring units closer together.
- Change the position of one or both units.
- Place units on a flat surface away from baby's reach.
- Recharge the Parent Unit.

STATIC, DISTORTION, INTERFERENCES**(FROM OTHER MONITORS, WIRELESS PHONES, WALKIE-TALKIES, ETC.)**

- Rotate Parent Unit or move it away from source of interference.

FEEDBACK (LOUD NOISES EMITTED FROM THE PARENT UNIT)

- Move units farther apart (at least 3 meters/10 feet apart)..
- Decrease volume on Parent Unit.

THE PARENT UNIT SCREEN IS BLACK (PARENT UNIT IS OFF)

- Turn the Parent Unit back **ON** by pushing and holding the **POWER ON/OFF** button for 2 seconds.
- Recharge the Parent Unit.

LIMITED WARRANTY

Angelcare® warrants to the original owner of this product that Angelcare® video, movement and sound monitor (model AC1300) is free from any material and workmanship defects, for a period of one (1) year from the date of the original purchase. If this monitoring system fails to function properly when used as directed under normal conditions within the one year period, Angelcare® will either repair or replace the product, at our discretion, free of charge. Please contact your distributor.

The product must be accompanied by a proof of purchase, either a bill of sale or other proof that the monitoring system is within the warranty period. Angelcare® will bear the cost of repairing or replacing the product and shipping it back to you.

This warranty does not apply to a product which has been damaged as a result of improper maintenance, an accident, improper voltage supply or any other form of misuse. The warranty is also void if the owner repairs or modifies the product in any way. Angelcare® is not liable for any incidental or consequential damages with regard to this product. The warranty also excludes any liability other than what is stated above. No other warranty is given.

REGISTER ANGELCARE PRODUCTS

Register your Angelcare® monitor and other Angelcare® products on our website:
www.angelcarebaby.com

ANGELCARE MONITORS INC.
201, Boulevard de l'Industrie, Local 104,
Candiac, Quebec
J5R 6A6 Canada

TECHNICAL SPECIFICATIONS

Range:

Open field (Open area without obstructions):
up to 820 feet (250 meters).

Number of Channels:

USA/Canada: 19 channels.
Europe: 64 Channels.

Power Supply:

3 AA alkaline batteries (not included) or
100-240VAC/5VDC adapter for Nursery Unit.

1 enclosed and non-detachable rechargeable Li-ion polymer battery and 100-240VAC/5VDC adapter for the Parent Unit.

Caution: To reduce the risk of electric shock, do not expose this product to rain or moisture.

Danger of explosion if batteries are incorrectly replaced. The included battery is an enclosed and non-detachable Li-ion polymer rechargeable battery, manufactured by SPRINGPOWER TECHNOLOGY (SHENZHEN) COMPANY LIMITED.



PROTECT THE ENVIRONMENT

This electronic equipment contains reusable material and must not be disposed with domestic waste. Please take it to a collecting point for electric and electronic equipment (ask your local authorities for details).

This warranty does not cover products and accessories which are not Angelcare-branded, or any problems that result from usage that is not in accordance with product instructions, failure to follow the product instructions, or problems caused by using accessories, parts or components not supplied by Angelcare.

LEGAL RIGHTS VARY FROM COUNTRY TO COUNTRY. SOME COUNTRIES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU.

WE ARE NOT IN ANY WAY RESPONSIBLE FOR ANY OUTCOME RESULTING FROM THE USE OF THIS PRODUCT.

IMPORTANT! This Angelcare® monitor is a personal care product. Please do not return this product to the store. If you have questions or need help, please call your distributor before returning this product.

The lithium rechargeable battery can NOT be replaced.

Batteries:

- DO NOT mix old and new batteries.
- DO NOT mix alkaline standard (carbon-zinc), or rechargeable (NI-CD, NIMH, etc.) batteries.

Transmission:

Data transmission between the Parent Unit and the Nursery Unit is made on the 2.4GHz frequency.

Hereby, Angelcare Monitors Inc., declares this baby monitor is in compliance with the essential requirements and other relevant provisions of Directive 1999/5/EC.

The EU declaration of conformity can be obtained at: www.angelcarebaby.com.

ErP COMPLIANCE

SOUTH AFRICA-BRAZIL-ECUADOR-EUROPEAN UNION-UK
USE ONLY ANGELCARE® BRAND POWER SUPPLY
- MODEL NUMBER: T05005V001 / T05005B001

GERMANY
USE ONLY ANGELCARE® BRAND POWER SUPPLY
- MODEL NUMBER: T05005F002

AUSTRALIA
USE ONLY ANGELCARE® BRAND POWER SUPPLY
- MODEL NUMBER: T05005S002

NORTH AMERICA
USE ONLY ANGELCARE® BRAND POWER SUPPLY
- MODEL NUMBER: T05005U002

Angelcare®

POUR VOTRE TRANQUILLITÉ D'ESPRIT

MANUEL DE L'UTILISATEUR

MONITEUR MOUVEMENTS,
VIDÉO ET SONS

Modèle AC1300



TABLE DES MATIÈRES

 AVERTISSEMENTS	21
INSTRUCTIONS D'INSTALLATION DU AC1300	24
ÉTAPE 1 - Charge de l'unité des parents (9 heures)	24
ÉTAPE 2 - Installation du détecteur de mouvements	25
ÉTAPE 3 - Installation de l'unité du nourrisson	27
ÉTAPE 4 - Mise en marche du moniteur	28
ÉTAPE 5 - Test et familiarisation avec le moniteur	29
ÉTAPE 6 - Activation et désactivation des fonctions supplémentaires	32
Résolution de problèmes	36
Garantie limitée	37
Fiche technique	38
ENGLISH	2
ESPAÑOL	39

MANUEL DE L'UTILISATEUR

VEUILLEZ CONSERVER VOTRE MANUEL DE L'UTILISATEUR POUR CONSULTATIONS ULTÉRIEURES.

VEUILLEZ LIRE ATTENTIVEMENT LES AVERTISSEMENTS ET LES INSTRUCTIONS AVANT D'ASSEMBLER ET D'UTILISER CE PRODUIT.

LE NON-RESPECT DES AVERTISSEMENTS ET DES INSTRUCTIONS POURRAIT ENTRAÎNER DES BLESSURES GRAVES OU LA MORT, OU ENDOMMAGER LE PRODUIT.

LE DÉTECTEUR DE MOUVEMENTS N'EST PAS CONÇU POUR LA SURVEILLANCE D'ADULTES OU D'ENFANTS ÂGÉS DE PLUS DE 24 MOIS.

LE DÉTECTEUR DE MOUVEMENTS N'EST PAS DESTINÉ À ÊTRE UTILISÉ POUR DÉTERMINER SI L'ENFANT EST SORTI DU LIT.

AVERTISSEMENT

L'assemblage doit être fait par un adulte.
Gardez les petites pièces hors de la portée des enfants lors de l'assemblage.

Chez Angelcare®, nous endossons fièrement tous nos produits.

Si vous n'êtes pas complètement satisfait ou si vous avez des questions :

États-Unis et Canada : Veuillez envoyer un courriel à : angelcare@cdecinc.ca

Autres pays : Veuillez communiquer avec votre distributeur local.

RÉGLEMENTATIONS DE LA FCC ET D'IC

Énoncé à propos de l'exposition aux rayonnements radioélectriques

L'unité du nourrisson est conforme aux limites d'exposition aux rayonnements radioélectriques de la FCC établies pour un environnement non contrôlé. **Attention :** Afin de préserver la conformité avec les lignes directrices de la FCC relativement à l'exposition aux rayonnements radioélectriques, placez l'unité du nourrisson à une distance minimale de 20 cm de TOUTE personne. L'émetteur de cet appareil ne doit pas être situé près de toute autre antenne ou de tout autre émetteur, ni utilisé conjointement avec toute autre antenne ou de tout autre émetteur.

Industrie Canada

Ce produit est conforme aux spécifications techniques applicables d'Industrie Canada.
CAN ICES-3 (B)/NMB-3 (B)