



Complete Home Alerting System

HA360M – USER GUIDE



Table of Contents

Important Safety Instructions.....	4
HomeAware Special Features.....	6
HomeAware Identification Features	7
Operations	8
Setting the Time of Day.....	8
Test Mode (Testing Audio/VIB/Strobe Flash /Lamp).....	8
LED Light Indicators	8
Setting the Alarm Time.....	9
Setting Snooze Time	9
Setting Alarm Duration	9
Setting Audio, Flash and Vibrator Alarm	10
Select 12 or 24 Hour Mode	10
Speaker Volume / Tone Setting	11
Testing Light Sensitivity.....	11
Power Failure and Battery Switching	11
Low Bat Screen	11
USB Charging.....	11
HomeAware Accessories	12
Regulatory Compliance	13
General Data	13
Warranty	16

IMPORTANT SAFETY INSTRUCTIONS

When using your HomeAware™ HA360M, basic safety precautions should always be followed to reduce the risk of fire, electric shock and persons including the following:

1. Read and understand all instructions.
2. Follow all warnings and instructions marked on the product.
3. Do not use this product near a bathtub, wash basin, kitchen sink or laundry tub, in a wet basement, near a swimming pool or anywhere else there is water.160
4. Avoid using a telephone (other than a cordless type) during a storm. There may be a remote risk of electrical shock from lightning.
5. Do not use a telephone to report a gas leak in the vicinity of the leak.
6. Unplug this product from the wall outlets before cleaning. Do not use liquid cleaners or aerosol cleaners on the product. Use a damp cloth for cleaning.
7. Place this product on a stable surface. Serious damage and/or injury may result if the product falls.
8. Do not cover the slots and openings on this product. This product should never be placed near or over a radiator or heat register. This product should not be placed in a built-in installation unless proper ventilation is provided.
9. Operate this product using the electrical voltage as stated on the base unit or the owner's manual. If you are not sure of the voltage in your home, consult your dealer or local power company.
10. Do not place anything on the power cord. Install the product where no one will step or trip on the cord.
11. Do not overload wall outlets or extension cords as this can increase the risk of fire or electrical shock.
12. Never push any objects through the slots in the product. They can touch dangerous voltage points or short out parts that could result in a risk of fire or electrical shock. Never spill liquid of any kind on the product.


IMPORTANT SAFETY INSTRUCTIONS

13. To reduce the risk of electrical shock, do not take this product apart. Opening or removing covers may expose you to dangerous voltages or other risks. Incorrect reassembly can cause electric shock when the appliance is subsequently used.
14. Unplug this product from the wall outlets and refer servicing to the manufacturer under the following conditions:
 - A. When the power supply cord or plug is frayed or damaged.
 - B. If liquid has been spilled into the product.
 - C. If the product has been exposed to rain or water.
 - D. If the product does not operate normally by following the operating instructions. Adjust only those controls that are covered by the operating instructions. Improper adjustment may require extensive work by a qualified technician to restore the telephone to normal operation.
 - E. If the product has been dropped or the case has been damaged.
 - F. If the product exhibits a distinct change in performance.
15. Never install telephone wiring during a lightning storm.
16. Never install telephone jacks in wet locations unless the jack is specifically designed for wet locations.
17. Never touch uninsulated telephone wires or terminals unless the telephone line has been disconnected at the network interface.
18. Use caution when installing or modifying telephone lines.
19. Use only the power cord and batteries indicated in this manual. Do not dispose of batteries in a fire. They may explode. Check with local codes for possible special disposal instructions.
20. The apparatus shall not be exposed to dripping or splashing and that no objects filled with liquids, such as vases, shall be placed on the apparatus.
21. The battery shall not be exposed to excessive heat, such as sunshine, fire or the like.
22. The MAINS plug is used as the disconnect device, which shall remain readily operable.

HomeAware™ Special Features

- D-RF Secure™ Digital Radio Frequency Technology
- Scrolling 2" extra large display
- Digital display scrolls the alert message across master unit and all deluxe remote receivers. Message will read as phone, door, baby, smoke etc.
- Digital display will scroll caller's name and number across display.
- Safe supervised monitoring: reports low battery and lost communication with all parts of the system
- Loud adjustable tone and volume
- Ultra bright strobe flash
- Full back up in the event of power outage
- Works with our powerful 12 volt Bed Vibrator.
- USB charger for all smart phones
- Bridge module allows for compatibility with Sonic Alert's traditional receivers, SA101, SA201, BL300, RH100, and SB1000.
- Smoke / CO2 detection*
- Digital long range remote pager*
- Weather radio transmitter*
- CellAware smart phone alert transmitter

HomeAware™ Identification Features

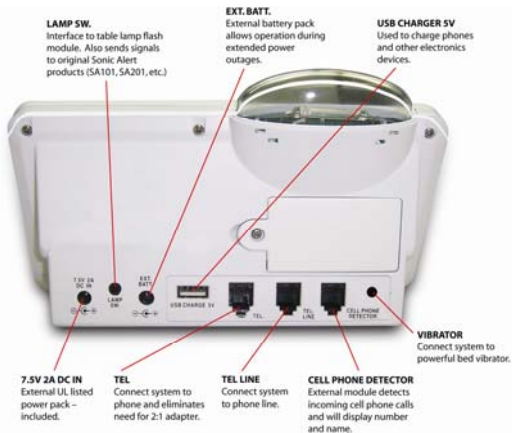

Operations

Setting the Time of Day:


1. Press the Time set button. Blinking dots indicate time set mode has been activated.
2. Press blue arrow Hour buttons up or down and the Minute blue arrow buttons up or down to set desired time and press the Enter button.
 - Dots will stop blinking to indicate time has been set.

Test Mode (Testing Audio/VIB/Strobe Flash /Lamp):


1. Press Enter and Back buttons together, the word TEST will be displayed. The audio, flash, vibrator and lamp will be activated. Press and hold the yellow SNOOZE button to exit and the word DONE will display.

LED Light Indicators

**First Column
LED Indications**

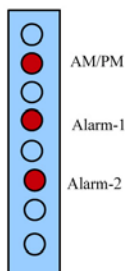

**Last Column
LED Indications**

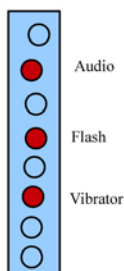

Setting Alarm 1

1. Press and hold AL.1 Set. Blinking dots indicate Alarm set mode has been activated.
2. Press the blue arrow Hour buttons up or down and the blue arrow Minute buttons to set desired alarm time and press the Enter button. Red LED light will appear on the left side indicating alarm 1 time is set.

Setting Alarm 2

1. Press and hold AL.2 Set. Blinking dots indicate Alarm set mode has been activated.
2. Press the blue arrow Hour buttons up or down and the blue arrow Minute buttons up or down to set desired alarm time and press the Enter button. Second red LED dot appears on the left side indicating alarm 2 time is set.
 - When alarm goes off, audio, flash, vibrator and amp will be activated in patterns to alert user.
 - Alarm Off or SNOOZE can be pressed to shut alarm off.

Setting Snooze Time

1. Press Set Up button and CLOCK will be displayed. Then press the Enter and SNOOZE will be displayed, press Enter and then press blue Tone button to set desired snooze duration, press Enter.

Setting Alarm Duration

1. Press Set Up button and CLOCK will be displayed. Then press the Enter and 12::24 will be displayed, then press the blue Tone button to see the BUZZER mode, hit Enter and then the blue Tone button to desired duration.

Setting Audio, Flash and Vibrator Alarm

1. Press Set Up button and CLOCK will be displayed. Then press the blue Tone button until ALARM1 is displayed. Press the Enter button and AUDIO will be displayed. Press the Enter button and ON::OFF will display, and then press the blue Tone to turn the audio Off or ON and then press the Enter button.
2. Press Set Up button and CLOCK will be displayed. Then press the blue Tone button until ALARM1 is displayed. Press the Enter button and FLASH will be displayed. Press the Enter button and ON::OFF will display, and then press the blue Tone to turn the FLASH Off or ON and then press the Enter button.
3. Press Set Up button and CLOCK will be displayed. Then press the blue Tone button until ALARM1 is displayed. Press the Enter button and VIB will be displayed. Press the Enter button and ON::OFF will display, and then press the blue Tone to turn the VIB OFF or ON and then press the Enter button.
4. Follow the same steps when setting ALARM2.

Select 12 or 24 Hour Mode

1. Press the Set Up button and the word CLOCK will be displayed. Then press the Enter button and 12::24 will display. Press the blue Tone button to toggle between 12 HR or 24 HR mode and then press the Enter button when desired mode is displayed.

Speaker Volume / Tone Setting:

1. Press the blue Volume up/down buttons to select volume level. There are 7 different volume levels and an OFF.
2. Press the blue Tone up/down buttons to select desired tone. There are 11 different tones.

Testing Light Sensitivity:

1. Place hand over sensor to create darkness. Different Led brightness levels can be seen in response.

Power Failure and Battery Switching:

Low Bat Screen

1. LOW BAT will be seen on display if battery is not connected. Connect Ext battery or press the BACK button to disable displaying LOW BAT. If battery voltage is less than 5V, LOW BAT will be displayed.
2. Connect external battery or install 5 AA batteries in battery compartment.
3. Disconnect mains power supply.
4. LED display will be dimmed. Upon any detection such as alarm or telephone the display will be at maximum brightness and after execution of events, display will go dim again.

USB Charging:

1. Connect the USB cable and verify charging with external device.

HomeAware™ Accessories

Transmitters

HA360DB – Doorbell
HA360SS – Sound Signaler
HA360BC – Baby Cry
HA360SA – Smoke Alarm
HA360C0 – CO2
HA360DK – Door Knock

Other

HA360CA – CellAware
HA360WA – Weather Alert
HA360P – Pager
HA360V – Vibrator
HA360BU – Bridge Unit
HA360LF – Lamp Flash

Regulatory Compliance

PART 68 OF FCC RULES INFORMATION

The HA360M complies with Part 68 of FCC Rules. You must, upon request, provide the FCC registration number and REN to you local phone company. Both numbers are shown on the HQA360M's rating label.

PART 15 OF FCC RULES INFORMATION

The HA360M complies with part 15 of the FCC rules, and with RSS-210 of Industry Canada. Operation is subject to the following two conditions; (1) This device might not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.


Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide responsible protect against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if jot installed and used in accordance with the instructions manual, may cause harmful interference to radio communications. However, there is no guarantee that interference will not interfere in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment on and off, the user is encouraged to thy to correct the interference by one or more of the following measures:

- i. Reorient or relocate the receiving antenna.
- ii. Increase the separation between the equipment and receiver.

- iii. Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- iv. Consult the dealer or an experienced radio /TV technician for help

Modification not expressly approved by the manufacturer could void the user's authority to operate the equipment under FCC rules.

GENERAL DATA

UL listed Power Adaptor: AC100-240V~, 50/60Hz
Power Consumption: Adaptor Output: DC9V ,
650mA



The Lightning flash and arrowhead within the triangle is a warning sign alerting you of dangerous voltage inside the product.

CAUTION:
TO REDUCE
THE RISK
OF ELECTRICAL
SHOCK, DO NOT
REMOVE THE
COVER
(OR BACK)
FROM THE
PRODUCT. NO
USER
SERVICEABLE
PARTS
INSIDE. REFER
SERVICING TO
YOUR SONIC
ALERT DEALER.



The exclamation mark within the triangle is a warning sign alerting you of important instructions accompanying the product.


**WARNING: TO PREVENT FIRE OR SHOCK HAZARD,
DO NOT EXPOSE THIS APPLIANCE TO RAIN OR
MOISTURE.**

The normal function of the product may be disturbed by Strong Electro Magnetic Interference. If so, simply reset the product to resume normal operation by following the instruction manual. In case the function could not resume, please use the product in other location.

GENERAL DATA

- Attention should be drawn to the environmental aspects of battery disposal.



-  This indicating separate collection for electrical and electronic equipment.
-

SONIC ALERT'S 1 YEAR LIMITED WARRANTY

The Sonic Boom Alarm Clock Model SBB500ss is warranted against manufacturing defects in materials and workmanship for one (1) year from the date of purchase. Within this period Sonic Alert will repair or replace at our option the SBB500ss without charge for parts and labor.

Simply contact our customer service department to obtain an RA (return authorization number) and instructions on how to process your product for return. Call 1-888-864-2446 or e-mail to RMA@sonicalert.info. You'll need a copy of your receipt to process the return authorization.

Sonic Alert Inc., Troy MI 48083, USA.



Made in China

Sonic Alert, Troy, MI 48083

www.SonicAlert.com 1-888-864-2446