




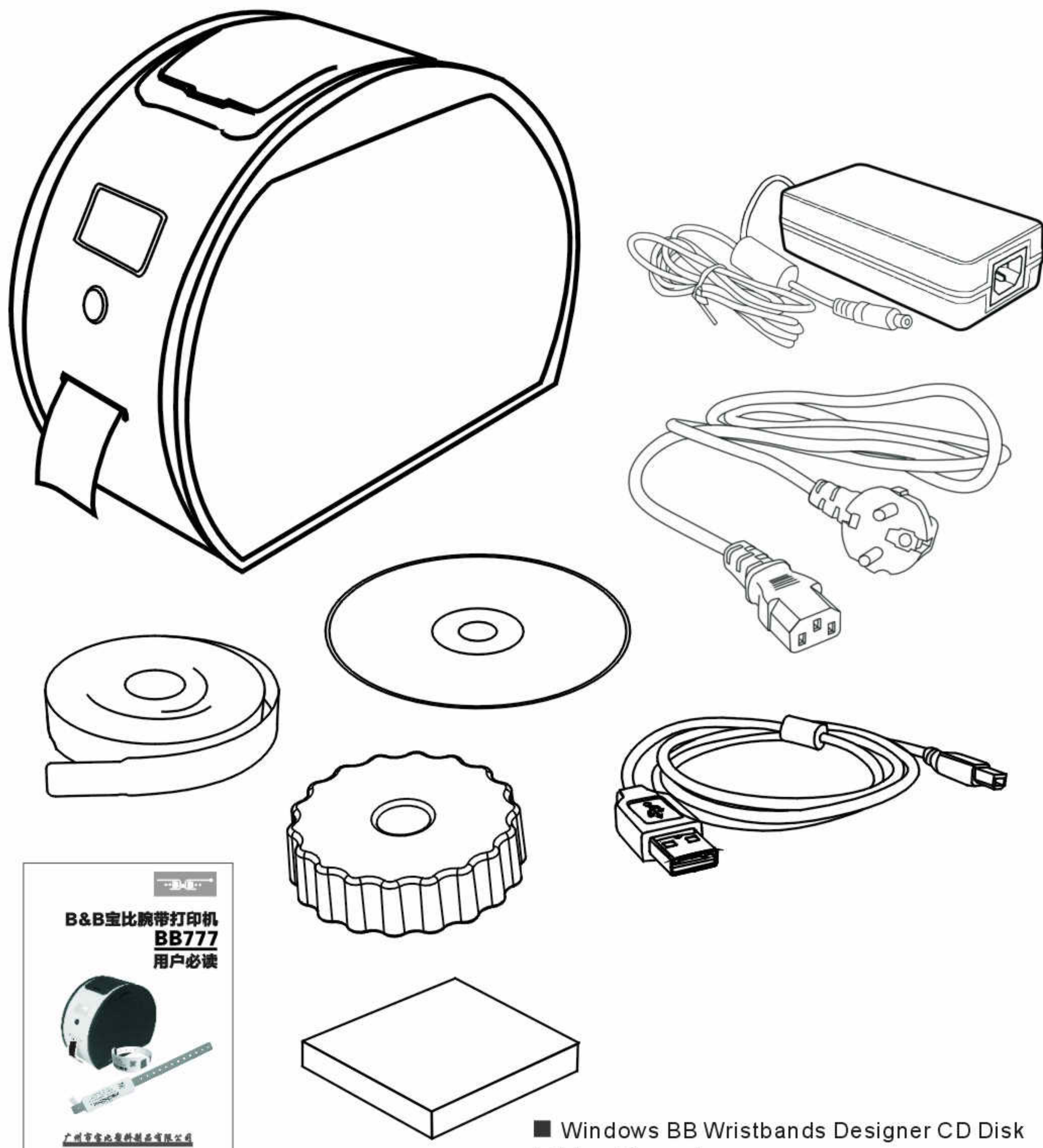

- Windows BB Wristbands Designer CD Disk
- User Manual
- Warranty Card
- USB port cable
- External universal switching power supply
- Power cord
- B&B Direct Thermal Printable Wristbands
- Print Head Cleaning Sponge
- Wristbands Rolls mounted case




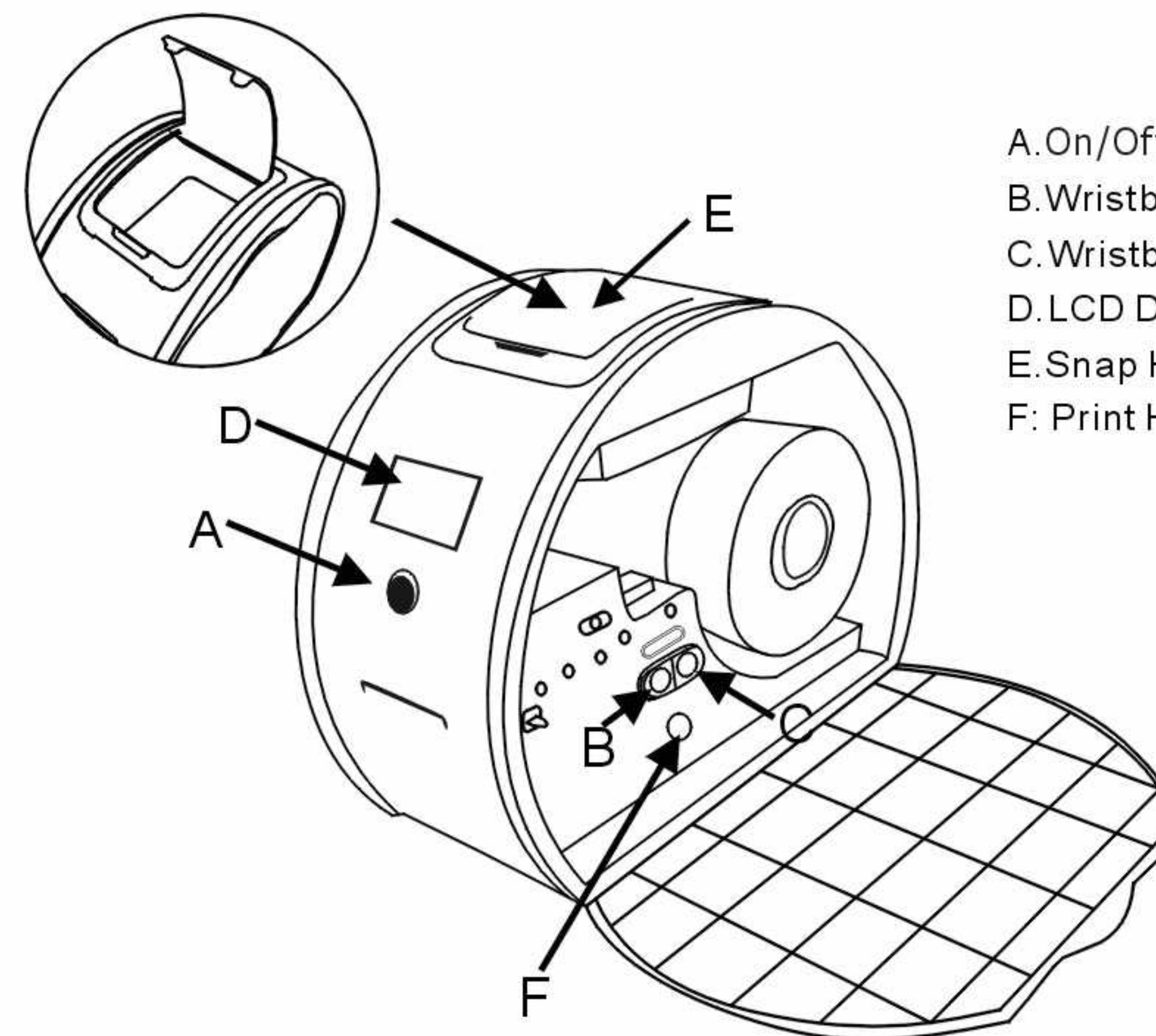


- A. On/Off button
- B. Wristband feed button
- C. Wristband back button
- D. LCD Display
- E. Snap Holder
- F. Print Head Clean button

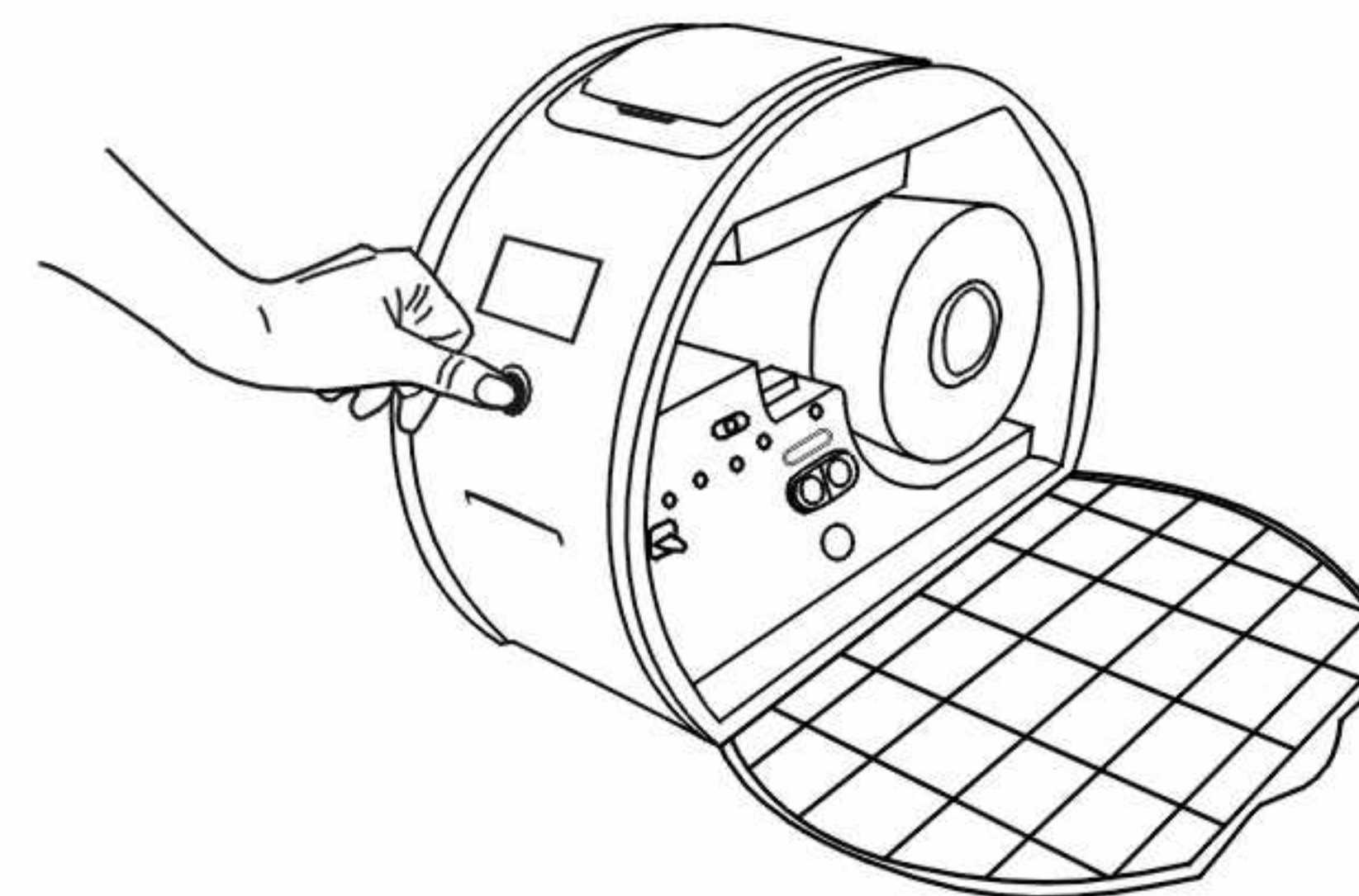


- START: to start printer, press the On/Off button once;
- TURN OFF: to turn off printer, press and hold the On/Off button for a few seconds;
- RESTART COUNTING: to restart printed quantity counting, press the On/Off button once when printer is ready
- Print Head Clean: to clean the print head, press this button once



Wristbands loading step

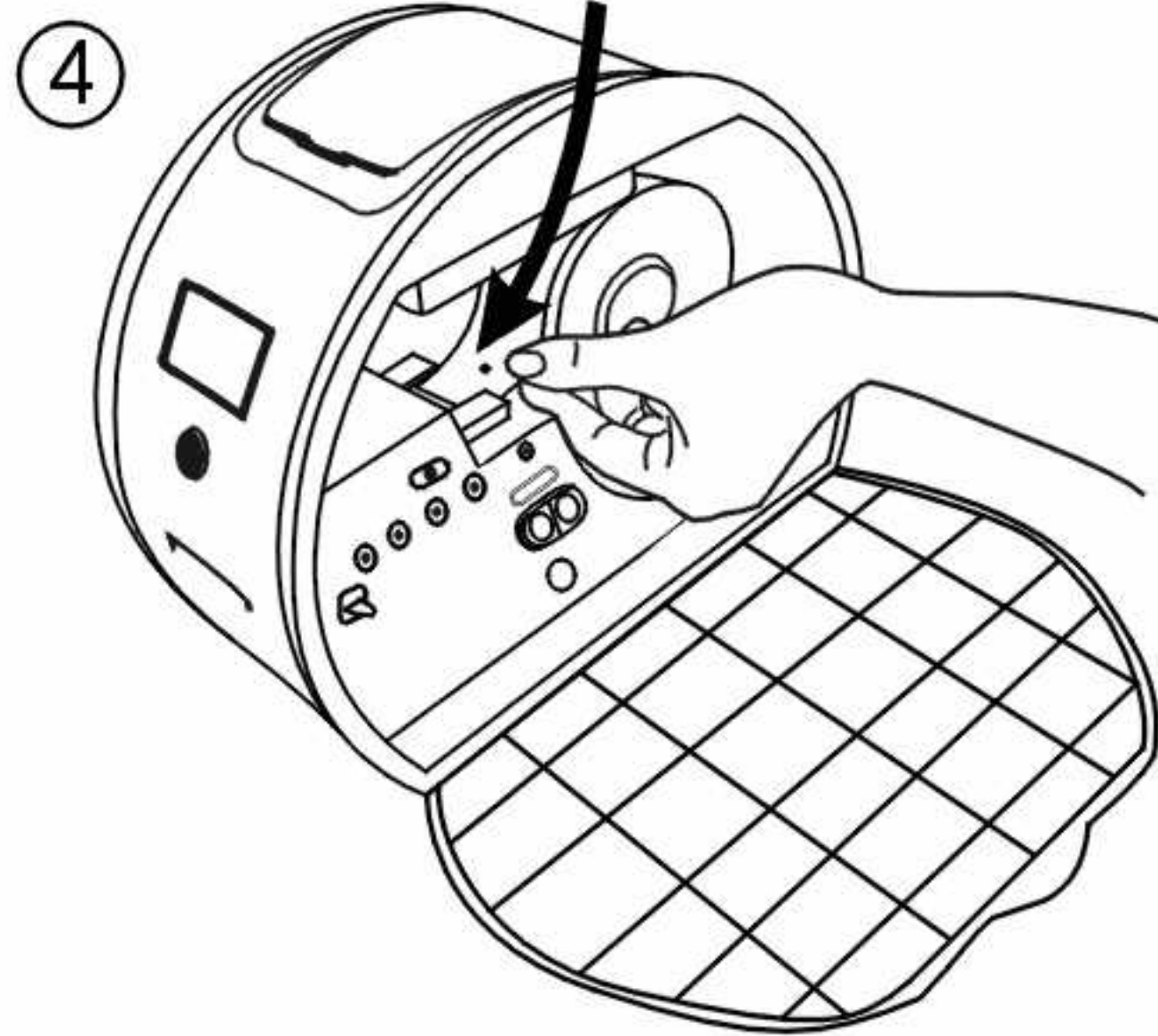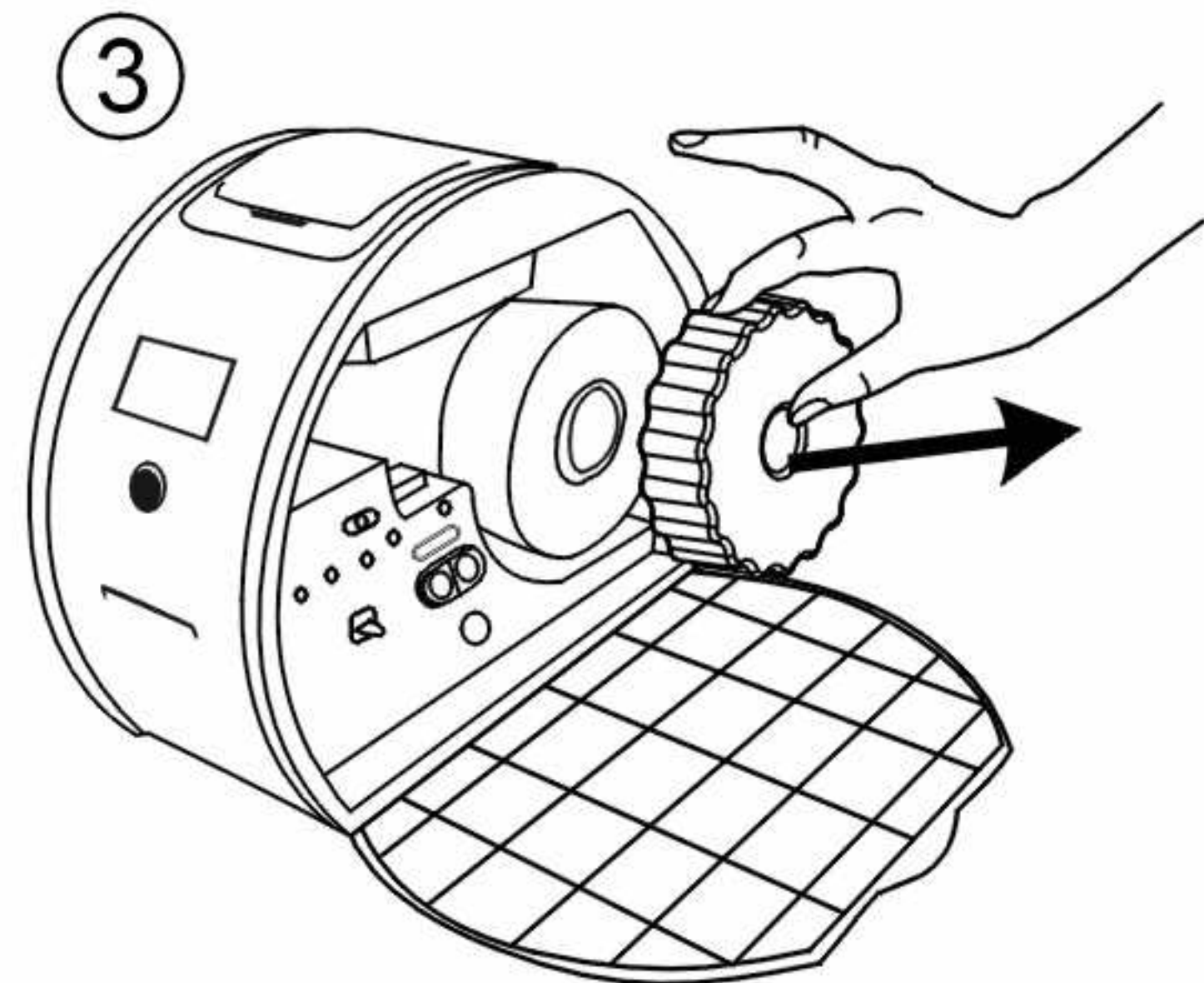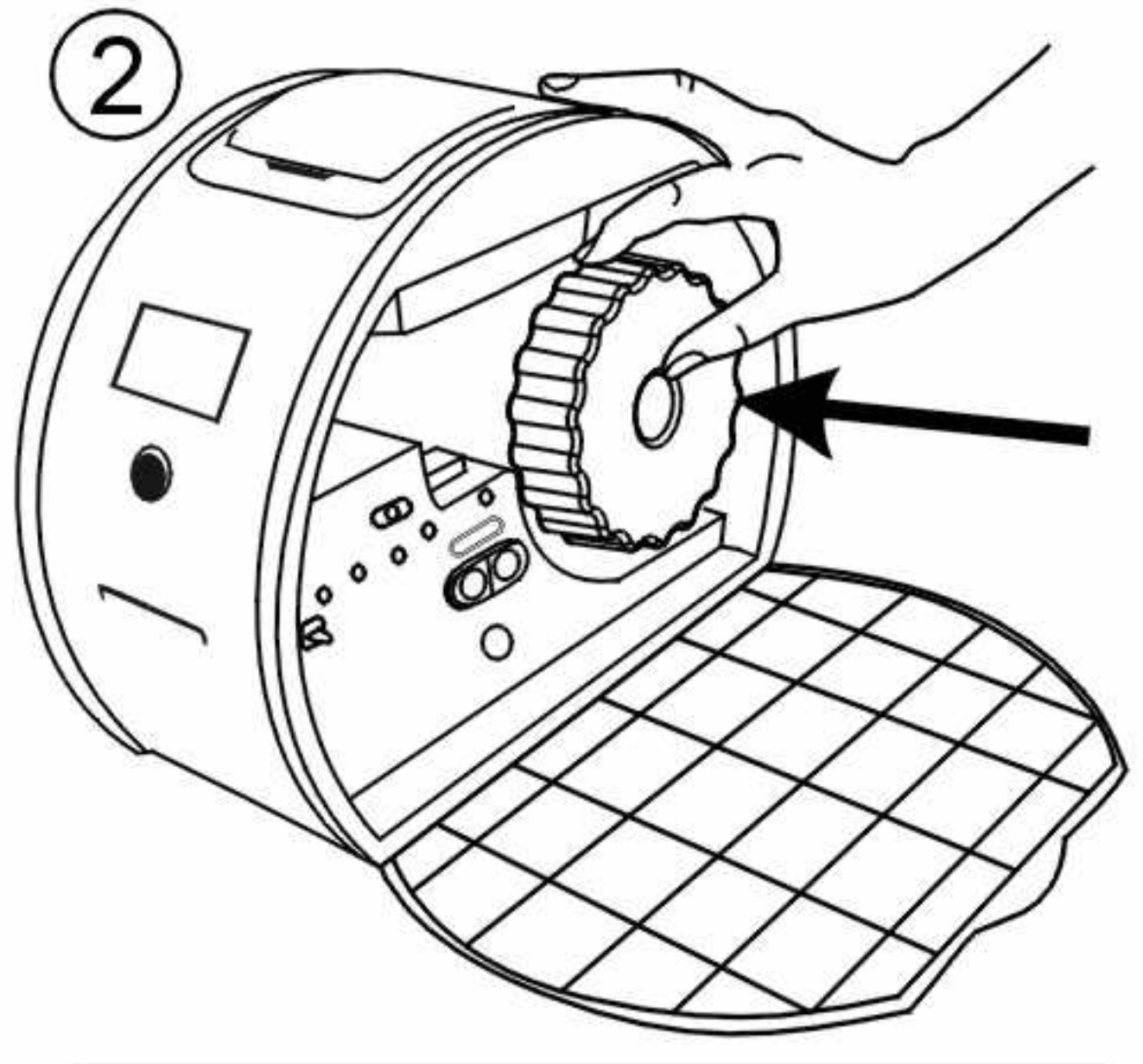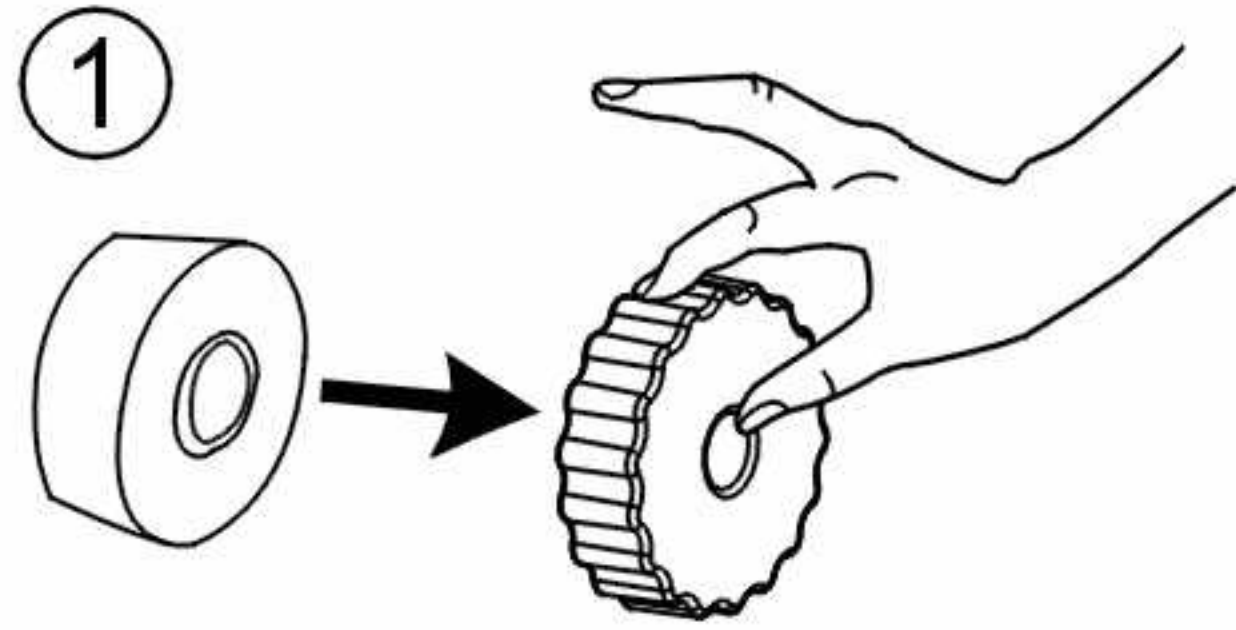


Press WRISTBAND FEED BUTTON,
wristbands automatically move to the
right printing place , LCD display
"PRINTER READY"

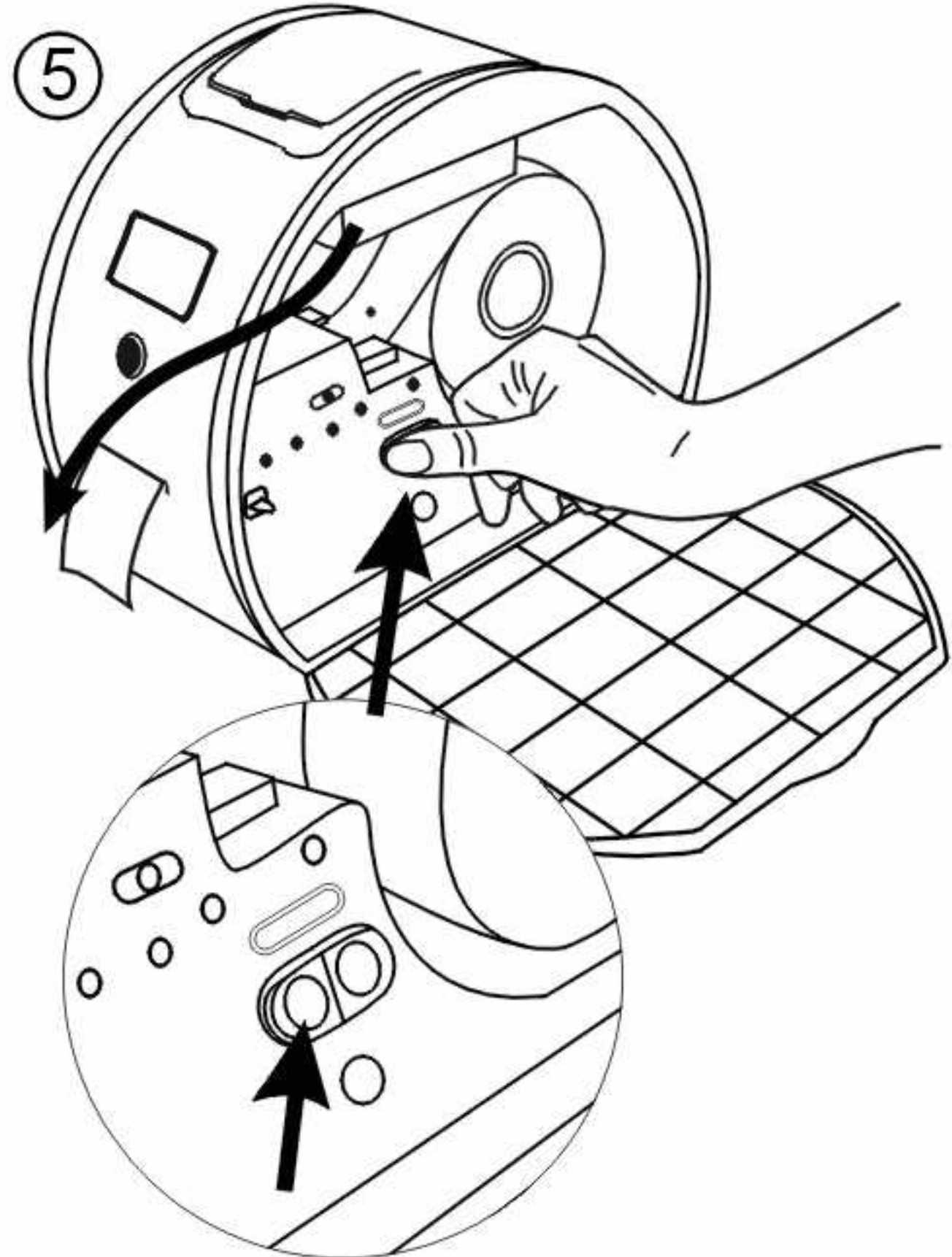


Change wristband rolls

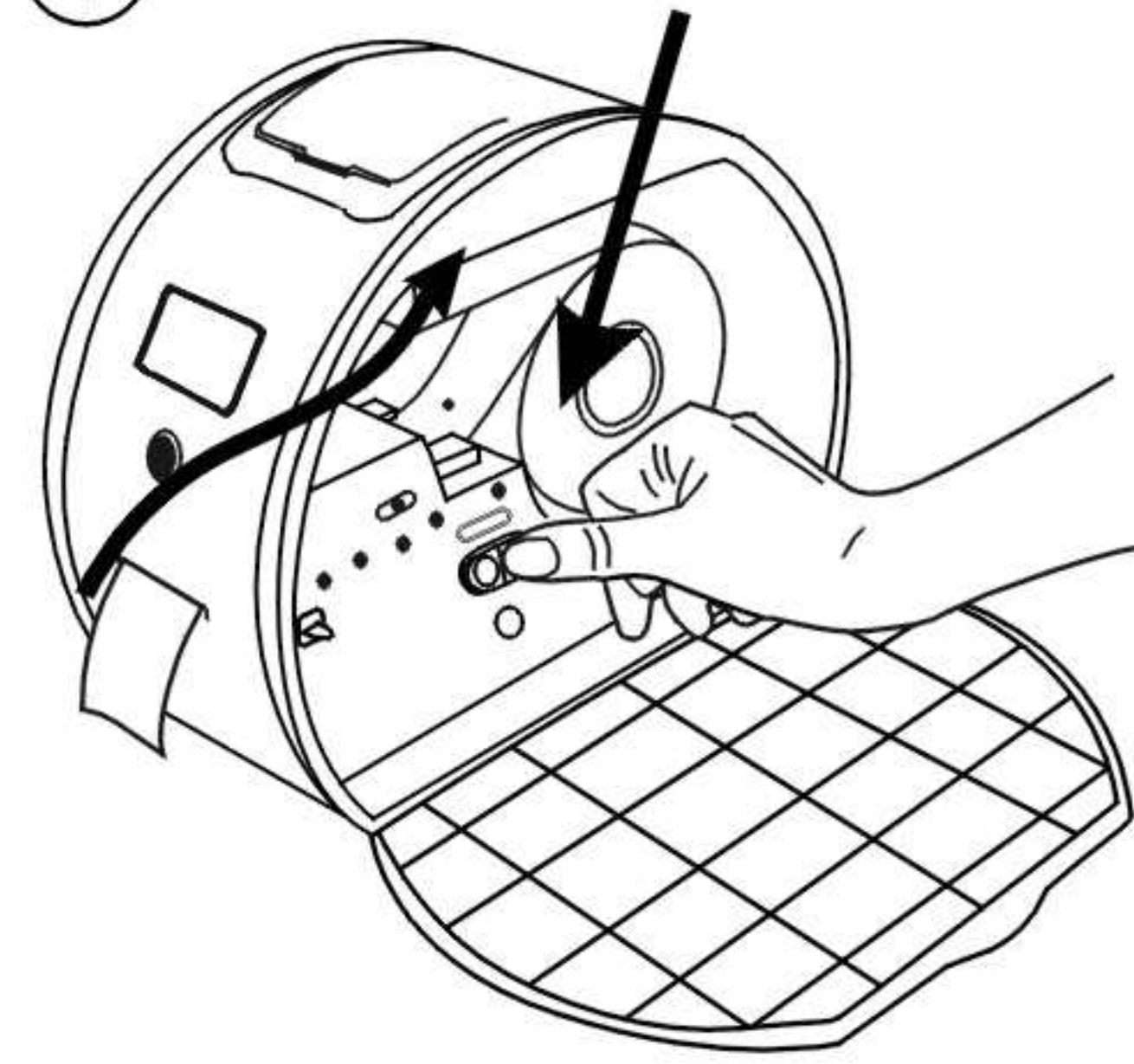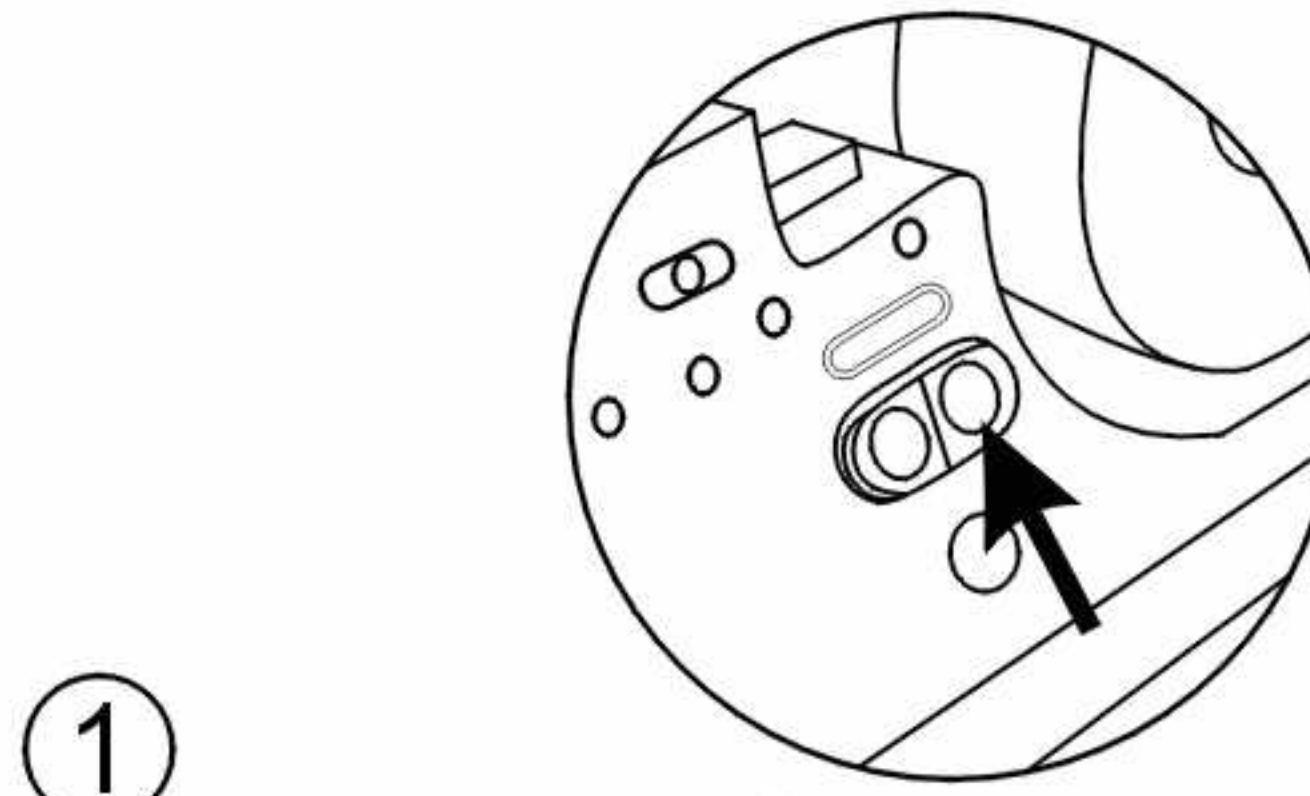

Press WRISTBAND BACK BUTTON,
wristbands automatically move back
to the roll.

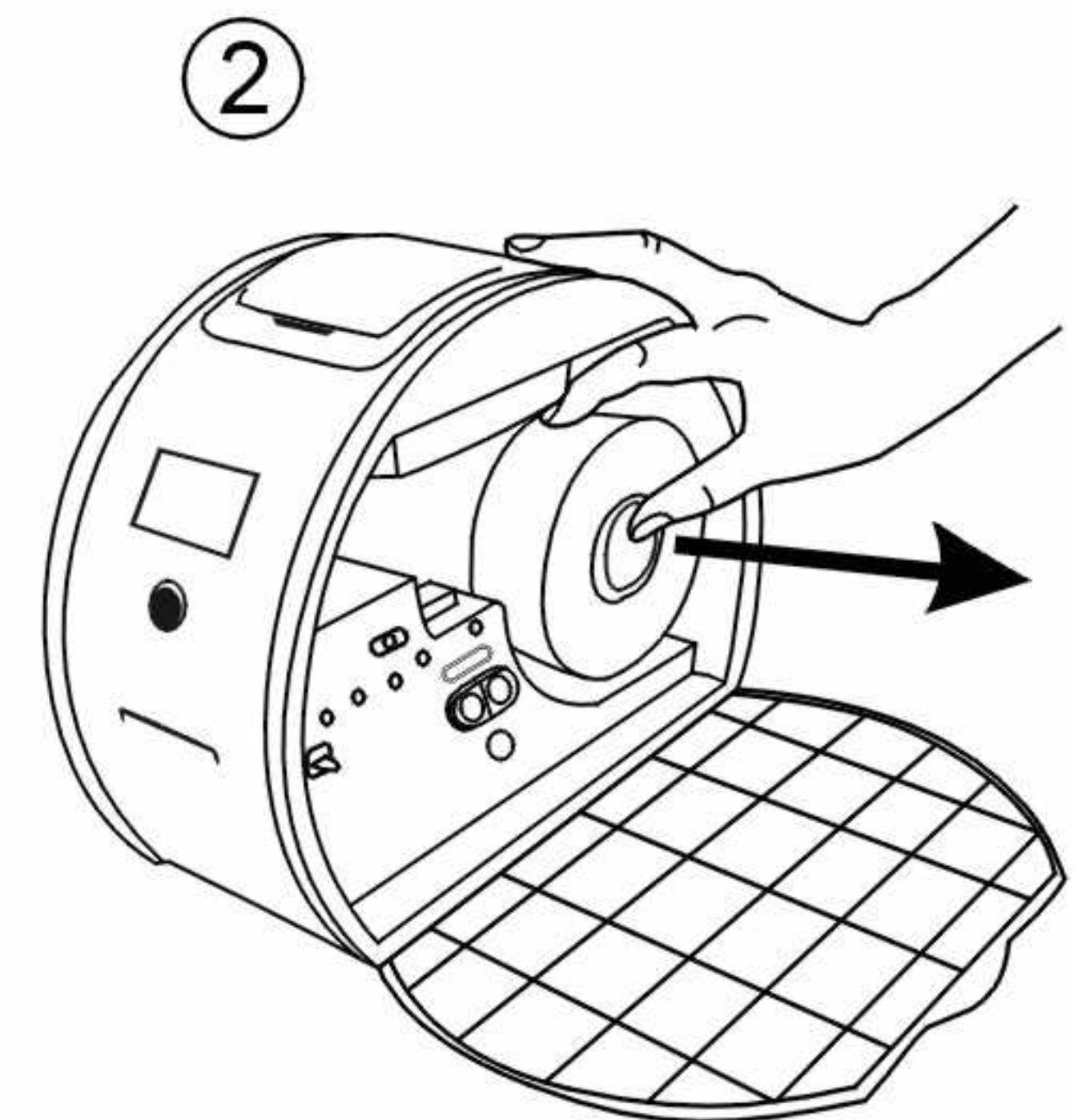


Print Head Cleaning Steps

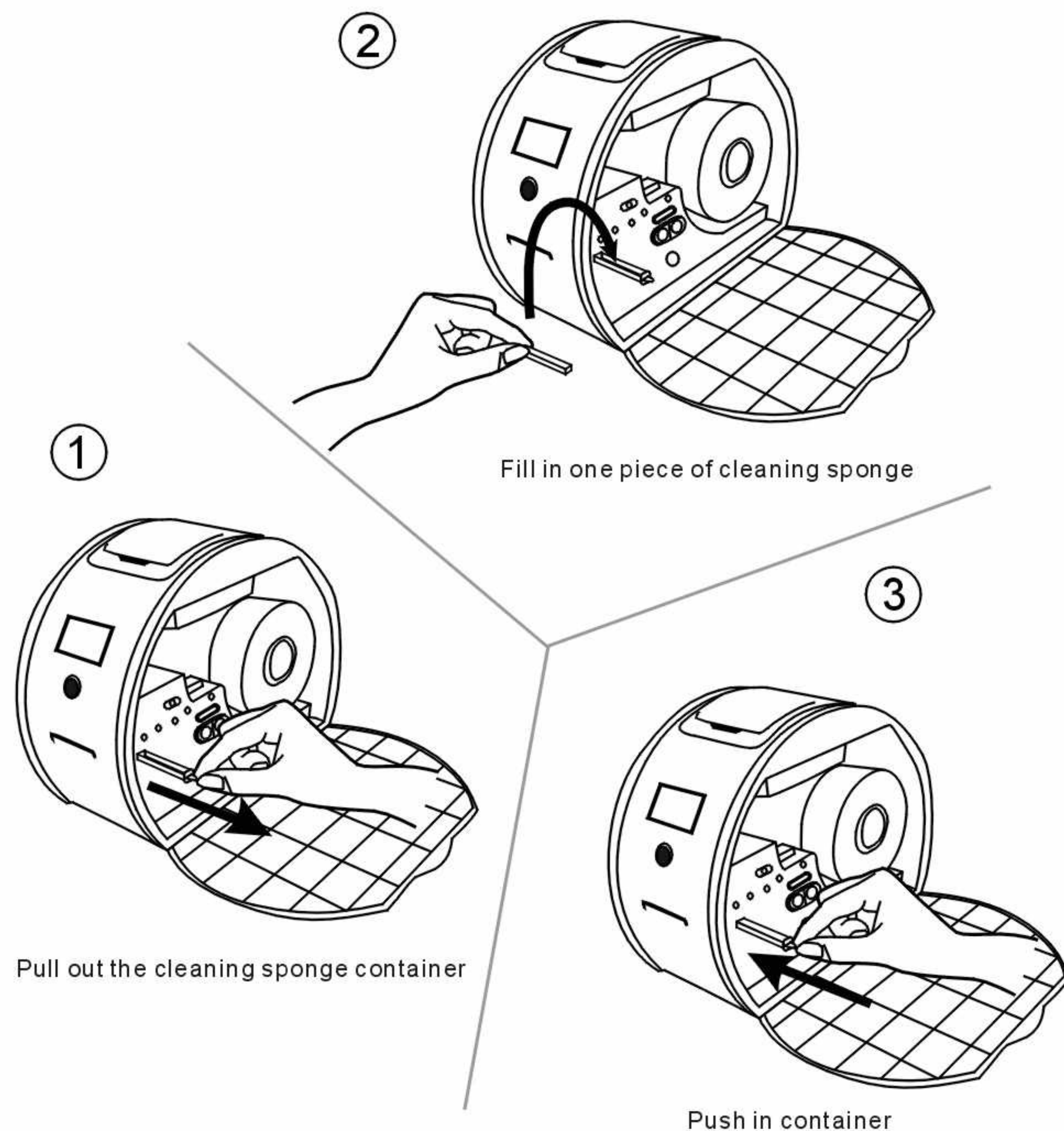

Caution : Do not clean the Print Head too often.


Print Head Cleaning Steps

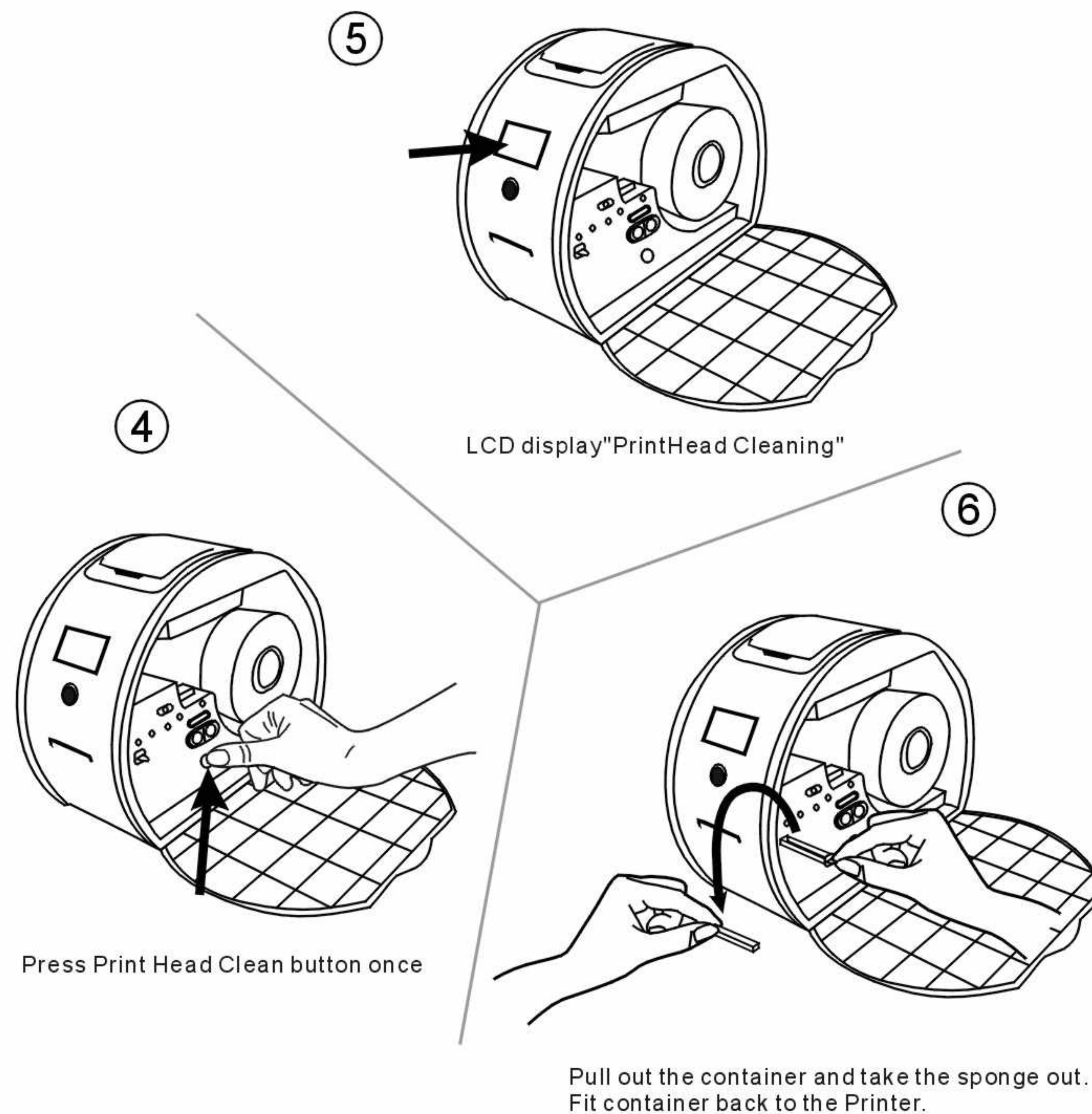


Caution : Do not clean the Print Head too often.

NO	Printer Error Status	Analysis of causes	Resolution	Remark
1	No LCD Display.	The printer is not receiving power.	Check the connection of Power Cord, Adapter and USB.	Need to disassemble the printer for check. Ask the authorized reseller for help.
		The LCD does not connect well.	Check LCD ribbon cable and reconnected.	
2	Press "Wristband Feed button, the wristbands do not move forward.	Check if the 2pcs rubber roll(platen) can turn around.	Either of the rubber roll can move.	
			One piece move, the other do not fit the right size.	
3	The printer turns on all right, but no wristband print out and printer do not respond.	Check if motor has had a critical failure.	Change the Motor.	Need to disassemble the printer for check. Ask the authorized reseller for help.
		Check if motor does not connect well.	Check the whole wiring.	
		The printer doesn't receive data.	Check if the USB cord connect well, and reinstall the Printer Driver.	
4	The Print Image does not look right.	The print head has worn out.	Change the Print Head	See the diagram in above page.
		The printhead is dirty.	Press "Wristband Feed button" and "Wristband Back button" 4-5 times.	
			Clean the print head.	
5	The printing does not start while everything's ready well.	Errors in wristband sensor.	Change the printer sensor.	Need to disassemble the printer for check. Ask the authorized reseller for help.
6	Missing print on several wristbands.			
7	No print on the wristband.	The media may not be direct thermal media.	Determining the thermal media types.	
		The Print head connect correctly.	Check if print head connect well to the key panel.	


Adhesive closure

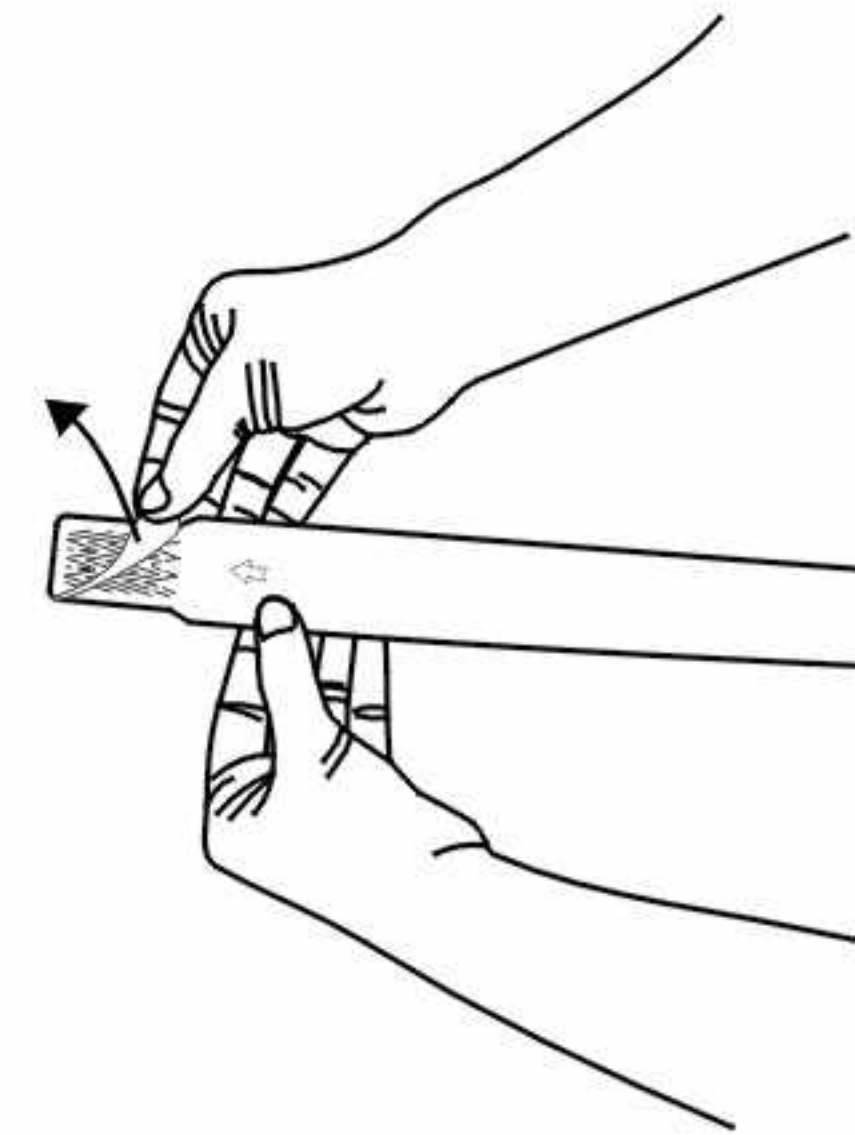


①

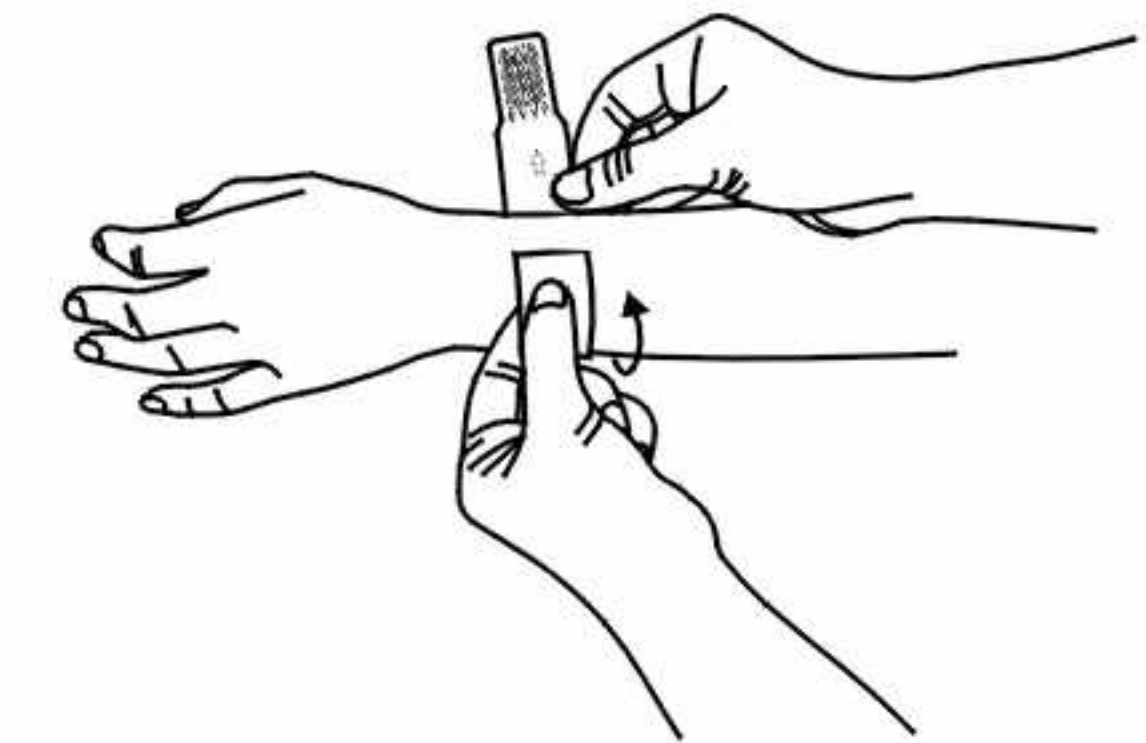

②

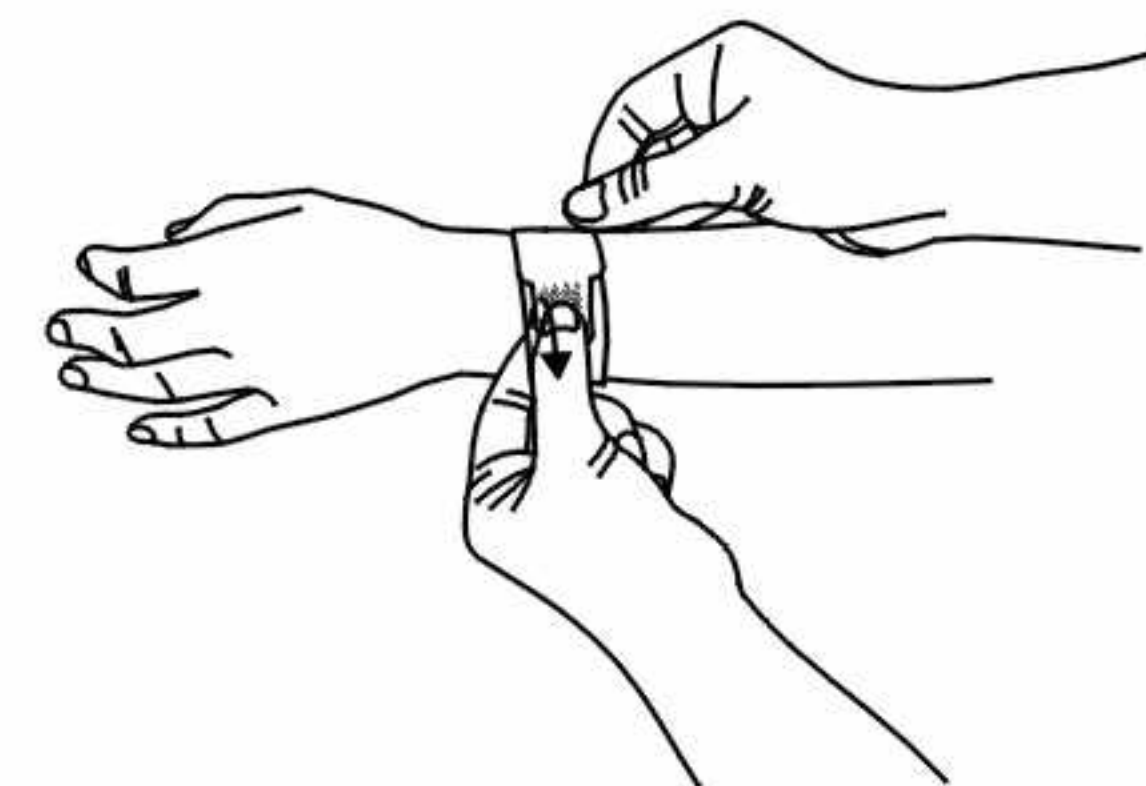


③

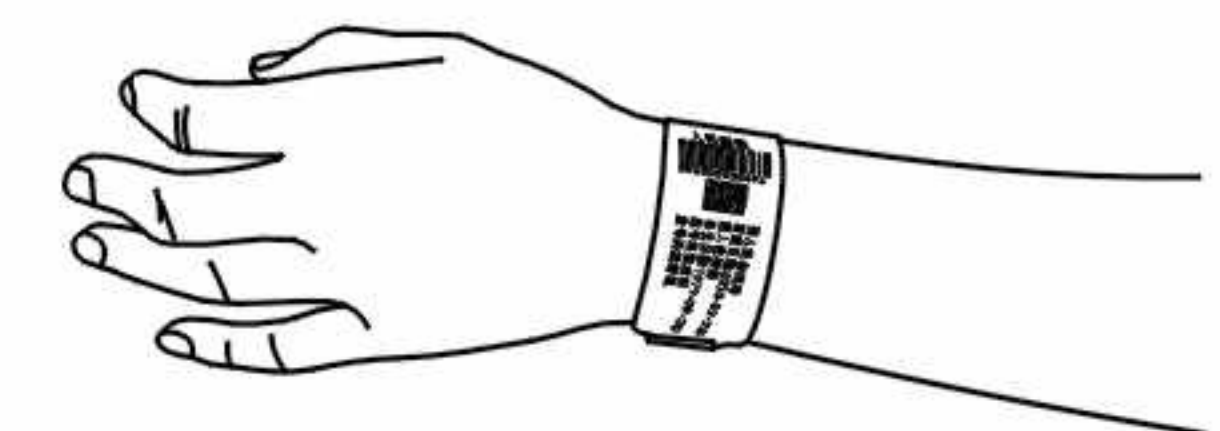


④


Plastic Snap

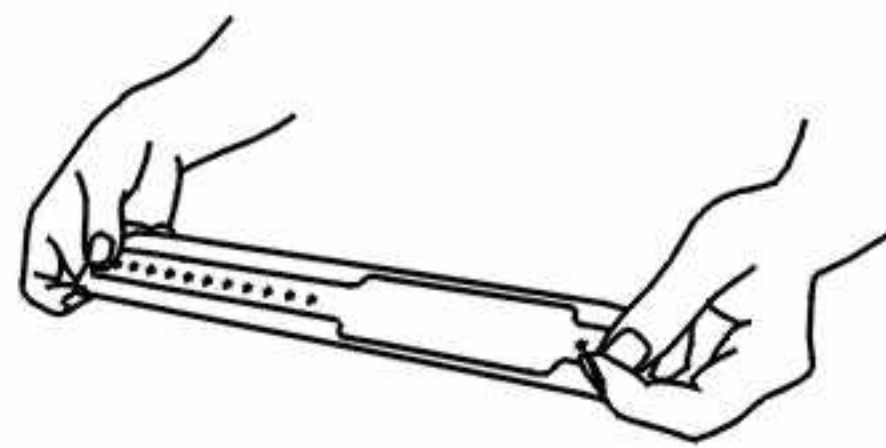


①

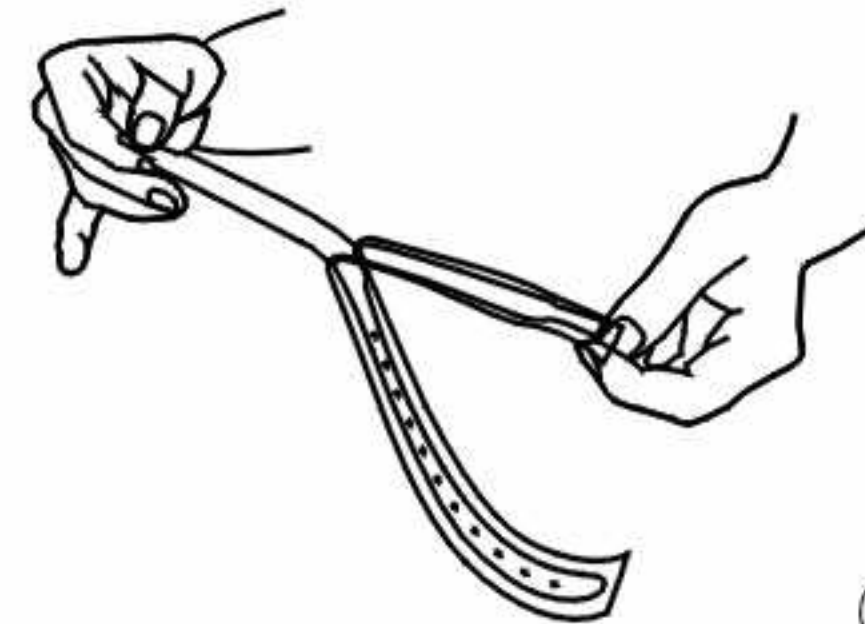


②

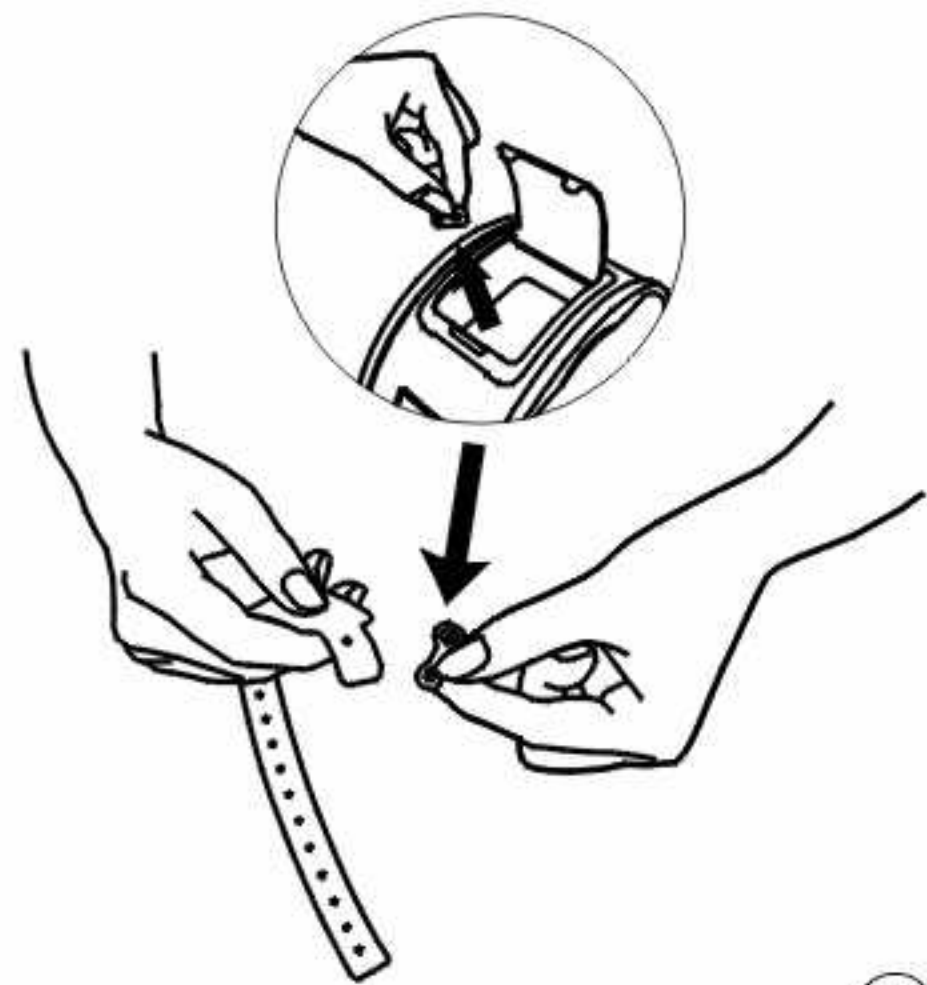


③

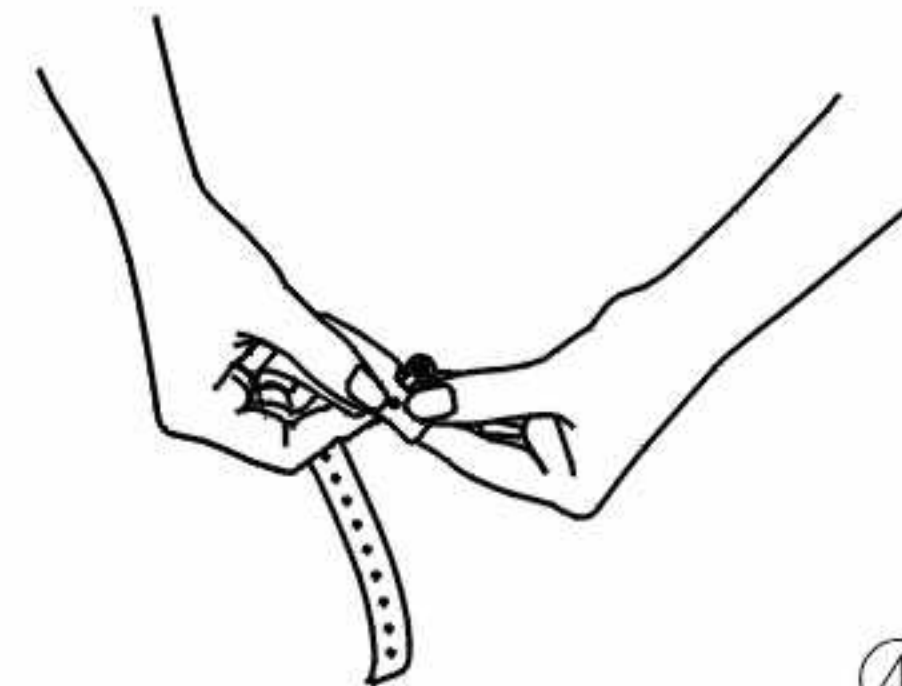

④

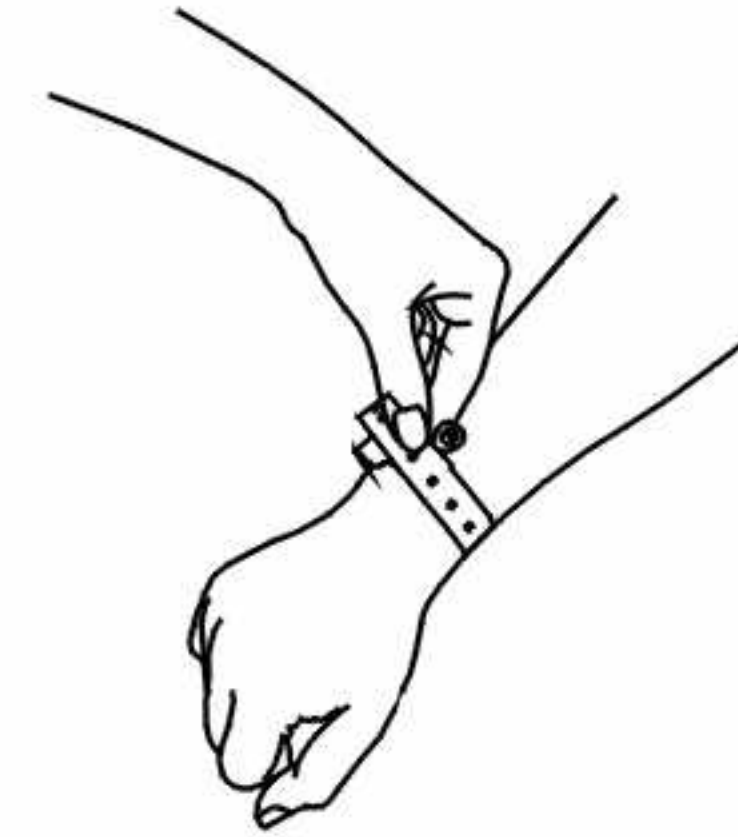

⑤

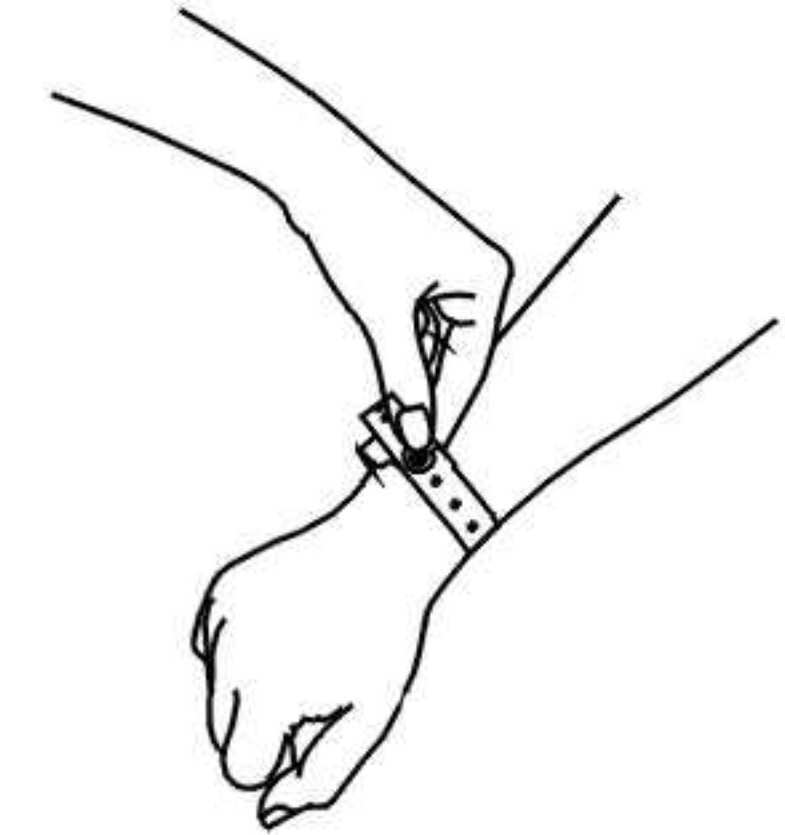

⑥



⑦



⑧

*B&B Identification Wristbands

Item No.	Size	Closure	Available Colors
BVP14210	Adult(295mm*25mm)	Adhesive	White/Red/Green/Blue Yellow/Orange/Purple/Pink
BVP14220	Adult(295mm*19mm)	Adhesive	
BVP14270	Infant(230mm*19mm)	Adhesive	
BVP14230	Adult(285mm*25mm)	Snap	
BVP14250	Adult(285mm*29mm)	Snap	
BVP14260	Adult(285mm*19mm)	Snap	

*B&B Patient Identification Wristbands

Item No.	Size	Closure	Available Colors
BVP14410	Adult(295mm*25mm)	Adhesive	White/Red/Green/Blue
BVP14420	Adult(295mm*19mm)	Adhesive	White/Red/Green/Blue
BVP14320	Adult(295mm*29mm)	Adhesive	White/Red/Green/Blue
BVP14370	Infant(200mm*19mm)	Adhesive	White/Light Blue/Pink
BVP14340	Adult(285mm*19mm)	Snap	White/Red/Green/Blue
BVP14350	Adult(285mm*29mm)	Snap	White/Red/Green/Blue
BVP14380	Infant(180mm*19mm)	Snap	White/Light Blue/Pink