

Users' Manual for Nasco Auscultation Remote Control

Summary:

The remote control is the main user interface, where the mode of operation is selected and the display gives the user feedback about current operation.

Operation:

- To begin using the Auscultation system, press the red Power button. This turns on the remote control and sends a signal to the stethoscope to activate it, as well. After the unit is activated, the display will be in the "status" mode, displaying the current menu settings for the heart and lung conditions.
- To select a new condition, press either the Heart or Lung button. This will put the display into the menu mode. The user can select a condition by either using the number buttons, or by viewing the conditions in sequence using the Scroll button.
- When the desired conditions for heart and lung are selected, press the Enter button to transmit the new set of conditions to the stethoscope. When the Enter button is pressed, the display will return to status mode. A transmission icon shows that the signal is being sent to the stethoscope.
- To turn off the unit, press the Power button.

Unit uses two AAA alkaline batteries

WARNING: NO USER SERVICEABLE PARTS INSIDE CASE