The pressure sensitive 'Baby Button' warns you if your baby is face down and at risk. A sound alarm and flashing light indicate that your baby has rolled over.

Up to 150 feet range and belt clip means you are free to move around your home.

REDUCING THE RISKS

3 keys to reducing the risk of SIDS for your baby

1. Check your baby's sleep position - back is best
2. Make sure your baby's head remains uncovered during sleep
3. Keep your baby in a smokefree environment, before birth and after.

For more information about SIDS and reducing the risks, contact the SIDS organization in your state.

Manufactured by Millennium Electronics, Melbourne, Australia.
Designed in Australia

Reducing the risks...

*Sound-Asleep...the only
baby monitor designed
to help reduce the risk of SIDS*

Instruction Booklet

3rd MILLENNIUM

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses and can radiate radio frequency energy and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment causes harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the equipment and receiver
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected
- Consult the dealer or an experienced radio TV technician for help

Warning: Any changes or modifications not expressly approved by the party responsible for compliance (Grantee) could void the user's authority to operate this equipment

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

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The #1 Risk

Worldwide research confirms that the greatest risk to your baby from Sudden Infant Death Syndrome (SIDS) is your baby's sleeping position.

Today, all new parents and carers are made aware of the risks and are advised that the safest sleeping position for babies (up to 24 months) is on their back.

Peace of mind

Sound-Asleep has been developed in association with leading pediatricians to warn if your baby has rolled over and is at risk. Sound-Asleep is simple to use. A small pressure sensitive button worn on baby's clothes, signals the remote alarm- up to 150 feet away - indicating that your baby is face down and at risk. Even slightly pressing the button sets off a warning light and sound alarm which continues to increase in volume until you reset the monitor. Your baby can be repositioned safely without waking - Safe and Sound-Asleep. Of course, the Sound-Asleep is not a "cure" for SIDS. It has been designed to assist in caring for your baby and is not intended as a substitute for regular supervision.

Features

- Compact, cordless and portable, Sound-Asleep's 150 feet range leaves you free to move around your home or garden
- Warning light and sound alarm
- Rechargeable (Batteries included)
- Simple to use
- Reliable - tested in leading hospitals
- Meets international quality standards
- Built in battery level indicator

ON/OFF Button

Press and hold button down until a short "Beep" sounds and indicator lights blink.
The unit is now on "Standby" (Battery Save mode) with no indicators lit or flashing.
Press and hold button down again until a long "Beep" sounds and indicator lights flash green. The monitor is now active.

RESET (Alarm Reset) Button

When alarm sounds, pressing this button for 2 seconds will stop the flashing light and sound alarm and reset the monitor.

Monitor Indicator Light

- | | | |
|--------------|--|--|
| Unlit | - battery is OK | |
| Flashing Red | - battery is low and should be charged | |
| Green | - battery is recharging | |

Baby Button Indicator Light

- | | | |
|----------------|---------------------------------------|--|
| Flashing Green | - monitor is active | |
| Flashing Red | - baby button battery needs replacing | |

Battery (ON OFF) Switch (Inside battery compartment)

When monitor is not in use for extended periods, switch off the rechargeable battery by sliding switch to the right.

REMEMBER

- When you push the "ON/OFF" button:
- Short "Beep" and no lights flashing means the monitor is on "Standby"
 - Long "Beep" with flashing green light means the monitor is "Active".

How To Use Your Sound-Asleep Monitor

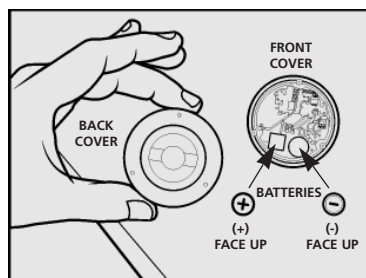
Before using your monitor the batteries will need charging. Typically, this will take 10-12 hours to fully charge.

1. Remove the pull out strip from the baby button.
2. Remove battery cover from the monitor case and locate the battery switch labelled "ON OFF". Slide switch to the left ('On' position) and push the adjacent "RESET" button and replace cover.
3. Connect recharger unit to the monitor and plug into the wall socket / power point - switch power on.
4. Check the monitor indicator lights are flashing green and red.
5. Gently press onto the baby button and the monitor will give a short "Beep" acknowledging the baby button.

Press and hold down external "ON/OFF" button on the monitor until a short "Beep" sounds.
7. Press and hold down external "ON/OFF" button on the monitor until a long "Beep" sounds. Check that the Monitor Indicator light is green to confirm that the monitor is charging. When fully charged (10-12 hours), disconnect charger from the monitor.
Battery will typically remain charged for 10 days.
8. The Baby Button Indicator Light will now be flashing green to indicate that the monitor is active. Gently press onto the baby button and the monitor should beep.
9. Clip the baby button to baby's clothing - middle of the chest is best.
10. When not in use, return the monitor to Standby (Battery Save mode). Press and hold the monitor external "ON/OFF" button until a short "Beep" sounds and indicator lights blink. The unit is now on "Standby" (Battery Save mode) with no indicators lit or flashing.
11. To reactive the monitor, press and hold down external "ON/OFF" button on the monitor until a long "Beep" sounds.

Baby Button Battery Replacement Instructions

1. Using a small screwdriver remove the three screws on the back of the baby button.
2. Carefully remove the back cover.
3. Remove the two batteries from the baby button.
4. Place the front cover of the unit on a flat surface. The printed circuit board should be facing upward.
5. Place the new batteries onto the terminals located on the printed circuit board ensuring that the batteries are correctly orientated. The round shaped terminal requires the battery to be located negative (-) side face up. The square shaped battery terminal requires the battery to be located positive (+) side face up.
6. Leaving the front cover on the table, carefully replace the back cover ensuring that the screw holes are correctly located on the screw bosses and the terminals on the back cover locate over the batteries.
7. Carefully pick up the baby button holding the two halves tightly and check that they fit together properly.
8. Place the baby button on the table face downwards and replace the screws. Do not over tighten.
9. Check the unit for correct operation by following the "how to use" instructions.
10. Your Sound-Asleep baby monitor is ready for use.

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Indicator lights for both "Baby Button" and monitor show battery charge level and indicate that the monitor is operating (left indicator "Monitor Indicator, right indicator "Baby Button")

Removable "Baby Button" clips to baby's clothing. When baby rolls over, the pressure sensitive button signals to the Monitor which emits a sound alarm and warning light

Button flashes and audible warning is heard when alarm is activated. "Alarm/Reset" button turns off the alarm, resetting the monitor after it has been activated

On/Off button

Monitor, complete with belt clip, can be worn by parent or guardian. Compact and lightweight, strong and portable

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