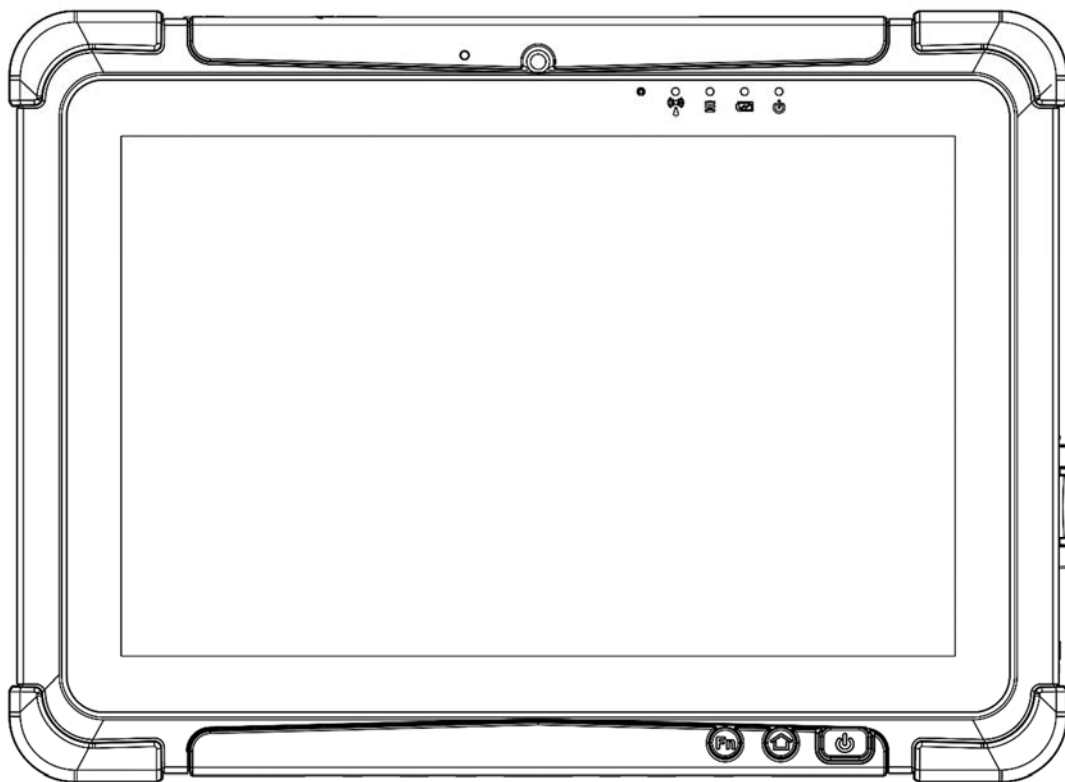
M101MDT-HUB / MD-M101

Healthcare Windows Tablet

Intel® Apollo Lake Pentium N4200

Windows® 10 IoT Enterprise

User Guide

Contents

Chapter 1: Read Me First	3
1.1 Advisory Conventions	4
1.2 Intention of Use	4
1.3 Safety Information	5
1.4 Handling and Maintenance	8
1.5 Declaration of Conformity (DoC)	9
Chapter 2: Getting Started	11
2.1 Introduction	12
2.2 Description of Parts	13
2.3 Hotkey for Specific Functions	15
2.4 First-Time Operation	15
Chapter 3: Setting Up Windows	18
3.1 Introduction	21
3.2 Setting up Windows 10 IoT Enterprise	21
Chapter 4: Cleaning the Device	24
4.1 Cleaning and Disinfection of the Device	25
4.2 Cleaning Agent Allowed	25
Chapter 5: Frequently Used Settings	48
5.1 Screen Resolution Scaling and Setting	49
5.2 Replacing Battery	52
5.3 Using Recovery Wizard to Restore Computer	52
5.4 Docking Connector Pin Definition	54
5.5 Basic Troubleshooting	55
Chapter 6: Accessories	56
6.1 Accessories Introduction	57
6.2 MD-M101 Medical Grade Docking Station	58

Chapter 1: Read Me First

This chapter provides safety information, handling and maintenance precautions and other important information about using the rugged tablet.

1.1 Advisory Conventions

Four types of advisories are used throughout the user manual to provide helpful information or to alert you to the potential for hardware damage or personal injury. These are Notes, Important, Cautions, and Warnings. The following is an example of each type of advisory.

Note: A note is used to emphasize helpful information

Important: An important note indicates information that is important for you to know.

Caution: A Caution alert indicates potential damage to hardware and explains how to avoid the potential problem.

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Warning!

An Electrical Shock Warning indicates the potential harm from electrical hazards and how to avoid the potential problem.

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1.2 Intention of Use

This tablet is intended to serve as a display-integrated computing platform for integration with hospital system. This device is designed for general purpose for hospital environment. For data collection and display for reference. It shall not be used for life-supporting system.

This Tablet is hand held equipment for health care professionals and can be mounted in a cradle with VESA mount for fix equipment.

Important:

This tablet can only use at landscape mode. Do not use at portrait mode.

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1.3 Safety Information

Follow and adhere to all warnings and instructions in this manual. For your safety, please read all safety and operating instructions before using the product. Keep this manual for future reference.

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When the Battery Pack Has Deteriorated, Replace It with a New Battery. Continued use of a damaged battery pack may result in heat generation, ignition or battery rupture.

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Do Not Expose the Battery Pack to Excessive Heat, or Extreme Heat (Near Fire, in Direct Sunlight for example) Generation of heat, ignition or rupture may result

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Do Not Use the Battery Pack With Any Other Products. The battery pack is rechargeable and is only designed for this specific product. If it is used with a product other than the specified product, electrolyte leakage, generation of heat, ignition or rupture may result.

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Do Not Use This Product With Battery Other Than the One Specified.

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General Guideline **Directive générale**

It is recommended to reboot the device when some functions are defect or inactive. If it still can't solve the problems please contact your dealer or agent.

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Discontinue using device immediately if you notice the battery is swelling, or looks puffy or misshapen. Do not attempt to charge or use the device. Do not handle a damaged or leaking battery. Do not let leaking battery fluid come into contact with your eyes, skin or clothing.

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WARNING: To avoid risk of electric shock, this equipment must only be connected to a supply main with protective earth.

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Warning! Do not modify this equipment without authorization of the manufacturer.

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Use a soft cloth to clean the surface.

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Please follow the local national requirement to dispose unit/accessories/waste products/residues etc.

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When you disconnect the AC adapter, disconnect from the socket-outlet First.

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The equipment should be installed near an easily accessible outlet.

For pluggable equipment, the power outlet must be installed near the equipment and must be easily accessible.

Accessory equipment connected to the analog and digital interfaces must be in compliance with the respective nationally harmonized IEC standards (i.e. IEC 60601-1 for healthcare equipment.) Furthermore all configurations shall comply with the system standard in IEC 60601-1. Anyone who connects additional equipment to the signal input part or signal output part is configuring a healthcare system, and is therefore, responsible that the system complies with the requirements of the system standard IEC 60601-1. The unit is for exclusive interconnection with IEC 60601-1 certified equipment in the patient environment and IEC 60950-1 certified equipment outside of the patient environment.

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Circuit diagrams, descriptions and component parts list will be made available only upon request when servicing is required. Please, contact the address below for related information.

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Warning! Tablet with cradle cannot be hand-held operated, and must be used with VESA mount as a fixed equipment.

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- Do not disregard the safety that a polarized or grounded connection can provide. A polarized connection consists of two plugs, one of which is larger than the other. A grounded connection has two blades and a third one that is grounded. If the main plug provided cannot be inserted into your outlet, consult an electrician to replace your obsolete outlet.
- [FR] Ne pas ignorer la sécurité qu'une prise polarisée ou une prise de terre peut prévoir. Une prise polarisée se compose de deux fiches, dont l'une est plus large que l'autre. Une prise de terre possède deux lames et une troisième qui est la terre. Si la fiche principale fournie ne peut être insérée dans votre prise, consultez un électricien pour remplacer votre prise obsolète.*

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- Refer all servicing to qualified service personnel. A technical intervention is required when the apparatus has been damaged in any way, for example if the power cord or plug is damaged, liquid has been spilled or objects have fallen inside the unit, if the product has been exposed to rain or moisture, if it does not operate normally or if it has been dropped.
- [FR] Confier toute réparation à du personnel qualifié. Une intervention technique est nécessaire lorsque l'appareil a été endommagé de quelque façon, par exemple si le cordon d'alimentation ou la fiche est endommagé, du liquide a été renversé ou des objets sont tombés à l'intérieur de l'unité, si le produit a été exposé à la pluie ou à l'humidité, s'il ne fonctionne pas normalement ou s'il a été jeté.*

- Grounding reliability can only be achieved when the equipment is connected to an equivalent receptacle marked "Hospital Only" and "Hospital Grade". Use a power cord that matches the voltage of the power outlet, which has been approved and complies with the safety standard of your particular country.
[FR] La fiabilité de la mise à la terre ne peut être atteinte que lorsque l'équipement est connecté à un réceptacle équivalent portant la mention «Hospital Only» et «Hospital Grade». Utilisez un cordon d'alimentation correspondant à la tension de la prise de courant approuvée et conforme aux normes de sécurité de votre pays.

1.4 Handling and Maintenance

General Guideline Guide Général

Keep the product at least 13 cm (5 inches) away from electrical appliances that can generate a strong magnetic field such as TV, refrigerator, motor, or a large audio speaker.

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Do not place heavy objects on top of the tablet computer as this may damage the display.

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The screen surface can be easily scratched. Use with care. Do not use paper towels to clean the display but use the included cleaning cloth. Avoid touching it with a pen or pencil.

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To maximize the life of the backlight in the display, allow backlight to automatically turn off as a result of power management. Avoid using a screen saver or other software that prevents the power management from working.

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Cleaning Guideline Guide de nettoyage

Use a soft cloth moistened with water or a non-alkaline detergent to wipe the exterior of the product.

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Gently wipe the display with a soft, lint-free cloth. Do not use alcohol or detergent on the product.

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Use a soft cloth to remove dirt on the display. The touchscreen surface has a special protective coating that prevents dirt from sticking to it. Not using a soft cloth may cause damage to the special protective coating on the touchscreen surface. To clean off hard to remove dirt, blow some steam on the particular area and gently wipe away using a soft cloth.

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Do not use excessive force on the display. Avoid placing objects on top of the display as this may cause the glass to break.

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1.5 Declaration of Conformity (DoC)

FCC Statement

This device complies with part 15 FCC rules. Operation is subject to the following two conditions:

1.5.1 This device may not cause harmful interference.

1.5.2 This device must accept any interference received including interference that may cause undesired operation

This equipment has been tested and found to comply with the limits for a class "B" digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

CE Notice (European Union)

Healthcare electrical equipment Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances

- EN 60601-1-2:2015+A1:2021
- IEC 60601-1-2:2014+AMD:2020
- EN 55011:2016+A2:2021

Healthcare electrical equipment Part 1: General requirements for basic safety and essential performance

- IEC 60601-1-6:2010+AMD1:2013+AMD2:2020

This equipment is in conformity with the requirement of the following EU legislations and harmonized standards. Product also complies with the Council directions.

Canada, Industry Canada (IC) Notices

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

(1) This device may not cause interference.

(2) This device must accept any interference, including interference that may cause undesired operation of the device.

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RF Exposure Information (SAR)

This device meets the government's requirements for exposure to radio waves. This device is designed and manufactured not to exceed the emission limits for exposure to radio frequency (RF) energy set by the Federal Communications Commission of the U.S. Government.

The exposure standard employs a unit of measurement known as the Specific Absorption Rate, or SAR. The SAR limit set by the FCC is 1.6 W/kg. Tests for SAR are conducted using standard operating positions accepted by the FCC with the EUT transmitting at the specified power level in different channels.

Radio Frequency (RF) Exposure Information

The radiated output power of the Wireless Device is below the Industry Canada (IC) radio frequency exposure limits. The Wireless Device should be used in such a manner such that the potential for human contact during normal operation is minimized.

This device has been evaluated for and shown compliant with the IC Specific Absorption Rate ("SAR") limits when operated in portable exposure conditions.

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Chapter 2: Getting Started

This chapter tells you step by step how to get the Rugged Tablet up and running.

2.1 Introduction

M101MDT-HUB is a 10.1-inch Windows-based tablet computer designed for healthcare applications with a housing enhanced with antimicrobial properties, weighting at just 1200 grams (2.7 pounds). This mobile and portable device is flexible enough to function as a standalone device and competent enough to be used for all sorts of data collection, patient care, information management and wireless communication in healthcare applications. With rugged features, a crystal clear 1920 x 1200 resolution display, projected capacitive touch screen, barcode scanner and superior processing power coming from Intel's Apollo Lake processor, this tablet can be used in hospital halls, waiting rooms or nurse stations for communicating and documenting patient information, allowing paramedics and nurses to instantly access and keep patient records.

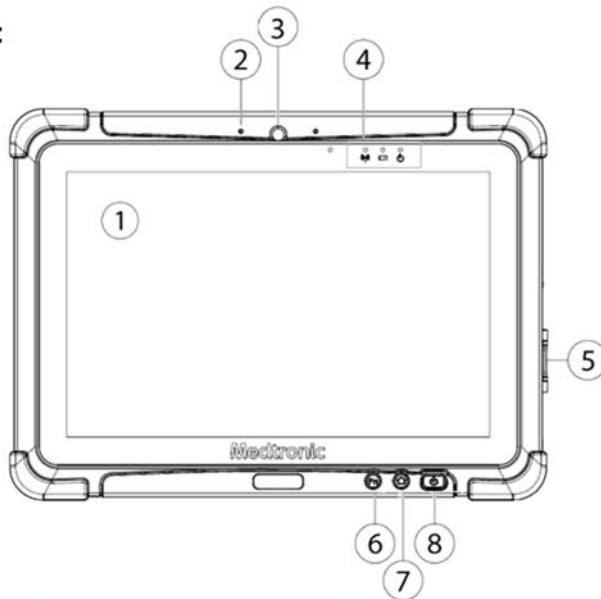
Highlights

- 10.1" 1920 x 1200 IPS LED Panel with direct optical bonding
- Intel® Apollo Lake Pentium N4200 processor
- Housing enhanced with antimicrobial properties
- 3 x Interface Port at rear side
- Lightweight, rugged, IP65 waterproof and dustproof
- IEC 60601-1, 60601-1-2 Certified for healthcare


2.2 Description of Parts

Dimensions: 271.8 x 197.2 x 19 mm

Front:

Right:



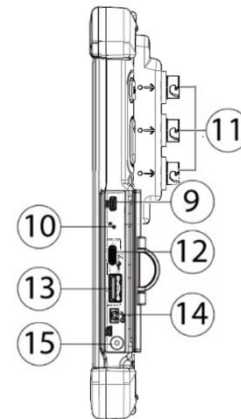
No	Item	Description
1	10.1" Touch Screen	Display and act as one of the inputs for the Tablet Computer
2	Digital Mic with noise cancellation	Use during video recordings and communications.
3	Front Camera	2 Megapixel Webcam
4	LED Indicators	Show the current status of the Tablet Computer. RF communications, battery status, power status.
5	I/O Cover	Protect I/O ports
6	Fn1 Button	Programmable function key configured by Hottab Utility
7	Menu Button	Hottab Quick Access
8	Power Button	Power on/off tablet

Front LED Indicator Details:

No	Item	Description
9	Micro HDMI	HDMI output to external display
10	Reset button	Hardware reset button.
11	Interface port	USB 2.0 thru 8-pin connector
12	USB 3.0 Type C	High speed USB port connects to USB devices, such as USB flash drive, mouse, keyboard, printer, etc. Note: The USB Type-C connector follows USB 3.0 standard, which does not support Alternate Mode (ALT) or USB Power Delivery (USB-PD).
13	USB 3.0 Type A	USB 3.0 port connects to USB devices, such as USB flash drive, mouse, keyboard, printer, etc.
14	Mic In/Line Out	Headset or headphone can be connected (Mic is only supported with 3.5mm 4-pin headset)
15	Power Jack	DC power input jack.

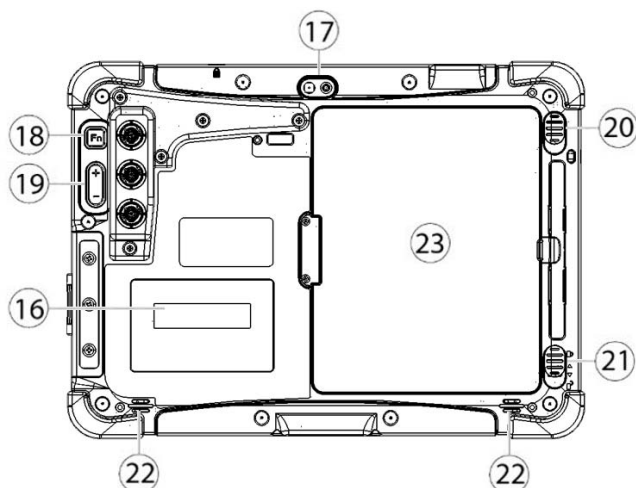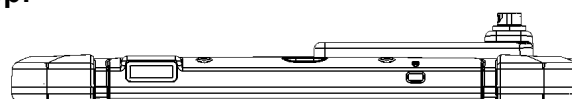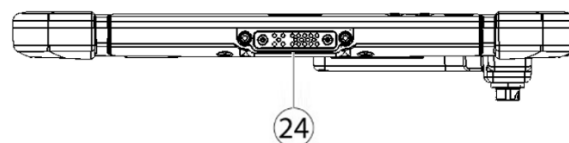Rear:**Top:****Bottom:**

No	Item	Description
16	UDI label recess	Recess reserved for UDI label.
17	Rear Camera with LED Flash	8MP Autofocus Camera. Captures photos and videos. The LED camera flash to provide flash lighting when use with camera.
18	Fn2 Button	Programmable function key configured by Hottab.
19	Brightness or Volume Buttons	Brightness/ Volume Control. (+) Up, (-) Down.
20	Battery Release Slider	Spring-loaded slider that latches the battery in.
21	Battery Lock Latch	Latch to lock the battery in place once battery is inserted. Not spring-loaded.
22	Speakers	Emit sounds.
23	Battery	7.4V, typ. 5140 mAh Li-Polymer Battery (2S1P)
24	Docking Connector	Port to docking solutions such as vehicle docking to provide power and external antenna

2.3 Hotkey for Specific Functions

For the physical buttons on the tablet computer, in addition to its dedicated functions detailed in the previous section, it can also be used to emulate commonly used keyboard key strokes. Under different scenario, a single physical button or a combination of physical buttons provide different keyboard key strokes, which are specified below:

In POST (before Windows boots up):

In Windows:

Buttons	Keystroke	Buttons	Keystroke
Fn1	F6 to enter Recovery Menu	Menu	Hottab Quick Access
Fn2	Esc to enter BIOS setup screen	Fn1	Default: Web Browser
Up (+)	Up	Fn2	Default: Camera
Down (-)	Down	UP (+)	Up.
Menu + Fn1	Enter	Down (-)	Down.
Menu + Up (+)	Right	Power	Power ON/ OFF
Menu + Down (-)	Left	Up (+) + Down (-)	Ctrl + Alt + Del

LED Indicators:

Item	Icon	Description
Power Indicator		Blue – Tablet is ON No light – Tablet is OFF
Battery Status		Orange – P.S. is plugged in and charging Green – P.S. is plugged in and battery is full charged Blinking Orange – Low battery (<10%) No light – Battery is in use and more than 10%
Wireless Communication		Green – WiFi/Bluetooth is ON No light – WiFi/Bluetooth are all OFF

2.4 First-Time Operation

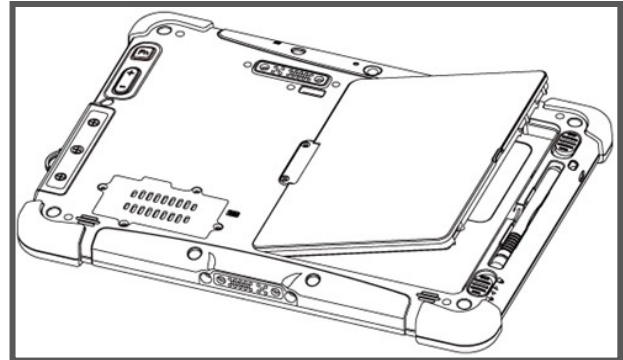
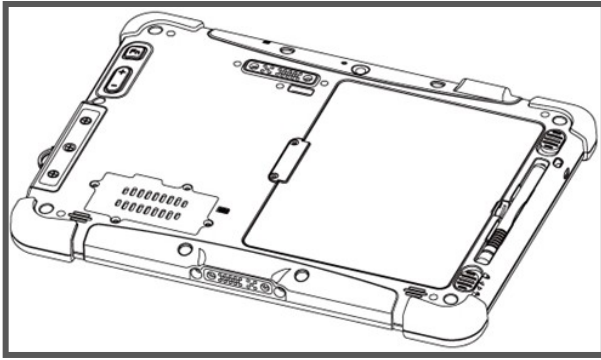
To get started:

1. Install the battery.
2. Charge your tablet.
3. Start the tablet.

2.5.1 Installing Battery

To install the standard battery pack:

1. Disconnect any cords connected to the tablet computer.
2. Turn the tablet over, back facing up.
3. Locate the Battery Lock Latch and push it to the “Unlocked” position.
4. Locate the Battery Release Slider that is spring-loaded. Pull back and hold.
5. Insert the battery with the connector-pins side first, and then lower the battery into the battery compartment. Continue to hold the Battery Release Slider until the battery is fully inserted.
6. Make sure the battery is completely lowered into the compartment before letting go of the Battery Release Slider. Battery will be flat against the housing once fully inserted.

To remove the battery pack:

1. Shut down the tablet (or put into sleep mode). Disconnect any cords connected to the computer.
2. Turn the tablet computer over, back facing up.
3. Locate the Battery Lock Latch and push it to the “Unlocked” position.
4. Locate the Battery Release Slider that is spring-loaded. Pull back and hold.
5. Lift the battery out using the opening on the side of battery compartment.

2.5.2 Charging Your Tablet

The M101MDT-HUB Healthcare Tablet Computer operates either on external AC power or internal battery power. It is recommended to use AC power when you start up the Tablet for the first time.

Caution: Use only the AC adapter included with your Tablet Computer. Using other AC adapters may damage the Tablet Computer and the Battery.

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Caution: For users start up the Tablet Computer for the first time with internal battery power, please make sure the battery is fully charged before using.

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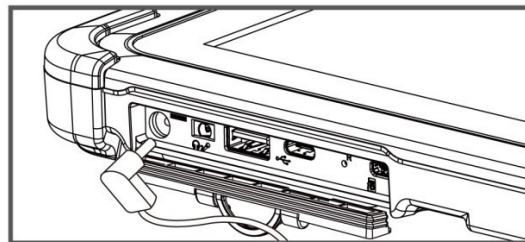
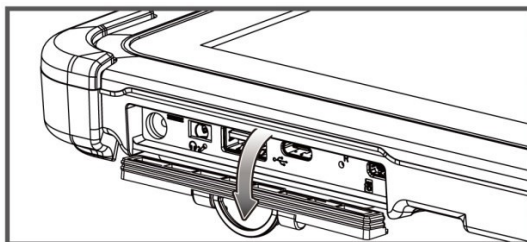
To connect a tablet to AC power:

1. Open I/O protective cover by pulling out the latch.
2. Plug the DC cord of the AC adapter into the power jack of the tablet computer.
3. Plug the AC power cord into the AC adapter.
4. Plug the AC power cord into the electrical outlet.

The battery LED indicator shows different battery states:

Orange - P.S. is plugged in and charging

Green - P.S. is plugged in and battery is full charged

Blinking Orange - Low battery (<10%)

No light - Battery is in use and more than 10%

2.5.3 Starting the Tablet

Important: When powering on the device for the first time, please make sure there's no interruption during the Windows setting up process, for example, accidentally powering off. If it happened, please access the recovery menu by pressing **Fn1** button on the front bezel to initiate recovery process.

Press and hold the power button for **1.5 to 3 seconds** until the blue LED power indicator light up.

Note: Do not press the power button repeatedly.

Do not perform the following operations until the Disk Activity Indicator off

- Connecting or disconnecting the AC adapter
- Pressing the power button
- Touching the tablet buttons, screen, or external keyboard/mouse

To power off the Tablet Computer:

1. Make sure you have closed any programs that are open on the Desktop. Start Screen apps don't need to be closed.
2. Tap on the **Settings** charm.
3. Tap on **Power**.
4. Tap on **Shut Down**.
5. Wait for the Tablet Computer to power off.

Note: The computer will forcibly be powered off if the power button is pressed and held for **6 seconds or longer**.

Chapter 3: Setting Up Windows

This chapter details how to setup the Windows that may be installed on the tablet for the first time.

3.1 Introduction

The M101MDT-HUB Tablet Computer operates on Windows 10 IoT Enterprise operating system.

Caution: Make sure the battery is full charged before starting the tablet computer for the first time with internal battery power.

- **Public network:** Select this location if you do not recognize all the computers on the network. (For example, you are in a coffee shop or airport, or you have mobile broadband.) This is a public network and is not trusted. Select the location where you will most often use the computer.

3.2 Setting up Windows 10 IoT Enterprise

Follow the steps below to complete the Windows 10 IoT Enterprise setup process.

1. Press the power button to turn on the computer.
2. First, select your country/region, your preferred app language, your preferred keyboard layout, and your time zone, then tap **Next**.

Hi there

Let's get a few basic things out of the way.

What's your home country/region?

United States

What's your preferred app language?

English (United States)

What keyboard layout would you like to use?

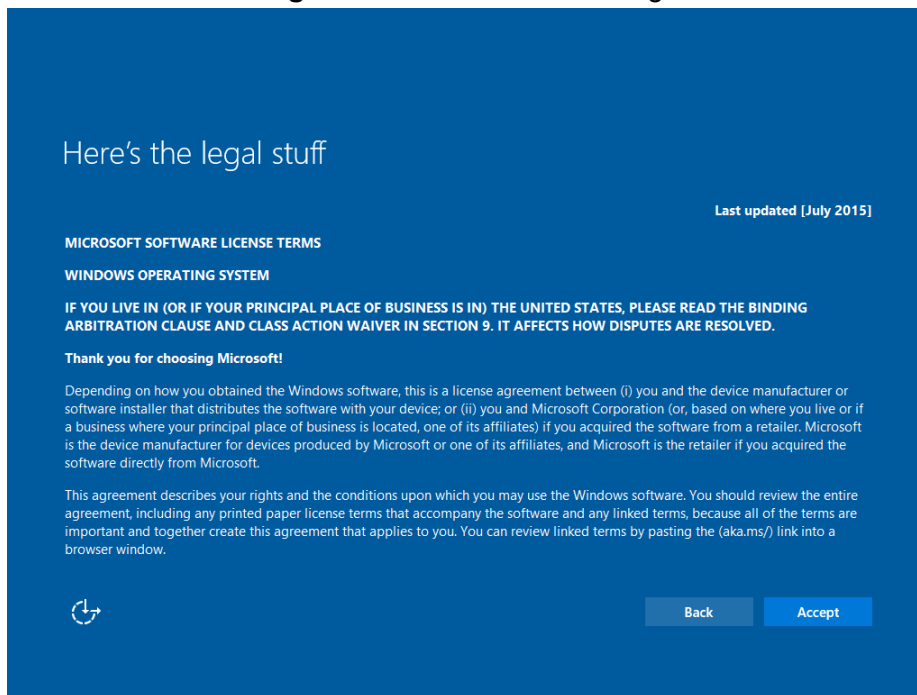
US

What time zone are you in?


(UTC-08:00) Pacific Time (US & Canada)

Next


3. On the **Here's the legal stuff** screen, read through the disclosures, then tap **Accept**.

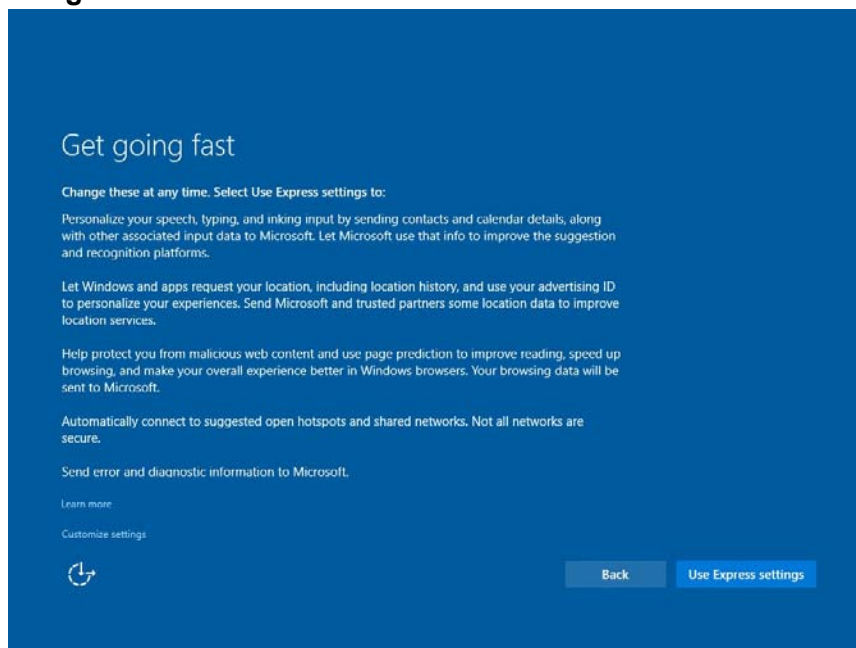4. Windows will then try to connect to network. Tap **Skip this step** to continue without a wireless connection as you can then set up the network later.
5. On the **Get going fast** screen, click **Use Express settings** for the fastest setup. If you would like more information, tap **Learn more**, or if you would like to select your own options, click **Customize settings**.

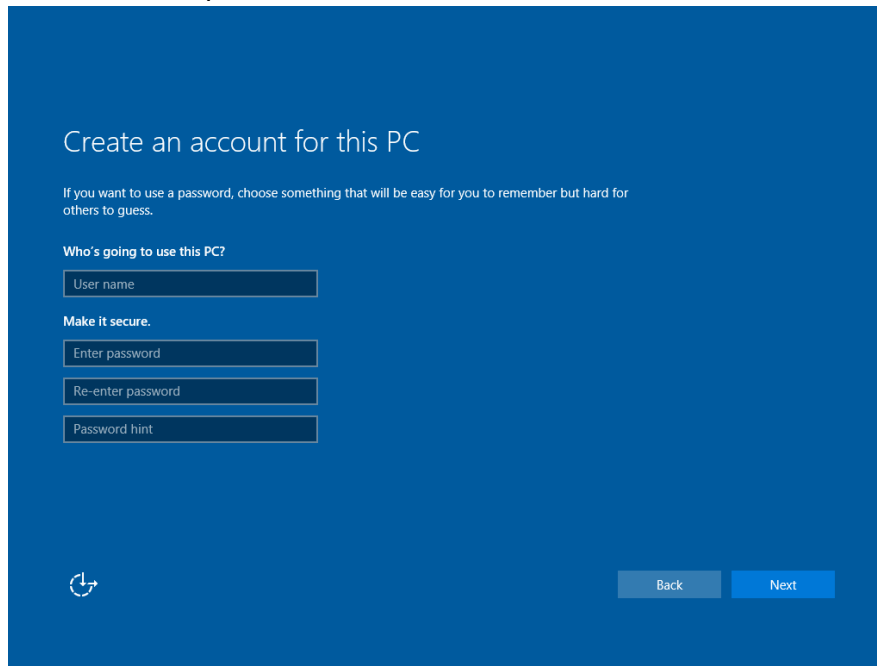6. To create a local account, type your User name, password, and Password hint in the fields provided, then tap **Next**.

The screenshot shows the 'Create an account for this PC' window. At the top, it says 'Create an account for this PC'. Below that, a note states: 'If you want to use a password, choose something that will be easy for you to remember but hard for others to guess.' The form is divided into two sections. The first section, 'Who's going to use this PC?', contains a single text input field labeled 'User name'. The second section, 'Make it secure.', contains three text input fields: 'Enter password', 'Re-enter password', and 'Password hint'. At the bottom left is a circular arrow icon. At the bottom right are two buttons: 'Back' and 'Next'.

7. The initial setup screens are complete, and the desktop is displayed.
8. Read any introductory screens describing features of your computer, and tap **Next** upon completing each. You are now done with the initial setup process.

Chapter 4: Cleaning the Device

This chapter describes how to clean the M101MDT-HUB Tablet Computer properly.

4.1 Cleaning and Disinfection of the Device

Lightly dampen a section of a microfiber cloth with water and gently wipe the surface of tablet and docking station, then go over it again with a dry section of the cloth to wipe away any moisture. Avoid getting moisture in any openings like the battery connector, I/O ports, speaker, or microphone.

4.2 Cleaning Agent Allowed

- Ethanol 98.8%
- Ethanol 70%
- Isopropanol 70%
- Hydrogen Peroxide solution 0.5-1% concentration
- Sodium Hypochlorite solution 6500 ppm (household bleach diluted ~1:10)

Chapter 5: Frequently Used Settings

This chapter frequently used settings such as changing resolution and scaling, using recovery wizard, and docking connector description, also contain basic troubleshooting guide.

5.1 Screen Resolution Scaling and Setting

The M101's 1920 x 1200 high resolution display is perfect for viewing image, video, or other applications, however, Windows user interface (icons, buttons, and other user interface elements) may be too small to read and operate at the default Windows settings. By changing the display scaling or the display resolution setting of the device, you can adjust the size of the text and other items on the desktop for optimal viewing. Display scaling is the recommended method because it has the effect of adjusting user interface element without having to move away from the optimal resolution setting at 1920 x 1200. This section will explain how you can adjust the built-in windows display scaling setting and resolution setting.

Note:

- The M101MDT-HUB Healthcare Tablet computer is shipped with display scaling set at **Larger - 150%**. We recommend the users set the scaling to this setting for optimal text and image viewing.
- Before proceeding to change the scaling option, make sure any opened document is saved and close, and all running applications are closed before proceeding to change the display scaling, because this procedure requires users to log out of their account to apply display scaling change.

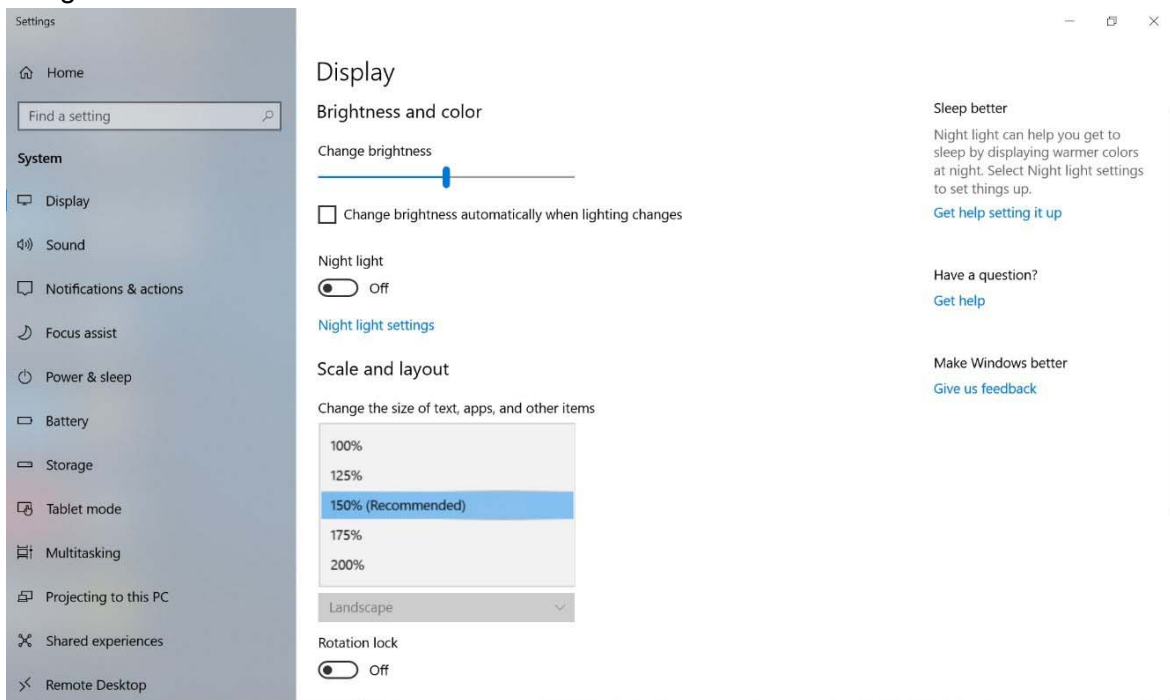
5.1.1 Changing Display Scaling (Recommended)

Perform the following to change display scaling setting:

1. Tap and hold anywhere on the desktop for 3 seconds and release to bring up the context menu, and select "Display settings" option as shown below:


2. A “Display” Window will show up. Click on the dropdown option under the “**Scale and layout**” section and change the scaling option to “**150% (Recommended)**” or any other desired display scaling.


5.1.2 Changing Screen Resolution

Alternatively, instead of changing the display scaling setting, user may choose to change the overall resolution setting to make screen elements bigger.

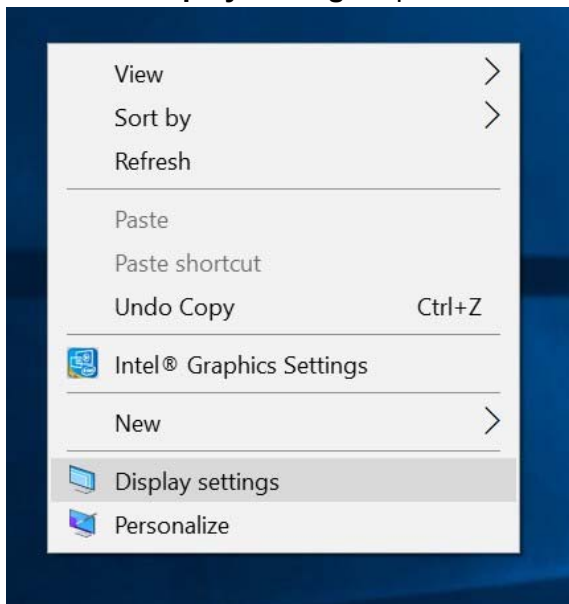1. Tap and hold anywhere on the desktop for 3 seconds and release to bring up the context menu, and select “**Display settings**” option as shown below:

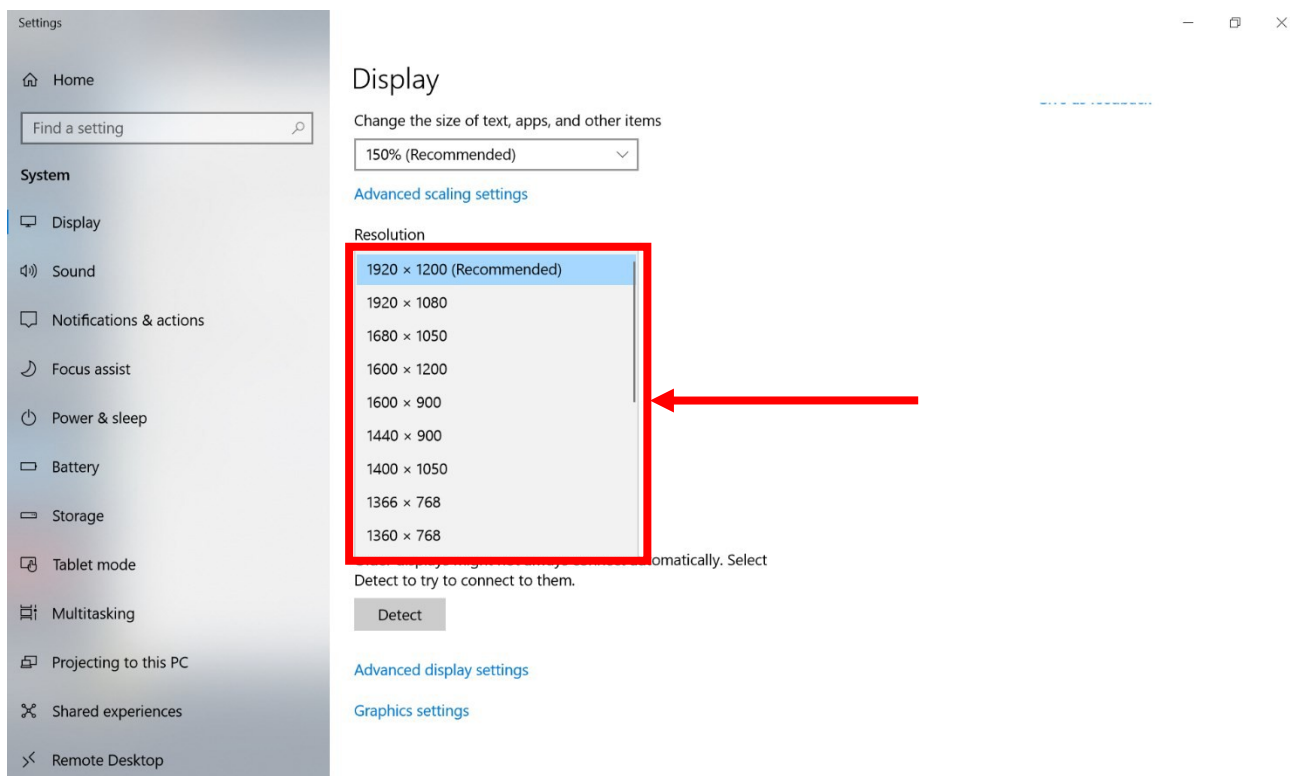- Click on the dropdown option next to “**Resolution**” and select the desired resolution by tapping on the scale:

- Hit “**Apply**”. A prompt “**Display Settings**” prompt will show up to ask the user if the applied resolution is usable.
- If the new resolution is the desired display setting hit “**Keep changes**” and continue to next step. If the new resolution is not desirable, hit “**Revert**” and select a different resolution to change to, and start again from step 2.
- The user may now use the device with the new resolution setting.

5.2 Replacing Battery

The M101MDT-HUB Tablet supports hot-swap, meaning the tablet doesn't need to be powered off to replace the main battery. To support hotswap, there is a backup battery inside the tablet that is charged together with the main battery. This battery cannot be removed, and requires adequate power for main battery hot swapping. When the backup battery is fully charged, the tablet can sustain up to 5 minutes of runtime for battery hot-swapping.

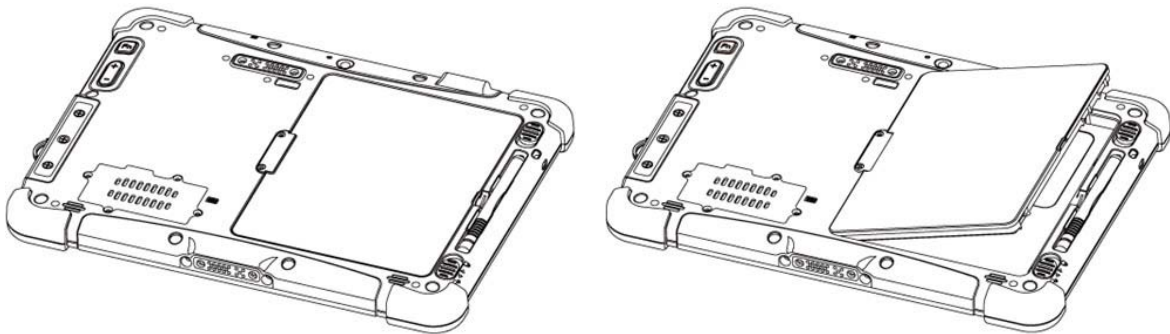
Perform the following to hot swap the batteries:

1. Power on the tablet, open Hottab to check the power level of the Backup battery.
2. If the Backup battery level is "Low", please charge the tablet with AC Power until the Backup battery level is "Medium" or "High", and do not remove the main battery.
3. If the Backup battery level is at "Medium" or "High", you can remove the main battery.
4. Insert a new fully charged battery into the battery compartment.
5. Pen Hottab to check Main battery level. If the battery level is displayed incorrect, check if the new removable battery is inserted properly.
6. If the battery level displayed correct, continue using the tablet.

Caution: Charge the built-in back-up battery every two months to keep battery health and prevent possible issues from battery over-discharge.

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5.3 Using Recovery Wizard to Restore Computer

The M101MDT-HUB Healthcare Tablet Computer has a dedicate recovery partition stored on the hard drive of the tablet to enable quick one-key recovery process. This partition occupies about 11GB of the storage space, and comes built-in to each M101MDT-HUB tablet computer.

Warning!

Before starting the recovery process, be sure to backup all user data, as all data will be lost after the recovery process.

To enable quick one-key recovery procedure:

1. Plug-in the AC adapter to the tablet computer. Make sure the tablet computer stays plugged in to power source during the recovery process.

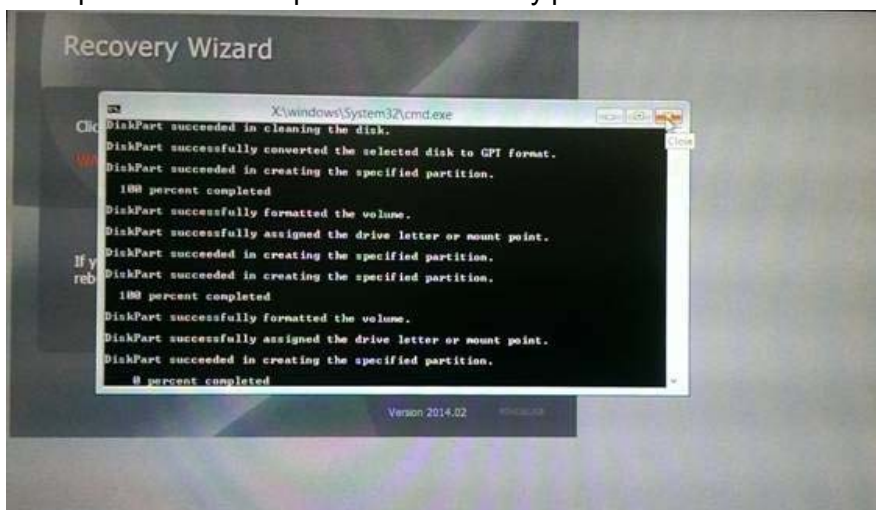
2. Power on the tablet computer, and when the boot screen shows up, press the **Fn1** button (Recovery Key) on the front bezel OR press **F6** when using an external USB keyboard to initiate the Recovery Wizard.
3. The following screen shows the Recovery Wizard. Click on "Recovery" button to continue.


4. A warning message about data loss will show up. Make sure data is backed up before recovery, and click on "Yes" to continue.

5. Wait till the recovery process to complete. During the recovery process, a command prompt will show up to indicate the percent of recovery process.

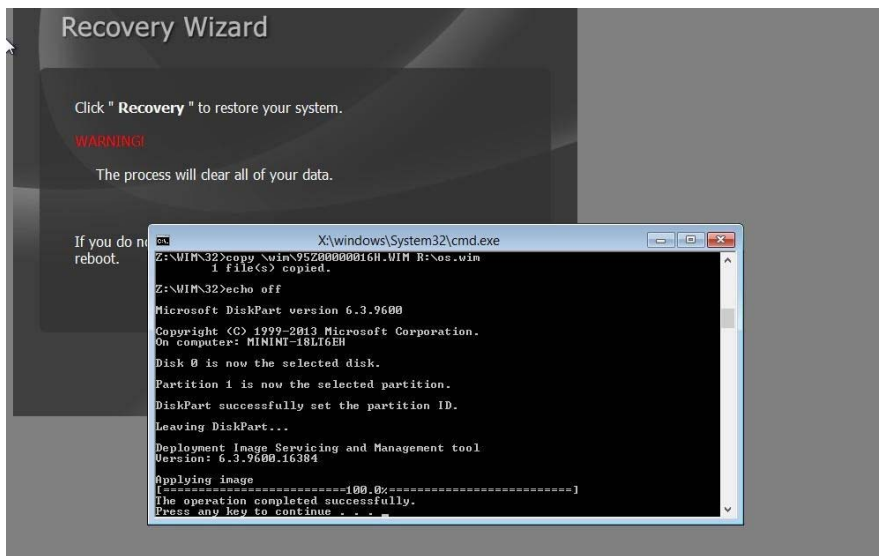
6. After recovery is completed, please press any key to continue, and the tablet computer will Shut down automatically.

5.4 Docking Connector Pin Definition

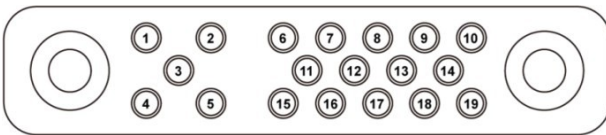The M101MDT-HUB Healthcare Tablet Computer supports one 19-pin docking connector at the bottom side. The pin definition is listed as the following table.

19 Pin Docking Connector

Pin No.	Pin Definition	Pin No.	Pin Definition
1	RF_GND	11	GPI
2	FAN_TACHO	12	GPO
3	NC	13	GND
4	RF_GND	14	Power
5	RF_GND	15	USB_D+
6	FAN_CONTROL	16	USB_D-
7	DOCK_DET (to EC)	17	GND
8	DET+ (for Docking)	18	GND
9	Power	19	DET- (for Docking)
10	Power		

Caution: Do not open when the expansion port is not used

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5.5 Basic Troubleshooting

Problem	Solution
The removable battery is not charging.	Make sure the AC adapter is properly connected, and that the removable battery is properly inserted.
Nothing appears on the screen.	- The tablet computer may be in sleep mode. Press the power button to wake up the display. - The brightness level may be too low. Increase brightness.
The screen suddenly turns blank and the power indicator goes off.	- Make sure that LED indicator is not powered off in Hottab. - Make sure the AC adapter is properly connected. - If running on battery power, the battery may be depleted, charge the battery.
Wireless LAN signal quality is poor.	- The tablet computer may be out of range. Move the tablet computer closer to the access point or another wireless LAN device associated with it. - Check if your surroundings have interferences, such as microwave ovens, cordless phones. Move the tablet computer away from objects causing interference.
I cannot connect to another wireless LAN device.	- Make sure that wireless LAN feature is powered on. - Make sure that the SSID setting is the same for every wireless LAN device in the network. - Restart the tablet computer - Make sure the IP address or subnet mask setting is correct.
I cannot connect to another device with Bluetooth.	- Make sure that both devices have Bluetooth powered on (For the Tablet Computer, check Hottab to see if Bluetooth is powered on). - Make sure that the distance between the two devices is within 10 meters and there are no walls or large obstructions between the devices. - Make sure the both devices are powered on and configured to be discoverable. - Make sure that both devices are compatible.
No sound is produced.	- Adjust volume of the tablet computer by pressing on the volume rocker. - If using an external audio device, make sure that the cable is properly connected.
I cannot project the tablet computer display to another monitor.	- Make sure the VGA driver is installed. - Make sure the output setting selected is correct.
Battery leaking or swollen.	- The lithium battery used in this tablet may swell after being charged/ discharged many times, but there is no problem regarding safety. - Continued use in such a state may result in the battery getting jammed, so please stop using the battery, and - purchase a new one.

Chapter 6: Accessories

This chapter lists available accessories of M101MDT-HUB healthcare tablet computer, and introduces MD-M101 medical grade docking station.

6.1 Accessories Introduction


The M101MDT-HUB provides a variety of product support capabilities. Table below lists the accessories available.

Accessory	Part Number	Description
Power Supply	98K000A000MI	Provides power 100-240VAC to 19VDC@3.12A
Replaceable Battery	98K000A000EW	7.4V, typ. 5140 mAh Li-Ion Re-chargeable Battery (2S1P)
Grade Tablet Dock	98DT00A0003I	Docking station for charging and I/O expansion.


6.2 MD-M101 Medical Grade Docking Station

MD-M101 is a medical grade docking compatible with M101MDT-HUB healthcare tablet. Use the MD-M101 docking station to charge the tablet, to extend the I/O ports, and to mount a tablet on the IP pole or other fixtures.

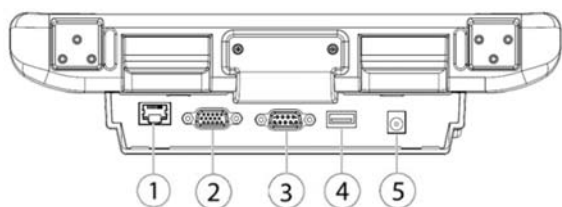Specifications

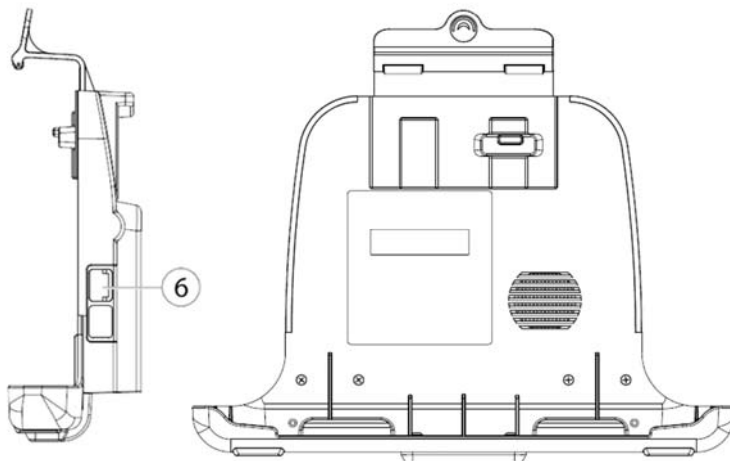1. LAN (10/100)
2. VGA output
3. RS232
4. USB 2.0 (Type A)
5. DC Jack (19V in)
6. USB 2.0 (Type A)

Mounting

The docking is designed to work with a standard 75mm x 75mm VESA mounting arm or base. Please mount the VESA plate on the back of the mount first before using it with the tablet.

Winmate Inc.
9F, No.111-6, Shing-De Rd., San-
Chung District, New Taipei City 24158,
Taiwan, R.O.C www.winmate.com

Copyright © 2021 Winmate Inc. All rights reserved.