

Crick Switch Box: User documentation (draft)

Contents of the Crick USB package

- Crick USB Switch Box
- Lead
- User manual

Requirements:

- PC running Windows 98 or above (with USB port)
Or
- Apple Mac Imac or above

Instructions

1. Connect the square end of the lead to the socket on the Crick USB Switch Box marked 'USB'
2. Connect the flat end of the USB lead to a USB port on your computer (you may do this with the computer switched on or off).
3. Connect your switch to a round socket on the Crick USB Switch Box

Note: If you are using Windows 98, you may need to install software drivers. These are on your Windows 98 CD. Please follow the enclosed instructions.

Testing the Crick USB Switch Box

- If you are using Windows 2000, please see the enclosed document for details of how you can test your switch box to see that it works.
- If you have a game that uses joystick buttons, start the game on your computer. Use the sockets on the Crick USB Switch Box that give you control of the relevant joystick buttons.
- If you are using Crick Software's 'Clicker 4' software, start the software, then take the following steps:
 - Open the **Options** menu and select **User Access**
 - Choose either **Single Switch** or **Two Switches** as the Access Method
 - Click the **Switch Connection** tab and select **Crick USB Box**
 - Click **OK** to close the dialog