


User Manual

Electronic Reading Magnifier


E-bot

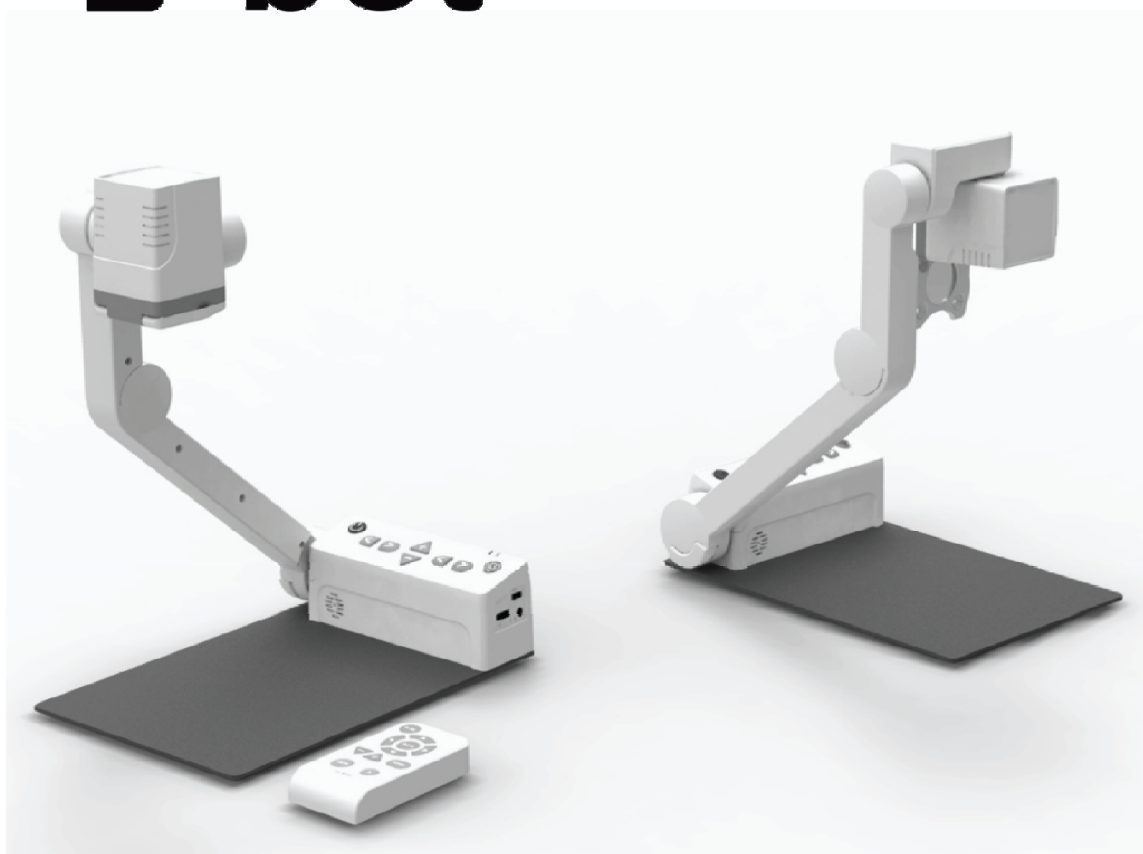

Easy to use, Versatile compatibility -

Advancing a new standard of transportable magnifier!

Intelligent electronic reading magnifier **E-bot**

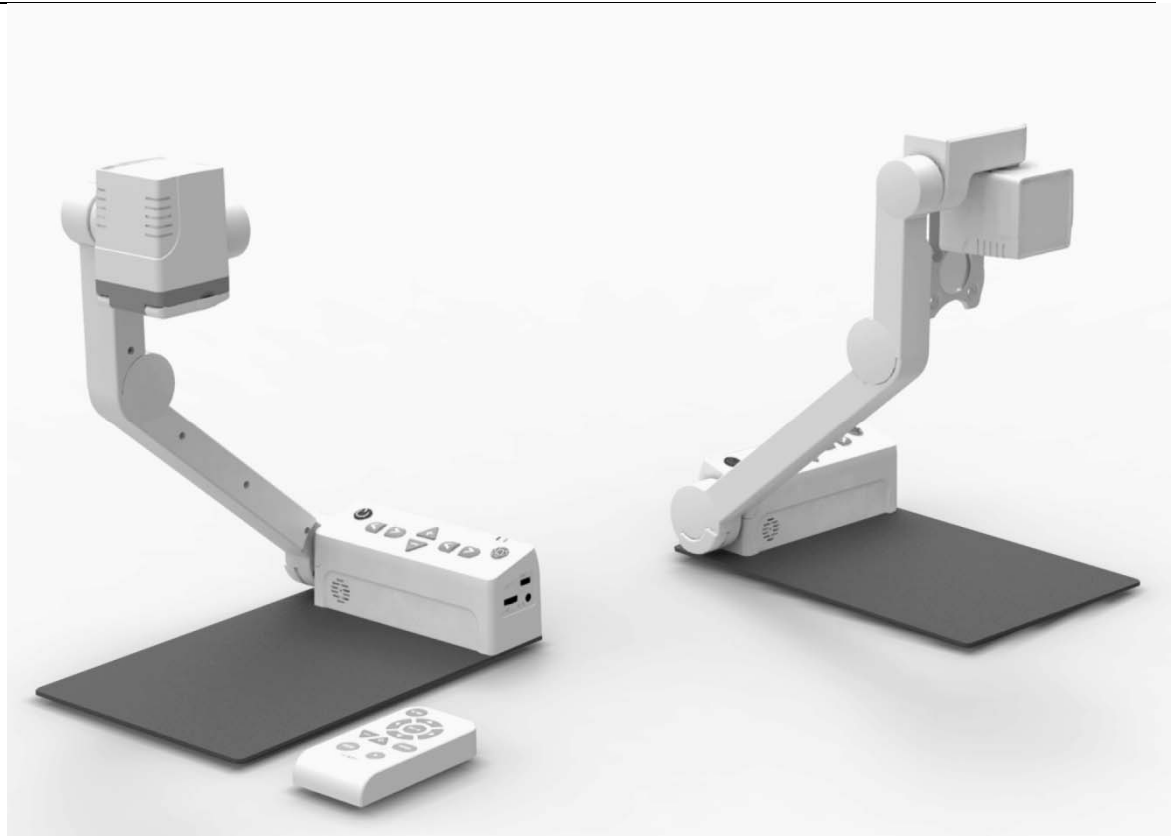

Intuitive to use for everyone

Display via PC, Mac, iPad/Android tablet, or any HDMI monitor

Wireless remote control for convenient operation

Quick and comfortable reading via the optical character recognition function

Thank you for purchasing E-bot.

E-bot HD is an electronic reading magnifier, which can enlarge the text of a wide variety of resources via a high-resolution camera.



ZOOM

You can change the magnification level for your convenience depending on the size of the text.



COLOR

E-bot provides you with various color modes so that you can select the color that best matches your needs and the material you are reading.



CONTRAST

You can adjust the contrast to see the image clearly no matter the material being viewed.



OCR

E_bot supports optical character recognition. You can hear the text via the built-in text-to-speech voices.

Table of Contents

1.	Safety Precautions.....	6
2.	Package Contents.....	7
3.	E-bot Orientation	8
1)	Product	8
2)	Back side of product.....	9
3)	Side of product.....	9
4)	Front of product.....	10
5)	Wireless remote controller.....	10
6)	Back side of Wireless remote controller.....	11
4.	Getting Started	12
1)	E-bot Setup	12
2)	Charging.....	13
3)	Charging via the AC Adapter.....	14
4)	Charging via USB.....	14
5)	Connecting to displays.....	14
6)	Connecting a monitor (HDMI)	15
7)	Connecting an On-Lap monitor (DC OUT).....	15
8)	Connecting to a PC (USB, Wi-Fi)	16
9)	Connecting to an Apple iPad or Android Tablet (Wi-Fi).....	18
5.	Basic Functions of E-bot.....	19
1)	Power On/Off.....	19
2)	Changing Camera Modes.....	19
3)	Magnification	20
4)	Changing the Color Mode.....	21
5)	Contrast.....	23
6)	LED light On/Off.....	24
7)	Finding your current Reading Position.....	24
8)	Switching to and from Natural Color mode	25
9)	Freeze.....	26
10)	Focus lock.....	26
6.	Advanced features	28
1)	Using the menu	28
2)	Color mode settings.....	29
●	Setting the color mode	29
●	To disable color modes	30
3)	Marker.....	31
●	Set Marker	31

●	Set Marker position and size	32
4)	OCR	33
●	OCR execution.....	33
●	Reading/navigating recognized text.	34
●	OCR views.....	34
●	OCR correction	35
●	OCR Language setting	35
5)	E-bot operating Language.....	36
6)	Voice Guide	36
●	Voice Guide On/off.....	36
●	TTS Volume	36
●	TTS Speed	37
●	TTS Pitch	37
●	TTS voice.....	38
7.	Using other functions	38
1)	Initialization	38
2)	Information	38
3)	Wi-Fi password change.....	39
4)	Keyboard Hot keys: (Only when using PC to display and control E-bot.)	40
8.	Troubleshooting	41
1)	When the wireless controller does not work	41
●	Check the Main body channel values	41
●	Checking the wireless remote control channel value	41
9.	A/S	42
10.	Specification	433
※	Smart Pad Viewer Gesture.....	444
※	How to control.....	488

1. Safety Precautions


- ※ For your safety and protection of the E-bot, please read and abide by the following important safety precautions.
- 1) Do not plug the AC adapter into a faulty electrical outlet. This is to prevent electrical shock or fire.
- 2) Do not touch the AC adapter with wet hands.
- 3) Use only the AC adapter provided with the E-bot. Using another AC adapter may damage the device. Be advised that damage caused by misuse is not covered by the warranty.
- 4) Please do not remove the power cable when the E-bot is turned on as this can damage the product.
- 5) Do not use the E-bot in a moisture-rich environment such as a bathroom, shower or a wet kitchen counter as moisture may get inside the unit and damage the components.
- 6) Do not use E-bot in extreme high or low temperatures. We recommend that you do not use this device in environments below 0°C or above 40°C, as the image may display inaccurately.
- 7) Always turn off the power before moving the product to another location.
- 8) When you press the release switch to fold the E-bot, please grip the arm and manually fold the unit. Allowing the arm to simply drop may cause impact damage to the unit.
- 9) Do not use the DC OUT at the back of the device as a power supply for devices other than authorized visual displays for E-bot as this can damage the product and/or the connected device.

2. Package Contents


- 1) Product**
- 2) 1 Battery**
- 3) Wireless remote controller**
- 4) Battery for Wireless remote controller**
- 5) Adapter**
- 6) Cable(USB 3.0, HDMI, HDMI to DVI gender)**
- 7) Carrying Case**
- 8) Product Case**
- 9) Earphones**
- 10) User Guide**
- 11) PC install program**

3. E-bot Orientation


1) Product

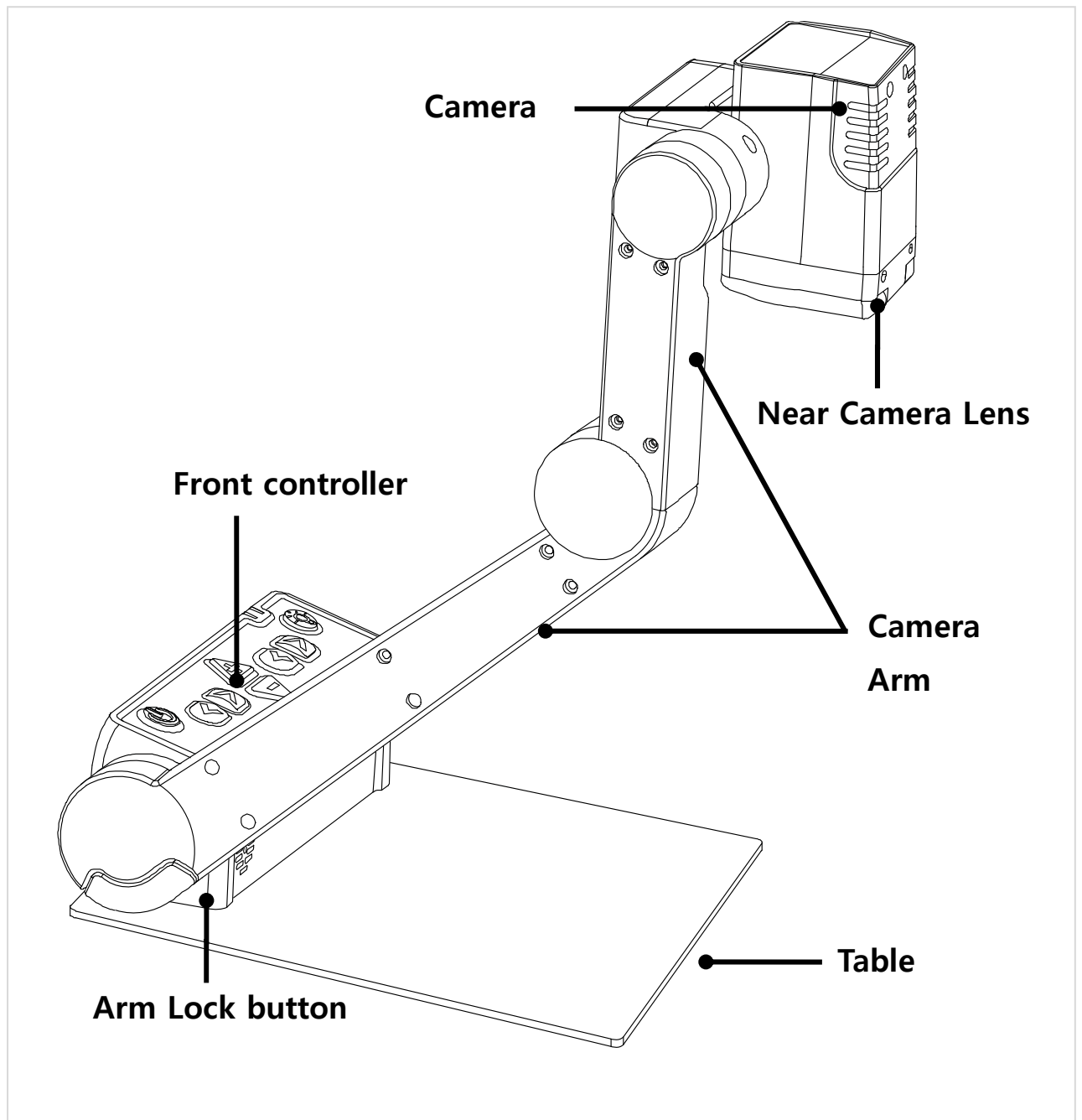


2) Back side of product

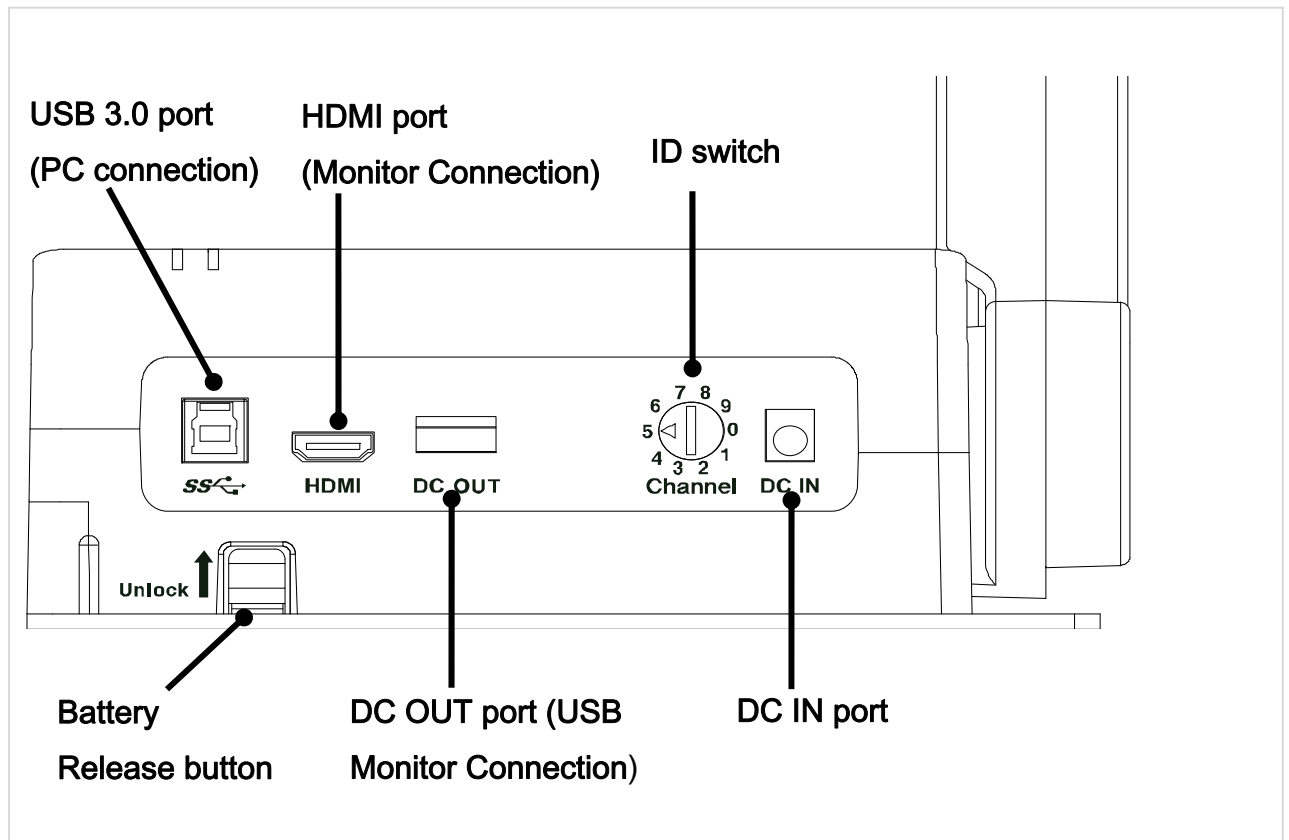


3) Side of product

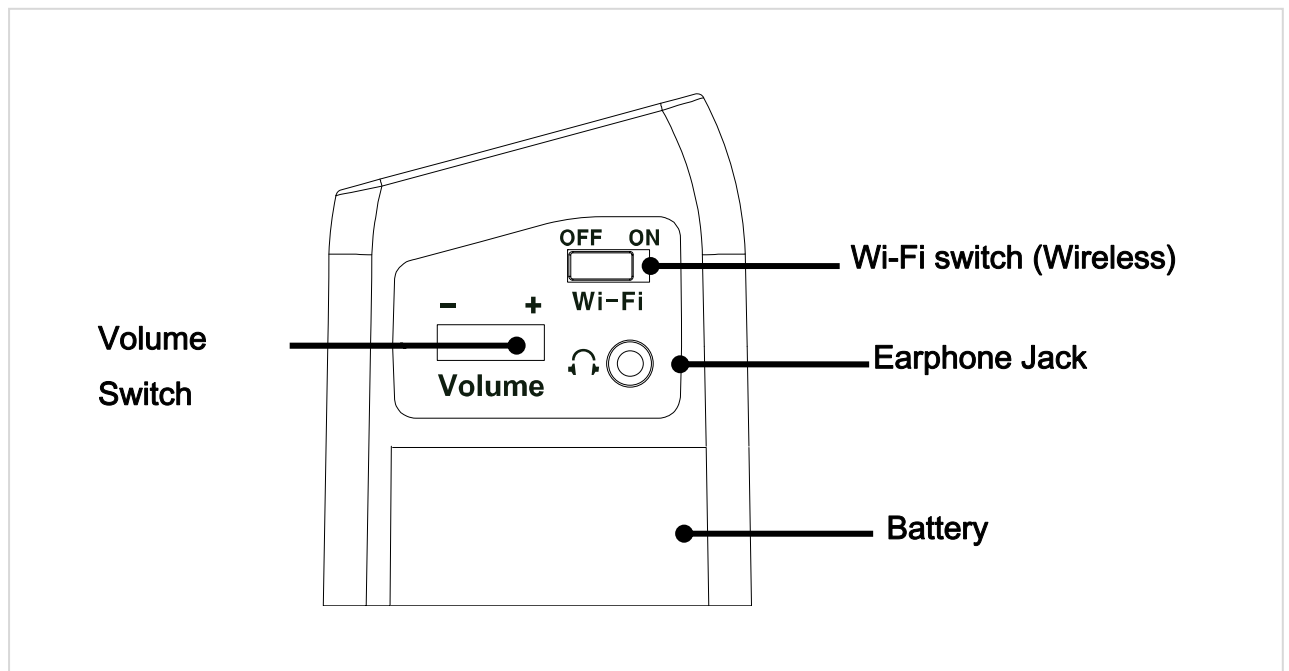


4) Front of product

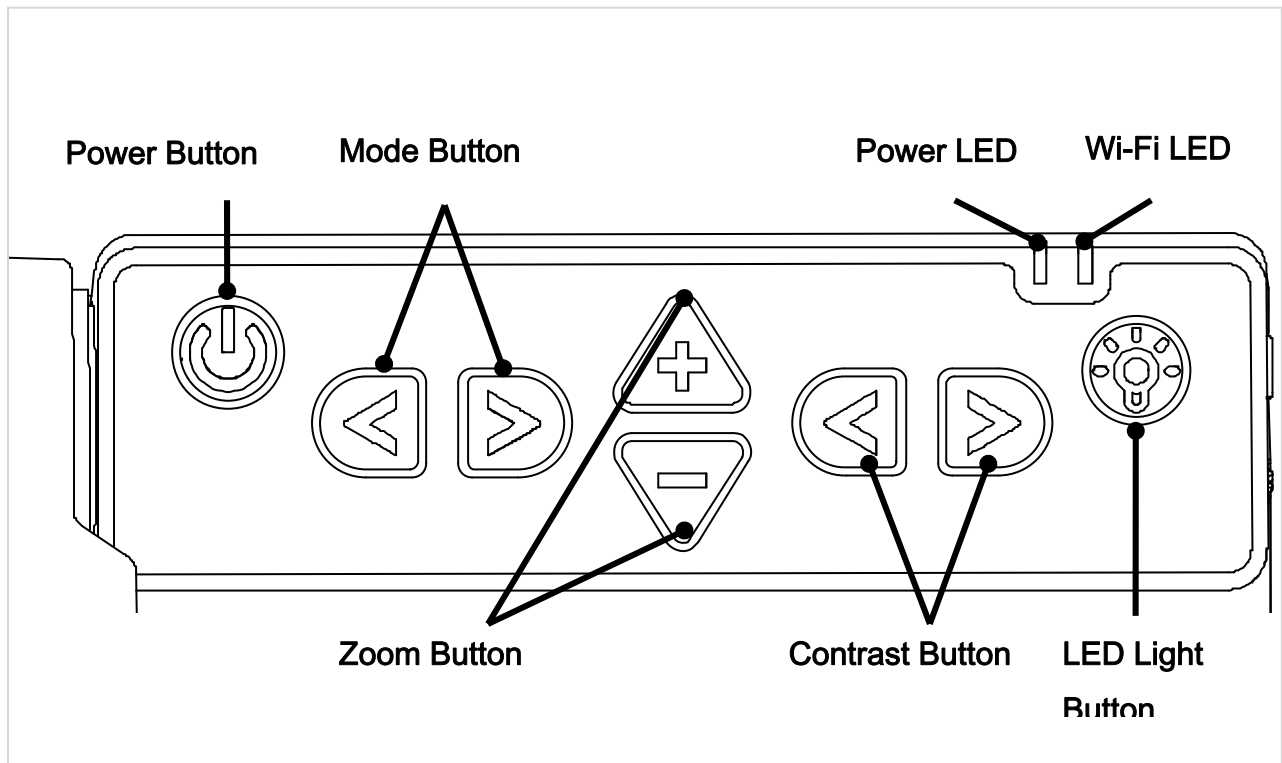

5) Wireless remote controller

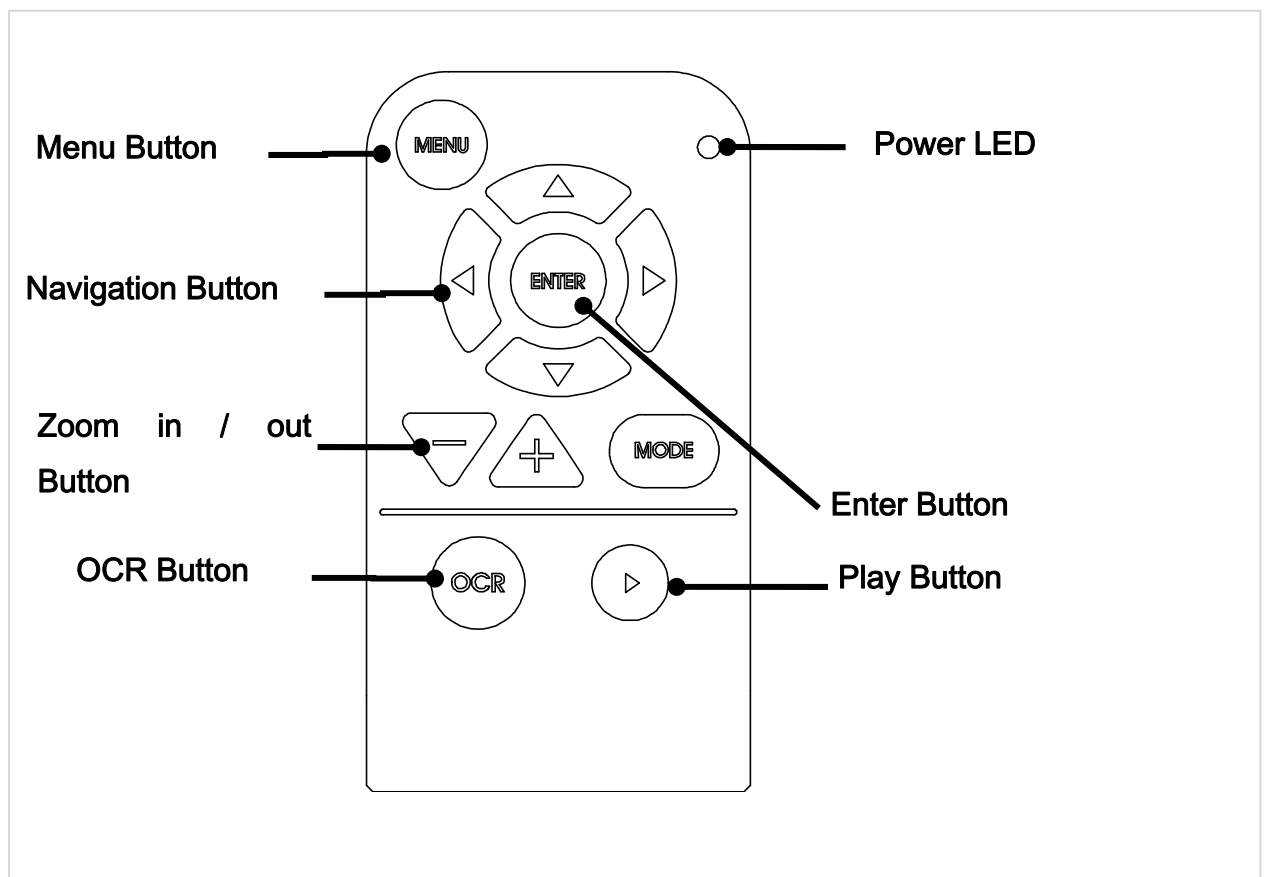

6) Back side of Wireless remote controller


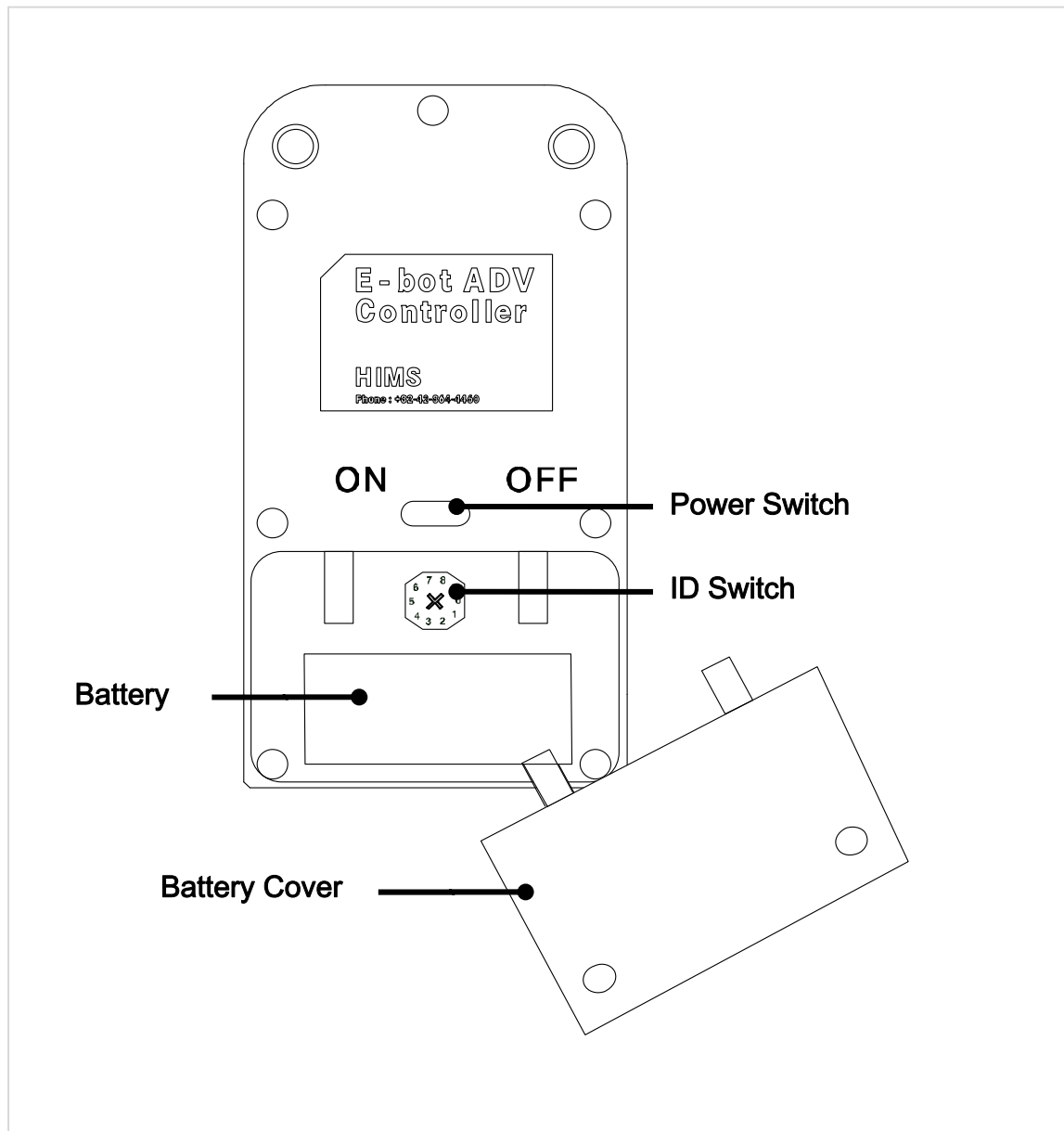


4. Getting Started


1) E-bot Setup

Follow the instructions below to setup the E-bot.


- ① Press and hold the Lock button on the left side of the E-bot.
While holding the lock button, unfold the longer arm upward.

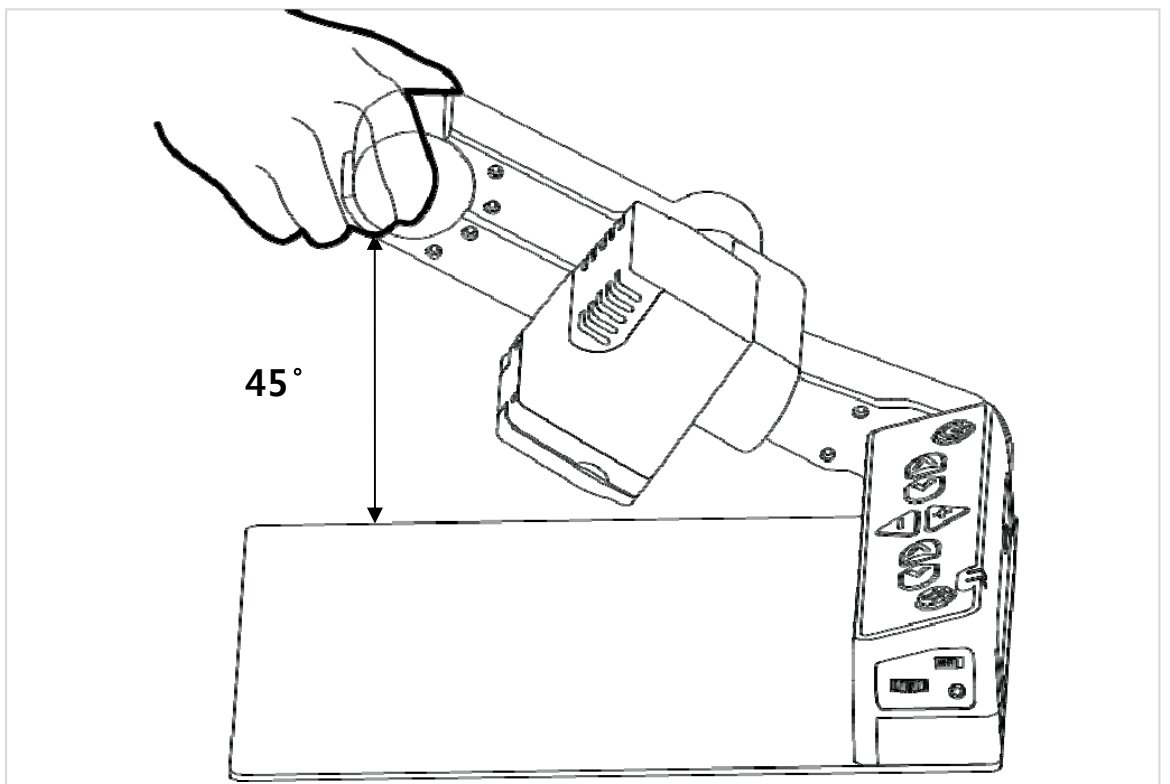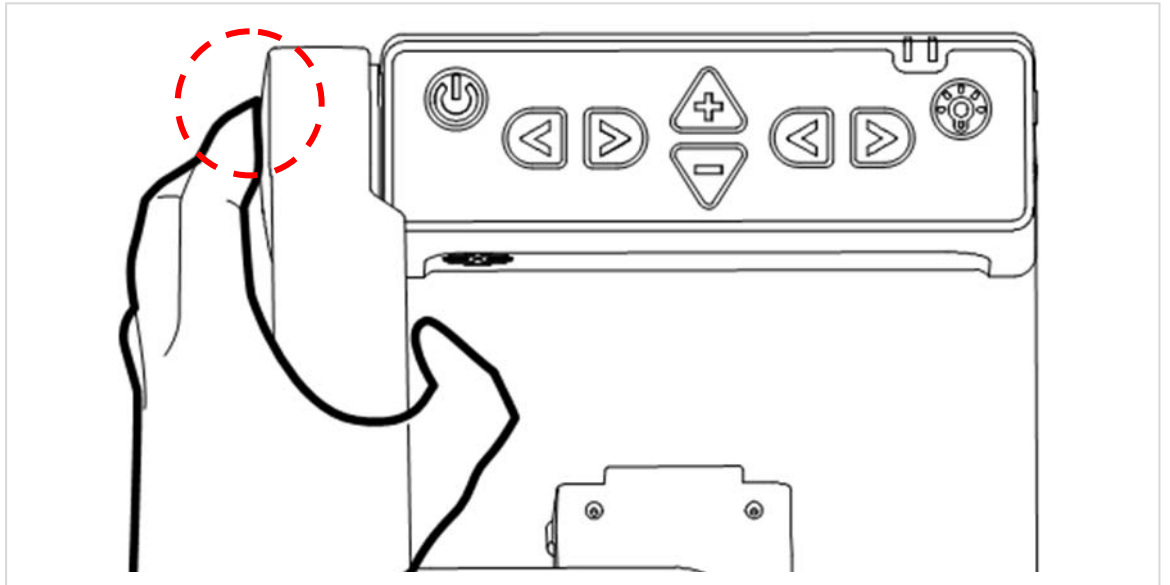

- ② Then, grip the shorter camera arm and unfold it upward as well.

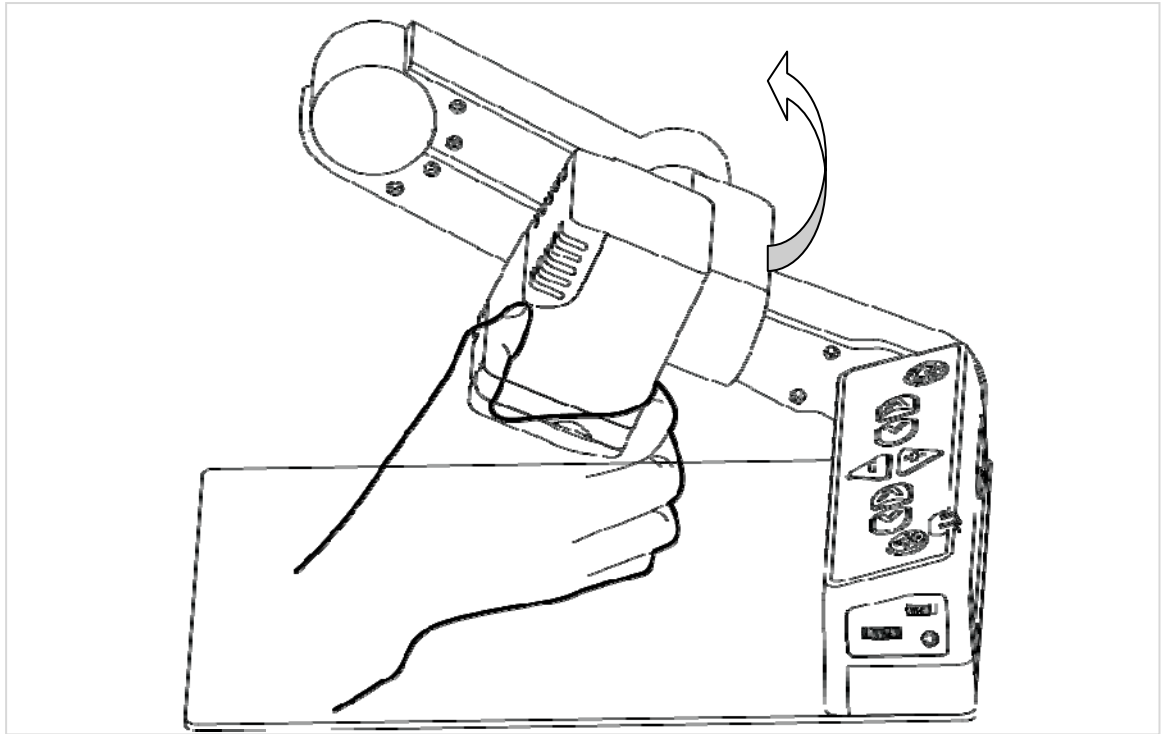

- ③ The setup to use E-bot is complete.

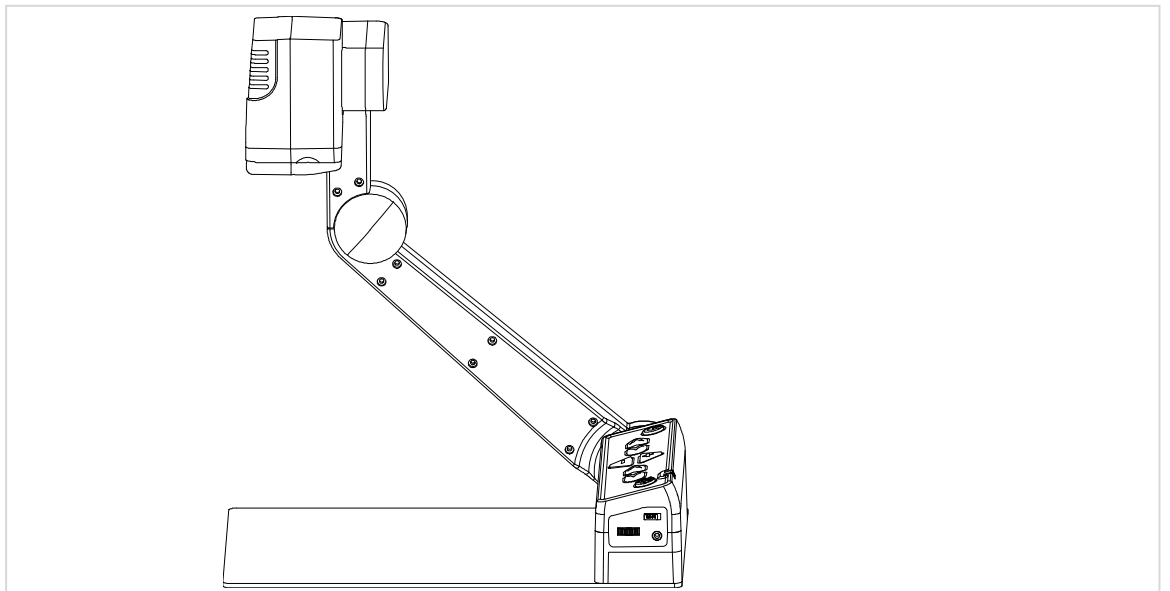


2) Charging

E-bot has a removable battery, allowing you to use the product without a power supply.

- ※ When you use the E-bot for the first time or after a long period of disuse, please allow the unit to charge for more than 5 hours


3) Charging via the AC Adapter

The DC jack is located on the back of the E-bot.

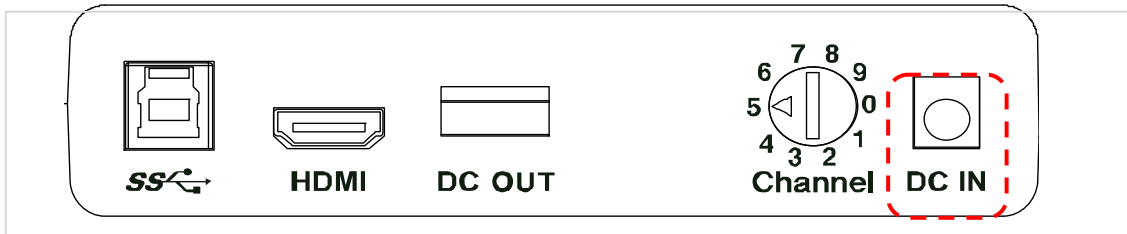

When E-bot is connected to the AC adapter, the battery is being charged. When the battery is being charged, the Power LED appears red. When the battery is fully charged, the power LED color is changed to green.

The power LED is located on the top right of the control panel of the main body. The power LED indicates status as follows:

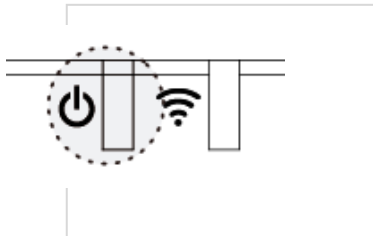

4) Charging via USB

- ※ If you connect the E-bot to a computer via USB while E-bot is powered off, the E-bot's battery is charged.

When E-bot is powered on, power is no longer supplied to the battery for charging, but rather, is used for outputting video data to the PC.


5) Connecting to displays

E-bot is compatible with various types of display devices, such as desktop video monitors, laptop computers, and Apple or Android

tablets.

6) Connecting a monitor (HDMI)

You can connect E-bot directly to a desktop monitor or TV via HDMI without the need for a computer or additional software as all of E-bot's programming and controls are on the main body of the unit and the wireless remote control.

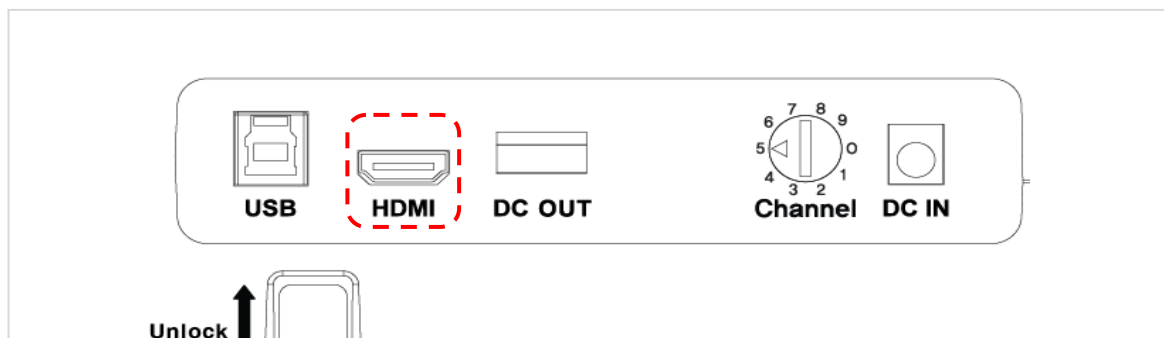

- ※ The monitor needs an external power source even though the E-bot is connected to AC power.
- ※ If the monitor has no HDMI port, please use the included HDMI to DVI gender adapter.

7) Connecting an On-Lap monitor (DC OUT)

The On-Lap Monitor is a portable, USB powered LCD, sold separately. Because its power comes from USB, it does not require an external power source. If you connect the On-Lap Monitor's USB power cable (USB 2.0) to the DC OUT port on the back of E-bot, you can use E-bot's power to power the On-Lap monitor. Connect On-Lap's HDMI cable to E-bot's HDMI port to display E-bot's output on the On-Lap monitor.




※ Note: The DC OUT port is used only for connecting the On-Lap monitor. Please do not connect USB drives or other USB devices to the DC out port.

8) Connecting to a PC (USB, Wi-Fi)

You can view E-bot's high-resolution image using a PC via the included USB 3.0 cable or via Wi-Fi. You can also control E-bot's functions via your PC's keyboard and/or mouse.

※ For instructions on controlling E-bot using a keyboard and mouse, please refer to the last section of this manual.

• USB

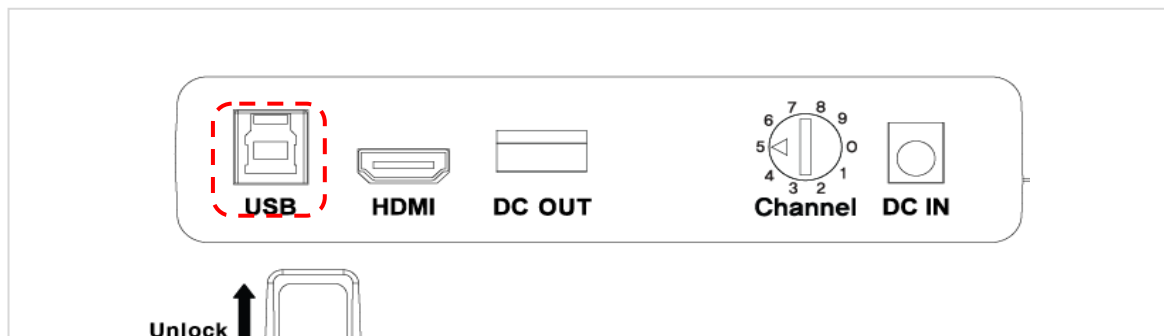

- ① Please install the E-bot Viewer program on your PC using the installation CD in your packing box.
- ② Connect the E-bot's USB 3.0 port to the USB port of your PC using the included USB 3.0 cable.
- ③ When the installation is complete, turn on the E-bot. At this point,

make sure the Wi-Fi switch is set to off. And run the E-bot Viewer on your PC. Select USB as the connection method.


※ If you connect E-bot to a PC which does not support USB 3.0, the enlarged image resolution is automatically adjusted to accommodate the difference in connection speed. Please note: E-bot cannot be supported on Mac computers which do not have USB 3.0 capability.

- **Wi-Fi**

E-bot can output magnified images via Wi-Fi connection to Windows PC's, Android tablets and Apple iPads. Note: Wi-fi output is not supported on Mac computers.

- ① To output to your PC via Wi-fi, install the E-bot Viewer program on your PC using the installation CD in your packing box.
- ② Set the E-bot Wi-Fi switch to ON. The Wi-Fi LED is changed to blue to indicate Wi-fi is on.

The Wi-Fi LED is located on the top right of the control panel on the main body. Wi-Fi LED status is shown as follows:

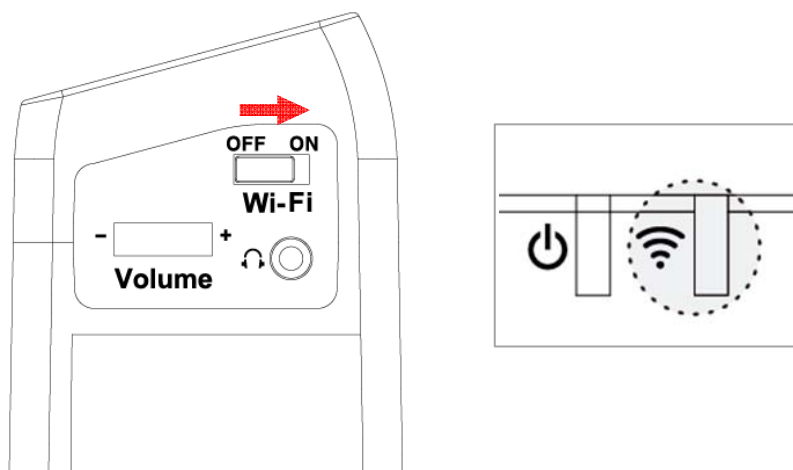


- ③ Enter the PC's wireless network connection area and connect to the E-bot network.
- ④ When you are asked for the network key, please enter the E-bot's Wi-fi password. The default password is '00000000 '.
- ⑤ Once your PC is connected to the E-bot network, Run the E-bot Viewer program on your PC, and select Wi-fi as the connection method.

9) Connecting to an Apple iPad or Android Tablet (Wi-Fi)

You can view E-bot's magnified image using an Apple iPad or Android tablet using a Wi-Fi connection. E-bot's functions can then be operated via the tablet's touch screen.

- ① To view and operate E-bot via your tablet, please connect to the Apple or Android app store, and install the E-bot Viewer application.
- ② When the installation is complete, turn on the E-bot. Then turn on the Wi-Fi switch on the right side. Once Wi-Fi is turned on, the Wi-Fi indicator light on the main body appears blue.

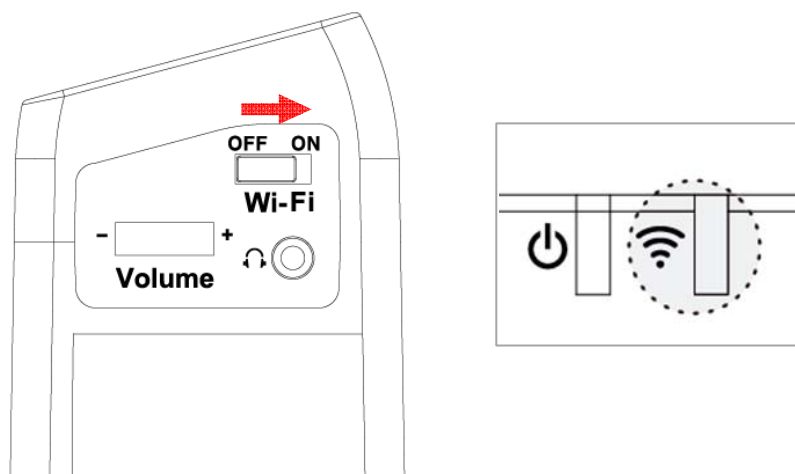

- ③ Be sure Wifi is on on your tablet, then select and connect to "E-bot" from the Wi-Fi networks list.


④ Connecting to E-bot requires a network key. By default, it is “00000000”. Enter the network key and connect as you would normally connect to a Wi-fi network on your tablet.

⑤ Run the E-bot Viewer app on your tablet, and make sure that the image is displaying correctly.

※ Note: In the E-bot's options menu, you can change the Wi-Fi password.

5. Basic Functions of E-bot

1) Power On/Off

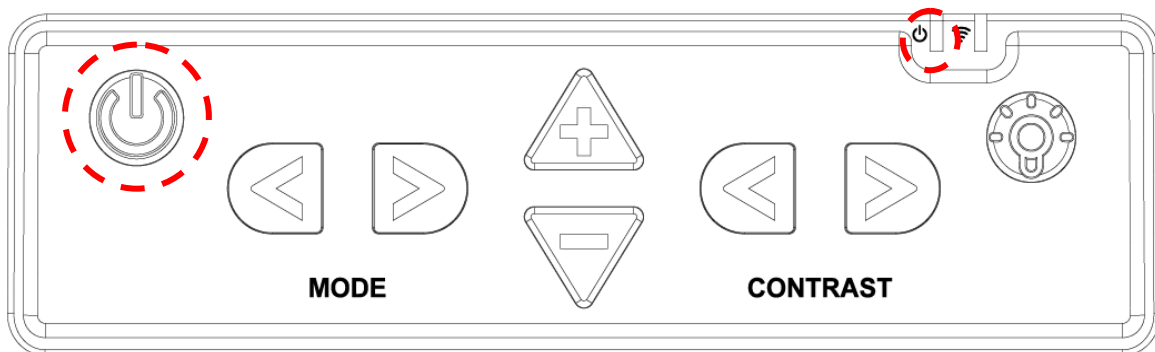



Press and release the power button on the E-bot's main body to power the unit on or off.

When the power is turned on, press and hold the power button for more than 2 seconds, the unit powers off. If you do not intend to use E-bot for a long period of time, please power it off.

Note: If the E-bot is in the folded position, it cannot be powered on.

2) Changing Camera Modes

E-bot has two camera modes; near viewing mode and distance viewing mode. The camera mode can be changed by rotating the camera.

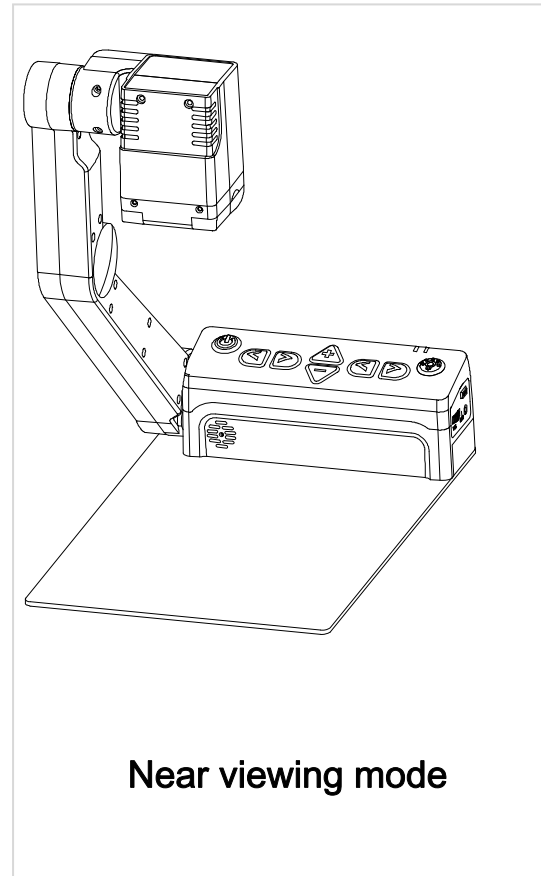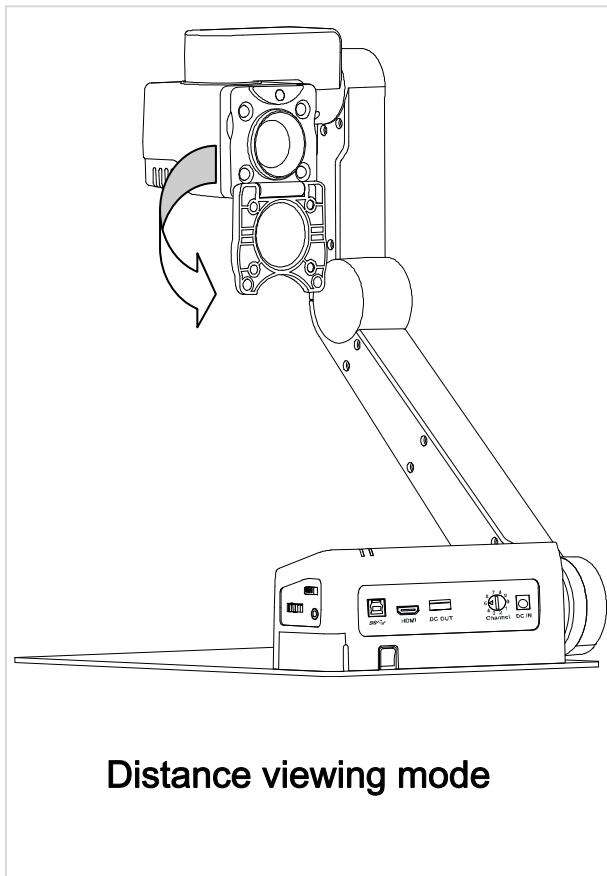


Flipping down the close-up lens switches to distance viewing mode.
Flip the close-up lens closed to switch back to near viewing mode.

※ Magnification, color mode, and function settings for each mode are saved separately. So when switching the camera mode, the settings are automatically changed to the settings you last used for that mode.


3) Magnification

You can change the magnification level to suit your visual needs and the size of the print you're currently reading.

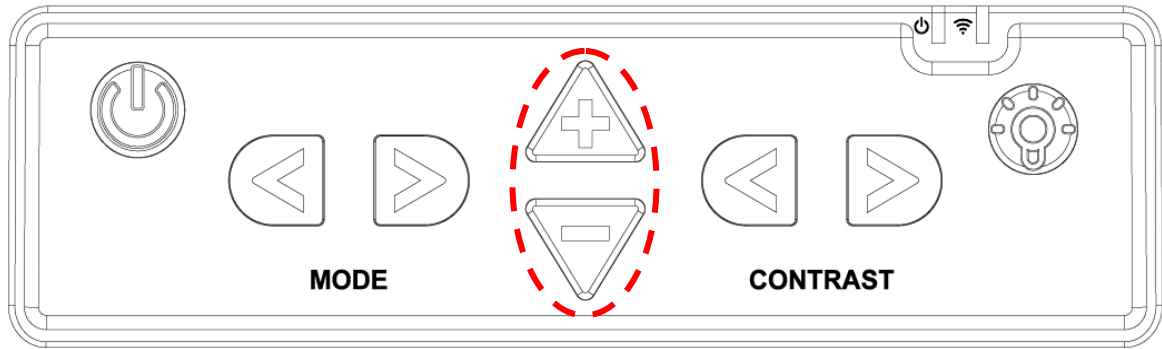


Press the Zoom In button (+) to increase the magnification level.
Press the Zoom Out button (-) to decrease the magnification level.
As you press and hold either of these buttons, the magnification level is adjusted continuously.

The zoom in / out buttons on the wireless controller are located as shown below:

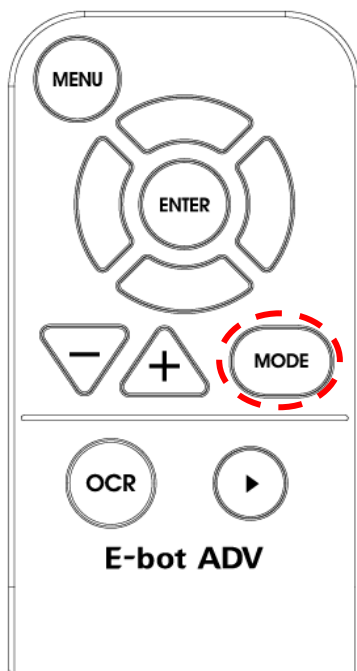


4) Changing the Color Mode

You can change the color of the text and/or background on the screen to accommodate your visual needs in regards to text type and your visual condition.

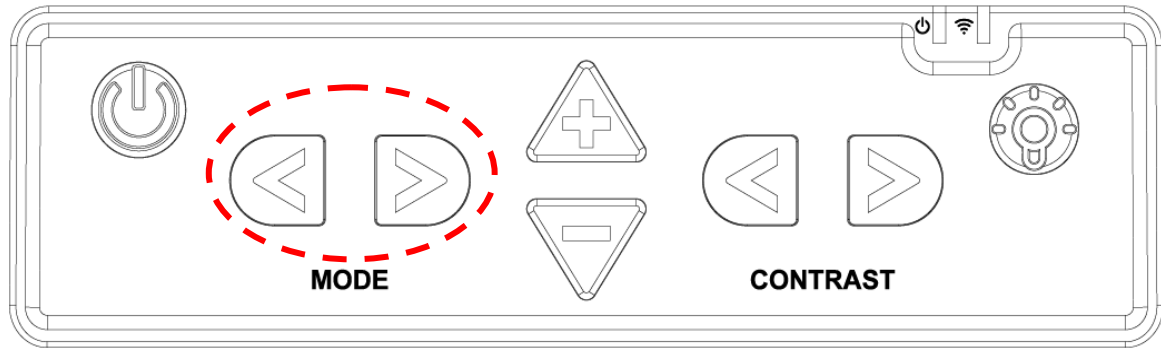

Press the Previous color button to change to the previous color mode. To change to the next color mode, press the next color button.

You can also use the Mode button on the wireless controller to cycle through the color modes.

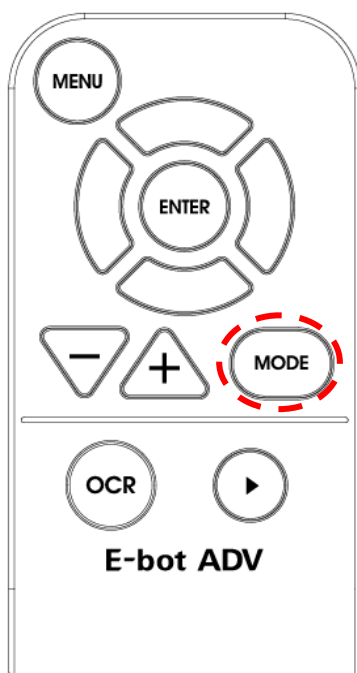

There are 7 color modes provided by default:.


1. Natural Color Mode
2. White text / black background
3. Black letters / white background
4. Black and White (Gray)
5. Yellow letters / black background

6. Yellow letters / blue background

7. Green letters / black background


5) Contrast

Due to variations in paper type, text type and color and the background color of documents or books that you are reading, you may need to adjust the contrast to read the documents or books clearly.

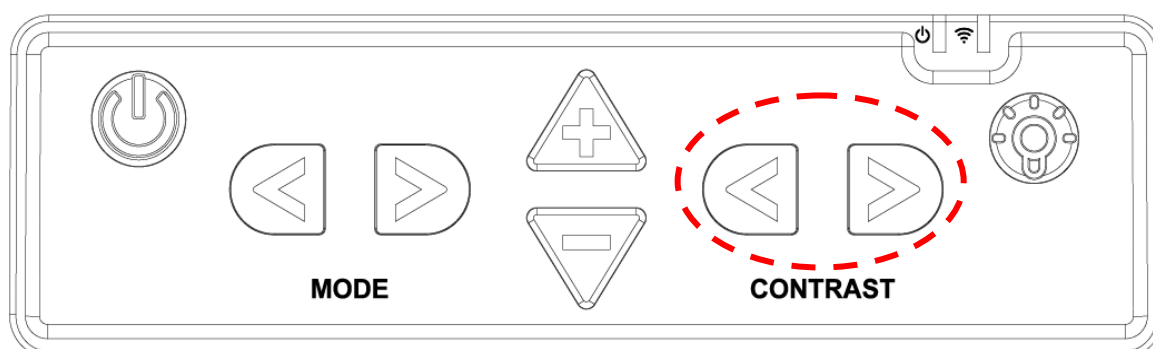


You can adjust contrast by using the Contrast Increase/Decrease buttons on the control panel on the main body.

Or press the Enter button and Zoom In/Out button simultaneously on the wireless controller.

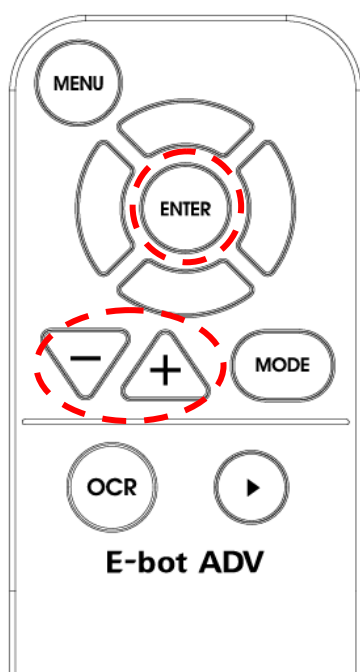

6) LED light On/Off

In the near view mode, the built-in light is automatically turned on to ensure a clear image. E-bot contains a sensor to detect the amount of illumination in your current surroundings, and lighting is automatically adjusted according to the ambient brightness.

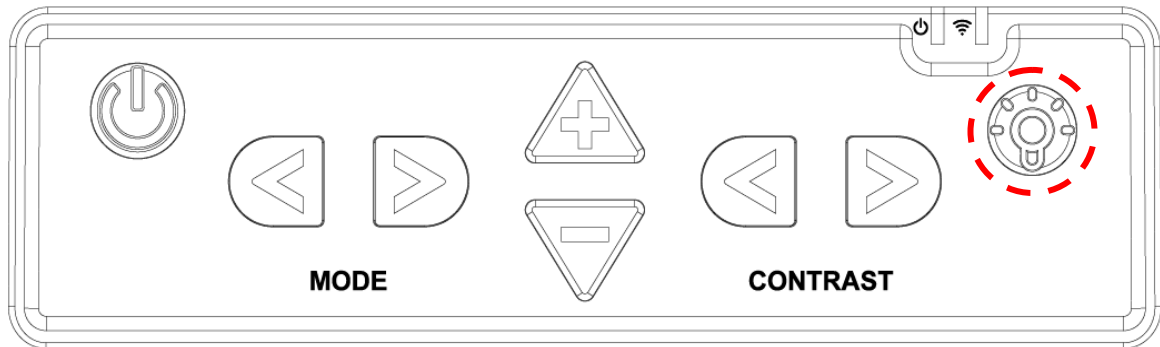


Press the LED button to turn the lights on or off. When viewing glossy objects, please turn off the LED light to avoid glaring.

- ※ If you turn the power off, when you turn it back on, the LED light is automatically turned off.
- ※ When using distance view mode, the LED light is not turned on.

7) Finding your current Reading Position


Use the Find location function, to find the position of the magnified text you are currently reading in relation to the document placed on the reading plate or the presentation screen or whiteboard you are viewing from a distance.

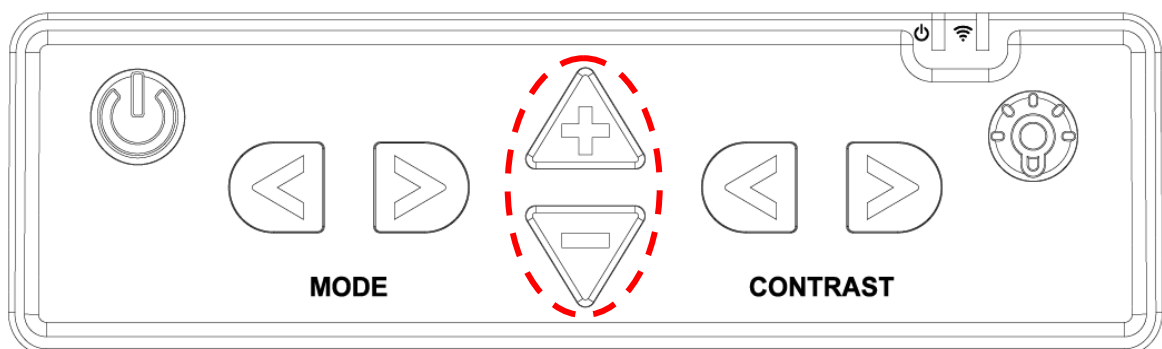

Press and hold the Zoom in button and Zoom out together, the magnification level is reduced back to normal and a red cross appears in the center of the display to assist your orientation so that you can locate the placement of the text in relation to the whole document.


Release both the Zoom in and Zoom out buttons, and the magnification level returns to the rate you were using before pressing the Zoom buttons.

Use the following key combination on the wireless controller to find your reading location:

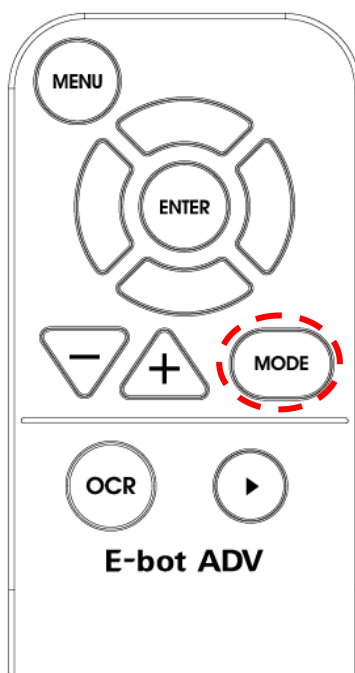


8) Switching to and from Natural Color mode

The E-bot allows you to alternate between Natural color mode and your current color mode setting. Press the Color Mode button on the wireless remote control for more than 2 seconds to switch between Natural Color Mode and your current color mode setting.

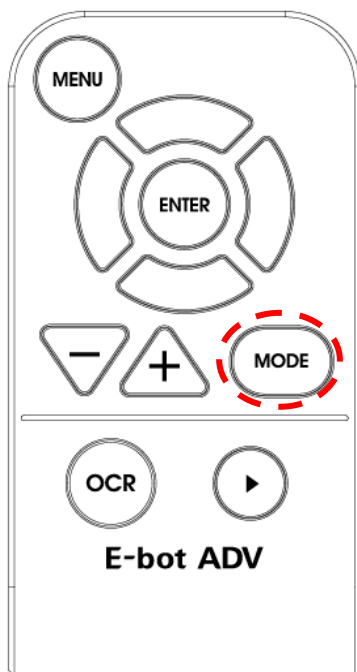

9) Freeze


Press and release the Play/Pause button to take a still shot.

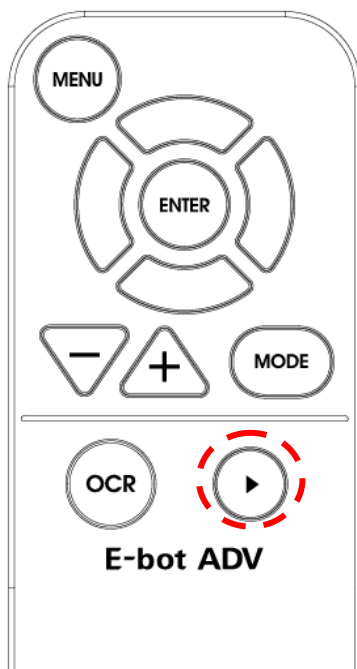

Press the Play/Pause button again to release the still shot and return to the live camera mode.


10) Focus lock

By default, E-bot uses automatic focus. However, if you're writing under the camera or doing something else on which you don't want the camera to focus, you can use the Focus lock to lock the focus on the text rather than allowing the auto focus to follow your hand or pen.

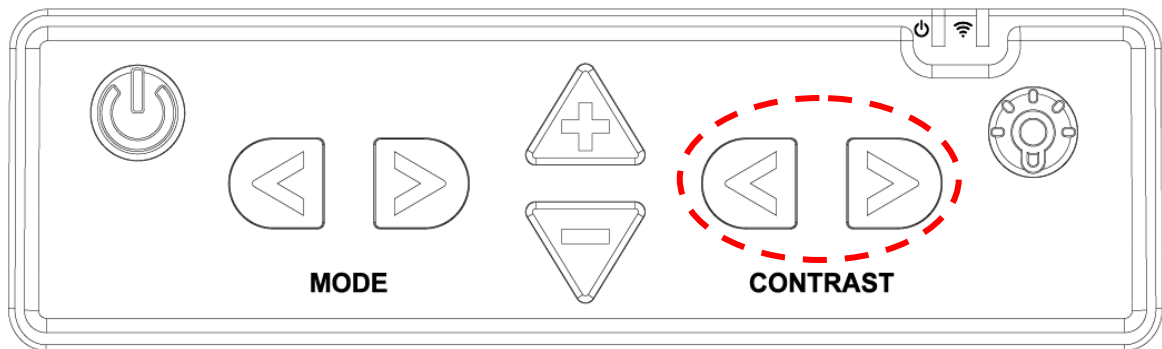

Press both the Contrast Increase button and Contrast Decrease button on the top of the main body.

Or press the Enter button on the wireless controller for more than 2 seconds.

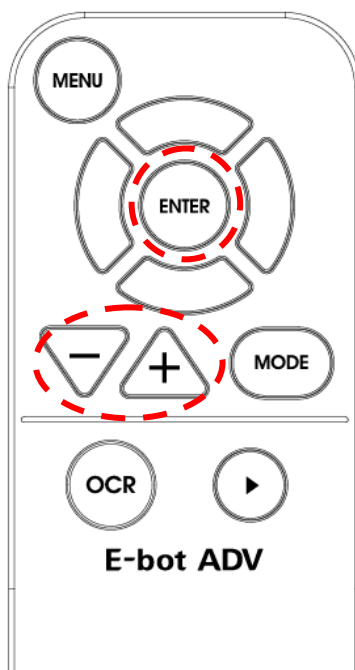

When focus lock is activated, the Focus Lock icon is displayed on the screen.




※ Focus is unlocked automatically when the camera is switched from Distance view mode to near view mode or powering off when focus is locked.

6. Advanced features

1) Using the menu

You may change many of the E-bot's settings via the Options menu.

Use the menu in the manner described below:

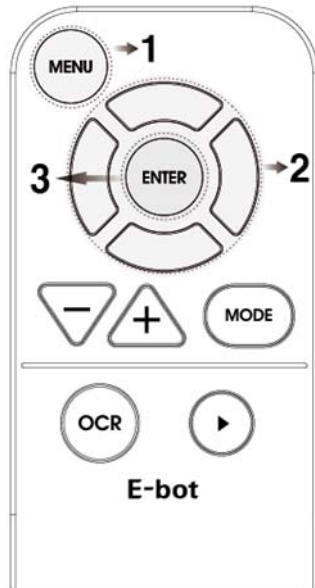

- 1. Menu button:** 1) Open Menu
2) Select item
3) Close Menu

- 2. Up/down button:** Move between items
Right/Left button : Change setting values

- 3. Enter Menu:** Select item

- ① Menu: Press the menu button on the wireless controller, and a menu window pops up on the left side of the screen.
- ② Tabs: The Menu consists of two tabs, 'Options' and 'OCR'. You can set the options related to magnification functions in the

'Options' tab, and can set the options related to OCR functions in the 'OCR' tab.

You can move between the Tabs using the Right and Left buttons.

- ③ Move between options: You can move through options in a tab using the Up and Down buttons. To select an Item, press the Enter button.
- ④ Moving back a level: to move back a level in the menu structure, press the menu button.
- ⑤ Closing the Menu: To close the menu, press the menu button from the top level of the menu structure.

2) Color mode settings


You can set the color mode to best accommodate your visual condition or the material you are reading. You can also disable color modes that are not useful to you.

- Setting the color mode

- ※ You can change to the frequently-used color except for Natural color mode. Up to 7 color modes can be set on the options setting.

- ① Bring up the menu while using the view in which you want to change (for instance, if you bring up the menu while using the near view, the color mode of the near view is changed)
- ② You can modify the color modes by choosing the “Color Mode” setting in the Options tab of the menu.
- ③ You can modify any of the color modes numbered 1 to 7. Select the mode you want to set.

- ④ The text Letter Color is displayed on the screen. Select the color of the text using the left and right arrows

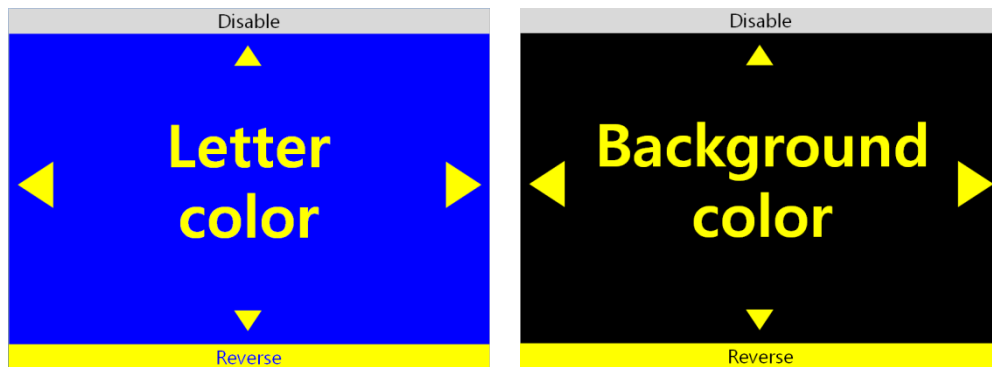


- * The text color and background color being displayed are exchanged when the [Reverse] button is pressed.
- ⑤ Select the Background color when you have finished selecting the text color. Use the same method as when setting the letter color..
- * If Grayscale is chosen when selecting the Letter color, the Color setting mode is exited without selecting a Background color.
- ⑥ After you have selected the background color, you are returned to the menu window.

● To disable color modes

You can disable color modes that are not useful to you so that you may cycle through useful color modes more quickly.

- ① Bring up the Menu, go to the Options tab, then select “Set color mode”.
- ② Select one of the color modes numbered 1 to 7.
- ③ If you press the Up arrow key when “Letter color” is displayed, a window appears asking whether to disable the current Color mode. Choose ‘Yes’ to disable the Color mode or ‘No’ to cancel.

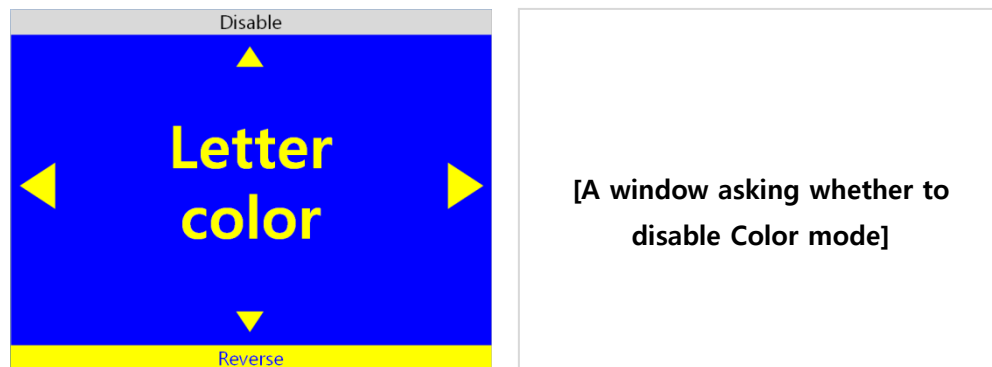

- ④ When you have made your selection, you are returned to the menu.

3) Marker

The Marker helps you to focus on the text that you are reading by narrowing the display area with window markers or by underlining the sentence that you are reading.

● Set Marker

There are 4 kinds of markers; Horizontal Line Marker, Vertical Line Marker, Horizontal Window Marker and Vertical Window Marker.


• Horizontal Line Marker

ABCDEFGHIJKLMN	OPQR
ABCDEFGHIJKLMN	OPQR
ABCDEFGHIJKLMN	OPQR
ABCDEFGHIJKLMN	OPQR


• Vertical Line Marker

ABCDEFGHIJ	KLMNOPQR
ABCDEFGHIJ	KLMNOPQR
ABCDEFGHIJ	KLMNOPQR
ABCDEFGHIJ	KLMNOPQR

• Horizontal Window Marker

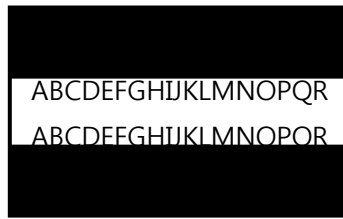

- Vertical Window Marker

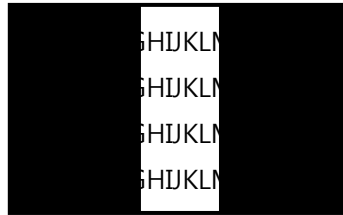

- ① Bring up the Menu, select “Options” then select “Marker”.
- ② Next, select the marker type; ‘Horizontal line marker’, ‘Vertical line marker’, ‘Horizontal window marker’, ‘Vertical window marker’. A check mark appears next to the selected marker.
- ③ After you have chosen the marker setting, the selected marker is displayed on the screen and you are returned to the Menu again.
 - * Select the marker setting from the menu again to disable the marker.

- Set Marker position and size

You can adjust the size of the window marker and location of the line marker.

- ① Bring up the Menu and select “Options” then “Marker”.
- ② If the line marker is selected, , the “Adjust location of marker” item appears. If the window marker is selected, the “adjust size of window marker” option appears. Select the relevant marker setting.
- ③ Use the arrow keys to adjust location or size.


- ④ Press Enter, and the setting is saved and you are returned to the menu.

※ Note: markers cannot be used in OCR mode.

4) OCR

Recognizes text under the camera when using near view mode and reads it aloud with built-in text-to-speech voices.

● OCR execution

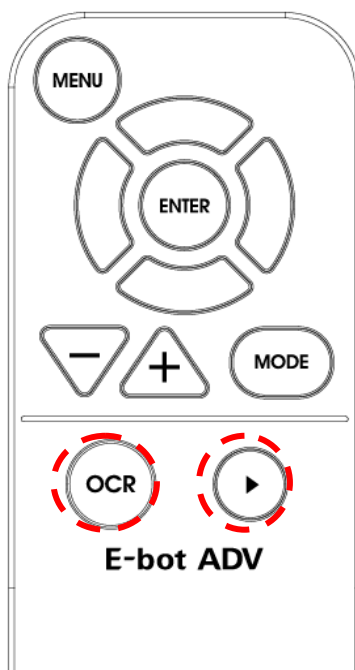

- OCR button : OCR execution
- ▶ button : OCR Play/Pause

When using near view, the image freezes when the OCR button is pressed. The text displayed on the frozen screen is recognized then read aloud using the internal TTS voices.

※ While recognition is being performed, a message indicating recognition is in progress appears. Please do not perform other functions while recognition is in progress.

Press the Play/Stop button to stop and start reading text aloud, if the Play/Stop button is pressed after text reading has completed, reading will start again from the beginning of the text.

Press the OCR button again to exit the OCR function and return to live camera mode.

※ If the magnification level is changed during viewing of recognized text, the magnification level is changed. However, if the the OCR view is set to image, the magnification level cannot be adjusted below the magnification level that was set when OCR was first played.

※ You may change the Color mode when viewing an OCR scan. The available color modes are the same as those available in near view mode.

- Reading/navigating recognized text.

Press the Right arrow key to move to the next sentence.

Press the Left arrow key to move to the previous sentence.

Press the Down arrow key to move to the next paragraph.

Press the Up arrow key to move to the previous paragraph.

Navigate to the place where you want to begin reading text aloud and press the Play/Stop button to begin reading at that point.

- OCR views

You may display OCR results as either an image, or as recognized text.

① Bring up the Menu then press the Right arrow key to move to the

OCR tab and select "OCR View".

- ② There are 4 options for displaying OCR results. , The current setting is indicated with a check mark. Select your desired view for OCR results.

- ✓ Image : Displays scanned image, highlights word being read aloud.
- ✓ Whole text : Extracts recognized text and shows it on the screen.
- ✓ Line : Extracts recognized text and shows the line currently being read aloud..
- ✓ Word : Extracts recognized text and shows each word as it is read aloud on the screen.

- OCR correction

The deskew function corrects the orientation of the text to the horizontal if it is not aligned correctly for more accurate text recognition. You can turn this option on or off.

- ① Bring up the Menu, and press the right arrow key to move to OCR tab, then "OCR Correction". Select the "Deskew" item.
- ② If the check box is checked Deskew is on. If it is not checked, Deskew is off.
- ③ Press enter to change the setting.

- OCR Language setting

This feature distinguishes and recognizes up to 3 different languages even if they are mixed on the screen.

- ① Bring up the Menu and press the Right arrow to move to the

OCR tab. Then, select the OCR language setting.

- ② Select the language you want to add. E-bot can recognize up to 3 languages simultaneously. . Note: the base language cannot be unchecked.

※ Language recognition is distinguished by paragraph if the selected language is based on Alphabet.

5) E-bot operating Language

You can change the basic language E-bot uses for its voice guide and menu system.

- ① Bring up the Menu and select “Language” from the ‘Options’ tab.
- ② Navigate to the language you wish to use with the Up and Down arrows, and press the Enter key on your chosen language.

.

6) Voice Guide

This option determines whether E-bot speaks as you navigate menus or press buttons.


● Voice Guide On/off

- ① Bring up the Menu and select “Voice guide” from the ‘Options’ tab
- ② If the checkbox is checked, the voice guide is on; if it is not checked, voice guide is Off. Press the Enter button to change the setting.

● TTS Volume

You can adjust the volume of the TTS voice in two ways.

- Adjusting on the main body

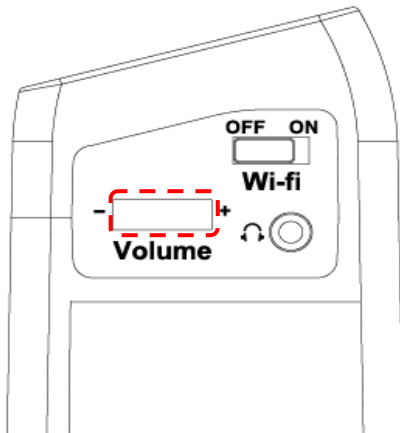

- ① E-bot's volume dial is located on the right side of the body.
- ② Turn the dial away from you, and the volume is increased, turn the dial toward you, and the volume is decreased.

- Adjusting in the menu

- ① Bring up the Menu, and select "Volume" in the "OCR" tab
- ② You can select from 14 volume levels. Use the Left and Right arrows to choose your desired volume level, then press Enter to save the settings. You can Mute the volume by setting the level to 0.

● TTS Speed

The TTS speed setting controls both the speed of the guide voice as well as the OCR reading voice.

- ① Bring up the Menu, and select "TTS speed" from the OCR tab.
- ② The TTS speed is adjustable using 14 levels. Use the Left and Right arrows to adjust the speed of the TTS then press Enter to save the setting.

● TTS Pitch

You can adjust the TTS voice pitch.

- ① Bring up the Menu, and select “TTS pitch” from the “OCR” tab.
- ② The TTS pitch can be adjusted among 14 levels. Use the Left and Right arrows to control the pitch of the TTS then press Enter to save the setting.

- TTS voice

Use this setting to select the gender of the voice.

- ① Bring up the Menu and Select “TTS type” from the “Options” tab.
- ② Use the Left and right arrows to select voice between male and female.

7. Using other functions

1) Initialization

- ① Set the settings back to the factory defaults.
- ② Bring up the Menu and Select “Initialization” from the “Options” tab.
- ③ You can initialize settings in the “Options” tab, in the “OCR” tab, or initialize all settings.
 - ✓ Initialize Options: Initialize settings in the Options tab of the Menu.
 - ✓ Initialize OCR: Initialize settings in the OCR tab of the Menu.
 - ✓ Initialize all settings: Initialize all settings including options tab and OCR tab settings.

2) Information

Use this option to check general information about the E-bot.

- ① Bring up the Menu and select “Information” from the “Options” tab.
- ② Here, you can check the following information about E-bot:
 - ✓ Software version
 - ✓ Products serial number
 - ✓ Remaining battery
 - ✓ Wi-Fi AP
 - ✓ Information of wireless remote control

3) Wi-Fi password change

An eight -digit number is needed to connect E-bot to Wi-Fi using an iPad, Android tablet or PC. The default password is 00000000. To change the password, follow the instructions below:

- ① Bring up the Menu and select “Wi-Fi password change” in the “Options” tab.
- ② A window asking for the current password appears. Input the password using the virtual keyboard on the screen. Use the arrow keys to navigate to the character you want to type, and the Enter button to input the character. Input the current password and press Enter on the virtual keyboard.
- ③ A window asking for the new password appears. Input the 8-digit password in the manner described above.
- ④ A window asking you to reconfirm the password to change appears. Input the new password again.
- ⑤ When you have finished the above steps, the new password is set.

4) Keyboard Hot keys: (Only when using PC to display and control E-bot.)

When using the E-bot Viewer application on your PC to display and control E-bot, you can use your keyboard and mouse to control e-bot's functions.

Users can even define the hotkeys to use E-bot more conveniently for them.

- ① Bring up the Menu and select "Hotkey change" in the "Options" tab.
- ② A list of hot keys is displayed. Select one of the hot keys in the list.


8. Troubleshooting

1) When the wireless controller does not work

The main body and the connected controller's channel setting value must be set the same for the controller to operate E-bot properly. If there are two or more E-bots in the same place, settings must be set differently, so that the controllers do not control the wrong E-bot, etc.


- Check the Main body channel values

- ① Please check the channel values on the rear of the Main body. An Arrow is pointing to the number currently set.

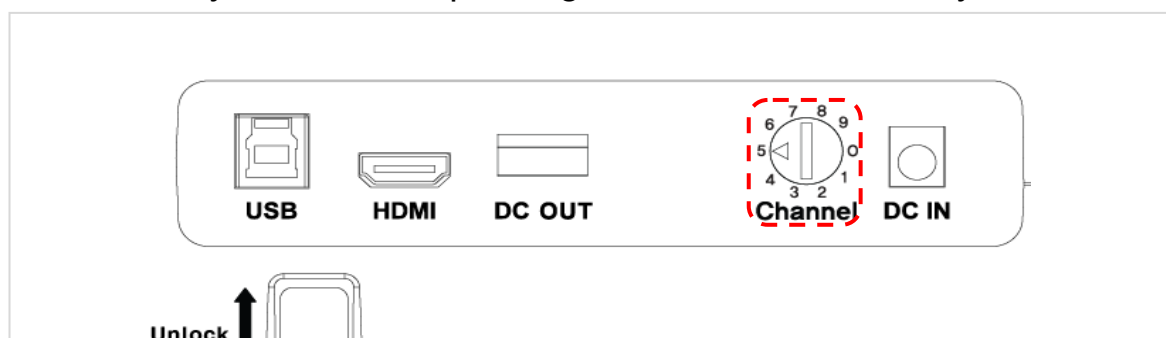

- Checking the wireless remote control channel value

- ② Remove the battery cover from the back of the remote control.
- ③ Check the channel setting switch. It can set the channel values as shown in the below picture. Make sure it is set to the same channel value as the one shown on the main body of your E-bot. If the channel value is different, please use (-) a screw driver or your fingernail to change the setting.

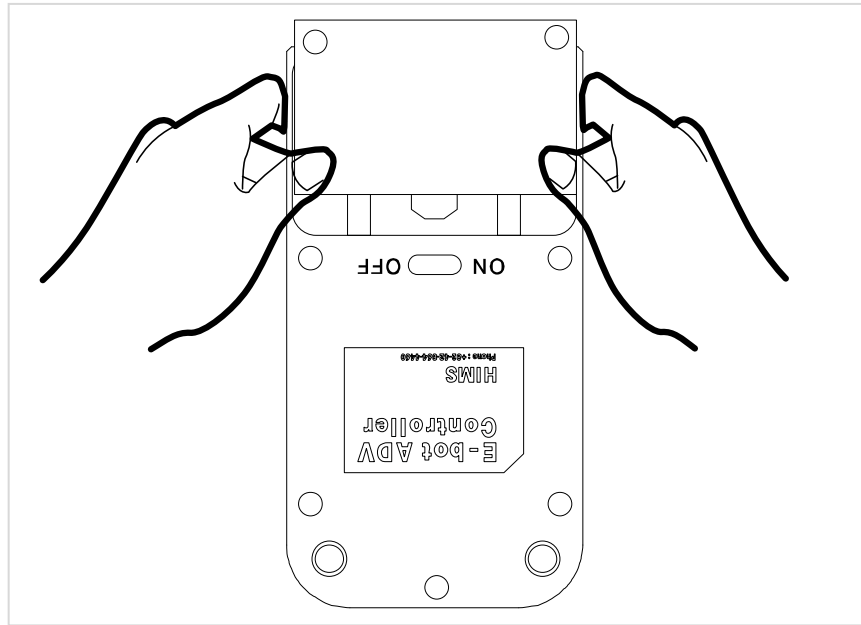

- ※ Changing the battery of the wireless controller, if the wireless controller does not operate even though the channel value is the same, please change the battery in the wireless controller.

9. A/S

Manufacturer: HIMS International

Manufacturer Address: 174 Gajeong-ro, Yuseong-gu,

Daejeon, Korea 305-350

E-mail: sale@himsintl.com


Phone Number: 042-864-4601




FAX: 042-864-0133




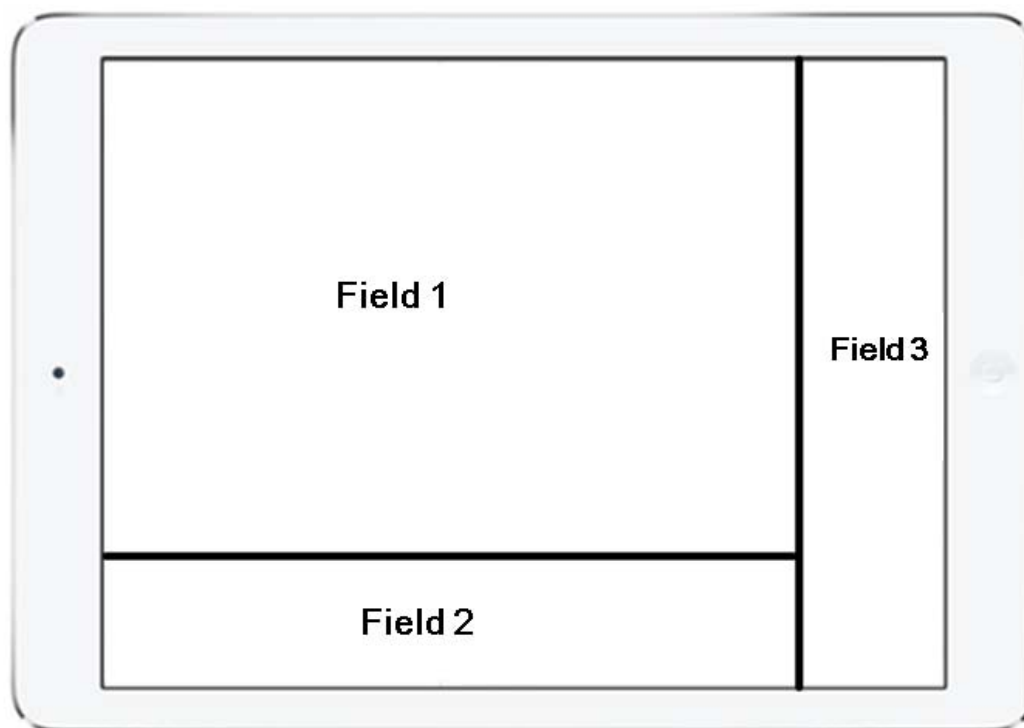


10. Specification

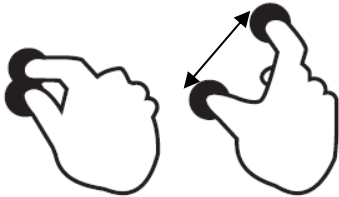


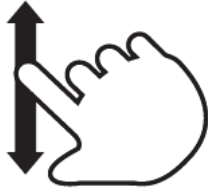


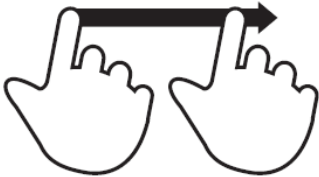
Camera	HD, 60fps
Video output	Wi-Fi : Android Tablet, iPad, Windows PC USB3.0 : Windows PC & MAC USB2.0 : Windows PC HDMI : TV, Monitor
Distance Viewing	Supported
Magnification rate	9.7"(Tablet) : 0.8x ~ 35x 13.3"(Monitor) : 1.2x ~54x 24"(Monitor) : 2.2x ~ 97x
Color modes	8 modes (7 modes are user-definable from color pallet and enable/disable modes independently)
Contrast	Supported
Marker(Line/Window)	Supported
Focus Lock	Supported
Find reading location	Supported
Still image	Supported
OCR	Supported
LED Light	* Support only Near view mode * Automatic brightness control
Battery	Rechargeable Lithium Ion battery More than 4 hours battery life
Size	Folded : 315mm(H) x 215mm(W) x 99mm(D) Unfolded : 315mm(H) x 215mm(W) x 369mm(D)
Weight	2.3kg / 5.0lbs



※ Smart Pad Viewer Gesture



Functions	Smart Pad
	Gesture
Zoom In	 (Field 1) Move fingers away from each other
Zoom Out	 (Field 1) Pinch fingers towards each other
Find Position	 (Field 2) Two-finger one-tap more than 2 seconds

Next color mode	 (Field 2) One-finger flick to right
Previous color mode	 (Field 2) One-finger flick to left
Switch to Natural Color Mode	 (Field 2) Two-finger double-tap
Contrast Up	 (Field 3) One finger flick to up
Contrast Down	 (Field 3) One finger flick to down
Freeze	 (Field 1) One finger Double-tap

OCR	OCR Execution	 <p>(Field 2) One finger Tap for more than 2 seconds</p>
	OCR Play/ Pause	 <p>(Field 1) One finger Double-tap</p>
	Move Sentence	 <p>(Field 1) One finger flick to left or to right</p>
	Move paragraph	 <p>(Field 1) One-finger flick up or down</p>
	OCR Exit	 <p>(Field 2) One finger Tap for more than 2 seconds</p>
	OCR mode ↔ Camera Preview	 <p>(Field 2) In OCR mode, Two-finger one-tap more than 2 seconds</p>
Menu	Open	

		One finger Tap on the left end of the screen and flick to right	
Moving tap	One finger Tap on the tap		
Move an item	One finger Tap on the wanted item		
Select / Set / Enter the sub-item	One finger Tap on the wanted item		
Save	One finger Tap on the confirm icon		
Cancel / Exit	One finger Tap on the cancel icon		
Close	 One finger Tap on the right end of the screen and flick to left		

⌘ How to control

Features		How to Control					Additional Description
		Main Body	Remote Controller	Keyboard	Mouse	Touchpad	
Power	Power On	Power BTN shortly (when power is off)	x	x	x	x	It takes around 45 seconds to boot E-bot up
	Power Off	Hold Power BTN over 3 seconds					
Sleep mode		Power BTN shortly	x	x	x	x	Enter/Release the Sleep mode
Light On/Off		Light BTN	x	x	x	x	Only in Near View mode
Zoom	Zoom In	Zoom In BTN	Zoom In BTN	[+]	Wheel scroll up (click Left click to	Move fingers away from each other	Continuous zoom

E-bot manual

					stop zooming in)		
	Zoom Out	Zoom Out BTN	Zoom Out BTN	[-]	Wheel scroll down (click Left click to stop zooming out)	Pinch fingers towards each other	Continuous zoom
	Find Position	Zoom in BTN +Zoom out BTN	Zoom In BTN +Zoom Out BTN	Ctrl + [f] need to be kept holding	x	Two-finger one-tap on the bottom of the screen for more than 2 seconds	Toggle between "Current zoom position" and "Minimum zoom position"
Color Mode	Mode changes	Previous Color Mode BTN/ Next Color mode BTN	Color Mode BTN	[m]/ [Shift]+[m]	Left click shortly	One-finger flick to right on the bottom	Changes in the stored order
	Switch to Natural Color Mode	x	Hold the Color Mode BTN for 2 seconds	[n]	x	Two-finger double-tap on the bottom	Toggle between "Natural color mode"

							and "previous color mode"
	Set Color Mode	x	Menu>Options >Set Color Mode	Menu>Options >set Color mode	Menu>Options >set Color mode	Menu>Options >set Color mode	Color modes from 1 to 7 can be set
	Reverse Color Mode	x	Menu>Options>Set Color Mode> select color mode want to change>[↓]	Menu>Options>Set Color Mode>select color mode want to change>[↓]	Menu>Options>Set Color Mode>select color mode want to change>[↓]	Menu>Options>Set Color Mode>select color mode want to change>[↓]	
	Disable Color Mode	x	Menu>Options>Set Color Mode> select color mode want to change>[↑]	Menu>Options>Set Color Mode>select color mode want to change>[↑]	Menu>Options>Set Color Mode>select color mode want to change>[↑]	Menu>Options>Set Color Mode>select color mode want to change>[↑]	
Contrast	Contrast Up	Increase contrast BTN	Enter BTN + Zoom In BTN	[c] or [↑]	Menu>Options >select Contrast	One finger Tap on right end of the screen and flick to up	
	Contrast Down	Decrease contrast BTN	Enter BTN + Zoom Out BTN	[Shift]+[c] or [↓]	Menu>Options >select Contrast	One finger Tap on right end of the screen and flick to down	
Freeze On / Off		x	Play/Pause BTN	[Space]	x	One finger Double- tap	
Focus Lock On / Off		Increase	Hold the Enter BTN	[Ctrl] + [I]	Menu>Options	Menu>Options	Only in Near

		e contra st BTN + Decrea se contra st BTN	for 2 seconds		>select Focus lock	>select Focus lock	View mode
Marker	Set Marker	x	Menu>Options >Marker	Menu>Options >Marker>select Marker	Menu>Options >Marker>select Marker	Menu>Options >Marker>select Marker	Select between the Horizontal / Vertical Line marker and the Horizontal / Vertical Window marker
	Marker Position	x	(When Marker is set) Menu>Options >Marker>Set line maker position	(When Marker is set) Menu>Options >Marker>Set line maker position	(When Marker is set) Menu>Options >Marker>Set line maker position	(When Marker is set) Menu>Options >Marker>Set line maker position	*Set the location of Line marker
	Marker Size	x	(When Marker is set) Menu>Options >Marker>Set window	(When Marker is set) Menu>Options >Marker>Set window	(When Marker is set) Menu>Options >Marker>Set window	(When Marker is set) Menu>Options >Marker>Set window	*Set the size of Window

			marker size	marker size	marker size	marker size	marker
Menu	Open	x	Menu BTN	[Ctrl] + [M]	Right click	One finger Tap on the left end of the screen and flick to right	-
	Moving tap	x	Left (BTN), Right (BTN) (In the top-level menu item)	(In the top-level menu item) [←], [→]	Left click on the wanted item	One finger Tap on the tap	*Move between Main Category of menu item. *Two kinds of tabs: 'Options', 'OCR'
	Move an item	x	Up (BTN), Down (BTN)	[↑], [↓]	Left click on the wanted item	One finger Tap on the wanted item	Move between the items at the same level
	Select / Set / Enter the sub-item	x	Enter BTN	[Enter]	Left click on the wanted item	One finger Tap on the wanted item	-
	Cancel / Exit	x	Menu BTN	[Back space] or ESC	Left click on the 'Exit' item	One finger Tap on the 'Exit'	-

	Close	x	Menu BTN (In the top-level menu item)	(In the top-level menu item) [Back space] or [ESC]	Left click on the 'Exit' item (In the top-level menu item)	One finger Tap on the right end of the screen and flick to left	-
OCR	OCR Execution	x	OCR BTN	[o]	Wheel click	One finger Tap on the bottom for more than 2 seconds	Execution / Play the OCR program
	OCR Play/Pause	x	Play/Pause BTN	[Space]	x	One finger Double-tap	-
	Move Sentence	x	Left BTN/Right BTN	[←], [→]	x	One finger flick to left or to right	-
	Move paragraph	x	Up BTN/Down BTN	[↑], [↓]	x	One-finger flick up or down	-
	OCR Correction	x	Menu>OCR>OCR correction>Correct Angle or Perspective	Menu>OCR>select OCR correction	Menu>OCR>select OCR correction	Menu>OCR>select OCR correction	Angle correction and Perspective correction
	OCR Exit	x	OCR BTN or Menu BTN	[ESC]	x	One finger Tap on the bottom for more than 2 seconds	Exit the OCR / Switch to the Real View mode
	OCR View	x	Menu>OCR>	Menu>OCR>select OCR View	Menu>OCR>select OCR View	Menu>OCR>select OCR View	Choosing among Image, Full

							text, Line and Word
	OCR Language Setting	x	Menu>OCR>OCR language	Menu>OCR>select OCR language setting	Menu>OCR>select OCR language setting	Menu>OCR>select OCR language setting	Up to 3 languages can be recognized at the same time
Voice guide	Voice Guide On/Off	x	Menu>Options >Voice guide On/Off	Menu>Options >select Voice guide On/Off	Menu>Options >select Voice guide On/Off	Menu>Options >select Voice guide On/Off	Set to ON/Off the Voice guide
	TTS Voice	x	Menu>OCR>Voice settings>TTS voice	Menu>OCR>TTS setting>TTS voice>select Male/Female	Menu>OCR>TTS setting>TTS voice>select Male/Female	Menu>OCR>TTS setting>TTS voice>select Male/Female	Select type of TTS playback voice
	TTS Speed	x	Menu>OCR>Voice settings>TTS speed	Menu>OCR>TTS setting>TTS speed>select TTS speed or] , [Menu>OCR>TTS setting>TTS speed>select TTS speed	Menu>OCR>TTS setting>TTS speed>select TTS speed	TTS playback voice speed adjustment
	TTS pitch	x	Menu>OCR>Voice settings>TTS pitch				
	Voice Volume Down/Up	Rotate wheel to the	Menu>OCR>Voice settings>TTS volume	Menu>OCR>TTS setting>TTS volume>select TTS	Menu>OCR>TTS setting>TTS volume>select TTS	Menu>OCR>TTS setting>TTS volume	

		left/right on the right side of main body		volume	volume		
Menu Language Setting	x		Menu>Options>Language	Menu>Options>Language>select language	Menu>Options>Language>select language	Menu>Options>Language>select language	Menu Settings and Guide Speech Language Settings
Wi-Fi Password Change	x		Menu>Options>Wi-Fi Password Change	Menu>Options>Wi-Fi Password Change	Menu>Options>Wi-Fi Password Change	Menu>Options>Wi-Fi Password Change	Wi-Fi default password is '00000000'
Initialize	x		Menu>Options>Initialize	Menu>Options>Initialize	Menu>Options>Initialize	Menu>Options>Initialize	*Option Reset *OCR Reset *All settings Reset (Wi-Fi password will not be reset)
Information	x		Menu>Options	Menu>Options	Menu>Options	Menu>Options	*Software

		>Information	>Information	>Information	>Information	Version *Serial number *Battery status *Wi-Fi AP
--	--	--------------	--------------	--------------	--------------	---

FCC Information to User

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Caution

Modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

FCC Compliance Information : This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation

IMPORTANT NOTE:

FCC RF Radiation Exposure Statement:

This equipment complies with FCC RF radiation exposure limits set forth for an uncontrolled environment.

The antennas used for this transmitter must be installed to provide a separation distance of at least 25 cm from all persons and must not be co-located or operating in conjunction with any other antenna or transmitter.

CAUTION

To reduce the risk of electric shock, do not remove the top cover (or the rear section).
No user serviceable parts inside, refer servicing to qualified personnel.



This symbol, wherever it appears, alerts you to the presence of uninsulated dangerous voltage inside the enclosure-voltage that may be sufficient to constitute a risk of shock.



This symbol, wherever it appears, alerts you to the important operating and maintenance instructions in the accompanying literature. Please read the manual.

- 1) Read these instructions.
- 2) Keep these instructions.
- 3) Heed all warnings.
- 4) Follow all instructions.
- 5) Do not use this equipment near water.
- 6) Do not use near any heat sources such as radiators, heat resistors, stove, or other equipment that produce heat.

CAUTION
RISK OF EXPLOSION IF BATTERY IS REPLACED
BY AN INCORRECT TYPE.
DISPOSE OF USED BATTERIES ACCORDING
TO THE INSTRUCTIONS

European CE notice to users and product statements

This product is CE marked according to the provisions of the R&TTE Directive(99/5/EC). Hereby, HIMS International Corporation declares that this product is in compliance with the essential requirements and other relevant provisions of Directive 1999/5/EC.

For further information, please contact [http:// www.himsintl.com](http://www.himsintl.com)

CE1177

HIMS International Corporation
174, Gajeong-ro, Yuseong-gu, Daejeon, KOREA 305-350
Tel: +82-42-864-4460
Fax: +82-42-864-4462
URL: www.himsintl.com

CONTACT INFORMATION

Manufacturer Address : 174, Gajeong-ro, Yuseong-gu, Daejeon, KOREA 305-350

To locate in-country HIMS International Corporation, distributors of the Electronic Magnifier (E-bot ADV)
please refer to the HIMS International Corporation Website [http:// www. www.himsintl.com](http://www.himsintl.com)

These distributor(s) represent local contacts for this product.

CORPORATE HEADQUARTERS:

HIMS International Corporation

174, Gajeong-ro, Yuseong-gu, Daejeon, KOREA 305-350

Tel: +82-42-864-4460

Fax: +82-42-864-4462

Web: [http:// www.himsintl.com](http://www.himsintl.com)

This device complies with Industry Canada license-exempt RSS standard(s).

Operation is Subject to the following two condition: (1) this device may not cause interference
,and (2) this device must accept any interference, including interference that may cause undesired
operation of the device.

EUROPEAN UNION “DECLARATION OF CONFORMITY”

DECLARATION OF CONFORMITY

HIMS International Corporation

174, Gajeong-ro, Yuseong-gu, Daejeon, KOREA 305-350

declare under our sole responsibility that the product(s)

Electronic Magnifier (E-bot ADV)– EB100A

to which this declaration relate(s) is in conformance with the following standards:

EN 301 489-1 V1.9.2:2011

EN 301 489-17 V2.2.1 :2012

EN 55022:2010

EN 55024:2010

EN 60950-1:2006/A2:2013

EN 300 328 V1.7.1:2006

following the provisions of the 1999/5/EC Directives.