



NX Patient Station Installation and Operation Manual

P206-001-OM
October 3, 2002
Revision 0.00.a

Table of Contents

A. Care of the NX Patient Station	3
C. Criteria Selection Screen.....	6
D. Steps For Using the Blood Pressure/Pulse Meter.....	6
E. Steps for Using the Scale	7
F. If Something Goes Wrong	8




A. Care of the NX Patient Station

1. You may clean the enclosure of your patient station with a soft cloth or a cloth dampened with tap water and a mild detergent.
2. To clean the video screen you may wipe it with a soft cloth. The use of house hold cleaning agents is not recommended.
3. Prevent direct sun light from shining onto the video screen.


B. Administration Setup

1. You must configure each peripheral that will be used with your NX Patient station. At the administration Setting screen you must select each device and then click on the Register button.

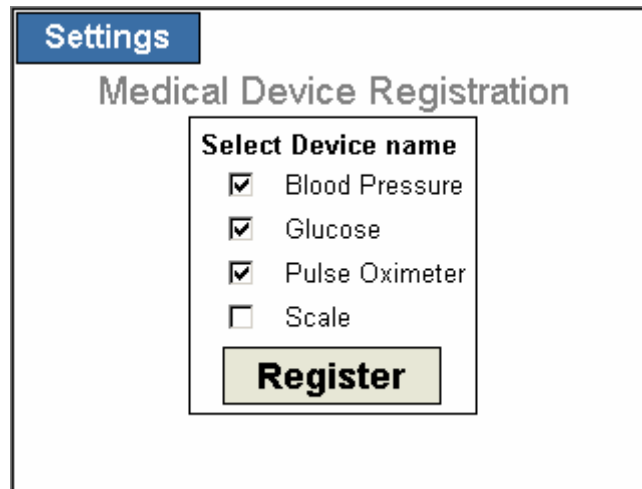

The screenshot shows a 'Settings' window with a 'Medical Device Registration' section. Inside this section is a box titled 'Select Device name' containing four items: 'Blood Pressure' (checked), 'Glucose' (checked), 'Pulse Oximeter' (checked), and 'Scale' (unchecked). Below this box is a 'Register' button.

If a device is already registered it will have a check mark in the Registered box.

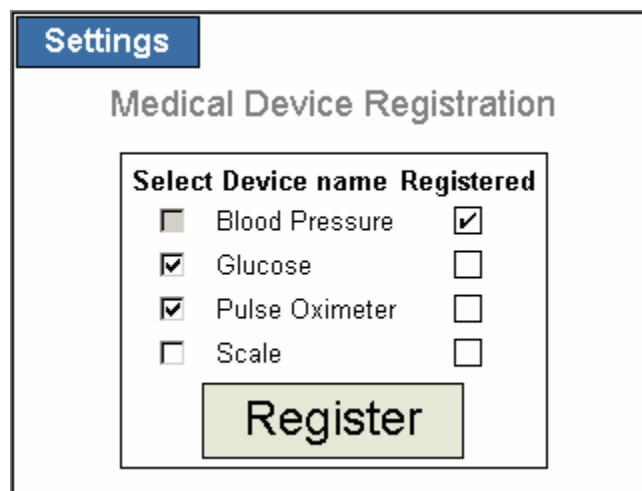

The screenshot shows the same 'Settings' window. The 'Medical Device Registration' section now has a box titled 'Select Device name Registered'. It contains four items, each with two checkboxes: 'Blood Pressure' (unchecked on the left, checked on the right), 'Glucose' (checked on the left, unchecked on the right), 'Pulse Oximeter' (checked on the left, unchecked on the right), and 'Scale' (unchecked on the left, unchecked on the right). Below this box is a 'Register' button.

Press Register to proceed with the device registration and follow direction shown on the screen.

Settings

Medical Device Registration

Please remove the batteries from the following devices:

- Glucose
- Pulse Oximeter

Press the OK button when you have done so.


OK

Settings

Medical Device Registration

Please insert the batteries back into the following devices:

- Glucose
- Pulse Oximeter



Please wait while the device registration is performed.

When the device registration is complete, all selected devices will be checked as Registered.

Settings

Medical Device Registration

Select Device name Registered


<input checked="" type="checkbox"/> Blood Pressure	<input checked="" type="checkbox"/>
<input checked="" type="checkbox"/> Glucose	<input checked="" type="checkbox"/>
<input checked="" type="checkbox"/> Pulse Oximeter	<input checked="" type="checkbox"/>
<input type="checkbox"/> Scale	<input type="checkbox"/>

Register

2. You must also set up an ISP account for your unit.
3. Patient ID will also be entered in the administration section.

C. Criteria Selection Screen


1. Once your patient station is active, you will be presented with the selection screen. You will make a selection by clicking on the appropriate box as seen below.

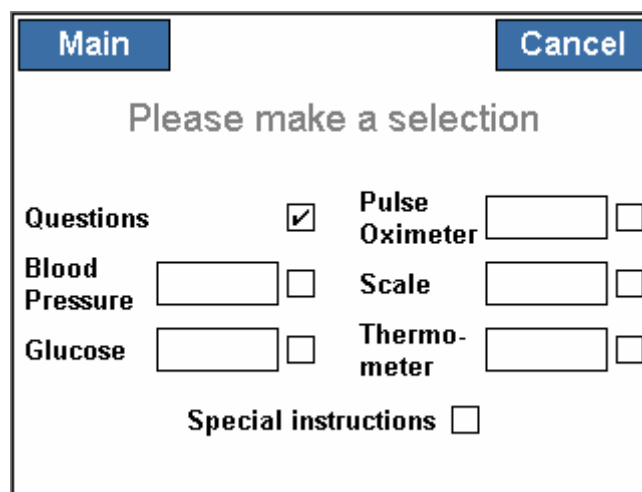

A screenshot of a software window titled "Criteria Selection Screen". It has a "Main" button in the top left and a "Cancel" button in the top right. The text "Please make a selection" is centered. Below this, there are several options with checkboxes and input fields: "Questions" with a checked checkbox, "Blood Pressure" with an empty input field and an unchecked checkbox, "Glucose" with an empty input field and an unchecked checkbox, "Pulse Oximeter" with an empty input field and an unchecked checkbox, "Scale" with an empty input field and an unchecked checkbox, "Thermo-meter" with an empty input field and an unchecked checkbox, and "Special instructions" with an unchecked checkbox.

D. Steps For Using the Blood Pressure/Pulse Meter

1. When you have chosen the Blood Pressure selection, the following screen will appear.

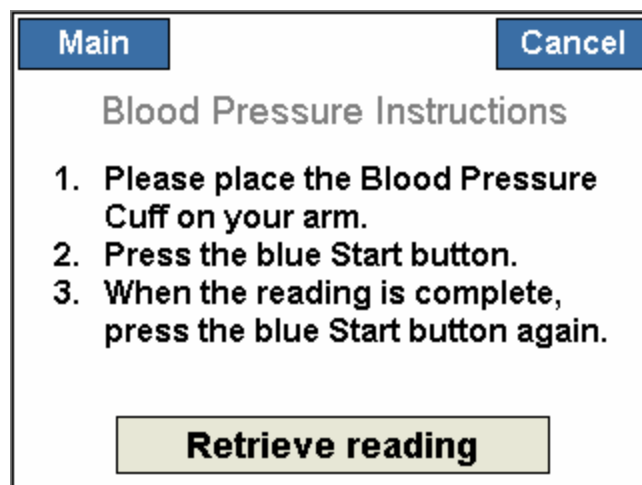

A screenshot of a software window titled "Blood Pressure Instructions". It has a "Main" button in the top left and a "Cancel" button in the top right. The text "Blood Pressure Instructions" is centered. Below this, there are three numbered instructions: "1. Please place the Blood Pressure Cuff on your arm.", "2. Press the blue Start button.", and "3. When the reading is complete, press the blue Start button again." At the bottom, there is a large button labeled "Retrieve reading".

2. Follow the instructions by placing the blood pressure cuff on your arm as previously shown by your healthcare provider. The tubing must be placed over the inner aspect of your arm and extend downward from the lower end of the cuff toward your hand.
3. Once a reading is displayed on your blood pressure meter, press the large blue button on the meter. This will turn your meter off and save the readings within the meter. You can then remove the blood pressure cuff.
4. Press the Retrieve Reading button to save the reading within your monitoring unit.

Helpful Hints:

1. It is important that you remain relaxed during your blood pressure check. Anxiety, movement, or talking may cause an increase in your blood pressure reading.
2. It is best to place the cuff directly over your skin. However, it may be placed over thin clothing.
3. It is important that friends and family members do not use your blood pressure meter. Their results will also be stored within the meter and transferred into your file when your healthcare professional calls.

E. Steps for Using the Scale

1. When you have chosen the scale selection the following screen will appear.

This screenshot shows a software interface for a scale. At the top left is a blue button labeled 'Main' and at the top right is a blue button labeled 'Cancel'. The word 'Scale' is centered in a large, light gray font. Below it, the text 'Please take a reading' is centered in a bold black font. At the bottom, the text 'Enter the reading' is centered in a bold black font.

2. If using a Wireless Adaptor, step on the scale and choose “Please take a reading”. If not using a Wireless Adaptor, choose “Enter the readings”. The following screen will appear.

This screenshot shows the next step in the scale's software interface. It features the same 'Main' and 'Cancel' buttons at the top. The word 'Scale' is centered. Below it, the text 'Enter a reading' is centered in a bold black font. Underneath this text is a small, empty rectangular input box. Below the input box is a numeric keypad consisting of three rows of buttons: the first row has buttons for 0, 1, 2, 3, and 4; the second row has buttons for 5, 6, 7, 8, and 9; and the third row has buttons for 'Enter', '.', and 'Clear'.

3. Enter in your weight by choosing the correct numbers. Once all numbers are entered, click on the enter button.

4.

F. If Something Goes Wrong

If, at any time, you feel your equipment is not working properly, please notify your healthcare provider immediately.

NX Patient Station

Revision Table:

Date	Change	Rev.	Appr v