



## Guardian Systems Pendant Users Guide

**The Guardian Systems Wireless Transmitter (GSY 4920-1, GSY 4912-1) is integrated into a Pendant encapsulation to provide an individual the ability to summon help.**



GSY 4920-2



GSY 4912-2

### Function

The user should press the center colored button on the Guardian Systems Wireless Pendant (GSY 4920-2, GSY 4912-2) when assistance is needed. The light on the device face will illuminate to indicate that the signal is being sent to the central attendant.



## **Guardian Systems Nurse Pullcord Station USER GUIDE**


***The Guardian Systems Nurse Pullcord station  
offers the latest in technology and ease of use.***

### **Function**

The Pull Cord Emergency Station contains one of the Guardian Systems Wireless Transmitter (GSY 4920-1, GSY 4912-1) and is intended for use in toilets, bath, tub rooms and other areas where priority calls require immediate attention. These consist of a pull cord, two-way toggle switch and a wireless transmitter. This station is intended for use in patient areas. Activating the pull cord by pulling down on the pullcord transmits a priority request for assistance. This priority call can only be cancelled at the point of origination.

### **Standard Features**

- Easily operated, heavy-duty, color-coded pull switch.
- Durable, five-foot long nylon Pullcord with an
- Easy-to-grasp knob, adjustable in length.
- Fire retardant, high impact, molded thermoplastic
- One-gang off-white faceplate.
- Mounts in a standard one-gang box made from plastic or fiberglass.

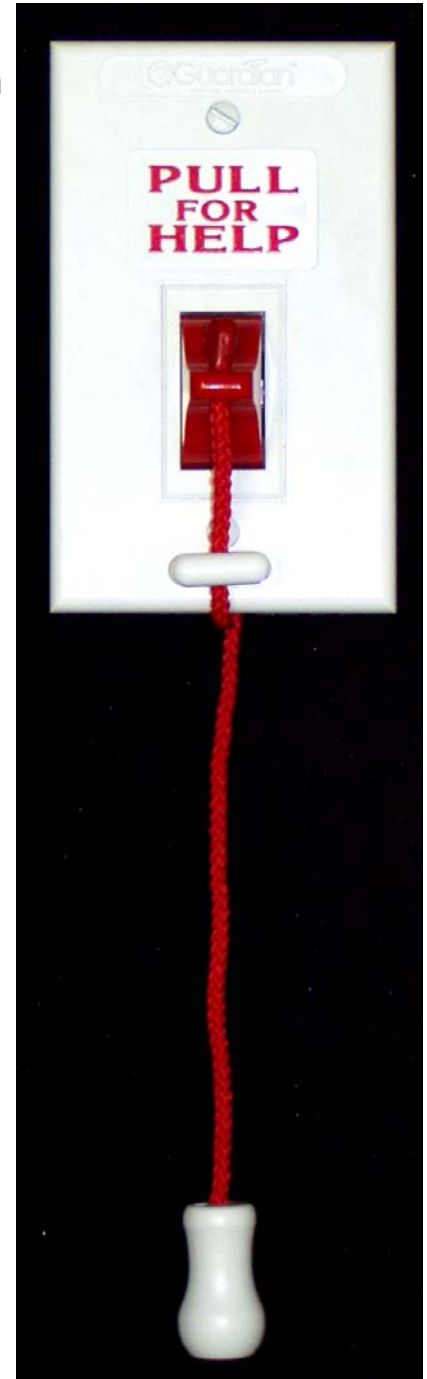

## Guardian System Waterproof Alarm Buttons

**The Guardian System Waterproof Alarm Button May be  
Configured in Three Convenient Styles\***



### Function

The #04901-60 Alarm Button transmits an emergency call for assistance signal to one or more #03500 Receivers. Each Alarm Button transmits a unique code which is defined for a particular resident. This unique code allows the system to immediately send the resident's name or room number, and location, to care givers.

### Dimensions

1  $\frac{11}{16}$ " H x 1  $\frac{3}{16}$ " W x  $\frac{1}{2}$ " D  
(Plus the dimensions of the mounting)

### Battery Life

Shelf Life – Approx. 10 Years

Activations – Over 100,000

### Mounting\*

- Necklace
- Clip
- Wrist band

\* All three attachments included with each Alarm Button.