The wireless control's "association" is preset at the factory. This procedure is only required if there is a malfunction, or if you are attempting to use the controller with a chair other than the one it was originally associated with.



Association Procedure:

- A. Using the wired hand control, press & hold the **Stop** & **Home** buttons simultaneously. (after two seconds you will hear a single "beep")
- B*. Using the wireless hand control, press & hold the Stop & Home buttons until you hear three "beeps". (one long "beep" indicates the association failed. Repeat Steps A & B)
- **C.** Check for proper operation of wireless hand control.
- * Note: Step B must be performed within 10 seconds of Step A, failure to do so will cause the procedure to fail.


Specifications

Range of Operation 20 ft (6 m) radius

Batteries:

Hand Control...... AA (qty: 2)

(Note: Battery access cover is located on back of controller)

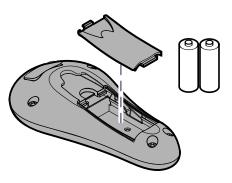

DA191300i

Troubleshooting

• When any control button is pressed, that controller sounds a single "beep". (All functions operate)

Cause: Low Battery

Solution: Replace batteries

• When any control button is pressed, nothing happens. (No "beeps", no movement, etc.)

Causes: 1. Batteries completely dead 2. Wireless controller has lost "association" to chair. *

Solution: 1. Replace batteries

2. Perform Association Procedure*.

* The wireless controller will only function with the chair it has been "associated" with. This prevents unwanted movement in locations with more than one chair.

If further service is required, contact your authorized Midmark dealer.

Compliance Information

FCC Compliance Statement

This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

Any modifications to this product may violate the rules of the Federal Communications Commission and make operation of the product unlawful.

Declaration of Conformity



Midmark Corporation declares that 002-1090-00 conforms to the applicable requirements of the Radio Equipment & Telecommunications Equipment Directive 1999/5/EC, Annex II.

Serial Number: ALL to present

Quality Assurance Manager

Gene Harshbarger Midmark Corp. 60 Vista Dr. P.O. Box 286 Versailles, OH 45380 U.S.A.

077-0520-00

Limited Warranty

SCOPE OF WARRANTY

Midmark Corporation ("Midmark") warrants to the original purchaser its new Alternate Care products and components (except for components not warranted under "Exclusions") manufactured by Midmark to be free from defects in material and workmanship under normal use and service. Midmark's obligation under this warranty is limited to the repair or replacement, at Midmark's option, of the parts or the products the defects of which are reported to Midmark within the applicable warranty period and which, upon examination by Midmark, prove to be defective.

APPLICABLE WARRANTY PERIOD

The applicable warranty period, measured from the date of delivery to the original user, shall be one (1) year for all warranted products and components.

EXCLUSIONS

This warranty does not cover and Midmark shall not be liable for the following: (1) repairs and replacements because of misuse, abuse, negligence, alteration, accident, freight damage, or tampering; (2) products which are not installed, used, and properly cleaned as required in the Midmark "Installation" and or "Installation / Operation Manual for this applicable product. (3) products considered to be of a consumable nature; (4) accessories or parts not manufactured by Midmark; (5) charges by anyone for adjustments, repairs, replacement parts, installation, or other work performed upon or in connection with such products which is not expressly authorized in writing in advance by Midmark.

EXCLUSIVE REMEDY

Midmark's only obligation under this warranty is the repair or replacement of defective parts. Midmark shall not be liable for any direct, special, indirect, incidental, exemplary, or consequential damages or delay, including, but not limited to, damages for loss of profits or loss of use.

NO AUTHORIZATION

No person or firm is authorized to create for Midmark any other obligation or liability in connection with the products.

THIS WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

SF-1487 REV. A1

General Information

Attention: Wired Hand & Foot Controls

Wired hand & foot controls are shipped standard with all models. A wired controller (either hand or foot) is required to perform the Association Procedure. This procedure may be required in the event of a malfunction, do <u>not</u> discard the wired controllers.

(Refer to: Association Procedure in this User Guide)

Intended Use

This product is intended to control patient positioning while providing better patient access by eliminating the cord from the hand & foot controls.

Authorized CE Representative

Countries in the EEC should direct all questions, incidents, and complaints to Midmark's authorized CE Representative listed below:

Promotal 22, Rue de St. Denis 53500 Ernee, FRANCE Phone: 33-2-430-517-76 Fax: 33-2-430-569-10

Calling for Service

If service is required, contact your Midmark dealer or to locate your nearest service provider, log onto: http://www.midmark.com

To contact Midmark directly:

1-800-Midmark (1-800-643-6275) 8:00 am until 5:00 pm Mon-Fri (EST)


Midmark Corporation
60 Vista Drive
Versailles, OH 45380-0286
Phone: 937-526-3662
Fax: 937-526-5542
midmark.com

Because we care.



User's Guide

Wireless Hand 002-1090-00

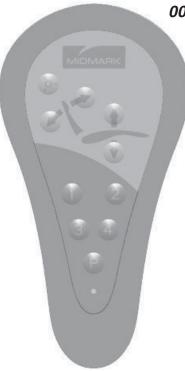

This guide outlines basic operation of the wireless hand / foot control and the associated base station accessory. If service is required, contact your authorized Midmark dealer.

Dealer (name / phone):	
Date of Purchase:	

© Midmark Corporation 2007 003-1764-00 Rev. A