

ScripTalk[®]

STATION 3



Quick Start Guide

STS3V1

Congratulations on your new ***ScripTalk Station 3***! This Quick Start Guide will help you get started on your way to safety, privacy and independence in managing your prescription medications.

What's In the Box?

***ScripTalk Station 3
Reader***

Sample Bottle

***USB-A to
USB-C Cable &
USB-A Power Adapter***



The Buttons



**On / Off
Volume
Switch**

Three buttons allow you to operate the reader and access the information on your prescription labels.

About the Electronic Label



The talking label may be found on the **bottom** or the **side** of your bottle or container.



Due to packaging or medicine content, your label may be attached as an **annex tag**.

To access the electronic label, lift it away from the package when scanning.



Battery Level

The device will report the battery level during start-up when below 20% charge.

To check the battery level at any time, unplug the device and turn it on.

Let the device fully start up. Press **all three buttons** (PREV, READ and NEXT).

Among other information spoken, the device will report the current charge level of the battery.

***Press
All Three
Buttons***



Recharging the Battery

Plug the charge adapter into a wall outlet, then plug the USB-C cable into the port on the **left side** of your ScripTalk.



When charging is complete,
your ScripTalk will let you know
“Battery Charging Completed.”

Max charge should take no more than
5-6 hours from a fully depleted level.

Power and Volume Control

Turn the thumbwheel switch clockwise to **power on** the unit.



To adjust the **volume**, turn the thumbwheel.



To **shut off** the unit, turn the thumbwheel counter-clockwise until you hear a click.



On-Board Help

To listen to the instructions on how to use the unit, press and hold **READ** for about three seconds. To stop the instructions, press **READ** again.



Press **PREV** or **NEXT** to move through the different topics.




Reading a Prescription Label

- To scan, press **READ**. You will hear beeping.
- Place your label on the top of the reader.
- Listen to the label information.
- To stop the speaking, press **READ**.



Scrolling Through the Information

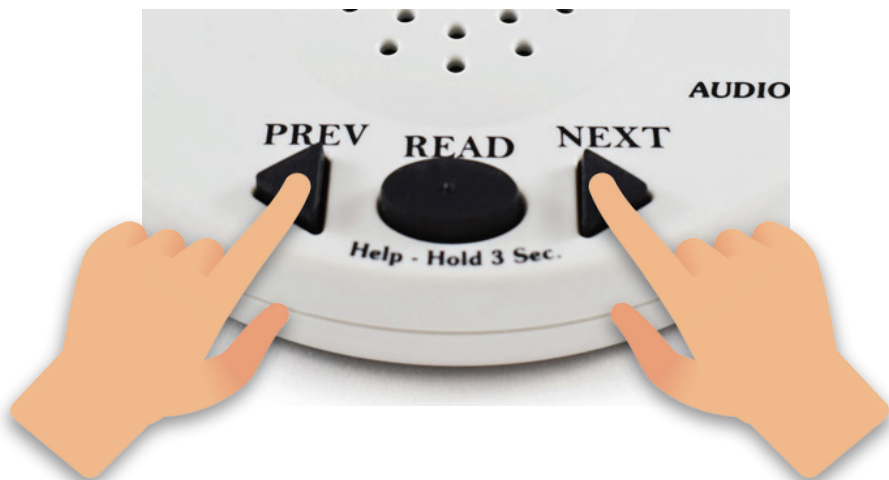

While the unit is speaking the label information, you can press **PREV** or **NEXT** to move through the information or to repeat an item.

Troubleshooting



Place your prescriptions on the reader individually. The unit can only read **one label at a time.**

If the unit cannot find or read a label, it will give you a warning. Have a sighted friend or a family member read the label for you, and talk to your pharmacist to see about a new label.

Changing the Voice Speed

**Press
Three
Seconds**



**Then
Select**



To change the voice speed, press and hold **PREV** or **NEXT** for three seconds, then press **PREV** or **NEXT** to find the speed you like.
Press **READ** to set the speed.

Save



Spell Mode

You can have the information spelled out if you don't understand a word. Select the slowest setting in voice speed. It will say "**Spell Mode.**" Press **READ** to select, then scan your label. Reset voice speed to exit.



Accessories

A handy travel bag is available for your convenience. The insulated zippered bag is designed to hold your reader as well as your medications. And it's perfect for trips or organization of your medications.



ScripTalk Travel Case

\$24.99

Headphones

\$15.99

FCC Compliance Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

CAUTION: The grantee is not responsible for any changes or modifications not expressly approved by the party responsible for compliance. Such modifications could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

For more information or assistance,
please contact us at:

En-Vision America

825 4th St W
Palmetto, FL 34221

Toll Free: (800) 890-1180

Fax: (309) 452-3643

frontdesk@envisionamerica.com
www.envisionamerica.com



En-Vision
AMERICA®

STS3V1