

- set to the correct input signal.
6. Remove viewfinder from the AIRTRAQ device and attach camera until retention clip is engaged.

II. Laptop Computer

1. Acquire a video capture device suitable for the computer being used, such as a portable device or PCMCIA card (Pinnacle, AverMedia, etc.)
2. Install on your laptop the software provided with the video capture device.
3. Connect the video capture device to the laptop computer, usually through a USB 2.2 port or PCMCIA.
4. Connect the video cable to the camera Video Output jack.
5. Connect the RCA jack to the video capture device in its Video Input jack. Some video input devices may require a different adapter. These are available at most electrical stores.
6. Switch the camera ON. The computer will recognize the device.
7. Open the software and proceed as indicated in the software's instructions.
8. The software may need to be customized to indicate that the input is set to Composite.

RECORDING

CD/DVD/VHS Recorder

1. Connect the video cable to the camera's Video Output jack.
2. Connect the RCA jack to the CD/DVD/VHS recorder in its Video Input jack.
3. Connect the CD/DVD/VHS recorder to an external monitor, using the appropriate cable.

CLEANING

The AIRTRAQ Clip-On camera is classified as a non-critical device. Recommended cleaning is low-level disinfection between each patient use. Be sure to follow your institution's specific cleaning procedures in consultation with this manual. Turn the camera OFF and remove from the AIRTRAQ. To wipe down the exterior of the camera, use clean cotton gauze pads that are saturated with enzymatic cleaning solutions (e.g. ENZOLTM Enzymatic Detergent), neutral pH soap and water, sodium bicarbonate solution (8-10%), Cidex™ OPA 0.08% peracetic acid solution, or isopropyl alcohol disinfection agents. Use soft brushes with the cleaning solution to remove any residue from areas that cannot be reached with the gauze pads. All visible contaminant or foreign matter must be removed. Be careful to keep running liquid off the camera's surface. Blot dry using an individual surgical towel.

CAUTION: Do not autoclave, rinse under run-

ning water, or soak in liquids. Avoid touching the lens of the camera and do not allow any liquid or moisture to enter the camera.

MANUFACTURER'S WARRANTY

King Systems Corporation warrants this product against defects in material and workmanship under normal use and service for one year from the original date of purchase. King Systems Corporation, at its option, shall repair or replace the defective unit covered by this warranty.

In order to keep this warranty in effect, the product must have been handled and used as prescribed in the instructions accompanying this warranty. This warranty does not cover any damage due to accident, misuse, abuse, or negligence.

Repair or replacement, as provided under this warranty, is your exclusive remedy. King Systems Corporation shall not be liable for any incidental or consequential damages.

AIRTRAQ CLIP-ON CAMERA

Supply Voltage: Rechargeable Lithium

Battery 7.2 V

Supply Current: 55 mA

Power Consumption: 0.8 Watts

Operating Temperature: from -10°C to 60°C

Dimensions: 85 mm x 80 mm x 55 mm

Weight: 120 grams

CAMERA STAND-ALONE BATTERY CHARGER

Supply Voltage: 110 V – 240 V

Output: 8.4 Volts, 550 mA

Dimensions: 72 mm x 60 mm x 25 mm

Weight: 40 grams

Manufactured in China for
King Systems Corporation

© 2007 King Systems Corporation
15011 Herriman Blvd
Noblesville, IN 46060
317-776-6823
www.kingsystems.com

Caution: Federal law restricts this device to sale by or on the order of a physician.

US Patent No 6,843,769

AIRTRAQ is the registered trademark of Prodol Meditec S.A.
Represented Trademarks are owned by their respective companies.