

Wireless Gateway CFC105

User Manual
Printed in Germany



Wireless RF Gateway
Model CFC105
Part no. 590087
User Manual

© 2008 Biocomfort Inc. All rights reserved.

The Biocomfort mark and logo are the exclusive properties and trademarks of Biocomfort Diagnostics GmbH & Co. KG in the United States and other countries. Windows® is a trademark or registered trademark of Microsoft Corporation in the United States and/ or other countries.

All other brands and marks referenced herein are the properties of their respective owners.

Biocomfort Diagnostics GmbH & Co. KG is the owner of all right, title, and interest, including copyright, in and to the software contained in the software packet included with this box and all software and firmware installed on any hardware included in this box. The software and firmware is protected by copyright under the U.S. and foreign copyright laws.

Limited Product Warranty Information see Chapter 5

Software License Information see Chapter 7

Disclaimer of Warranties see Chapter 8

Limitation of Liability and Exclusion of Statutory Remedies see Chapter 9

Table of Contents:

1	Safety Information	6
1.1	Intended use	7
1.2	Warnings	7
2	Using the Wireless RF Gateway for the first time	11
2.1	Minimum System requirements	11
2.2	Installing the software	11
3	Technical Data	12
4	Electromagnetic Compatibility	13
5	Symbols and abbreviations	15
6	Limited Product Warranty Information	16
7	Software License Information	16
8	Disclaimer of Warranties	17

9	Limitation of Liability and Exclusion of Statutory Remedies	19
----------	--	-----------

Contents of Package

- Wireless RF Gateway CFC105
- CD ROM
- User manual
- Biocomfort Limited Product Warranty
- Biocomfort End-User License Agreement

Info at www.biocomfort.com

For help getting started or any questions you may have, please do not hesitate to give us a call at 1-866-294-8267 (available from 9 a.m. to 5 p.m. Monday through Friday EST)

1 Safety Information

Dear Customer,

we are pleased that you have chosen this Biocomfort product.

Please read these safety and operation instructions before using this Biocomfort product or any accessories provided with this product or allowing any other person to use this product. When using this Biocomfort product, these basic safety precautions and warnings must be followed to reduce the risk of injury to your person and health, the risk of damage to the Biocomfort product and the risk of injury or damage to other persons or property. Retain all safety, operating and use instructions for future reference.

If you allow any other person to use this product, you must ensure that the person has

received and read and understands this user manual, including in particular the safety instructions herein.

Keep this user manual in close proximity to the equipment.

All warnings on the product and in the user manual must be strictly adhered to. You are responsible to ensure that any other person using this product adheres to these warnings and instructions.

All operating, use and assembly instructions must be followed by you. Failure to do so could result in health injury, bodily harm and/or damage to the equipment or other property.

The product is only intended for use as described in this manual.

1.1 Intended use

The Wireless RF Gateway CFC105 is designed to manage wireless networks of Biocomfort devices such as a blood pressure monitor, a blood glucose monitor or a diagnostic scale, to configure these devices and to receive the readings measured with these devices.



1.2 Warnings

The symbols below identify and alert the user to the presence of important safety, operating, and service instructions.

WARNING:

Indicates that non-compliance with the safety instructions may lead to health injury or bodily harm of the user or other persons.

CAUTION:

Indicates that non-compliance with the safety instructions may lead to damage to the product or other property and/or to bodily harm of the user or other persons.

WARNING:

This product is only intended for adults, not for children or infants. Do not allow any child or infant to use this product.

WARNING:

This product contains one or more chemicals known to the State of California to cause cancer and birth defects or other reproductive harm.

CAUTION:

Do not open, disassemble or tamper with the product in any way. Opening the device automatically cancels all warranties. If you have technical or troubleshooting questions, please contact us via web, email and phone.

CAUTION:

Never push objects of any kind into the product or accessories through openings as

this action might short-circuit parts and result in a fire or electric shock.

Do not locate or use the product or accessories near water (for example, near a bathtub or a sink, in a wet basement, or near a swimming pool). Never immerse the product in any liquid and never spill liquid of any kind on the equipment or cables or cords.

CAUTION:

Do not use liquid, aerosol cleaners, or solvents on or near the product or any accessory. Clean only with a dry cloth. Disconnect any cables from the computer and before cleaning either the product or any accessory.

CAUTION:

This product should never be placed near or over a radiator or heat register.

CAUTION:

Do not use mobile phones, microwave ovens or other instruments that emit an electromagnetic field near this device as they may cause this device to malfunction.

CAUTION:

Use only with Biocomfort parts and accessories with this device.

Parts and accessories not approved by Biocomfort for use with this device may damage the unit.

CAUTION:

Do not activate or use the wireless device where the use of wireless devices (e.g. phones) is prohibited or when it may cause interference or danger.

CAUTION:

The wireless device may not work properly if it is near a wireless device or equipment that has not been approved by or does not comply with the regulations of the Federal Communications Commission (FCC). Any such device or equipment may interfere with the operation of the Biocomfort wireless device.

CAUTION:

Do not use the wireless device in hospitals or near other medical equipment or devices that is not manufactured by Biocomfort. Follow any restrictions imposed by hospitals on the use of wireless devices.

CAUTION:

Do not use the wireless device on aircraft. Wireless devices can cause interference in aircraft. Follow instructions by authorized aircraft personnel.

WARNING:

Do not use the wireless device at a refueling point. Do not use near fuel or chemicals.

WARNING:

Do not use the wireless device near blasting. Follow any restrictions. Do not use the device where blasting is in progress

WARNING:

Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment.

2 Using the Wireless RF Gateway for the first time

Caution:

Please install the software before you connect the wireless gateway with the computer.



2.1 Minimum System requirements

PC/Laptop

- Windows® 2000/XP/Vista
- PC min. 400 MHz or higher
- USB 1.1 or higher
- CD-ROM drive
- ActiveSync (only for PDA installation)

PDA

- Windows® Mobile 2003 or higher
- CompactFlash Type II slot

2.2 Installing the software

As the Wireless RF Gateway can be part of various Product Packages the installation of the drivers for the CFC105 may be different from case to case. Therefore please refer to the software installation instructions of the package the CFC105 came with.

3 Technical Data

Power Supply

Operating Voltage	5 VDC
Operating current	48 mA at 3 VDC

Radio

Operating frequency	2.4 GHz
Transmit power	0 dBm (max. EIRP)
Receive sensitivity	-85 dBm (typ.)
Range (Line of Sight)	max. 20 m
Data rate	250 kbps
FFC ID:	VUB590087

PC interface

Type	USB 1.1
------	---------

Mechanical data:

Dimension (L x W x D)	1,68" x 1,43" x 0.2/0.43"
Weight	0.6 oz.

Operating conditions

Temperature range	32 °F ... 104 °F
Rel. Humidity	20 % ... 85 %
No mechanical shocks or vibrations.	

Storage and transport conditions

Temperature range	-4 °F ... 140 °F
Rel. Humidity	10 % ... 95 % (no condensation)

4 Electromagnetic Compatibility

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- This device may not cause harmful interference.
- This device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the manufacturer's instructions, may cause interference harmful to radio

communications. There is no guarantee, however, that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver
- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio or TV technician for help.

Warning:

Changes or modifications to this unit not expressly approved by the party responsible

for compliance could void the user's authority to operate this equipment.

EU Regulatory Conformance:


Biocomfort hereby declares that the Wireless RF Gateway CFC105 is in compliance with the essential requirements and other relevant provisions of EMC-Directive 89/336/EEC and R&TTE-Directive 1999/5/EC. The Declaration of Conformity is available for viewing at the following location in the EU Community:

Biocomfort Diagnostics GmbH & Co. KG
Bernhaeuser Strasse 17
73765 Neuhausen a.d.F.
Germany

5 Symbols and abbreviations

On the device, packaging material and in the operating manual you will find the symbols shown below with the following meanings:

The serial number of the device is composed as follows and can be found under the barcode.

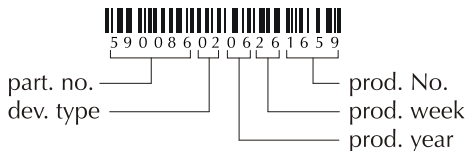

Please follow the operating manual

REF

Part/Order number

FCC ID

FCC registration number



CE symbol



Symbol for separate collection of electric and electronic devices (EU only)



FCC symbol

SN

Serial number

6 Limited Product Warranty Information

This product (other than the software and documentation) is subject to the Biocomfort One-Year Limited Warranty separately included herewith and also available from Biocomfort or any authorized Biocomfort dealer or from Biocomfort's website at <http://www.biocomfort.com/support>.

If you do not agree with, and do not want to be bound by, the terms of the Biocomfort one-year limited warranty, return the box and its contents, including the hardware and accessories unused and the unopened software packet, to Biocomfort, or to the authorized Biocomfort dealer where you purchased the product, within fourteen (14) days of purchase, together with proof of purchase, and your full purchase price will

be refunded. By using this product or letting the 14-day period lapse, you agree to the terms of the Biocomfort one-year limited warranty.

7 Software License Information

The software in this box, and any software or firmware installed on the hardware in this box, is subject to the terms and conditions of the Biocomfort End-User License Agreement separately included herewith and also available from Biocomfort or any authorized Biocomfort dealer or from Biocomfort's website at <http://www.biocomfort.com/support>.

If you do not agree with, and do not want to be bound by, the terms and conditions of the Biocomfort End-User license agreement, return the box and its contents, including the

unopened software packet, together with the hardware and accessories unused, to Biocomfort or to the authorized Biocomfort dealer within fourteen (14) days of purchase, together with proof of purchase, and your full purchase price will be refunded. By opening the sealed software packet inside this box or letting the 14-day period lapse, you agree to the terms of the Biocomfort end-user license agreement.

8 Disclaimer of Warranties

All software and firmware installed on any hardware sold to you, or furnished with or referenced in, this user manual is provided “as is” without any warranty of any kind, whether express or implied.

This document is provided “as is” and Biocomfort Inc. (and Biocomfort Diagnostics GmbH & Co. KG) assumes no responsibility for any typographical, technical, or other inaccuracies in this document. Biocomfort reserves the right to periodically change information that is contained in this document; however, Biocomfort make no commitment to provide any such changes, updates, enhancements or other additions to this document to you in a timely manner or at all.

BIOCOMFORT AND ITS DIRECT, INDIRECT AND ULTIMATE PARENT COMPANIES, ITS OTHER AFFILIATES, OR THEIR RESPECTIVE LICENSORS, SPECIFICALLY DISCLAIM ALL WARRANTIES, EXPRESS OR IMPLIED, INCLUDING WITHOUT LIMITATION IMPLIED WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE, TITLE AND NON-INFRINGEMENT, WITH RESPECT TO THE SOFTWARE, INCLUDING ALL THE COMPUTER PROGRAMS, SOURCE CODE CONTAINED THEREIN, FIRMWARE, AND ALL IMAGE OR AUDIT FILES OR OTHER CONTENTS OF ANY CD-ROM OR OTHER PHYSICAL MEDIA, INSTALLED ON ANY HARDWARE SOLD TO YOU, OR FURNISHED WITH OR REFERENCED IN THIS USER MANUAL, AND/OR THE TECHNIQUES DESCRIBED IN THIS USER MANUAL AND OR THE CONTENTS OF THIS USER MANUAL. BIOCOMFORT DOES NOT WARRANT THAT THE FUNCTIONS CONTAINED IN THE SOFTWARE OR DESCRIBED IN THIS USER MANUAL WILL MEET YOUR REQUIREMENTS, APPLICATIONS, USES OR PARTICULAR PURPOSES, WILL HAVE ANY BENEFICIAL

EFFECT OR THAT THE OPERATION OF THE SOFTWARE WILL BE ERROR FREE OR THAT THE CONTENTS OF THIS USER MANUAL ARE ERROR FREE. BIOCOMFORT SPECIFICALLY DISCLAIMS ANY WARRANTY TO THE EFFECT THAT THE SOFTWARE AS USED BY YOU WILL REDUCE YOUR STRESS LEVEL, TREAT OR HEAL ANY ILLNESS YOU MAY HAVE OR IMPROVE YOUR HEALTH OR WELL-BEING. THE ENTIRE RISK ARISING OUT OF USE OR PERFORMANCE OF THE SOFTWARE REMAINS WITH YOU.

Some states may not allow a disclaimer of all warranties, express or implied, with respect to software. In that case, the Biocomfort limited product warranty applicable to the hardware sold to you shall apply to the software except that Biocomfort disclaims all implied warranties to the maximum extent permissible by applicable law.

9 Limitation of Liability and Exclusion of Statutory Remedies

To the maximum extent permissible by applicable law, in no event shall Biocomfort or its direct, indirect and ultimate parent companies, its other affiliates, or their respective licensors, be liable for any consequential, indirect, special, incidental, punitive or other damages whatsoever, including without limitation damages for loss of future or anticipated business profits, loss of goodwill, loss of revenue, loss of business opportunity, business interruption, loss of business information, or any other pecuniary loss, and including under any claim of negligence, strict liability, design defect, or other theory, arising from (1) any defect in the software or the use of or inability to use the software or firmware installed on any

hardware sold to you, or furnished with or referenced, in this user manual or from the licensing of the software or firmware by Biocomfort or the provision of any services by Biocomfort or Biocomfort USA, (2) any defect or error in this user manual or any other documentation, even if Biocomfort and/or Biocomfort USA have been advised by you of the possibility of such damages.

To the maximum extent permitted by applicable law, Biocomfort and its direct, indirect and ultimate parent companies, its other affiliates, its suppliers and its contract manufacturers shall have no liability for punitive, special, incidental, consequential or indirect damages resulting from the purchase or use or non-use of any Biocomfort product or any part thereof, from any product defect, from any damage to real or personal property, from any use in disregard of any instructions or any warnings

attached to or furnished with the product, or from breach of warranty, breach of contract, negligence, tort liability, strict liability, or any other legal or equitable theory, and you hereby irrevocably waive all claims to that effect. Excluded damages shall include, but not be limited to, product (re-)installation expenses, loss of anticipated benefits, profits, savings or revenue, loss of goodwill, business interruption damages, loss of time or use of the product, any third party claim, and injury to property, even if Biocomfort knew of the likelihood of such damage.

The maximum liability of Biocomfort, its parent company, affiliates, and suppliers with respect to all applicable express and implied warranties shall not exceed the purchase price amount paid for the Biocomfort product.

Some states do not allow the exclusion or limitation of special, incidental, consequential or indirect damages, so the above limitation and exclusion may not apply to you.

Distributed by:

Biocomfort Inc.
23 Third Avenue
Burlington, MA 01803
USA
Telephone: 1-866-294-8267
E-mail: info@biocomfort.com
Internet: www.biocomfort.com

Manufactured by:

Biocomfort Diagnostics GmbH & Co. KG
Bernhaeuser Strasse 17
73765 Neuhausen a.d.F.
Germany
Telephone +49-7158-980160
E-mail: info@biocomfort.de
Internet: www.biocomfort.de

Biocomfort Inc.

23 Third Avenue
Burlington, MA 01803
USA

Telephone: 1-866-294-8267

E-mail: info@biocomfort.com

Internet: www.biocomfort.com

5300 Explorer Dr.

Mississauga, ON L4W 5G4

Canada

Telephone: 1-905-614-0550

E-mail: info@biocomfort.de

Internet: www.biocomfort.com