

# PTX1 Safety Instruction

## IMPORTANT

### FCC NOTICE

---

This device complies with Part 15 of the FCC Rules, Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

### Caution:

---

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

## CAUTION

---

Use of controls or adjustments or performance of procedures other than those specified herein may result in hazardous radiation exposure.

The laser beam emitted from the optical pickup is invisible. Accordingly:

- **Do not open the optical pickup housing.**

## LASER INFORMATION

---

This product has been designed and manufactured according to IEC 60825-1 on the Safety of laser products.

This Laser product comes under "Class 1 Laser Product".

## IC NOTICE

---

This Class B digital apparatus complies with Canadian ICES-003.

Cet appareil numérique de la classe B est conforme à la norme NMB-003 du Canada.

---

# For your safety

---

The Plectalk has been designed with the utmost consideration for safety. Nevertheless, incorrect use can result in serious injury or death, due to fire or electrical shock. Be sure to observe the following precautions.

## Important Note

Failure to observe the following warnings may invalidate the guarantee and could cause serious injury.

- \* If the Plectalk's operation seems impaired or damaged in any way, please contact local dealer as soon as possible.
- \* If you hear a strange noise, smell smoke or an odor coming from the Plectalk, turn off the power immediately.
- \* Use the Plectalk in a horizontal position only.
- \* Do not allow sand or dust to come in contact with the Plectalk as this may lead to a malfunction and could make repair impossible.
- \* Do not use the Plectalk in locations where strong radio waves are emitted or where there is radiation.
- \* Do not use the Plectalk near electrical equipment such as TVs, radios, or tuners as it may pick up additional noise from these devices.
- \* Do not use a power supply not made for use with the Plectalk or "electronic transformers" of the type designed for overseas travelers.
- \* Do not use or store the Plectalk in any of the following places, as this may cause damage.
  - Inside cars with the windows closed in the hot sun.
  - In direct sunlight or close to heaters, radiators.
  - Locations subject to severe vibration such as on top of a washing machine.

## Danger

Observe these danger warnings:

- \* Do not use the unit during thunderstorms.
- \* Do not damage the power cord of the AC adapter.
- \* Do not use the device in a location exposed to water, such as a bathroom.
- \* Do not insert or remove the power plug with wet hands.

## Warnings

Observe these warnings to prevent possible injury:

- \* Never disassemble, attempt to repair, or modify your player yourself.
- \* If liquids or metal fragments should enter the Plectalk, remove the power plug immediately.
- \* Do not listen at high volumes for extended periods as this can lead to impaired hearing especially when using headphones.
- \* Do not pull the plug out by its cord when removing the power plug from the Plectalk.

## Precautions

If the following precautions are not observed, you could cause damage to the Plectalk, recording media, or your data:

- \* Make sure that the Plectalk, mains adapter, power cord and UK mains plug are securely connected.
- \* The CD rotates at high speed, do not glue anything that is particularly thick, such as a Braille label, onto the surface.
- \* The Plectalk is only compatible with round 12 cm CDs. 8 cm CD and irregularly shaped CDs, such as the business card type, will cause a malfunction and should not be used.
- \* Do not touch the bottom of the Plectalk for an extended period of time. It may become hot, and could cause a low-temperature burn.

## Battery Safety Warnings

Be sure to observe the following precautions to prevent accidents such as burns or fires due to leakage, heating, combustion, or explosion of the battery:

- \* Only use the specified battery that was included with Plectalk. Replacements are only available by Plector or local dealer.
- \* Do not expose the battery to fire.
- \* Do not disassemble the battery.
- \* Do not heat the battery in a microwave oven, on a cooker, or anything similar.
- \* Do not put the battery with metal objects such as coins, hairpins, or necklaces; these can cause a short.
- \* Do not leave the battery in hot locations such as under the hot sun, or in a hot car.
- \* Do not use the battery, if it becomes wet.
- \* If battery fluid gets into your eyes, wash them with water and consult a physician immediately.
- \* If the fluid gets on your body or clothing, flush the area well with water.
- \* Do not subject the battery to shocks, such as dropping or stepping on it.

## Plectalk Care

- \* Wipe dust and dirt from the Plectalk with a soft cloth dampened with a diluted neutral detergent, then wipe dry with a cloth.
- \* Always remove the power cord from the player's outlet when cleaning your Plectalk.

## CD Care

- \* Wipe CD-ROM discs with a clean, soft cotton cloth to remove surface dirt such as fingerprints.
- \* Do not wipe the disc in a circular motion. When cleaning, use a straight-line motion, wiping from the center outwards.