



**ALARME DE POSITION
DU COUCHAGE DU BEBE**

Baby Sleep Posture Monitor



**ALARME DE POSITION
DU COUCHAGE DU BEBE**

Baby Sleep Posture Monitor

**MANUEL D'INSTRUCTIONS
A LIRE ET A CONSERVER**

Instruction Manual

CONTENTS

Know Your Bell Nuit
Package Content

Getting Started
Introduction

Operating Instructions
Main Switch
Music Selection
Day or Night Mode
Luminous Indicators
Pairing Procedure

Precautions
Safety Precautions
Cautions on Water-resistance
Precaution on Usage

Specifications

Warranty

FCC Statement

Trouble Shooting

Know Your Bell Nuit

1. Package Content



Receiver



Baby Clip



Instruction Manual



AAA Battery

2. Battery Replacement for the Receiver



【Important Note】


1. The Baby Clip is designed for single baby use.
2. Always place the Baby Clip into the Receiver when not in use.
3. When the Baby Clip is in low battery, the Receiver is unable to detect baby's movement. Please contact your retailer for new Baby Clip replacement.

Getting Started

Introduction

Bell nuit is a new device that monitors baby's sleeping posture. It is composed of 2 units, the Receiver and the Baby Clip. Attach the Baby Clip to baby's diaper, and the Receiver will automatically alarm if a baby turns his/her tummy with the following postures.


- Sleeps flat on his/her tummy

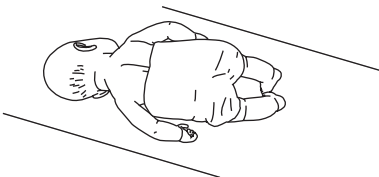

- Attempts to sit up over 80 degrees.



- Turns his/her tummy over 90 degrees

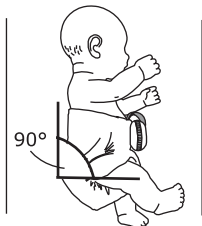

Operating Instructions

Attach the Baby Clip to baby's diaper

Make sure it is firmly attached. (See Figure 1-3)

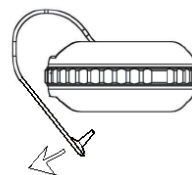


Figure 1

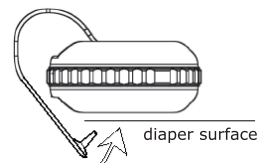


Figure 2

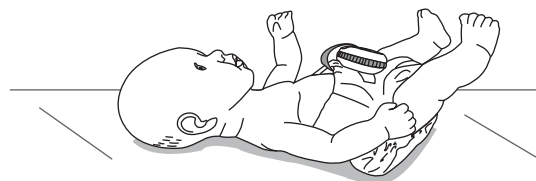


Figure 3

Main Switch

Music Selection

There are 3 types of music. Select different types of music by switching the knot back and forth between "MUSIC" and "OFF" positions.


Day or Night Mode

Day Mode: Standard volume.

Night Mode: Low volume.

Luminous Indicators

There are 2 luminous indicators on the Receiver.

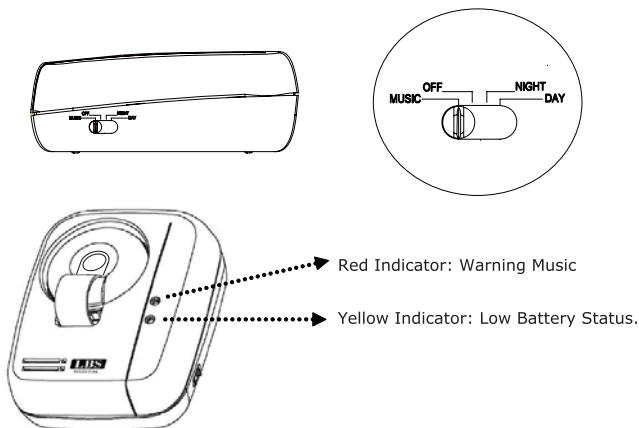

【Note】

1. When the Receiver is in low battery, yellow indicator flashes.
2. When the Baby Clip is in low battery, yellow indicator stays on.

Pairing Procedure

Pairing Procedure is required for every new Baby Clip purchased.

1. Ensure the switch knot is at "OFF" position. (See Fig. 4)
2. Insert batteries and leave the battery cover open. (See Fig. 5)

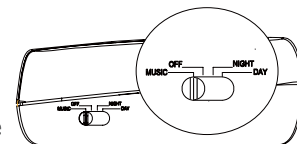


Figure 4

3. Position the blue button of the Baby Clip downwards. (See Fig. 6).
4. Switch the knot to "Day" position.
5. Use pen or needle to press "RESET" button located next to the batteries. (See Fig. 5).

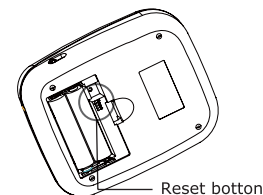


Figure 5

6. The Receiver shall alarm and the red indicator shall lit ON.
7. Turn over the blue button of the Baby Clip upwards. (See Fig. 7).

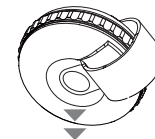


Figure 6

8. Switch the knot to "OFF" position to complete the pairing procedure. (See Fig. 4)

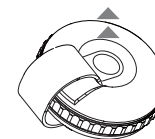


Figure 7

Precautions

Safety Precautions

- Do not tamper with the baby clip as it may cause malfunction of Bell Nuit device.
- To prevent fire or shock hazard, do not expose the device to rain or moisture.
- Do not place objects on top of Bell Nuit device. It could damage the device.
- Do not store the device in the following ambient conditions:
 - Locations exposed to direct sunlight.
 - Locations subject to high temperatures and high humidity.
 - Wet or damp locations where water may get on the device.
 - Near fires or open flames.
 - Locations exposed to strong vibration.

Cautions on Water-resistance (for the Baby Clip only)

The Baby Clip is designed so that it can be used with slight exposure to water, rain, snow and other forms of moisture. When it gets dirty, clean it with a cloth dampened with water. But there are several points on which caution should be observed.

- Do not allow it to come into contact with large amounts of water.
- Do not immerse it in water.

Precaution on Usage

This product cannot replace responsible adult supervision of a baby. It is designed to assist adults in baby supervision. Do not use it for medical.

SPECIFICATIONS

Model Number	TLM-101
Radio Frequency	433.92MHz
Unobstructed Effective Range	50m (maximum)
Power Source	
Baby Clip	23A Battery *1
Receiver	AAA Battery*2
Environmental Conditions	
Storage Temp.	-25°C ~ 75°C
Operating Temp.	10°C ~ 40°C
Humidity	Up to 95%
Warning Tilt Angle	
Sit Up Right	> 80 degrees
Left Tilt	> 90 degrees
Right Tilt	> 90 degrees
Dimensions (LxWxH mm)	
Baby Clip	47 x 47 x 22
Receiver	105 x 84 x 40
Weight	
Baby Clip	27g
Receiver	115g

Note: Duration of the transmission of this device is no greater than one second and the silent period between transmissions is at least 30 times the duration of the transmission and greater than 10 seconds, complies with part 15 regulation section 15.231.

Warranty

Your Bell Nuit device is warranted to be free from defects in materials and workmanship appearing within 1 year from the date of purchase, when used in accordance with the instructions provided. The above warranties extend only to the original retails purchaser.

FCC Statement

You are cautioned that changes or modifications not expressly approved by the part responsible for compliance could void the user's authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- this device may not cause harmful interference and
- this device must accept any interference received, including interference that may cause undesired operation of the device.

Trouble Shooting

Problem	Possible Cause	Solution
The Receiver alarms and the red indicator lit on regardless of position of the Baby Clip.	"Pairing Procedure" incomplete.	Perform "Pairing" Procedure" again.
The Receiver does not automatically alarm and the red indicator does not lit on while the Baby Clip is over 90 degrees left/right or up over 80 degrees position.	Interference from a device with the same frequency as Bell Nuit.	Check your surrounding devices.
	Yellow indicator flashes for low battery status of the Receiver.	Replace both AAA batteries with new ones.
	Yellow indicator stays on for low battery status of the Baby Clip.	Replace the Baby Clip with new one and perform the "Pairing Procedure" again.
Other Malfunction	Unexpected radio signal exists.	Switch the knot at "OFF" position and then press the "RESET" Button.

MANUFACTURER:

digiO2 International Co., Ltd.

4F-13, No.79, Sec.1, Hsin Tai Wu Rd,
Hsi-Chih, Taipei Hsien 221, Taiwan.

DISTRIBUTOR:

LBS Medical

101 Chemin de Ronde 78290 CROISSY SUR SEINE

EU REPRESENTATIVE:

Niish Technologies Ltd.

Office 8, Marcus House, Park Hall Business Village,
Longton, Stoke-on-Trent, ST3 5XA, United Kingdom